



**Renfert**

making work easy

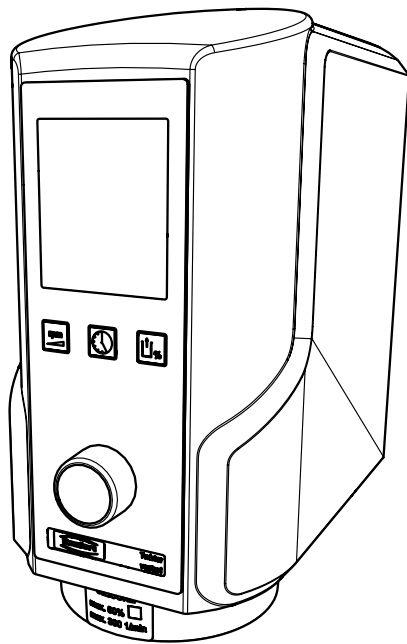

**1827 x000**

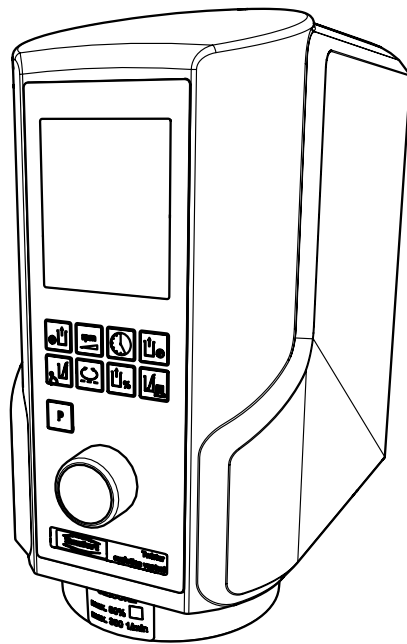

**1829 x000**

# TWISTER venturi TWISTER evolution venturi

Service Instructions

# Contents

## 90003 4675

Filterset • Set of filters • Jeu de filtres • Kit di filtri • Juego de filtros  
Набор фильтров

## 90003 4715

Sicherungsset • Fuse set • Dispositifs de sécurité • Kit di fusibili • Juego de fusibles  
Набор предохранителей

## 90003 4155

O-Ring-Set • Set of O-rings • Jeu de joints thoriques • Kit di guarnizioni  
Juego de anillos tóricos • Набор уплотнительных колец

## 1820 0210 / ..0510 / ..0710 / ..1010

Rührwerk • Paddle • Mixeur • Spatola • Agitador • Смесительный механизм

## 90003 6121 / ..6122 / ..5938 / ..2769 / ..6070 / ..2767 / ..2771 / ..2768

Netzkabel • Power cord • Cordon d'alimentation • Cavo di rete • Cable de red  
Сетевой кабель

## 90003 4137

Zellgummimatte • Prot. rubber mat • Tapis en caout. • Cusc. di gomma cellul.  
Alfombrilla de caucho • Резиновый коврик

## 90003 4700

Kabelsicherung Twister • Cable safety catch for Twister • Arrêt de câble pour Twister  
Fermacavo di sicurezza Twister • Lazo de sujeción para cable de red Twister  
Фиксатор для кабеля Twister

## 90013 2653

EingangsfILTER Set • Input filter Kit • Filtre d'entrée Jeu  
Filtro d'entrata Kit • Filtro de entrada Juego • Входной фильтр

## 216606

Quick-Referenz-Karte Twister evolution / evolution venturi • Quick reference card  
Carte référence rapide • Sommario rifer. rapido • Guía referencia rápida  
Краткий справочник-карта

## 90003 5700

Ablufffilter • Exhaust air filter • Filtre air en sortie • Filtro aria di scarico  
Filtro aire de escape • Фильтр отработанного воздуха

## 90003 4203

Anmischpaddel 65 ml • Wing mixer 65 ml • Mélangeur à ailettes 65 ml  
Pala di miscela 65 ml • Mezclador de paletas 65 ml • Смесительная лопасть 65 мл

## 90003 4679

Frontfolie Twister venturi • Frontfoil Twister venturi • Membrane souple Twister venturi • Membrana comandi p.Twister venturi • Lámina p. elementos de mando Twister venturi • Фронтальная пленка Twister venturi

## 90003 4680

Frontfolie Twister evolution venturi • Frontfoil Twister evolution venturi • Membrane souple Twister evolution venturi • Membrana comandi p.Twister evolution venturi  
Lámina p. elementos de mando Twister evolution venturi • Фронтальная пленка

## 90013 4747

Leiterplatte kpl. 1827 • Printed circuit compl. 1827 • Carta imprimée compl. 1827  
Circuito stampato compl.1827 • Circuito impresso compl. 1827  
Печатная плата в комплекте 1827

**90033 4747**

Leiterplatte kpl. 1829 • Printed circuit compl. 1829 • Carta imprimée compl. 1829  
Circuito stampato compl.1829 • Circuito impresso compl. 1829  
Печатная плата в комплекте 1829

**90003 4745**

Getriebemotor • Geared motor • Moto-réducteur  
Motoriduttore • Motor reductor • Редукторный двигатель

**90013 4162**

Dichtung Motorwelle • Seal for motor shaft • Joint de l'arbre du moteur  
Guarnizione per albero motore • Obturación para árbol del motor  
Уплотнитель вала двигателя

**90003 4719**

Feinfilter • Micro filter • Filtre fin • Microfiltro • Filtro fino • Фильтр тонкой очистки

**90003 4690**

Schaltnetzteil • Power supply • Alimentation sur secteur • Alimentatore di rete  
Fuente de alimentación • Сетевой переключатель

**90003 5989**

Netzschalter • Mains switch • Interrupteur de réseau • Interruttore di rete  
Interruptor de red • Сетевой выключатель

**90002 1527**

Magnetventil • Solenoid valve • Electrovanne • Valvola elettromagnetica  
Válvula magnética • Электромагнитный клапан

**90002 1586**

Belüftungsventil • Air valve • Soupape d'aération • Valvola di aerazione  
Válvula de aeración • Клапан вентиляции

**90003 4273**

Venturidüse • Venturi nozzle • Buse Venturi • Ugello Venturi  
Tobera de Venturi • Сопло Вентури




**90003 4823**

Display Twister • Display Twister • Afficheur Twister • Display Twister  
Pantalla de visualización Twister • Дисплей Twister




## 90003 4675


Filterset • Set of filters • Jeu de filtres • Kit di filtri • Juego de filtros  
Набор фильтров

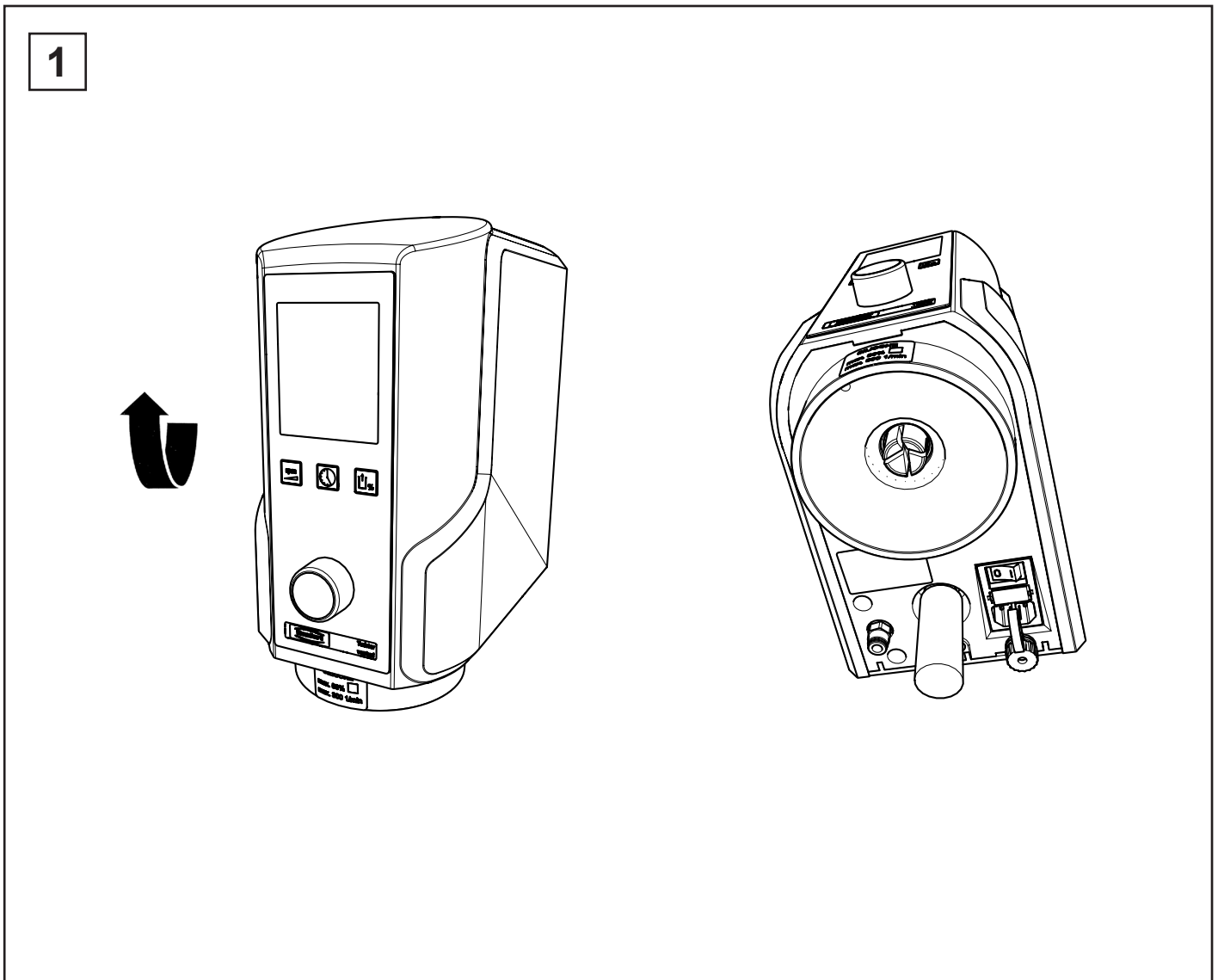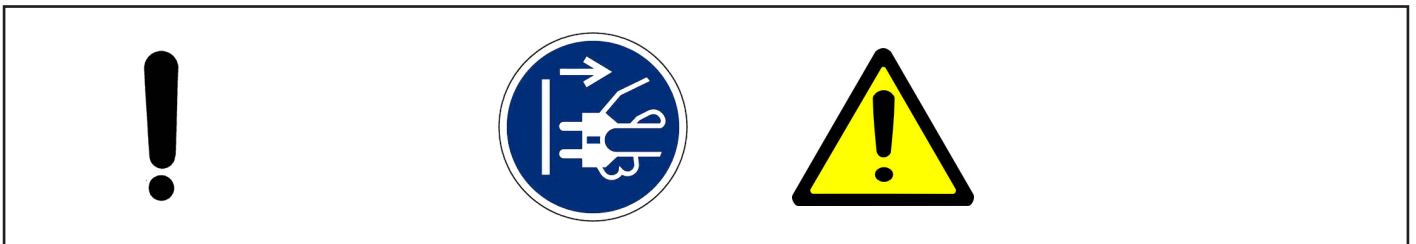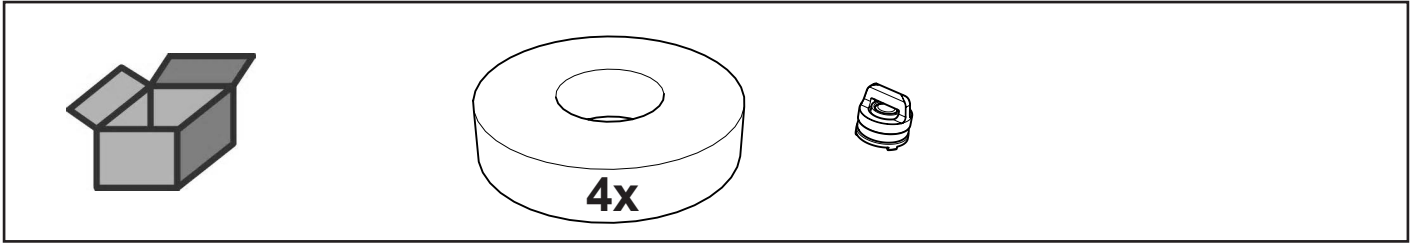



2

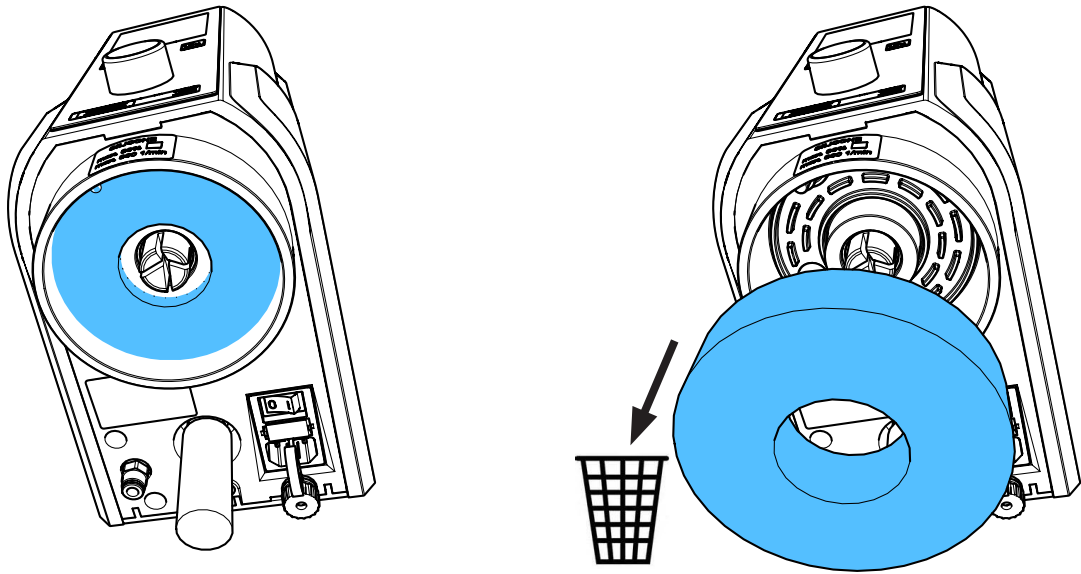



3

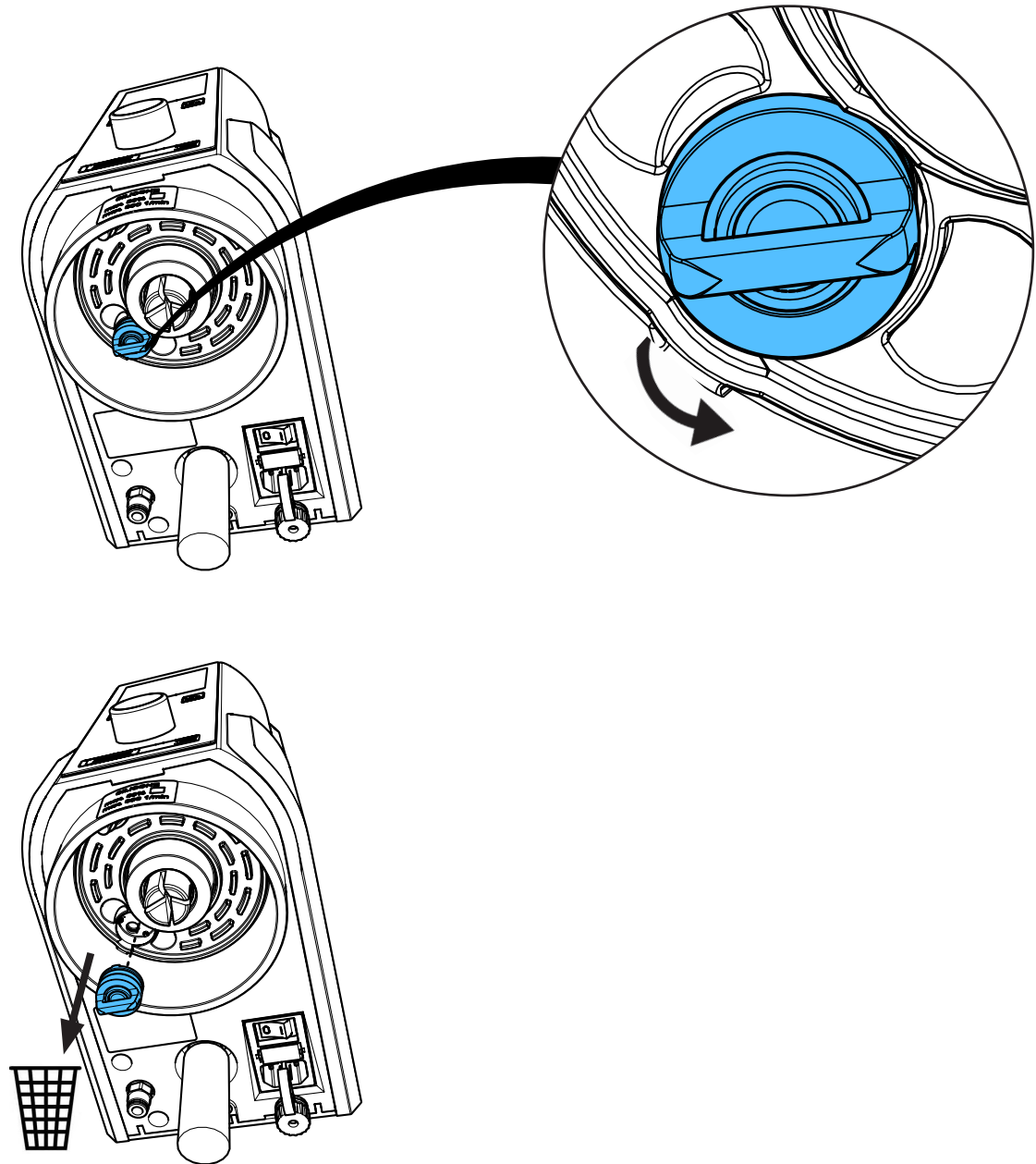


4

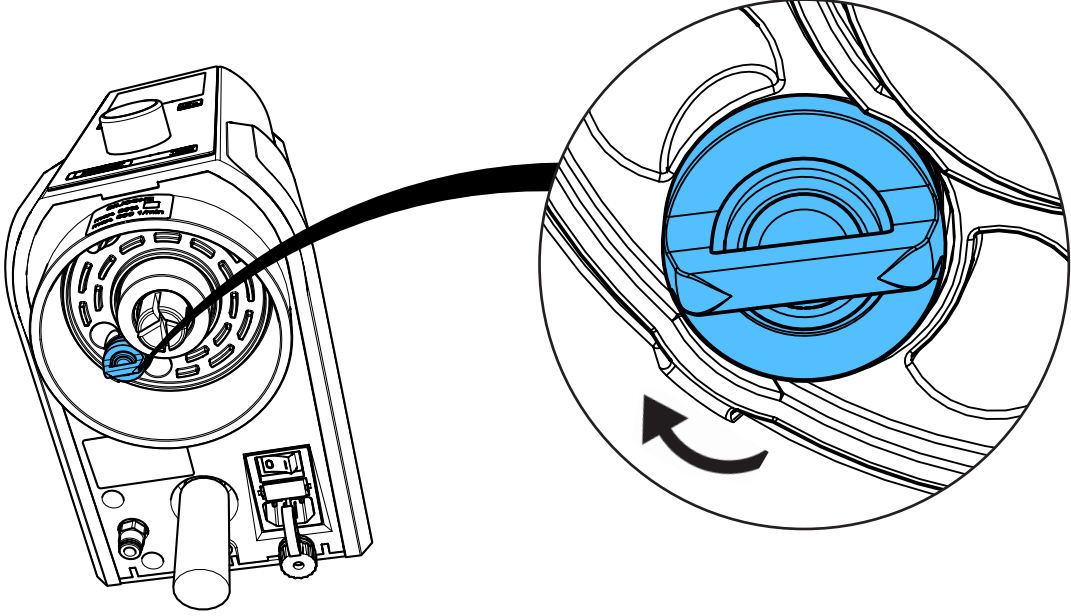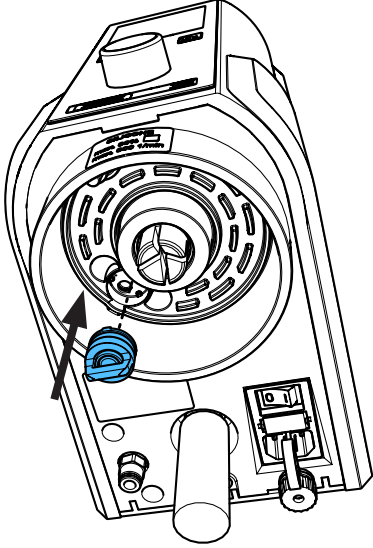

5

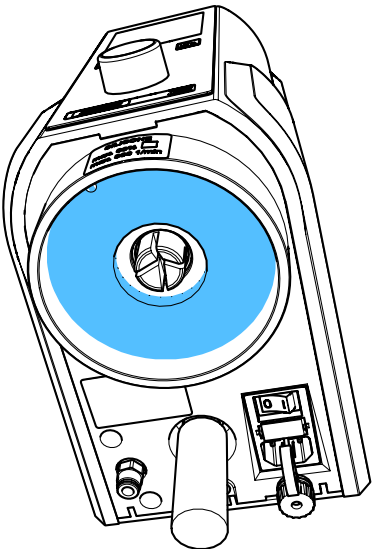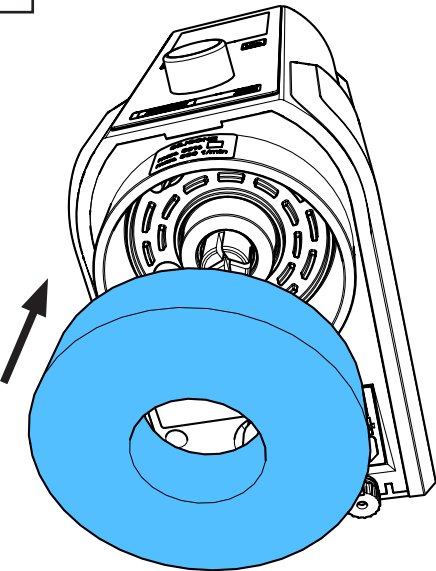



6

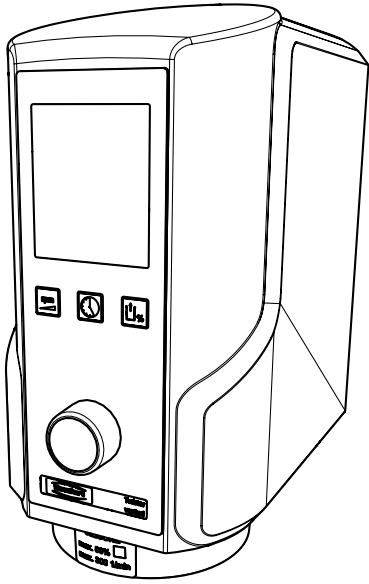




## 90003 4715


Sicherungsset • Fuse set • Dispositifs de sécurité • Kit di fusibili • Juego de fusibles  
Набор предохранителей

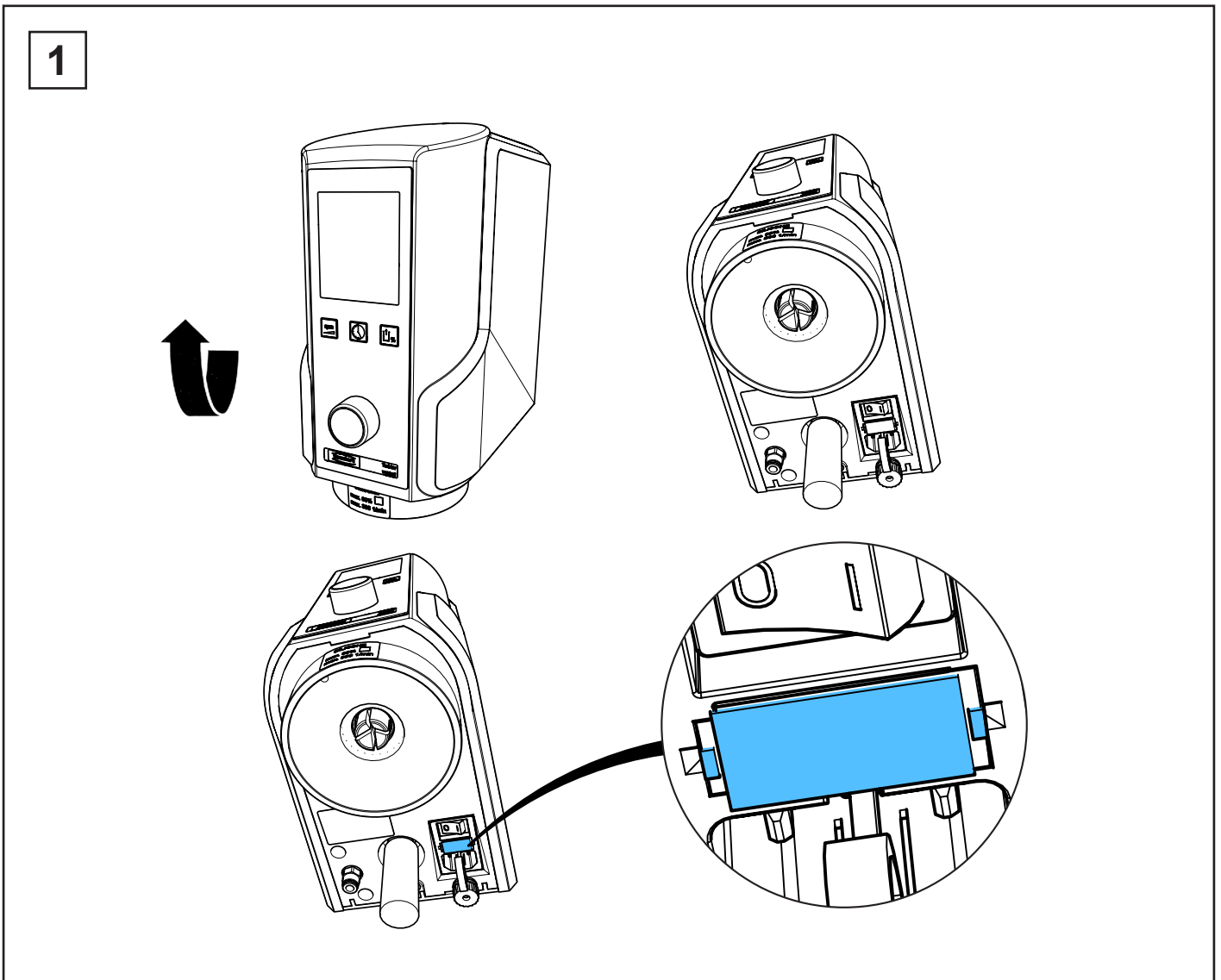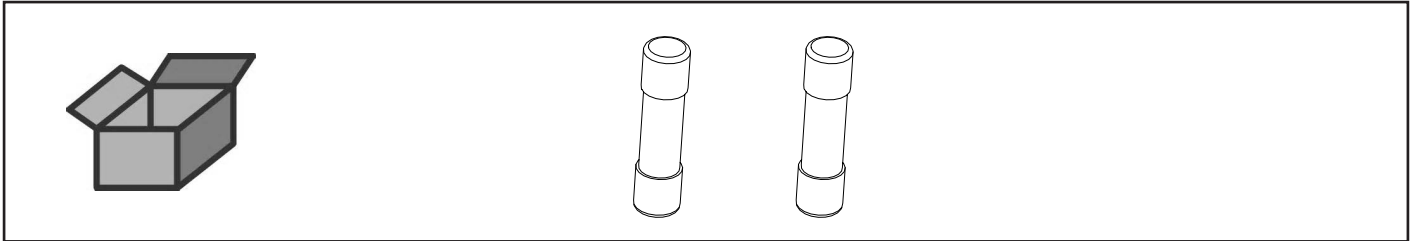


2

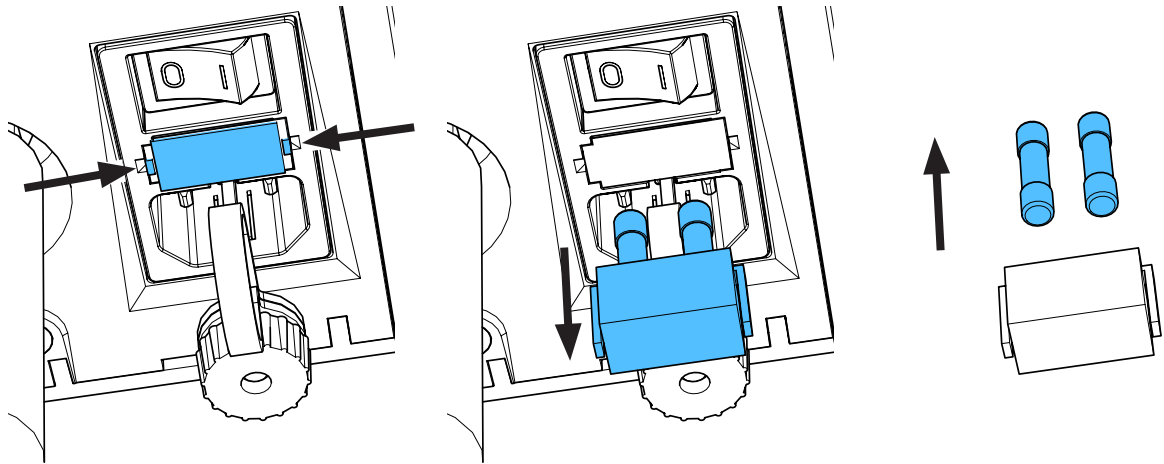


3

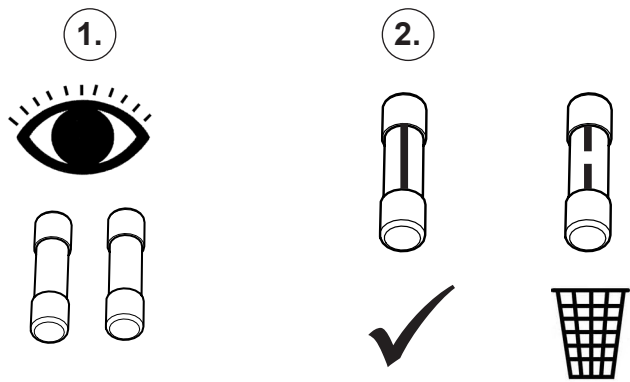

4


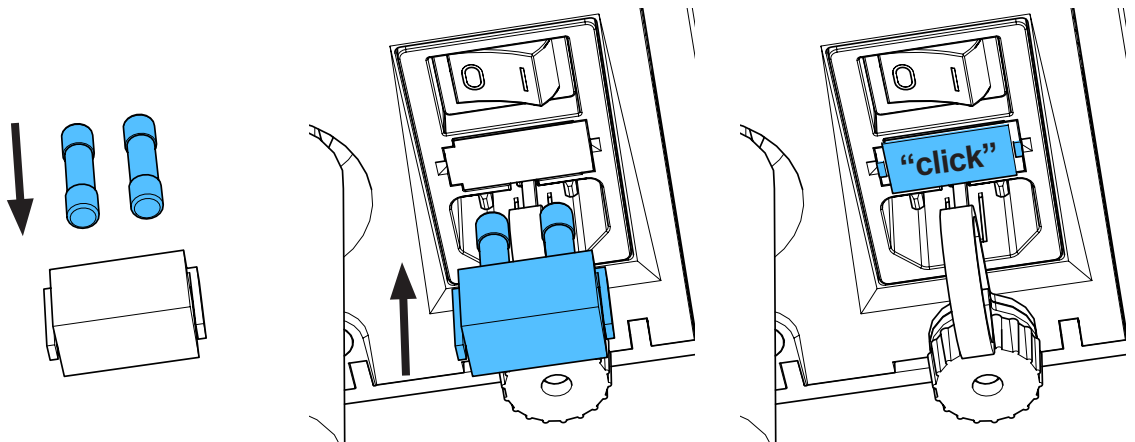


5

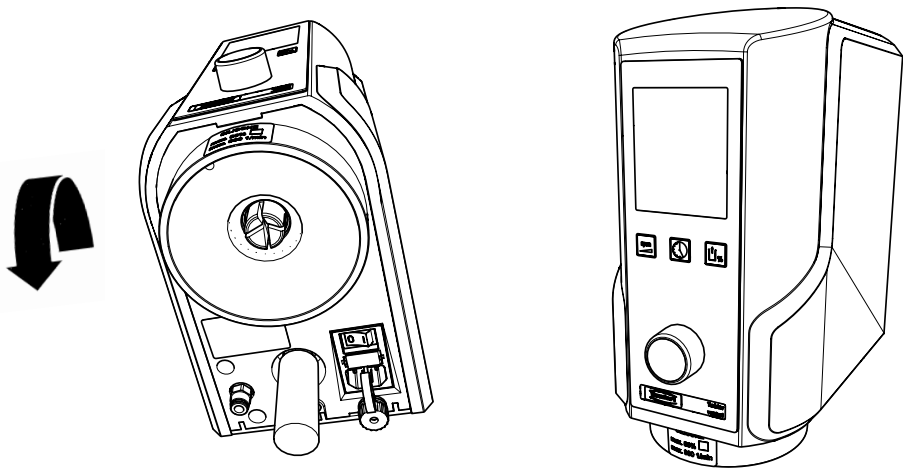


## 90003 4155


O-Ring-Set • Set of O-rings • Jeu de joints thoriques • Kit di guarnizioni  
Juego de anillos tóricos • Набор уплотнительных колец

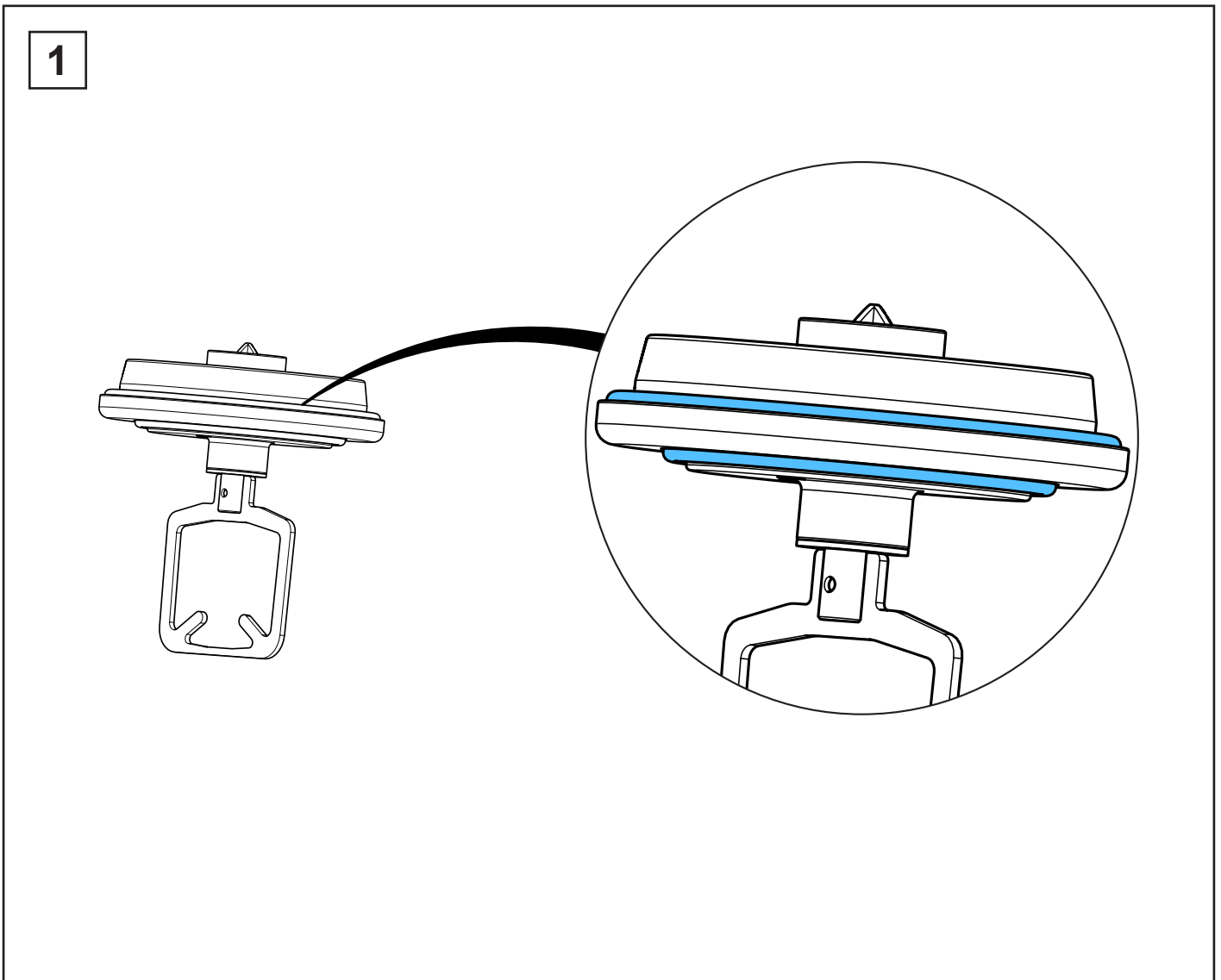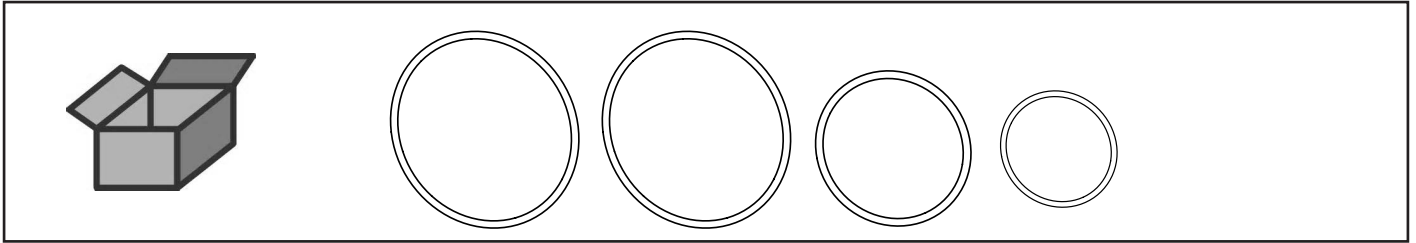


2

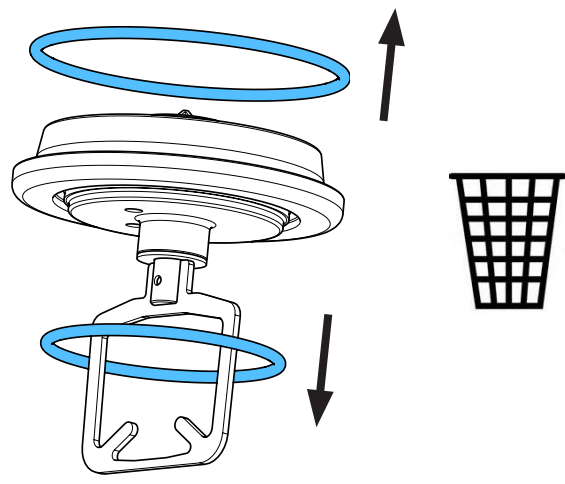

3


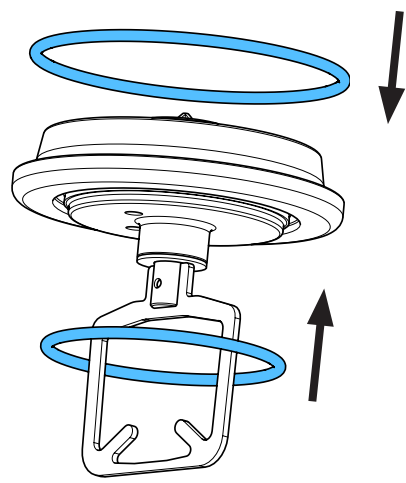


4

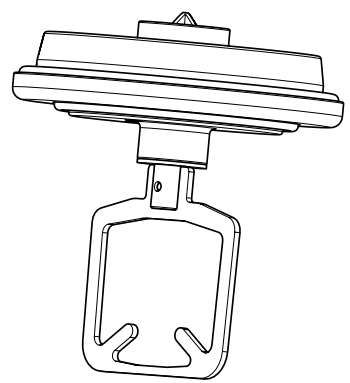

**1820 0210 / ..0510 / ..0710 / ..1010**


Rührwerk • Paddle • Mixeur • Spatola • Agitador • Смесительный механизм

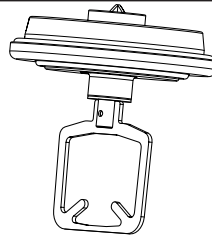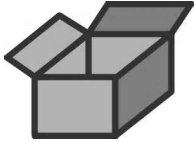

**1820 0210 = 200ml**  
**1820 0510 = 500ml**  
**1820 0710 = 700ml**  
**1820 1010 = 1000ml**





**1**

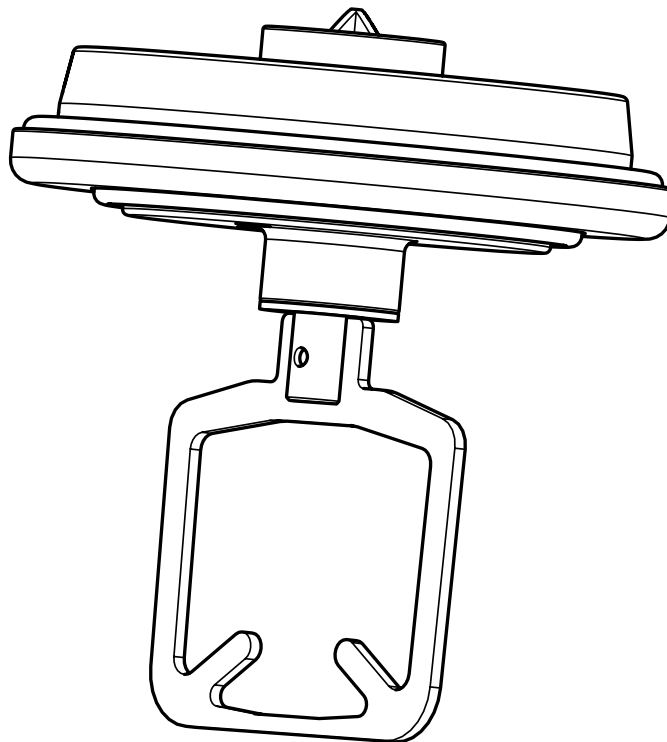








**90003 6121 / ..6122 / ..5938 / ..2769 / ..6070 / ..2767 / ..2771 / ..2768**




Netzkabel • Power cord • Cordon d'alimentation • Cavo di rete • Cable de red  
Сетевой кабель

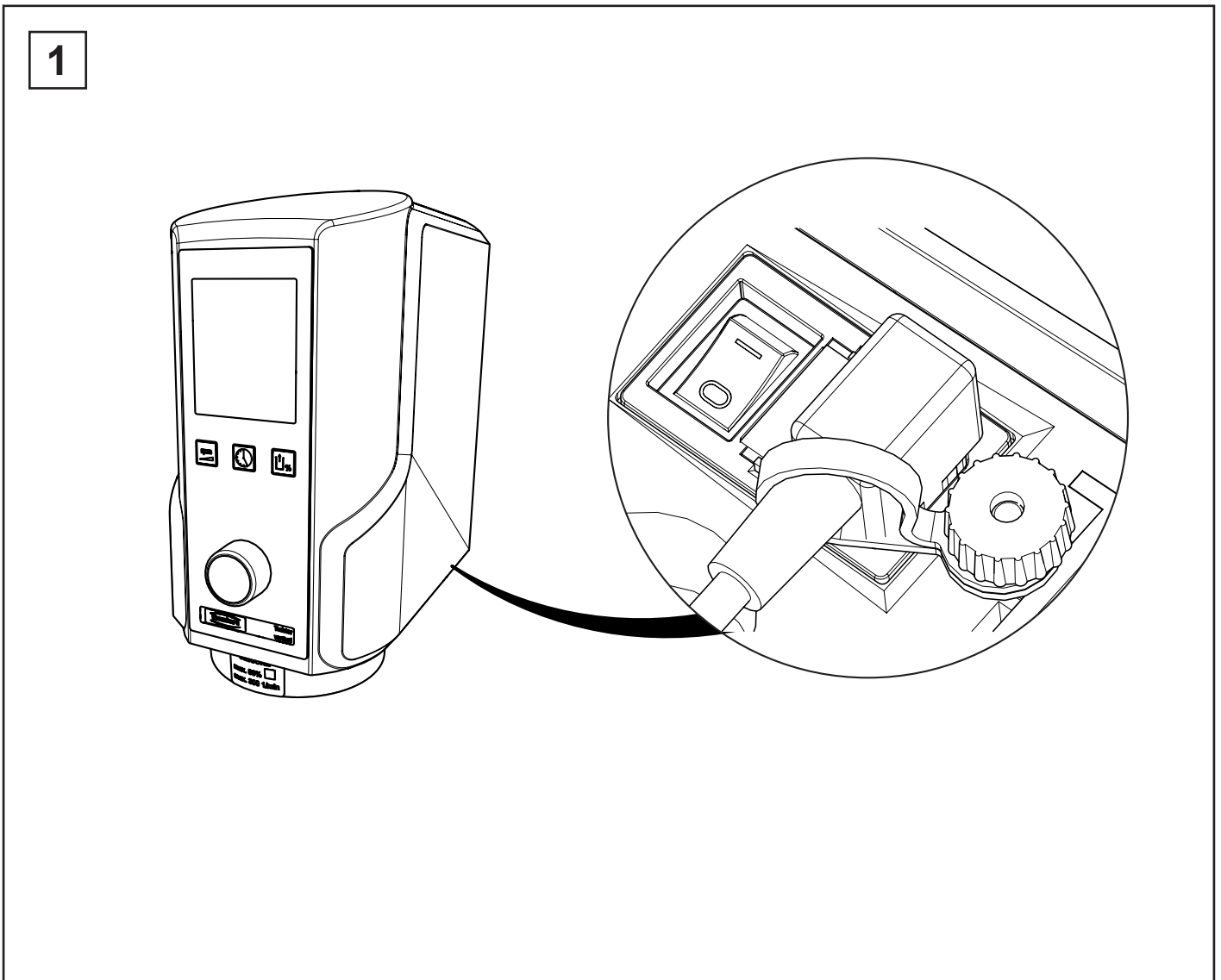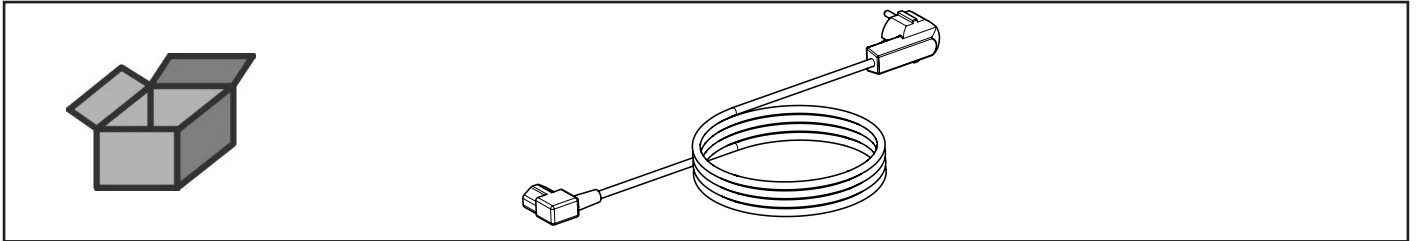


2

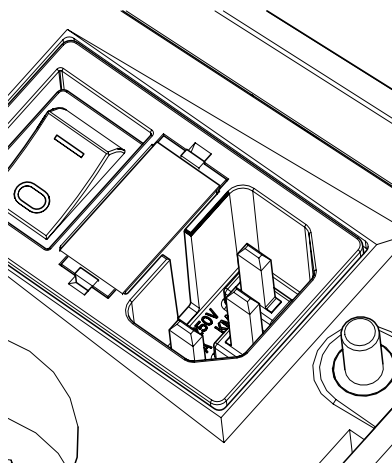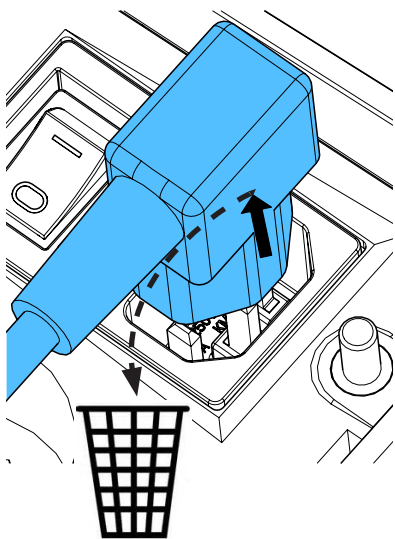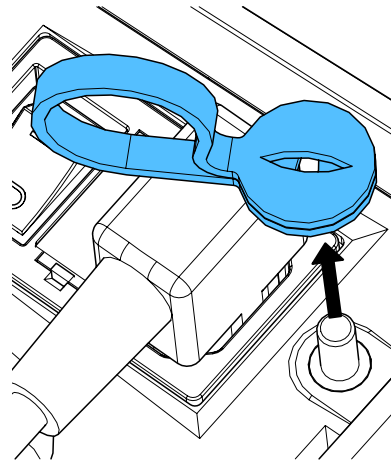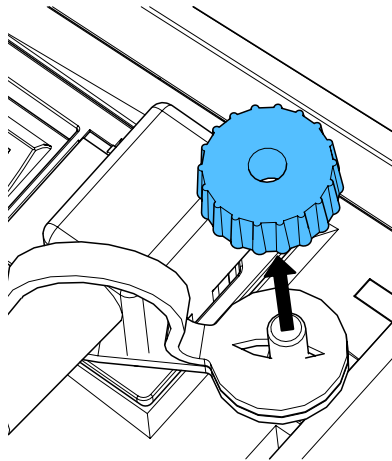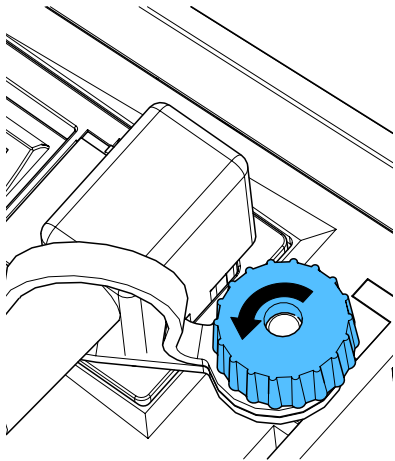

3

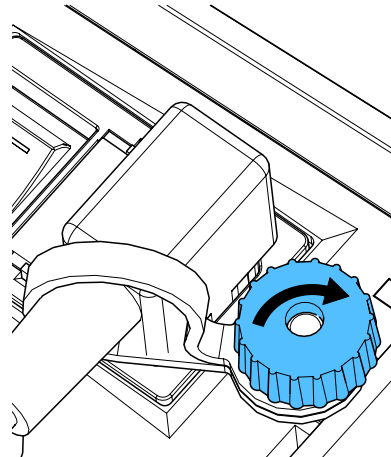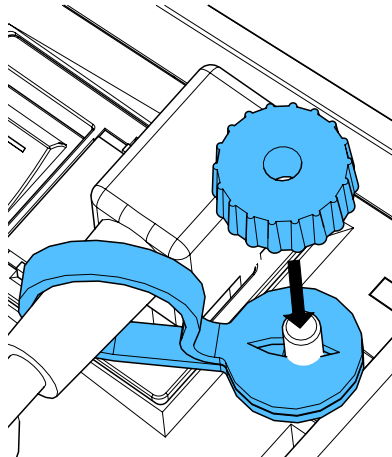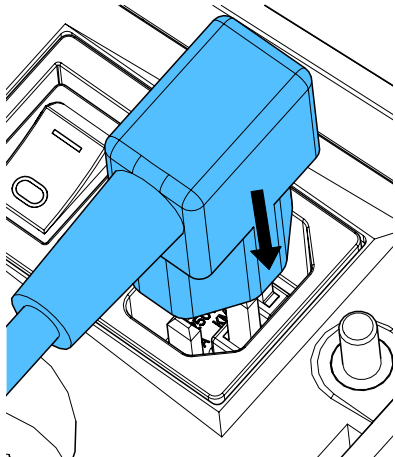




4

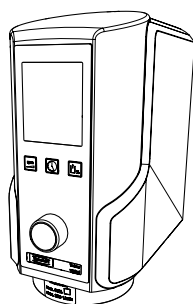




## 90003 4137


Zellgummimatte • Prot. rubber mat • Tapis en caout. • Cusc. di gomma cellul.  
Alfombrilla de caucho • Резиновый коврик

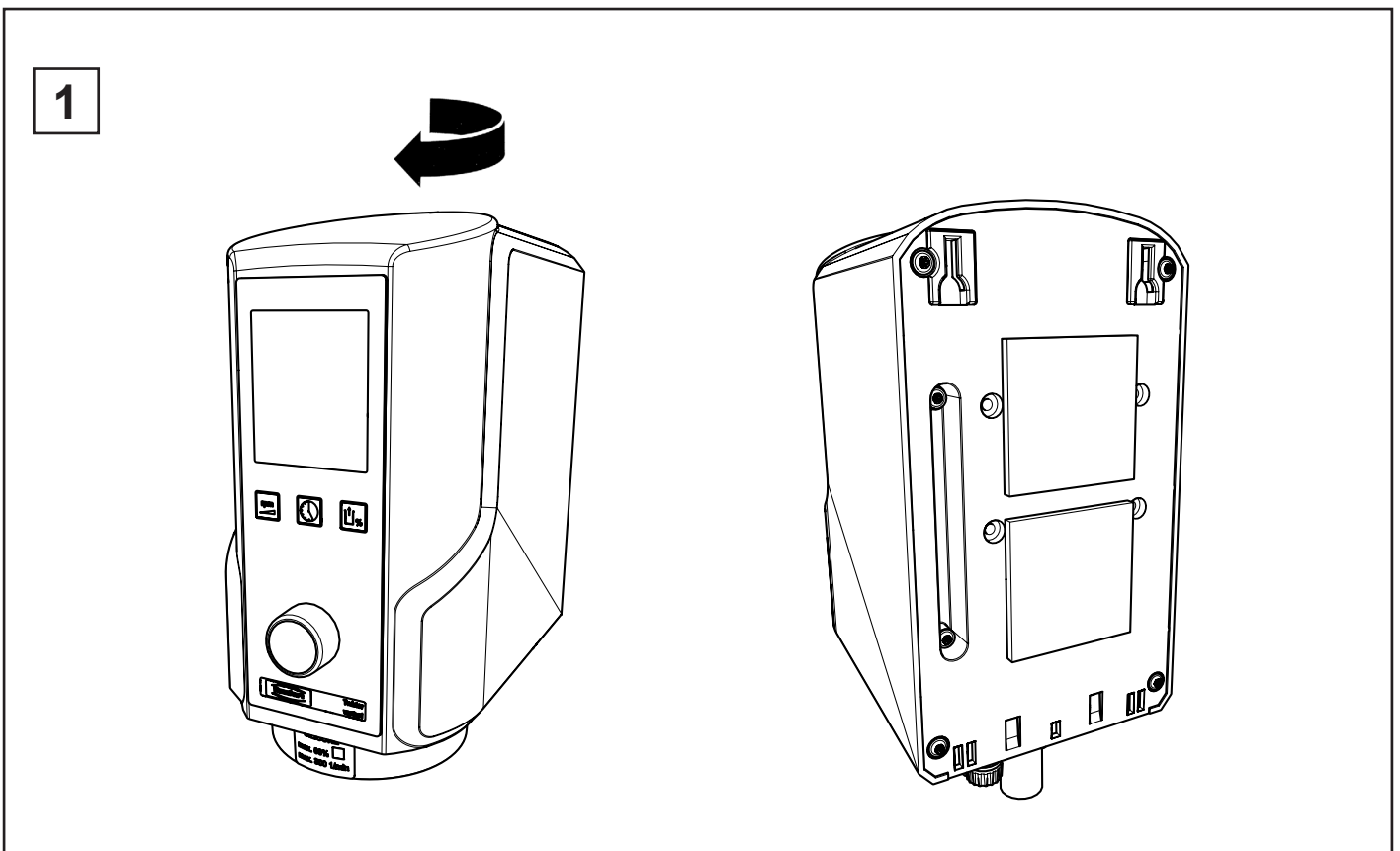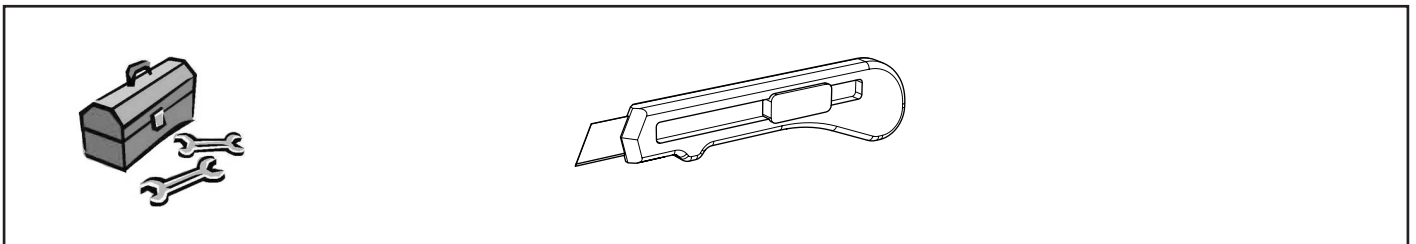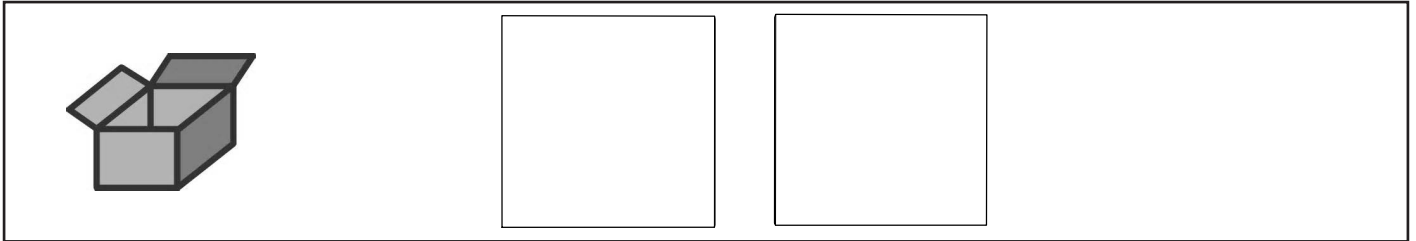


2

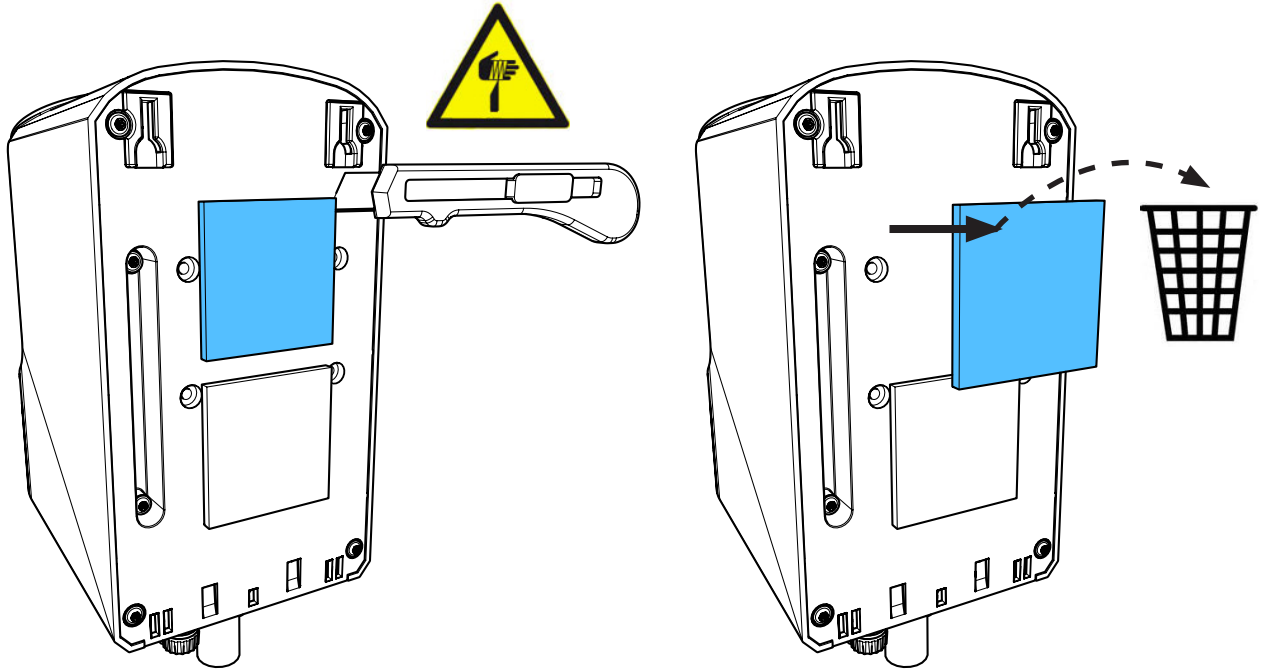

3


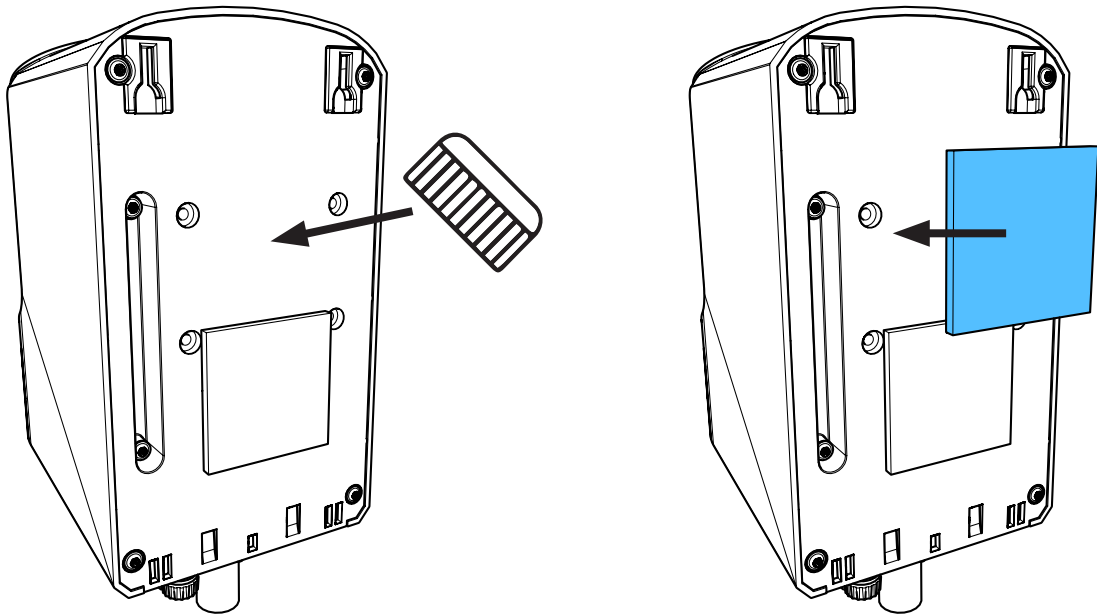


4

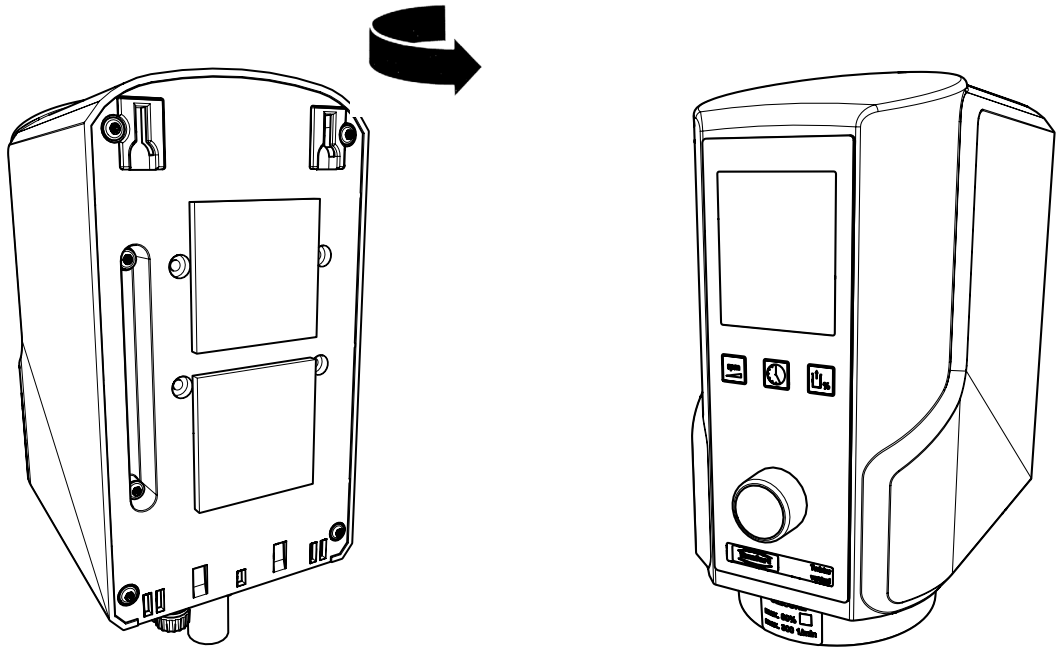


## 90003 4700


Kabelsicherung Twister • Cable safety catch for Twister • Arrêt de câble pour Twister  
Fermacavo di sicurezza Twister • Lazo de sujeción para cable de red Twister  
Фиксатор для кабеля Twister

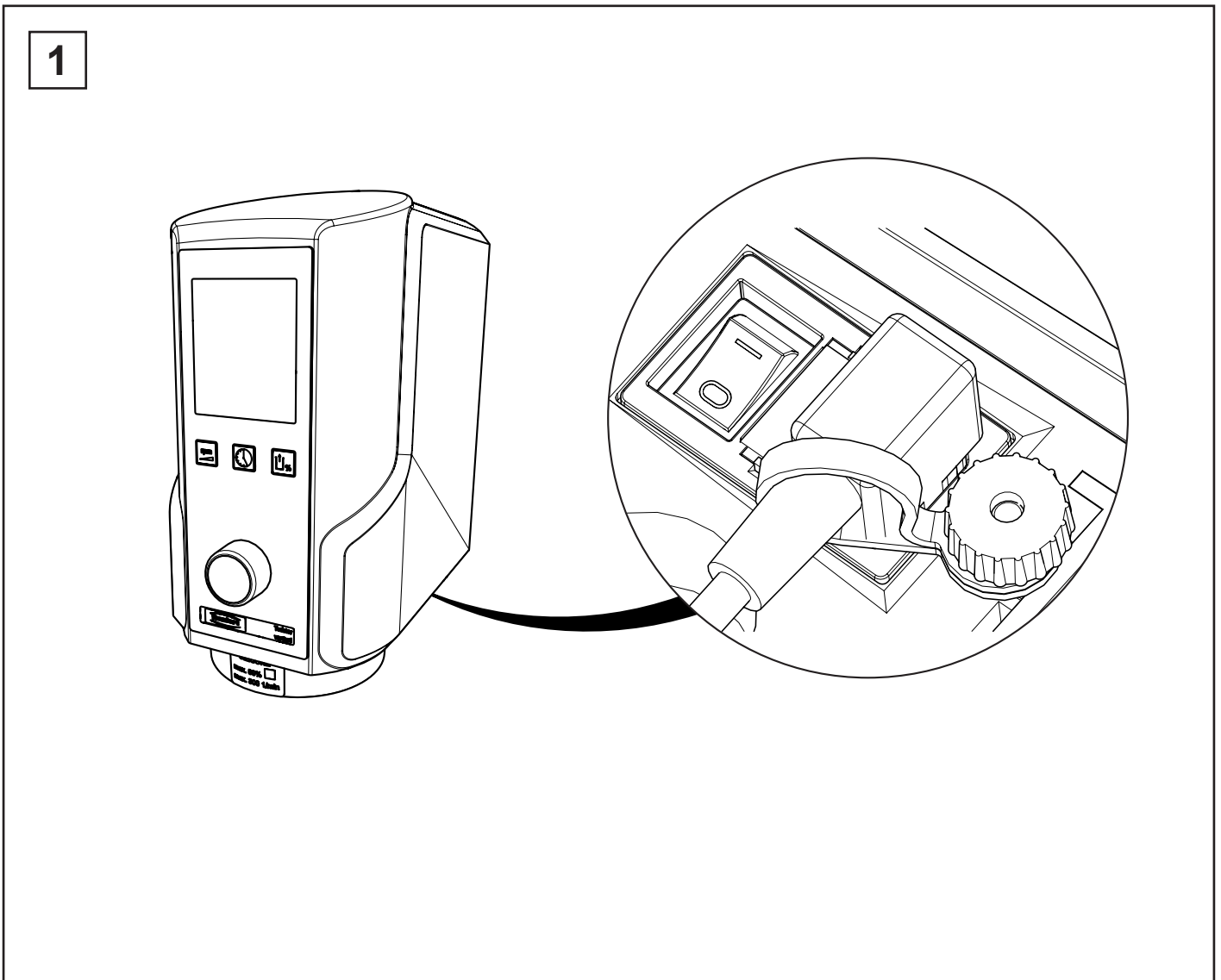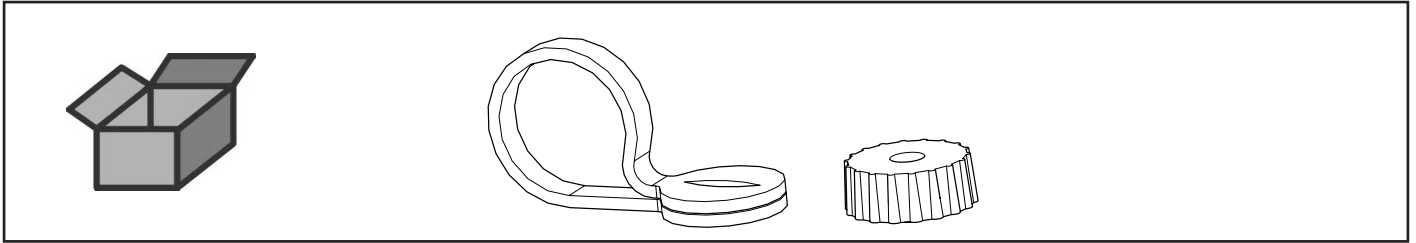


2

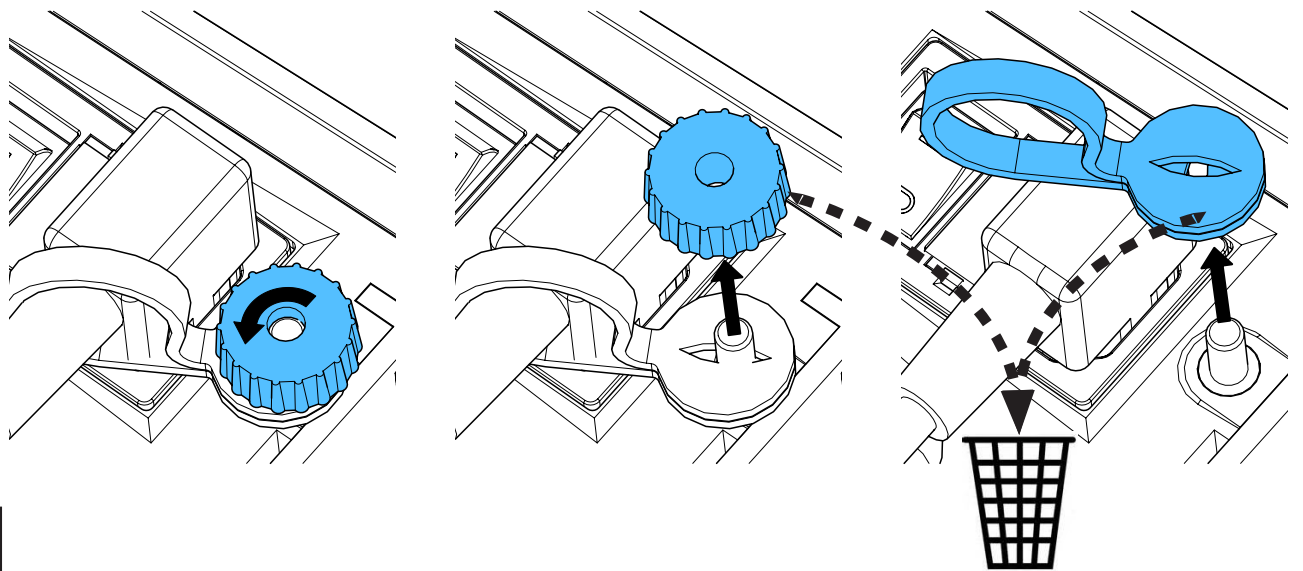

3



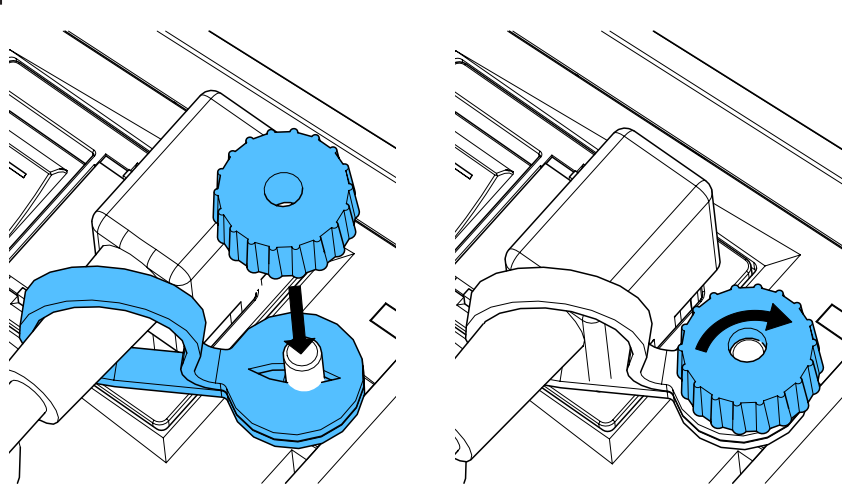


4

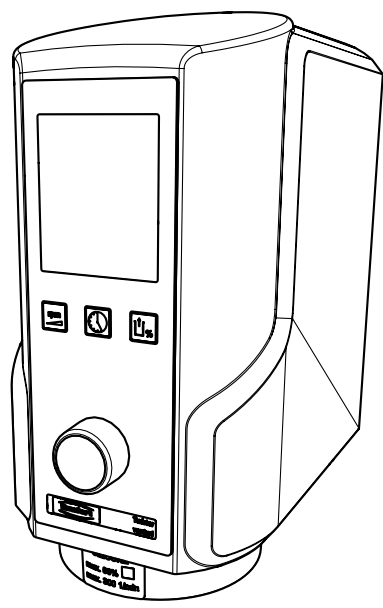


## 90013 2653


EingangsfILTER Set • Input filter Kit • Filtre d'entrée Jeu  
Filtro d'entrata Kit • Filtro de entrada Juego • Входной фильтр

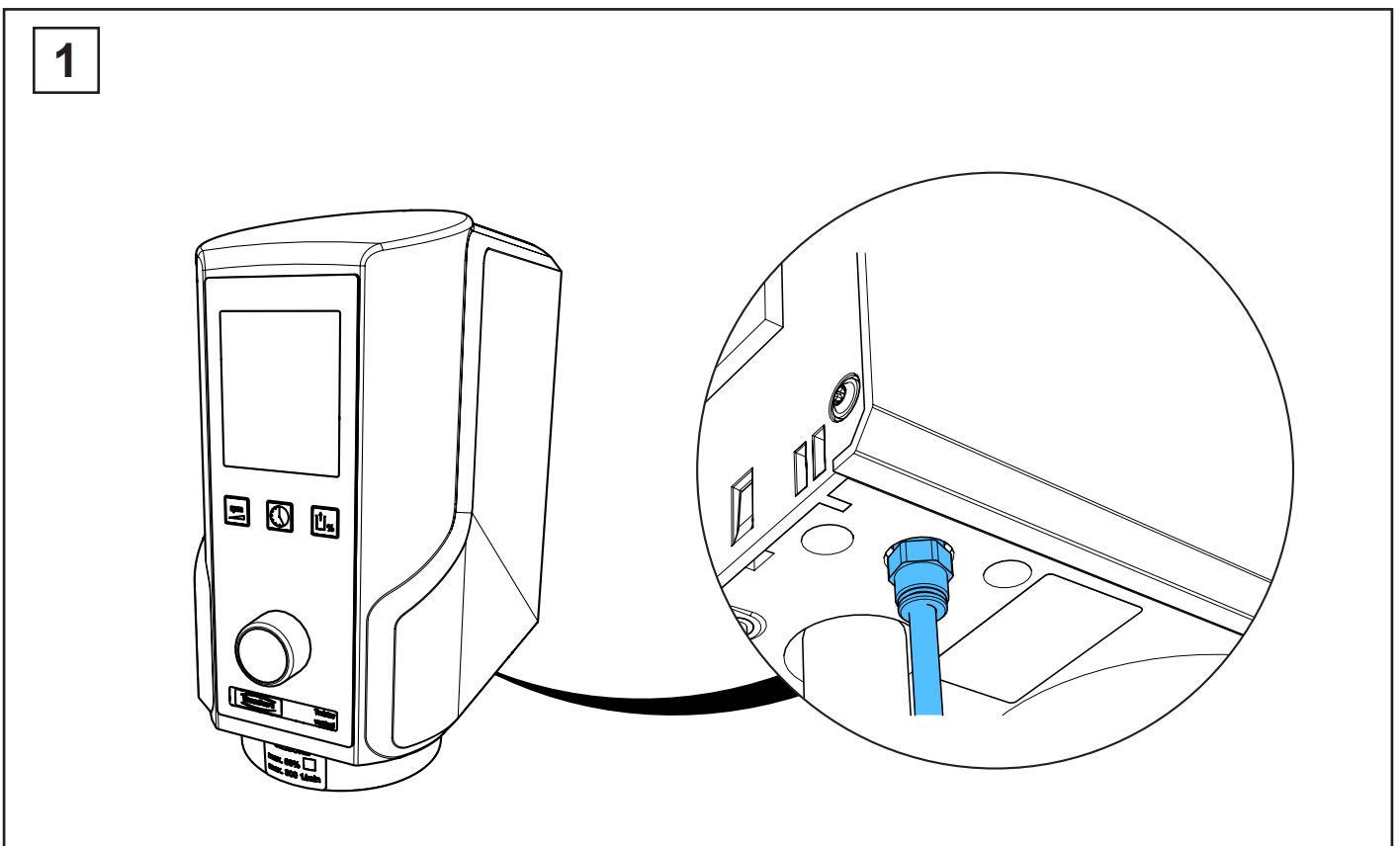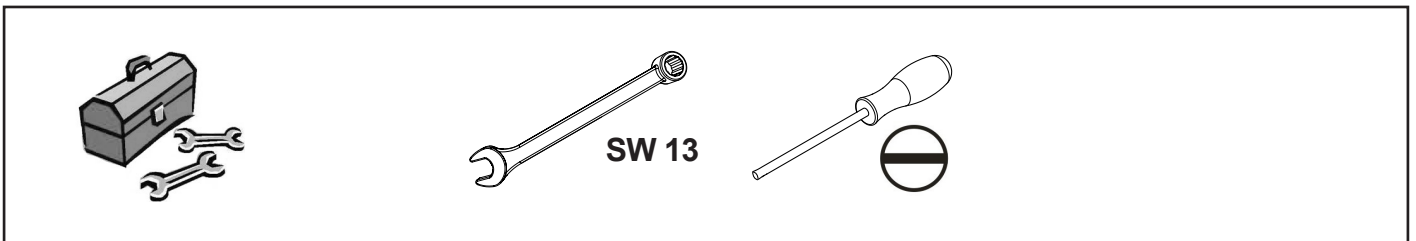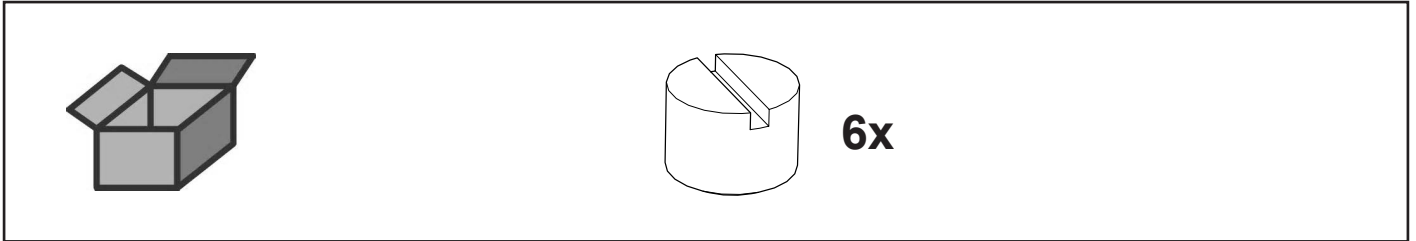




2

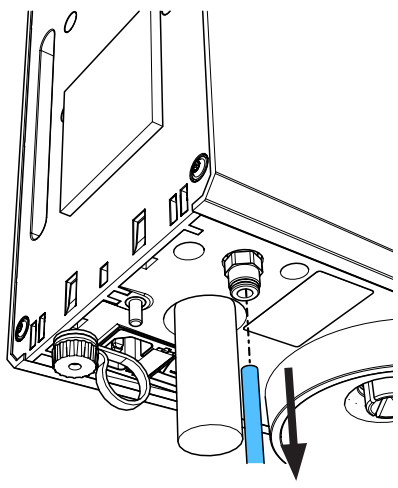



*i*

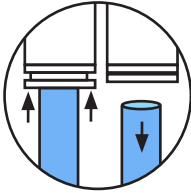


3

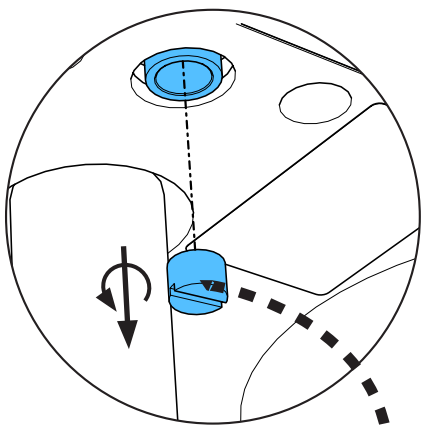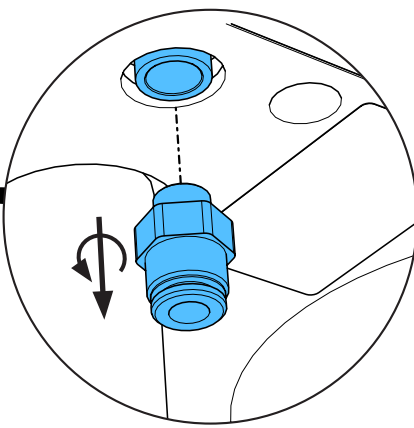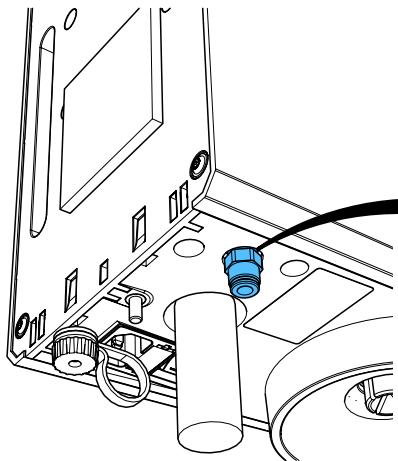

4

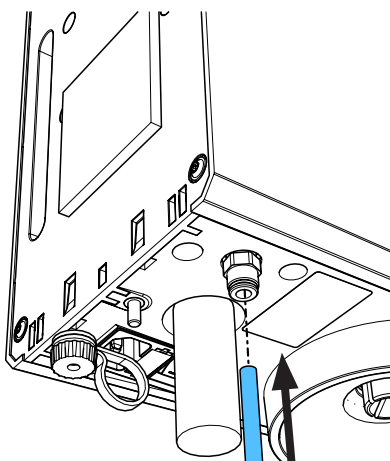
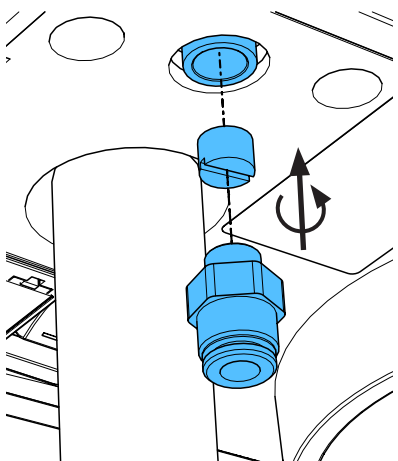


*i*

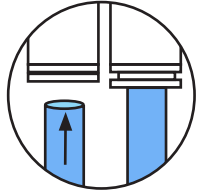

## 216606

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
Quick-Referenz-Karte Twister evolution / evolution venturi • Quick reference card  
Carte référence rapide • Sommario rifer. rapido • Guía referencia rápida  
Краткий справочник-карта

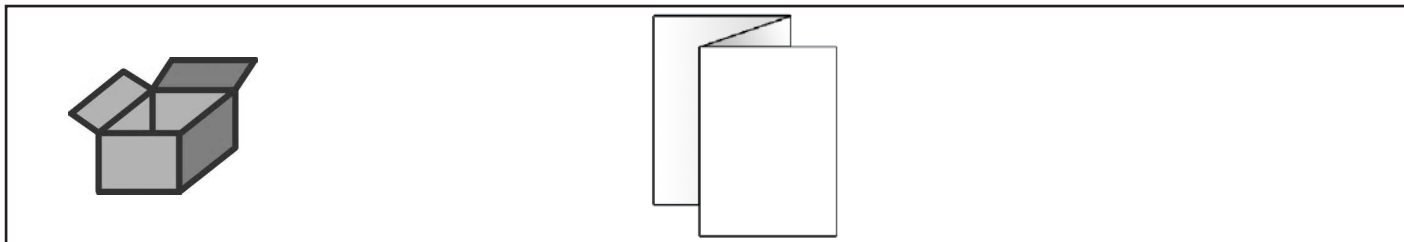


## Select program



## Auto Start

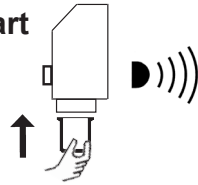

## Stop

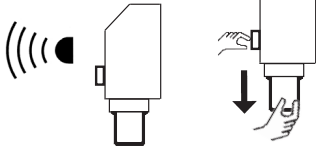

P	
P	
P	
P	
P	
P	
P	
P	
P	
P	

	Zeit Time Temps Tempo	Tiempo Время смешивания 攪拌時間 시간
	Drehzahl Speed Vitesse Numero di giri	Velocidad de giro Число оборотов 回転数 속도
	Intervall Interval Intervalle Intervallo	Intervalo Интервал インターバル 간격
	Vakuum Vacuum Vide Vuoto	Vacio Вакуум バキューム 진공
	Vorvakuum Pre-vacuum Pré vide Prevuoto	Prevacio Предварительный вакуум 前バキューム 진공 전
	Nachvakuum Post-vacuum Après vide Postvuoto	Postvacío Дополнительный вакуум 後バキューム 진공 후
	Gips Plaster Plâtre Gessi Yeso Гипс 石膏	Vorspateln Premix Pré malaxage Pre-spatolazione Premezclar Предварительное смешивание 前攪拌 프리믹스
		Einbettmasse Investment Revêtements Rivestimenti Revestimientos Паковочная масса 埋没材 매몰재



0310 21-6606

Auswählen  
Select  
Sélectionner  
Selezionare

**1.**

Seleccionar  
Выбор  
選擇する  
선택



Einstellen  
Adjust  
Réglage  
Impostazione

**2.**


Ajuste  
Установка  
設定する  
조정



Speichern  
Save  
Mémorisation  
Memorizzazione

**3.**




Memorización  
Занесение в память  
メモリーする  
저장

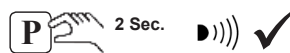




## 90003 5700


Ablufffilter • Exhaust air filter • Filtre air en sortie • Filtro aria di scarico  
Filtro aire de escape • Фильтр отработанного воздуха

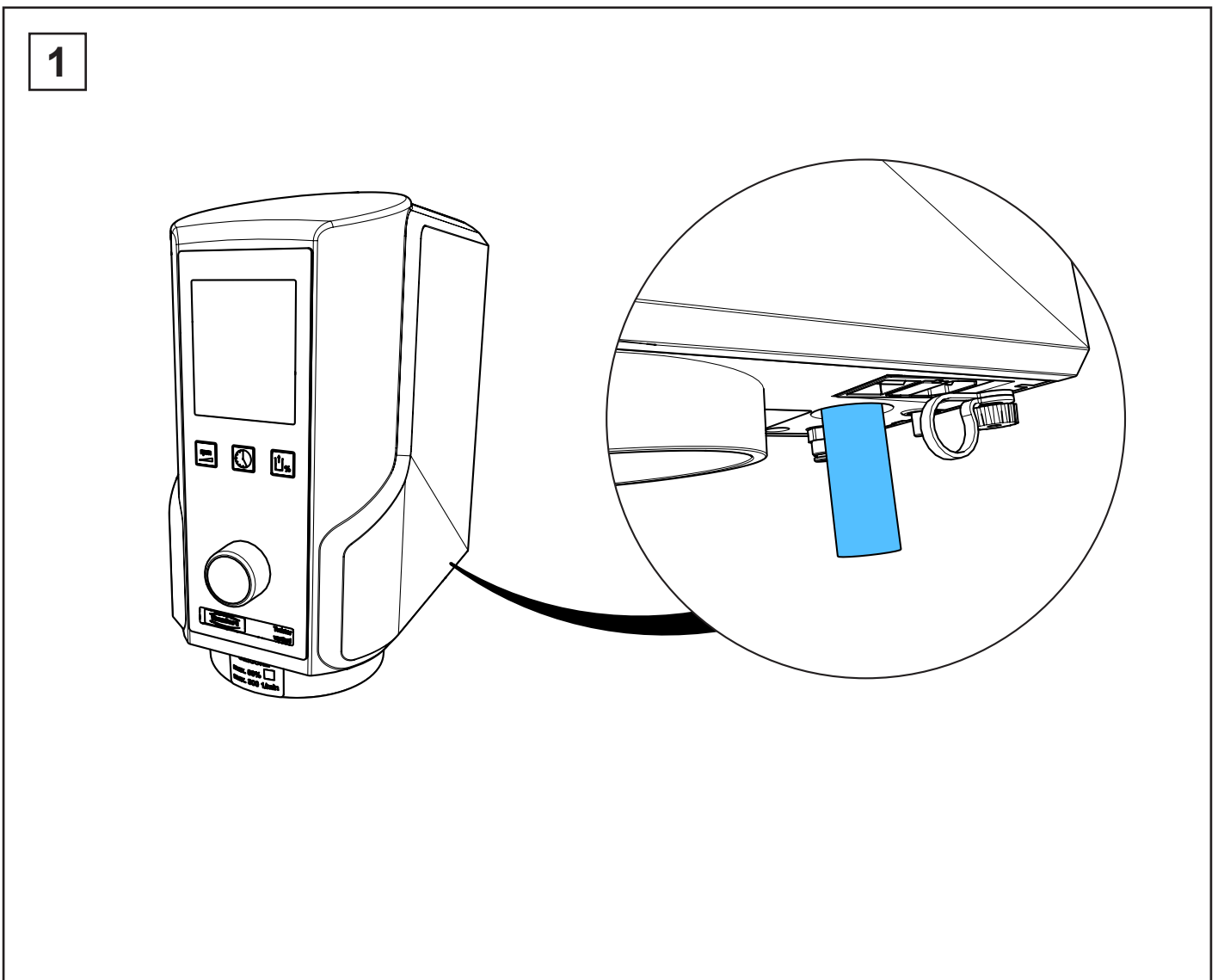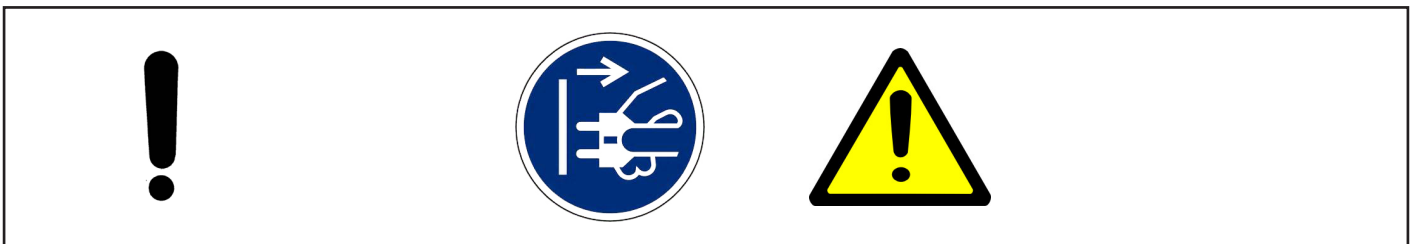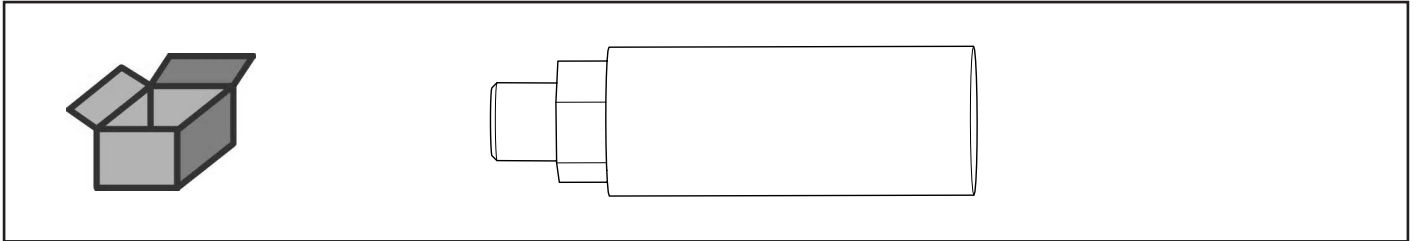


2

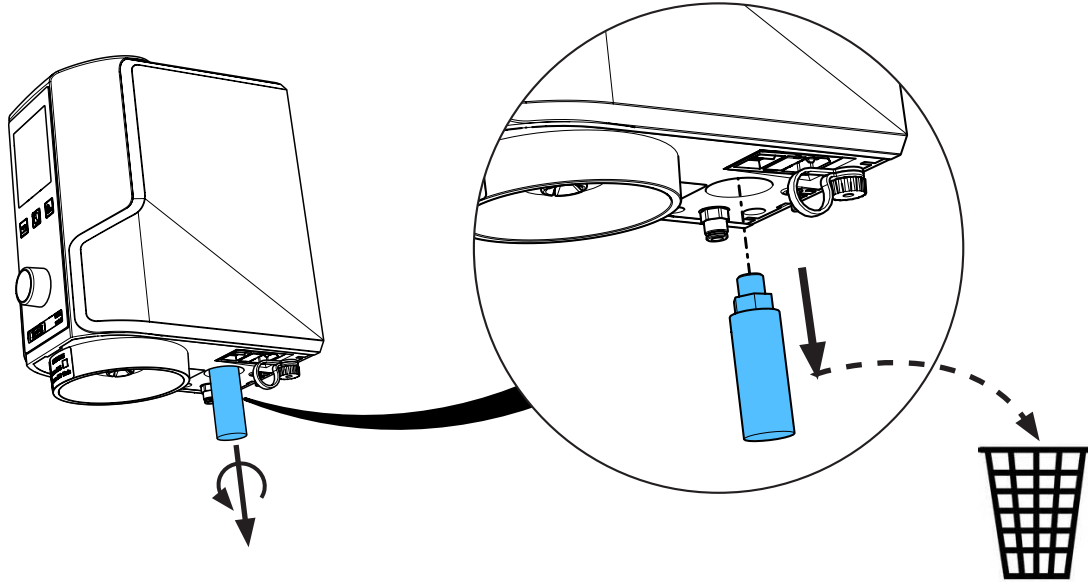

3



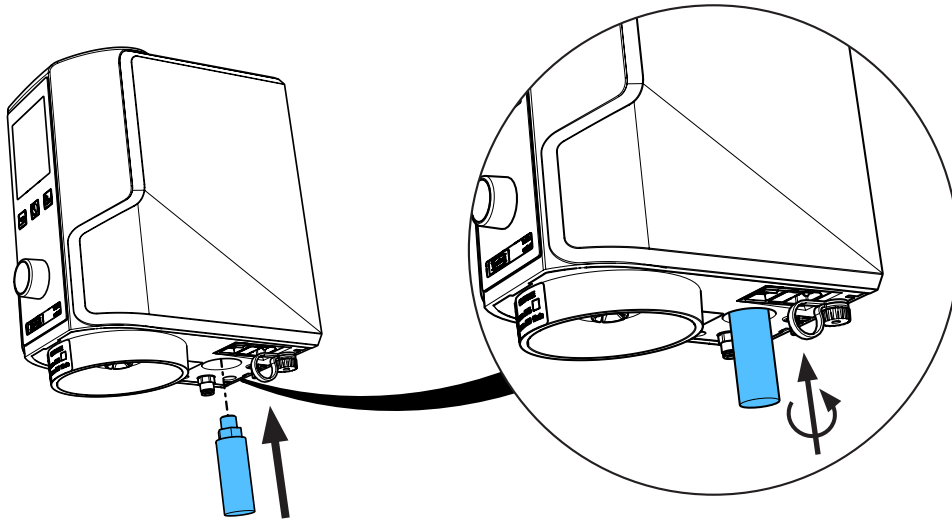


4

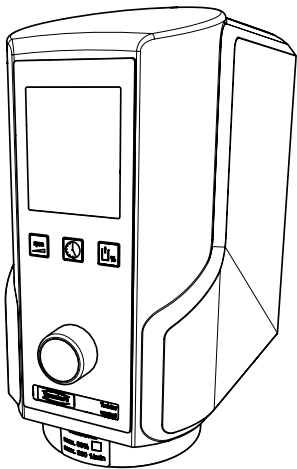


## 90003 4203


Anmischpaddel 65 ml • Wing mixer 65 ml • Mélangeur à ailettes 65 ml  
Pala di miscela 65 ml • Mezclador de paletas 65 ml • Смесительная лопасть 65 мл

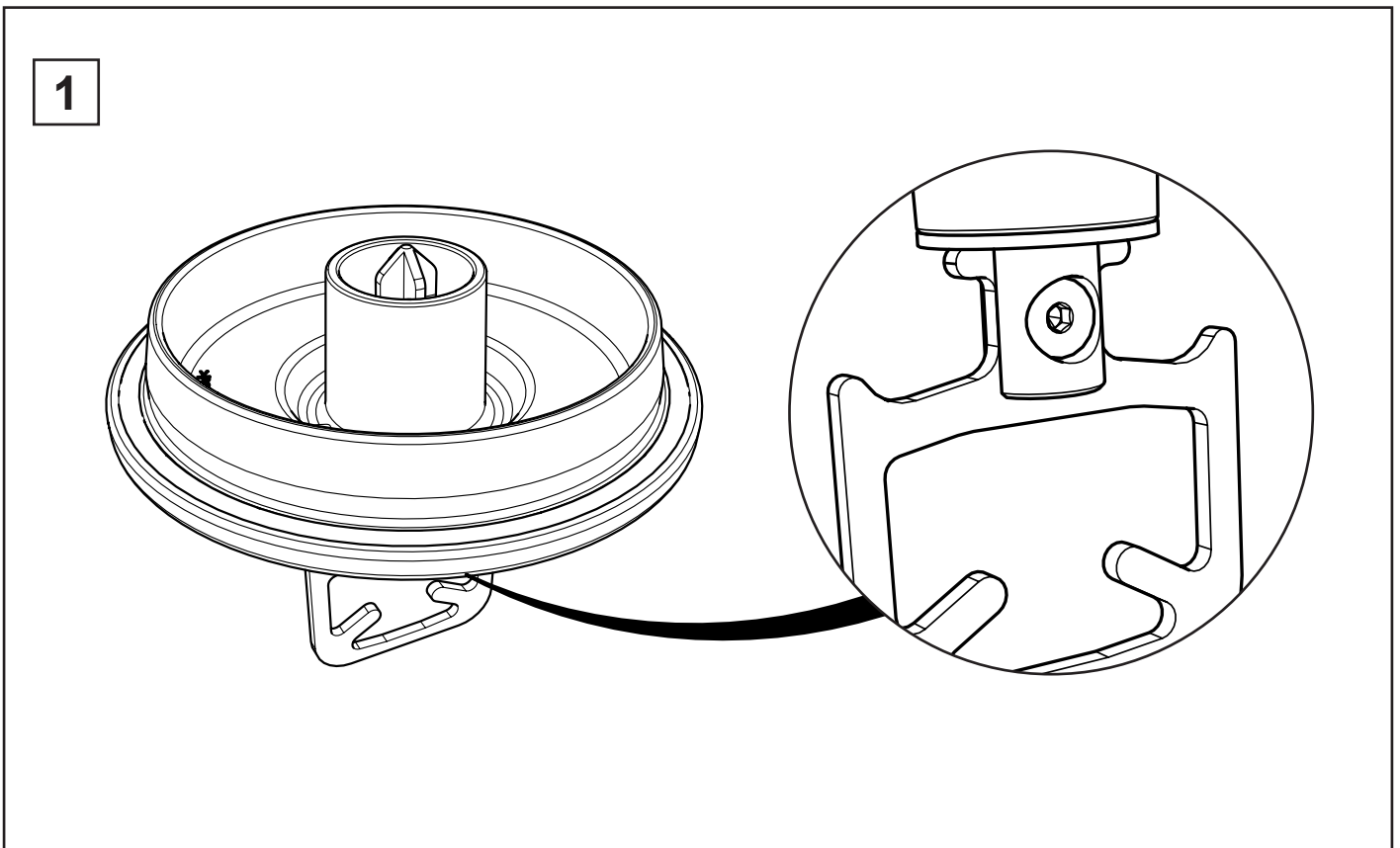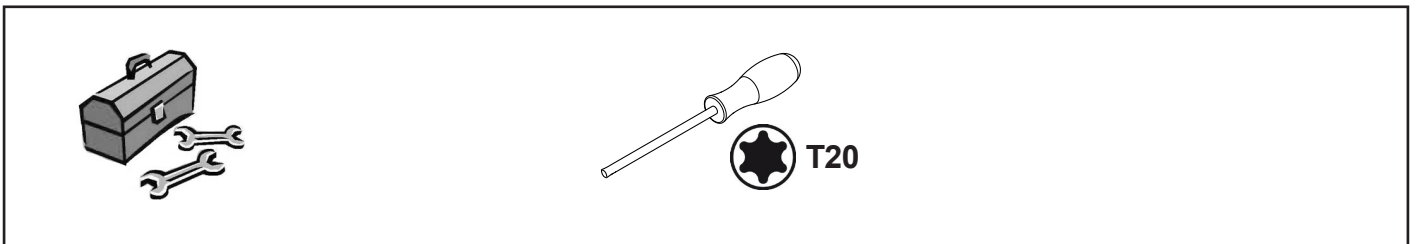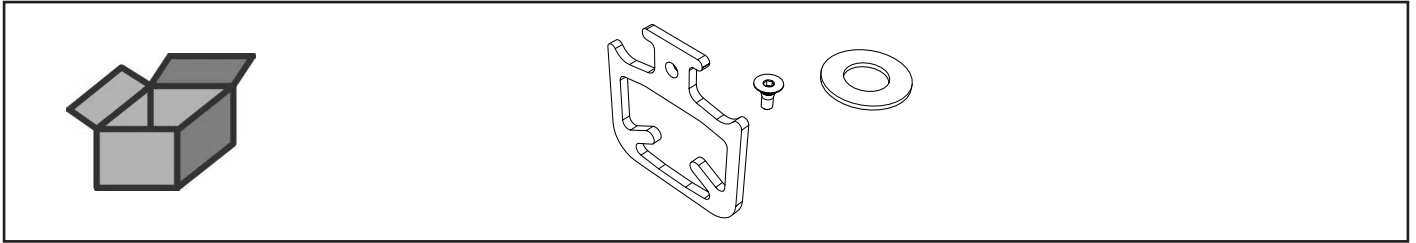

2


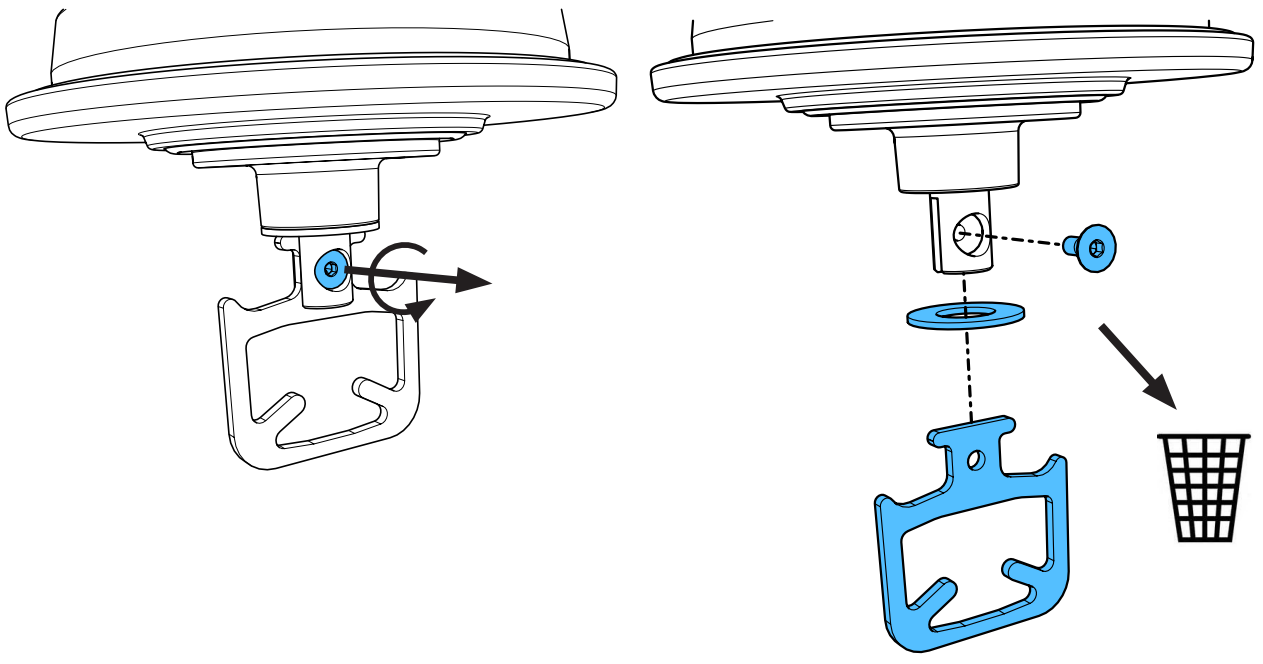


3

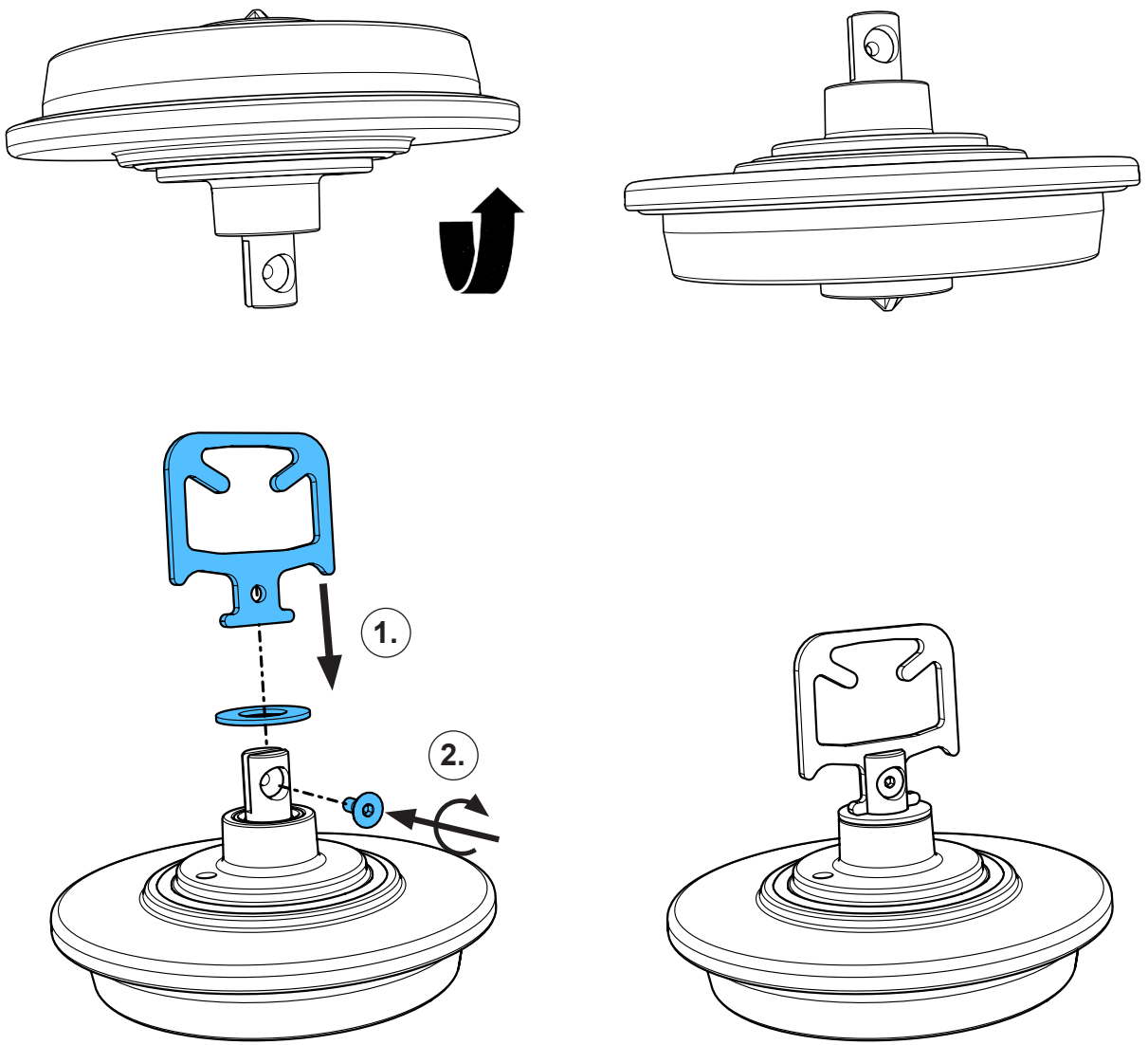


## 90003 4679


Frontfolie Twister venturi • Frontfoil Twister venturi • Membrane souple Twister ven-  
turi • Membrana comandi p.Twister venturi • Lámina p. elementos de mando Twister  
venturi • Фронтальная пленка Twister venturi

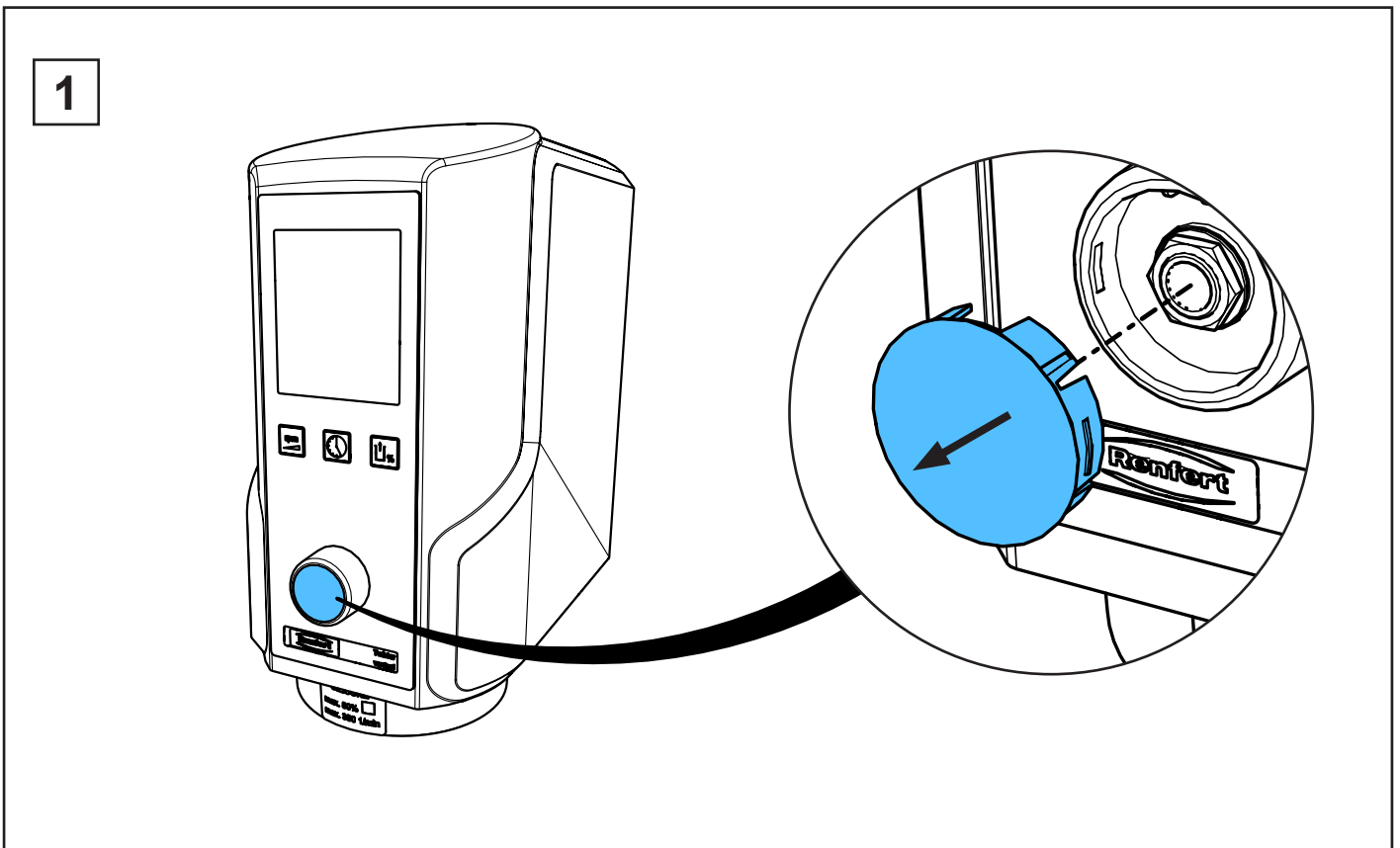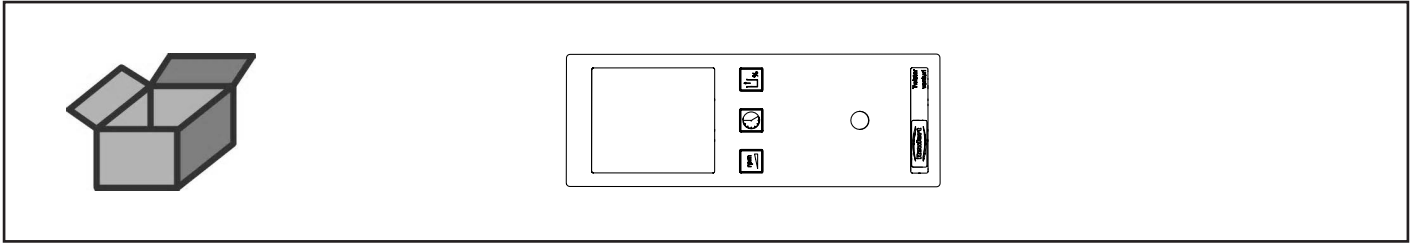


2

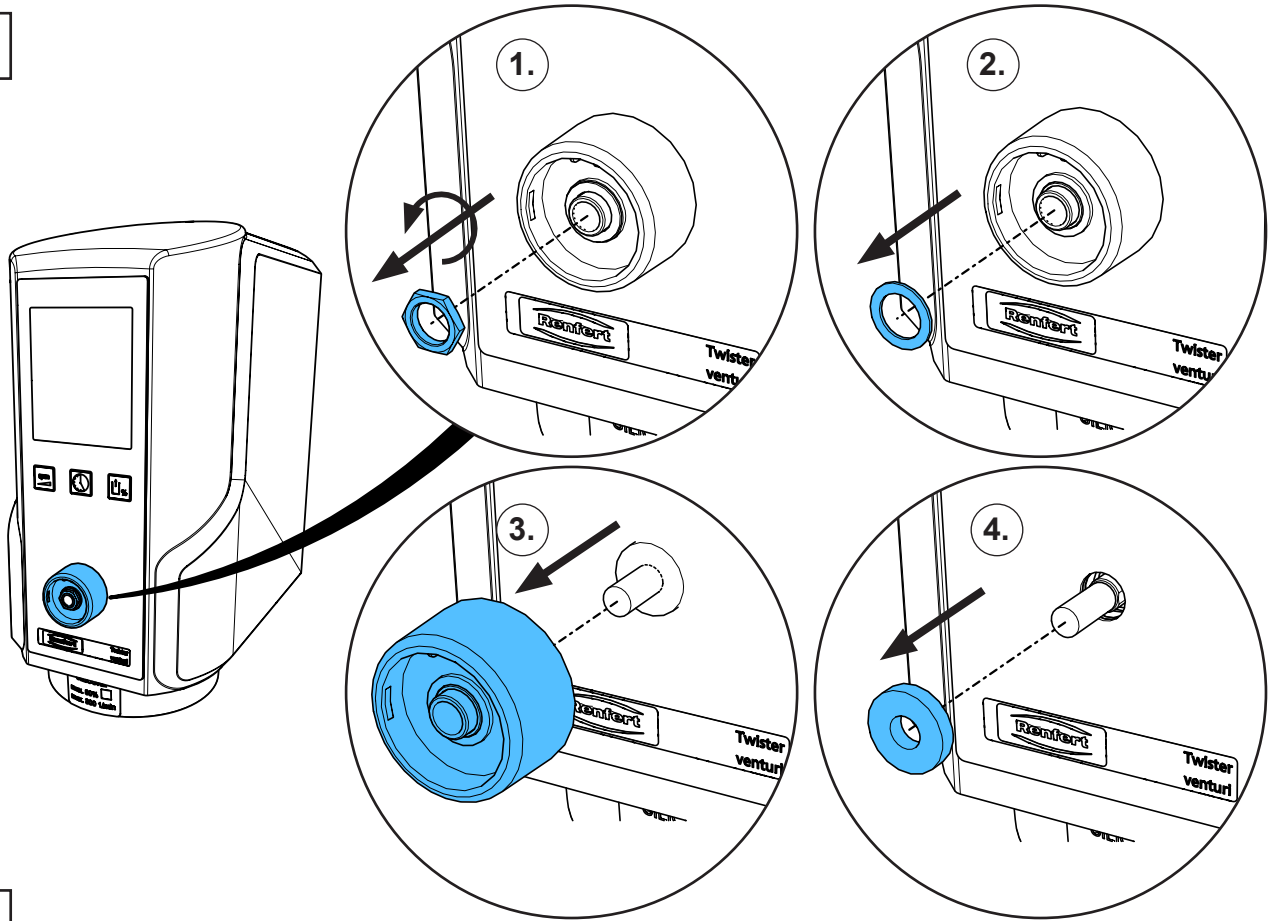


3

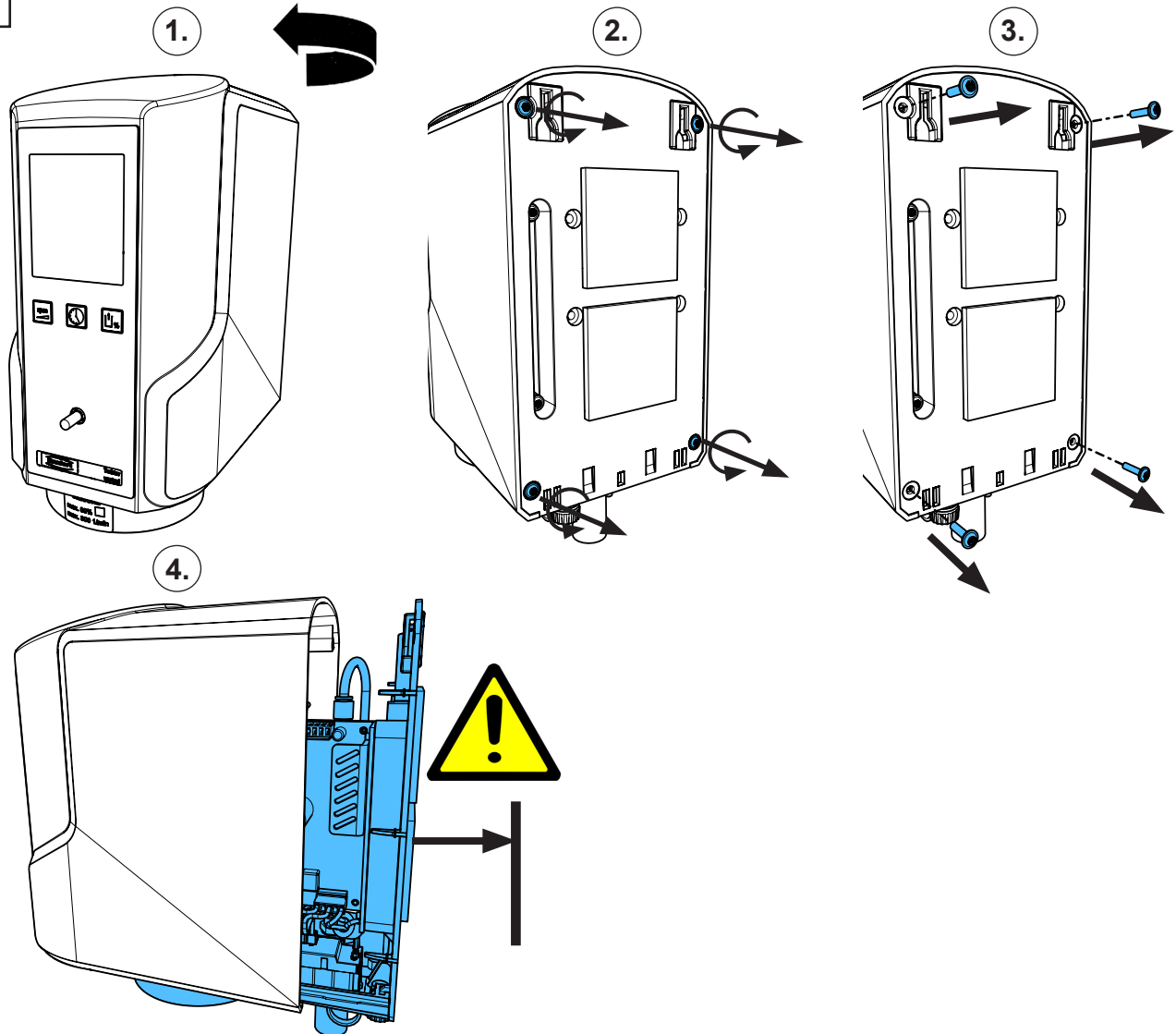


4

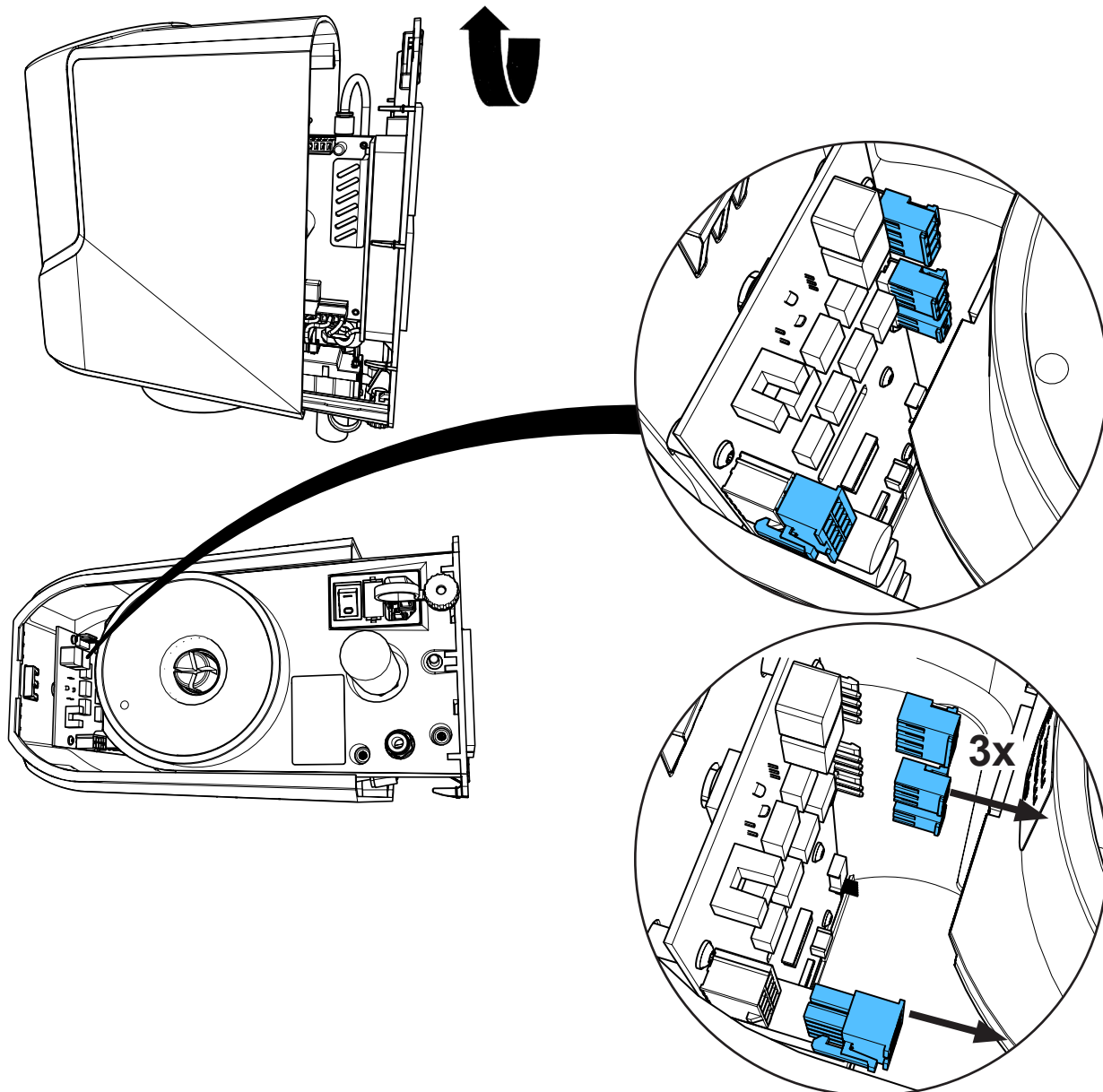


5

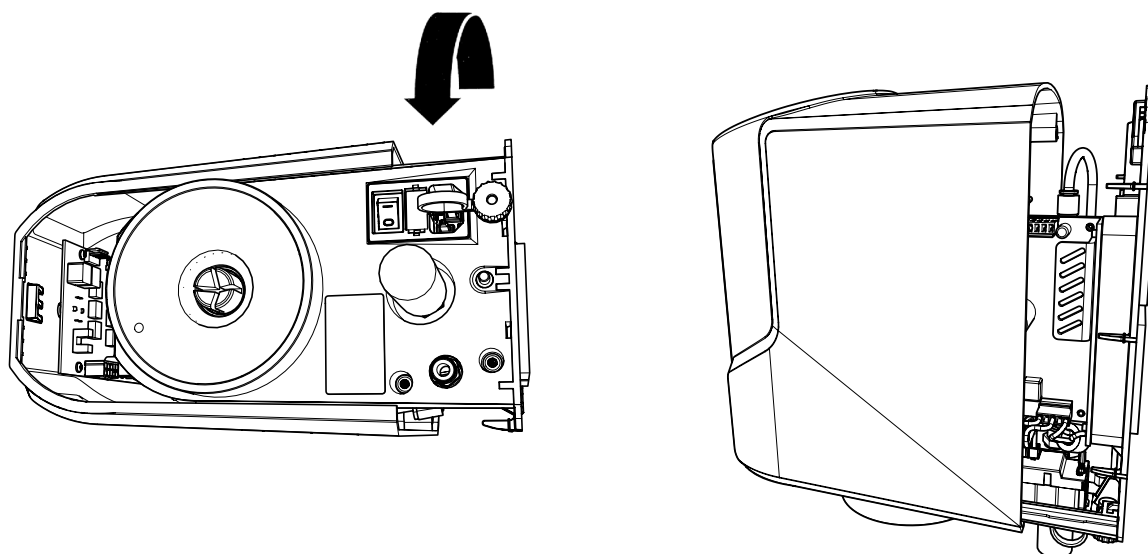





6

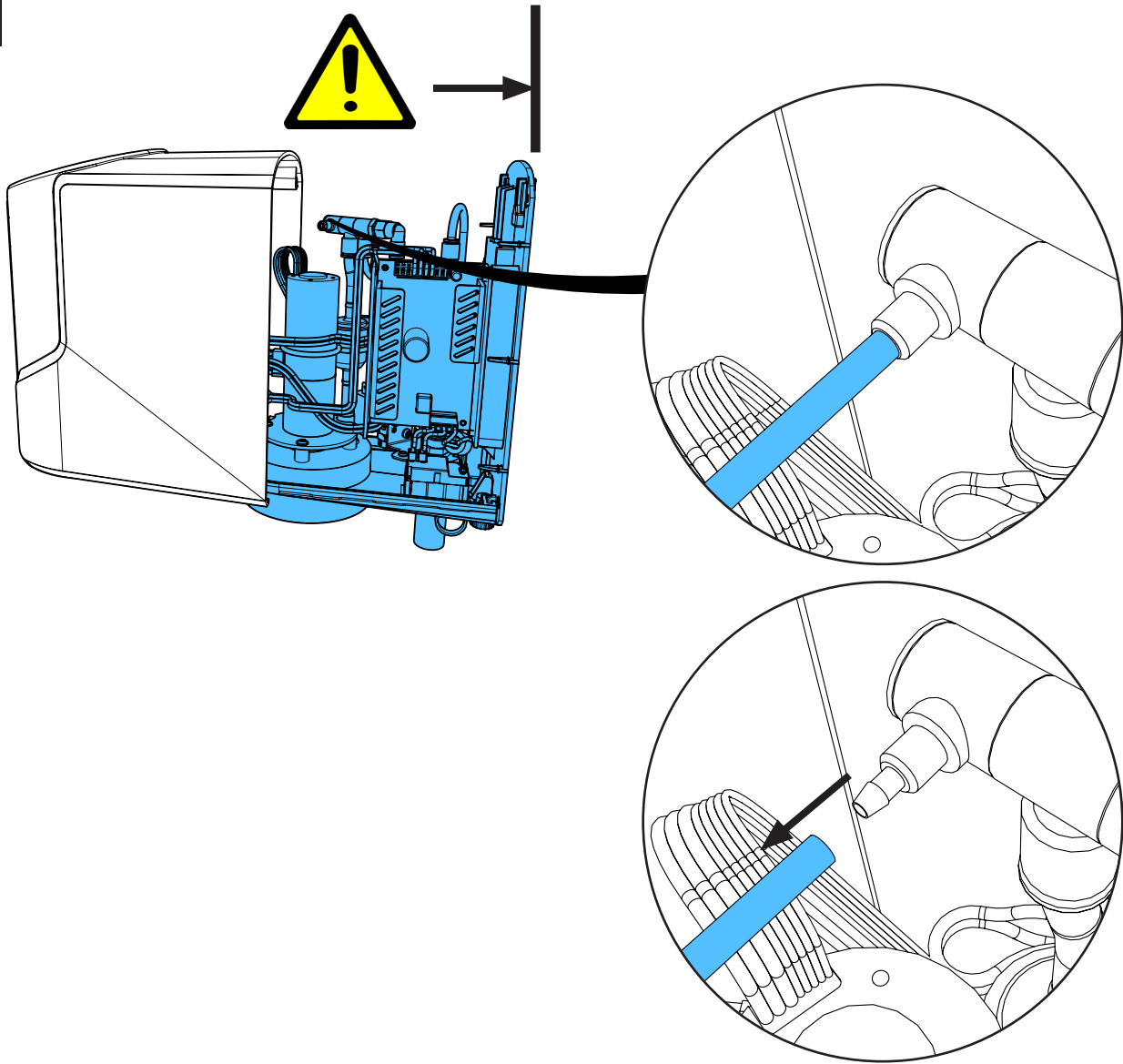






7

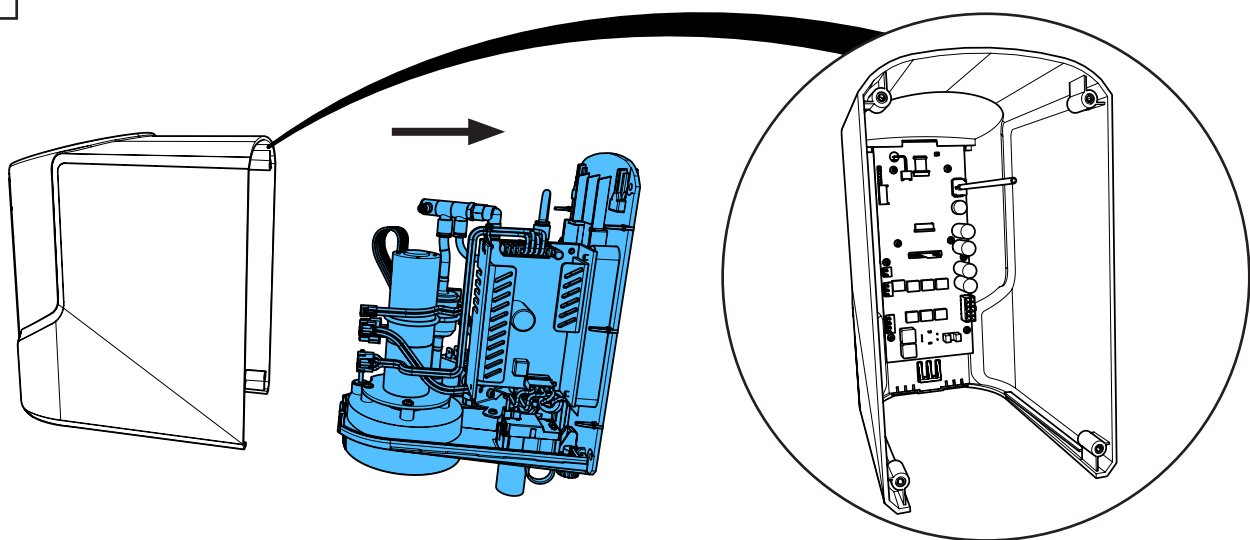




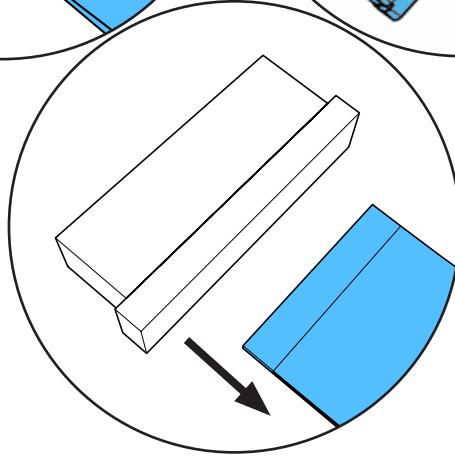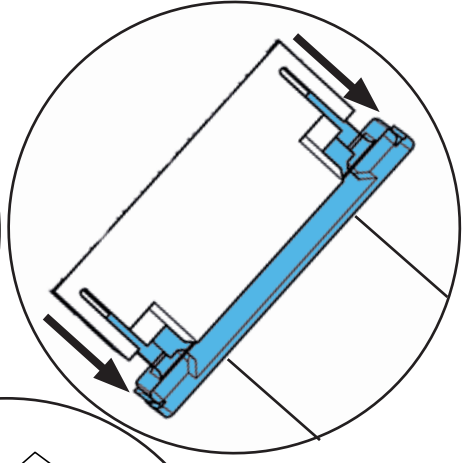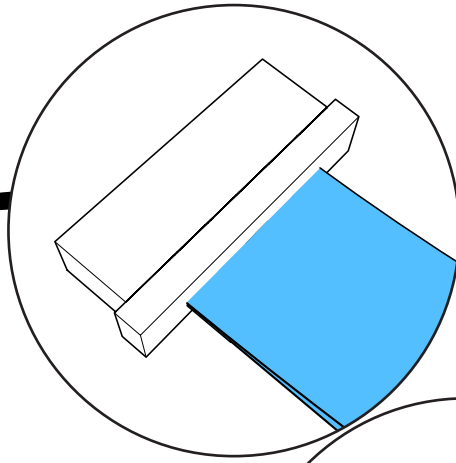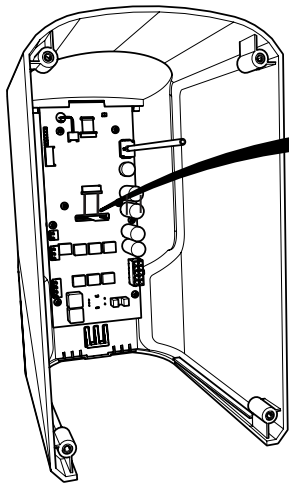






8

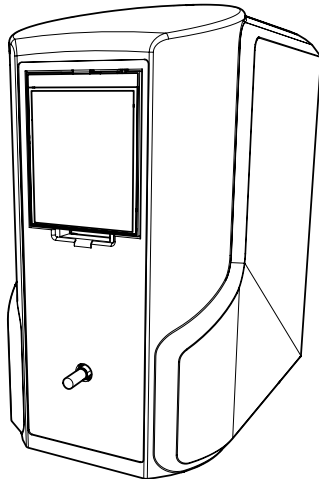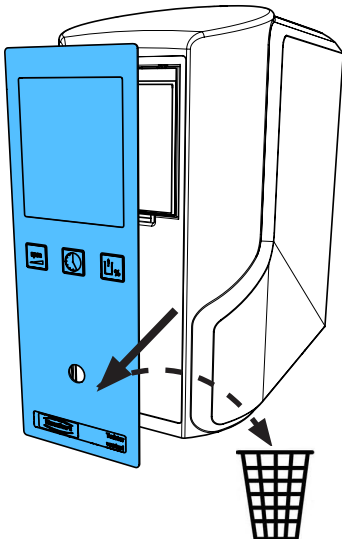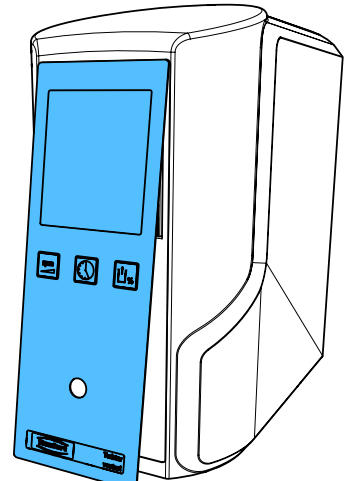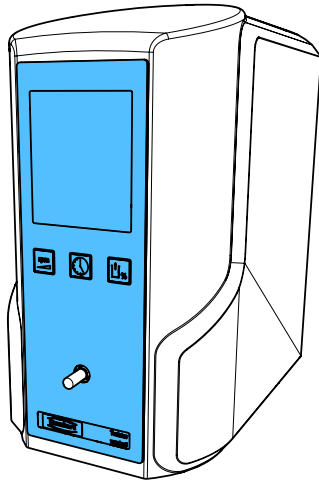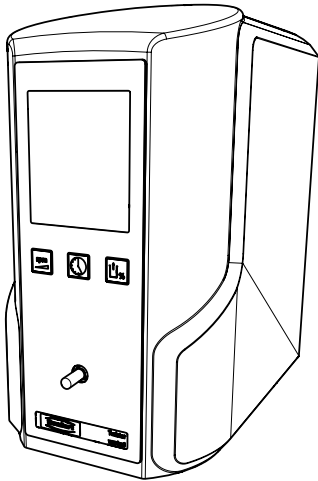



9

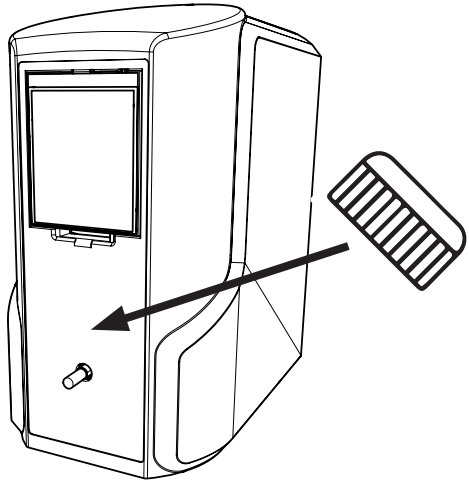







10

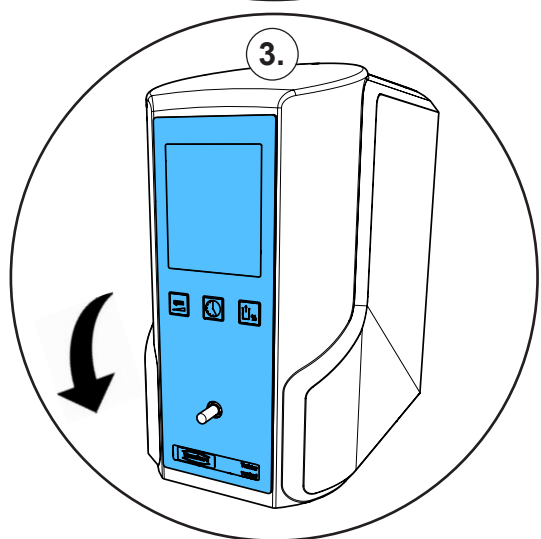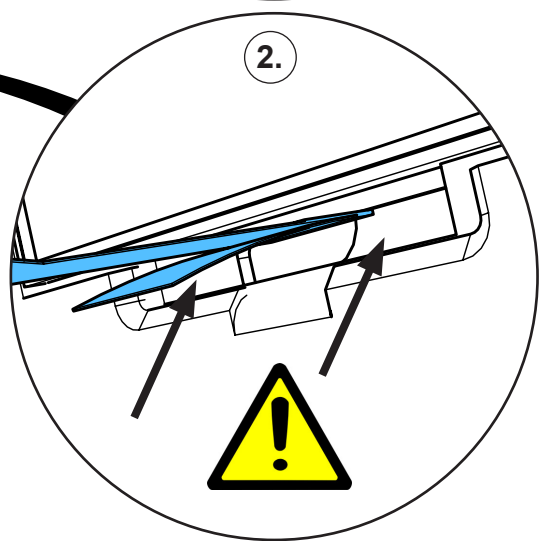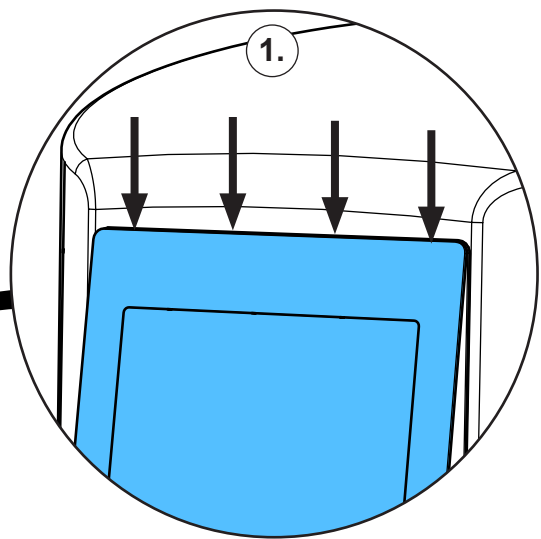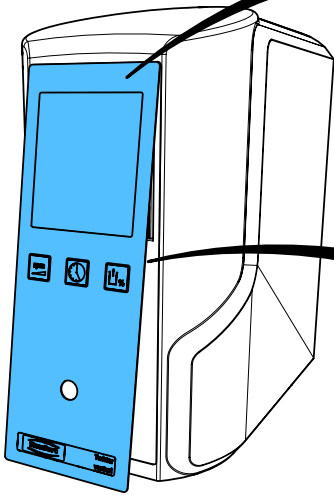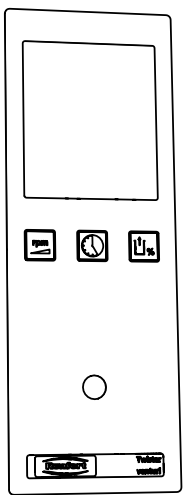




11

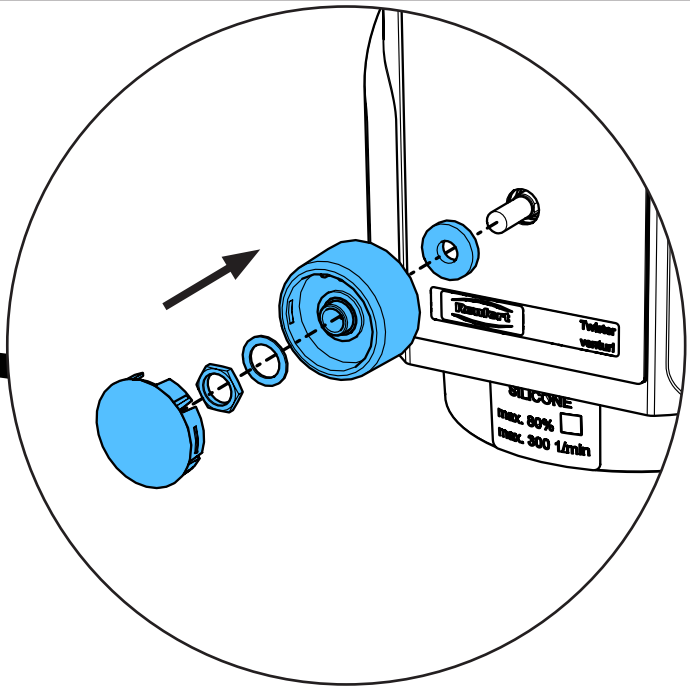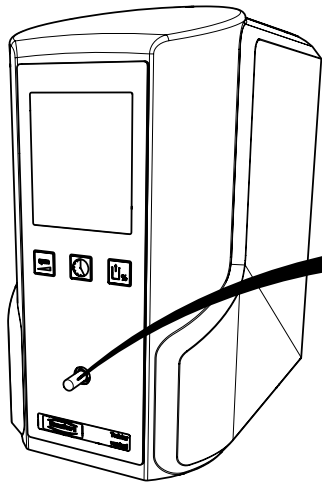


12

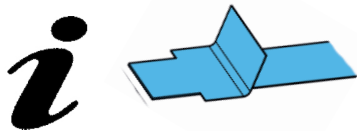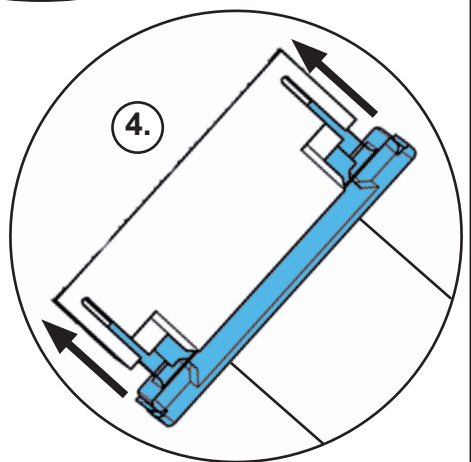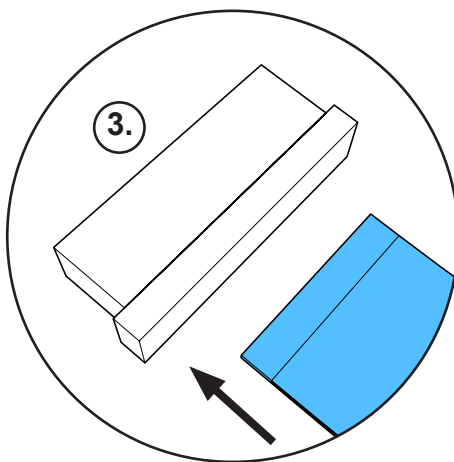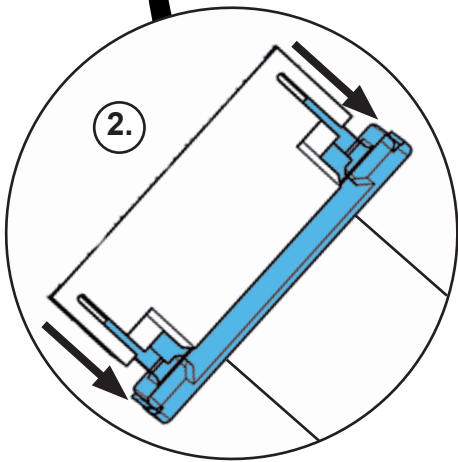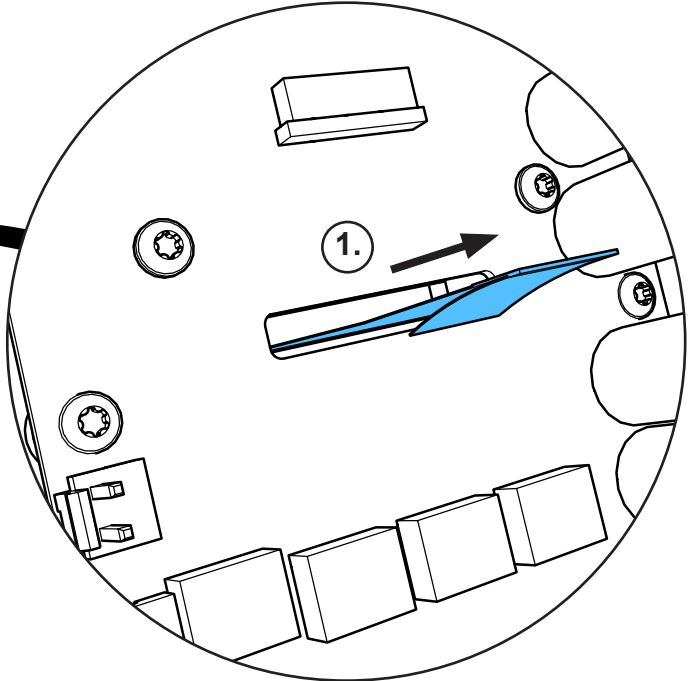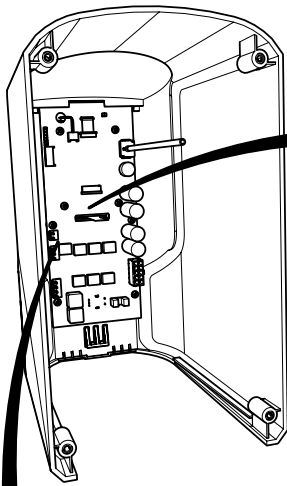


13

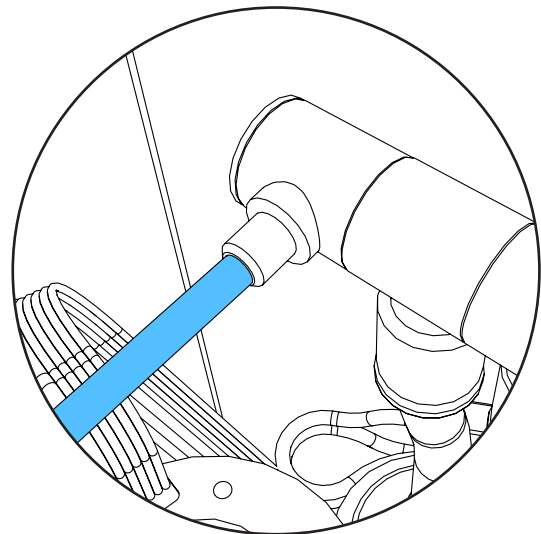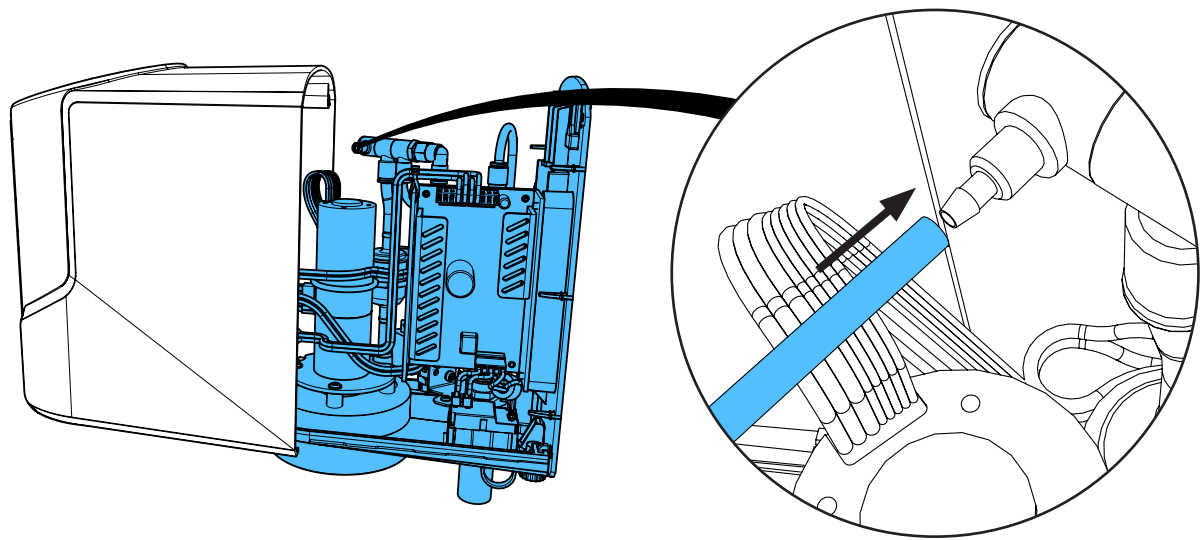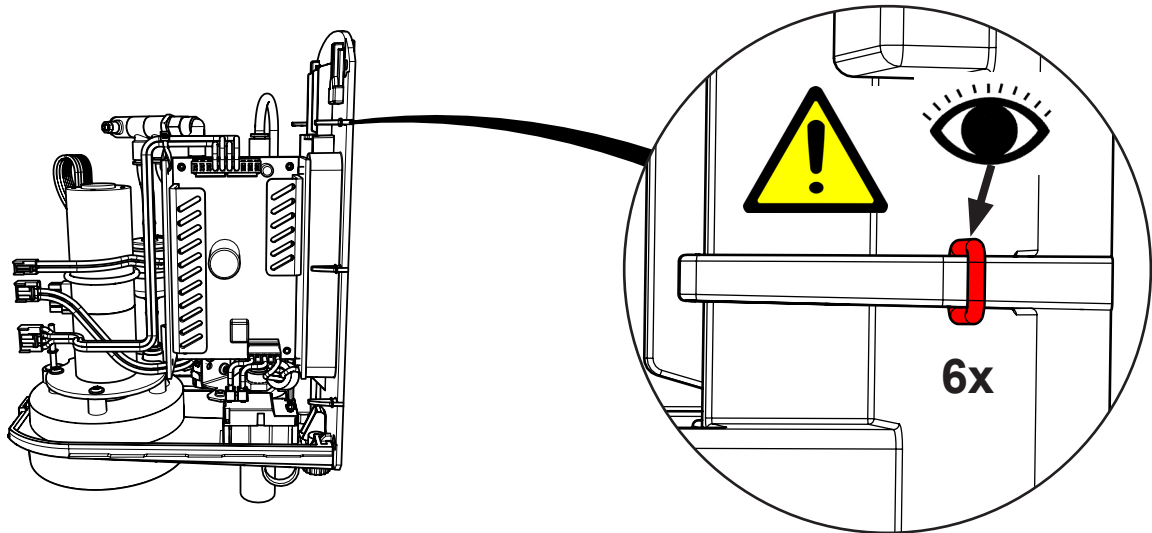


14

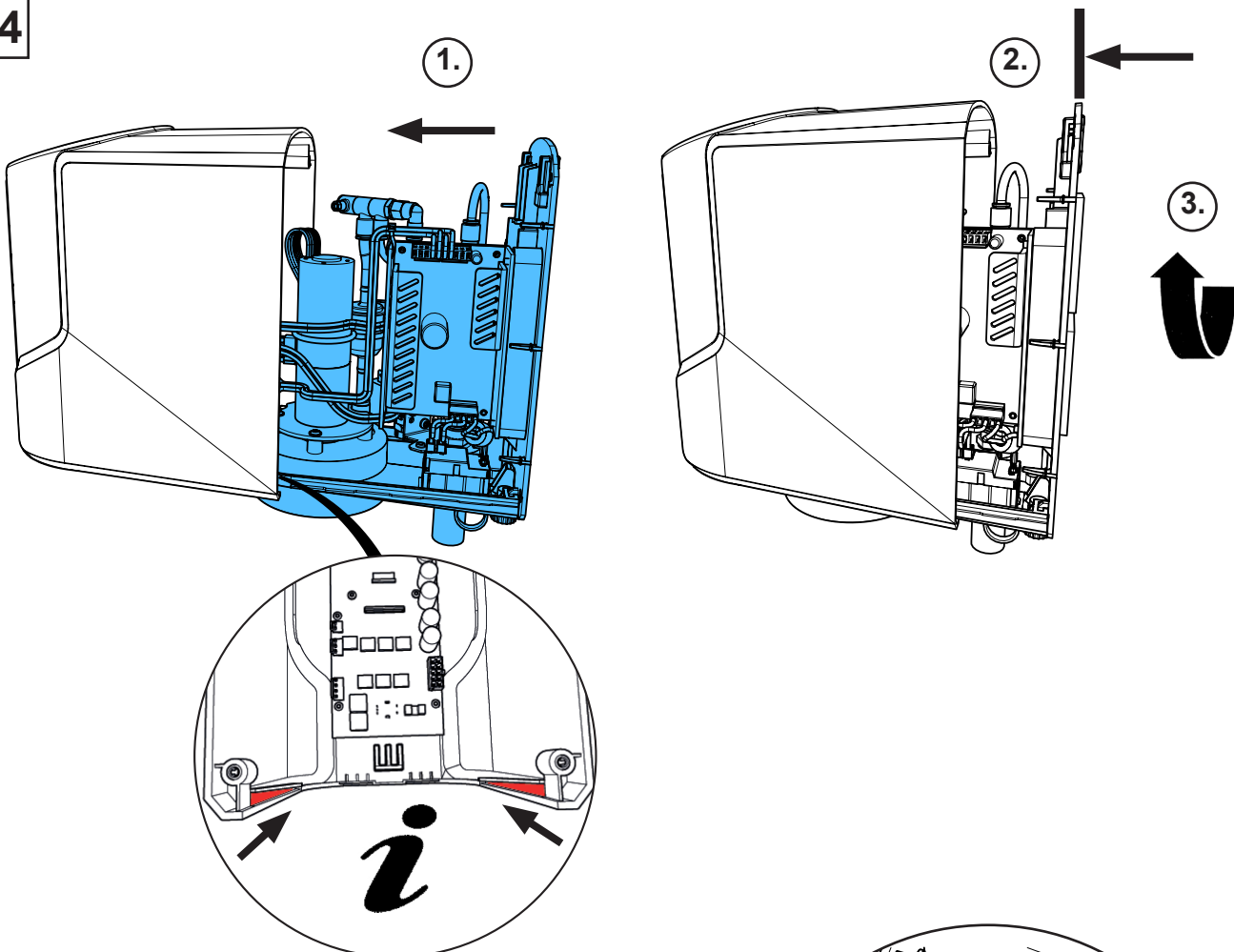


15

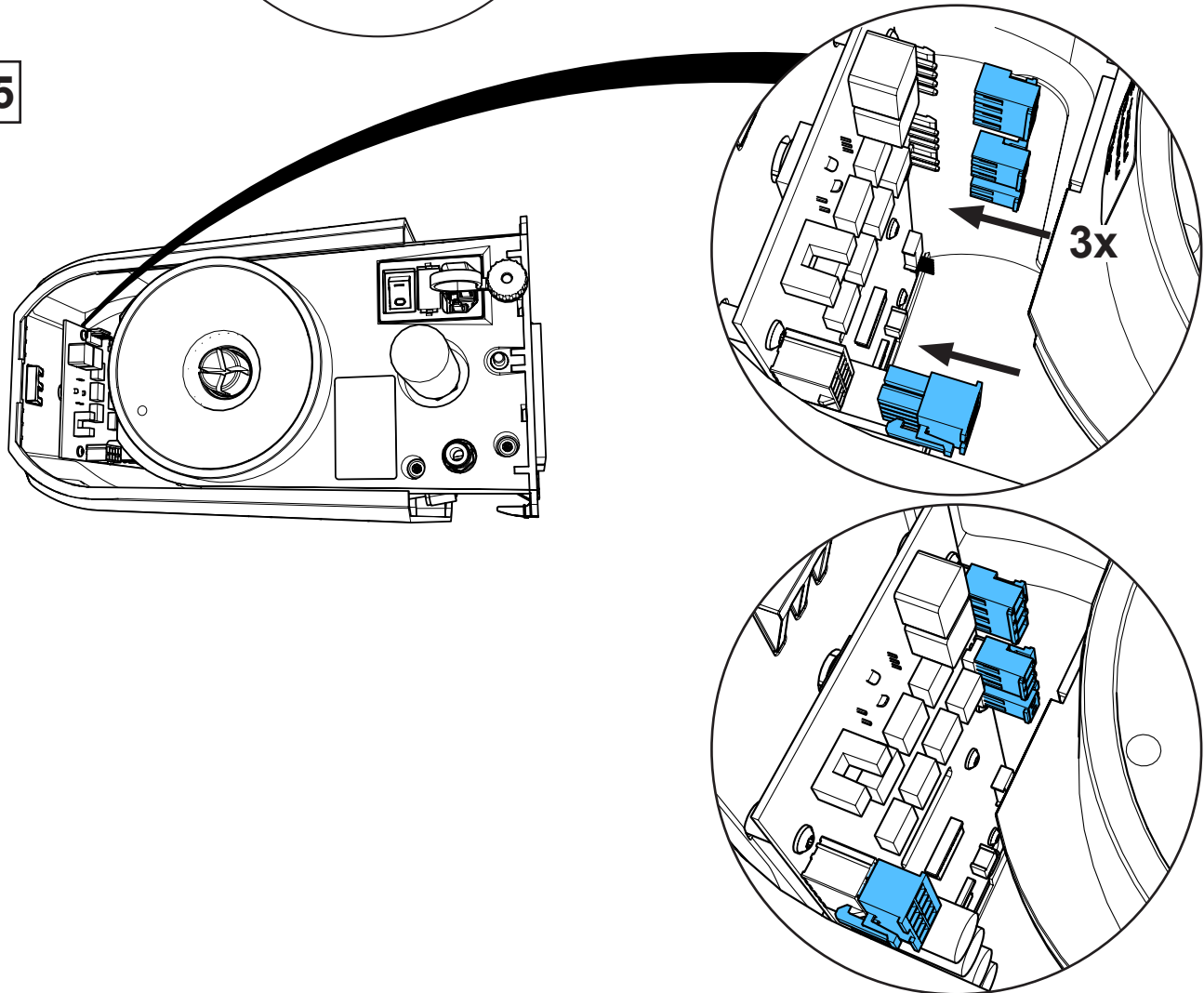


16

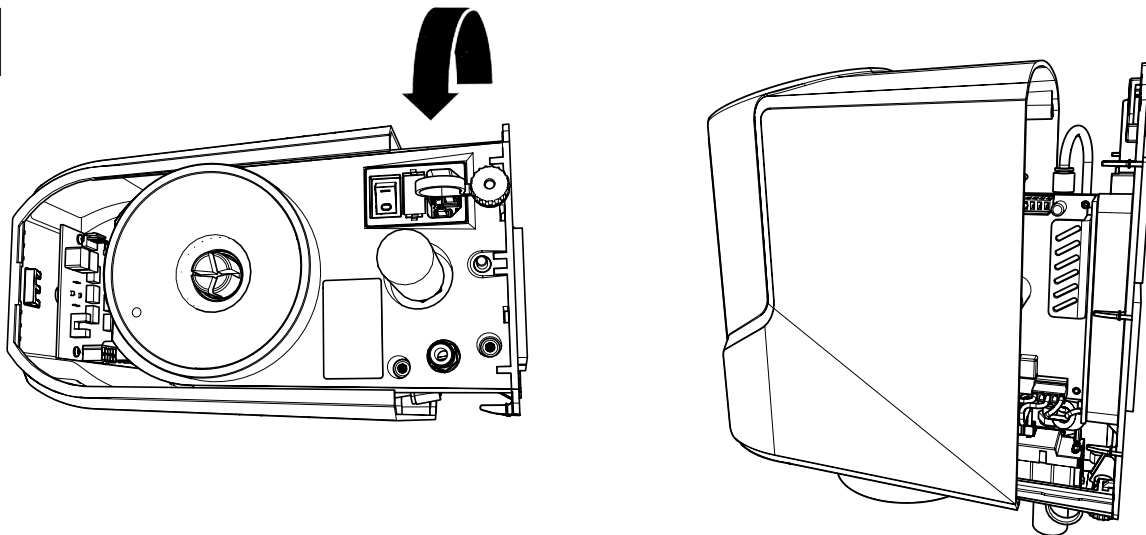

17


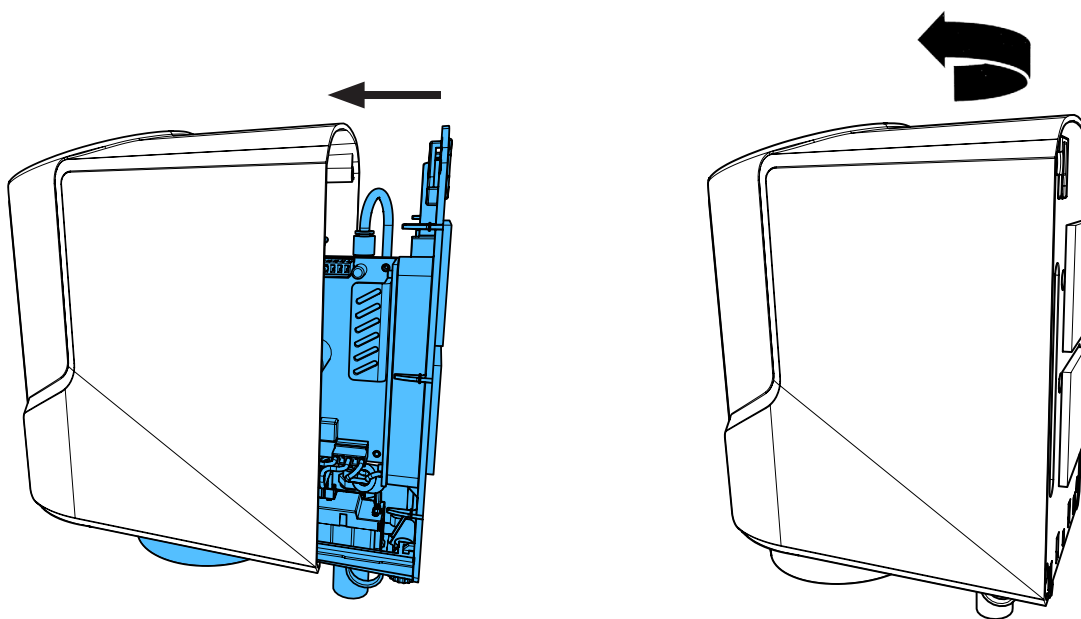


18

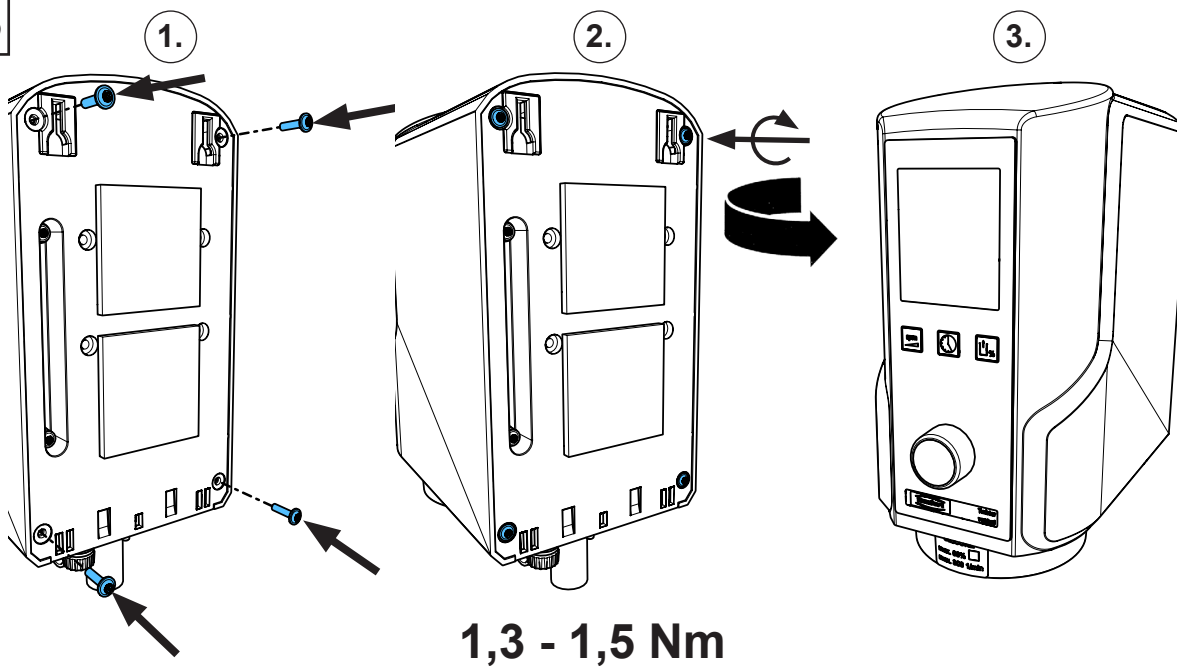


## 90003 4680


Frontfolie Twister evolution venturi • Frontfoil Twister evolution venturi • Membrane  
souple Twister evolution venturi • Membrana comandi p. Twister evolution venturi  
Lámina p. elementos de mando Twister evolution venturi • Фронтальная пленка

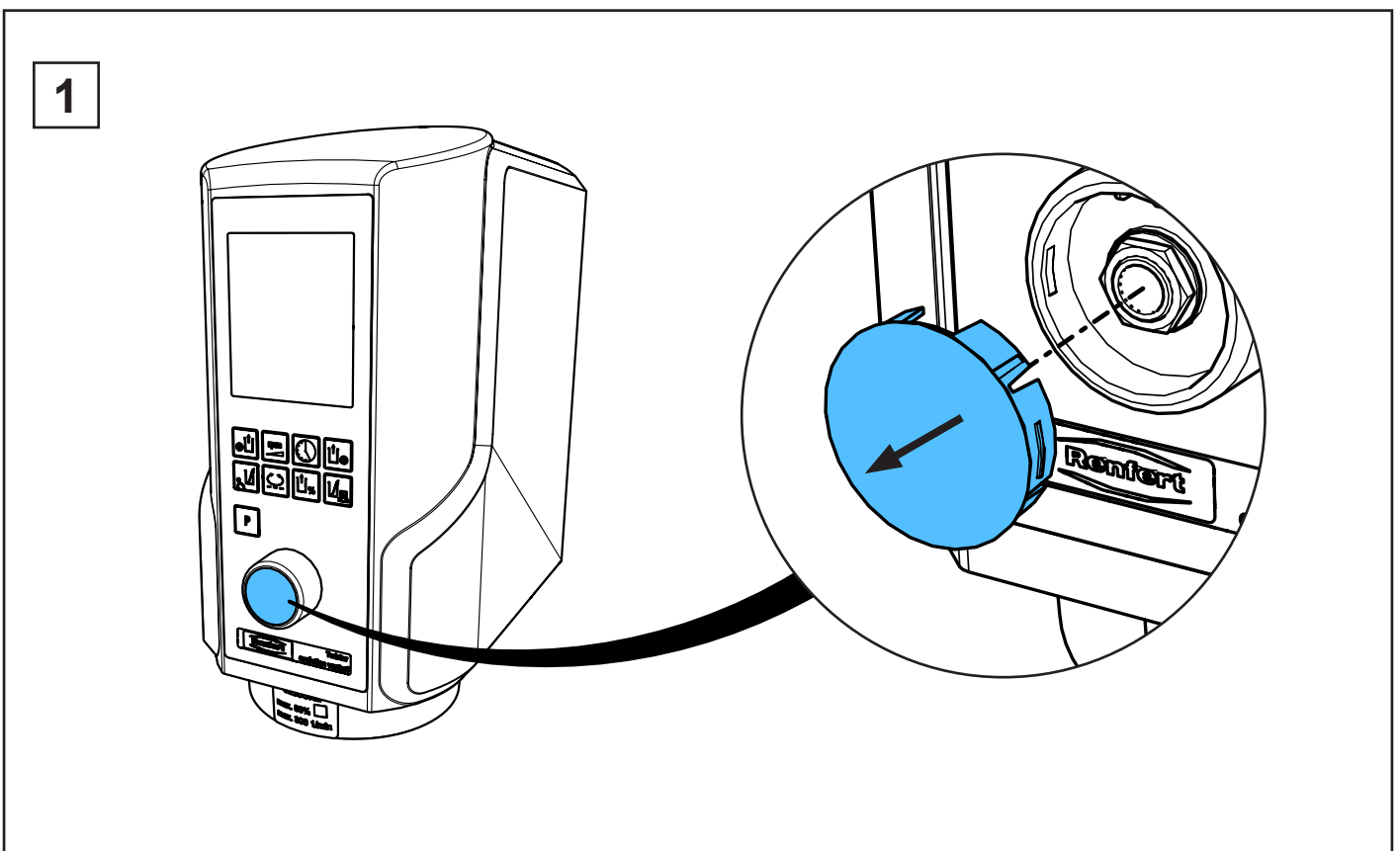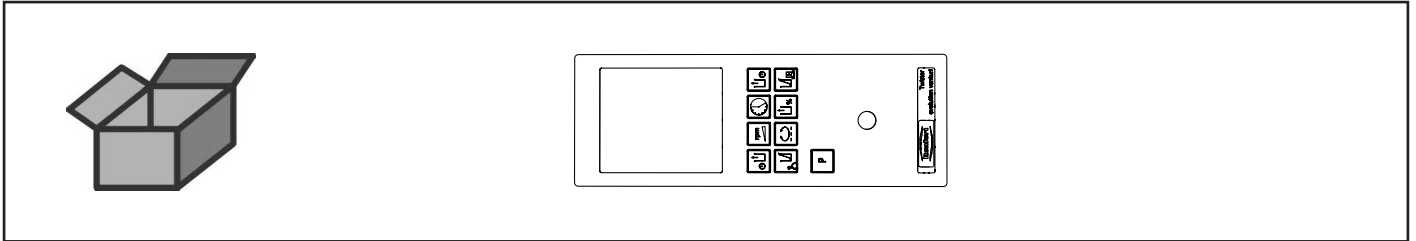

2




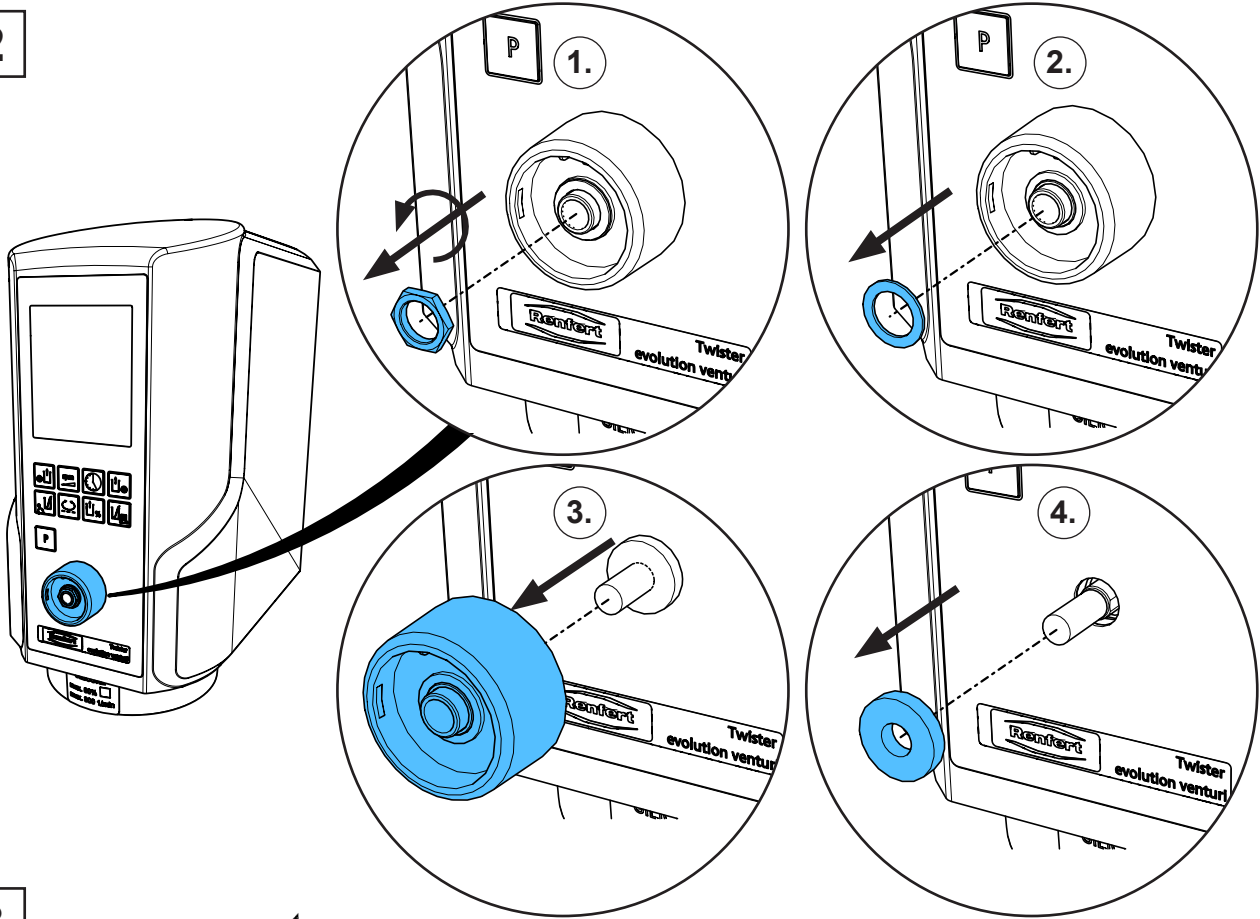


3


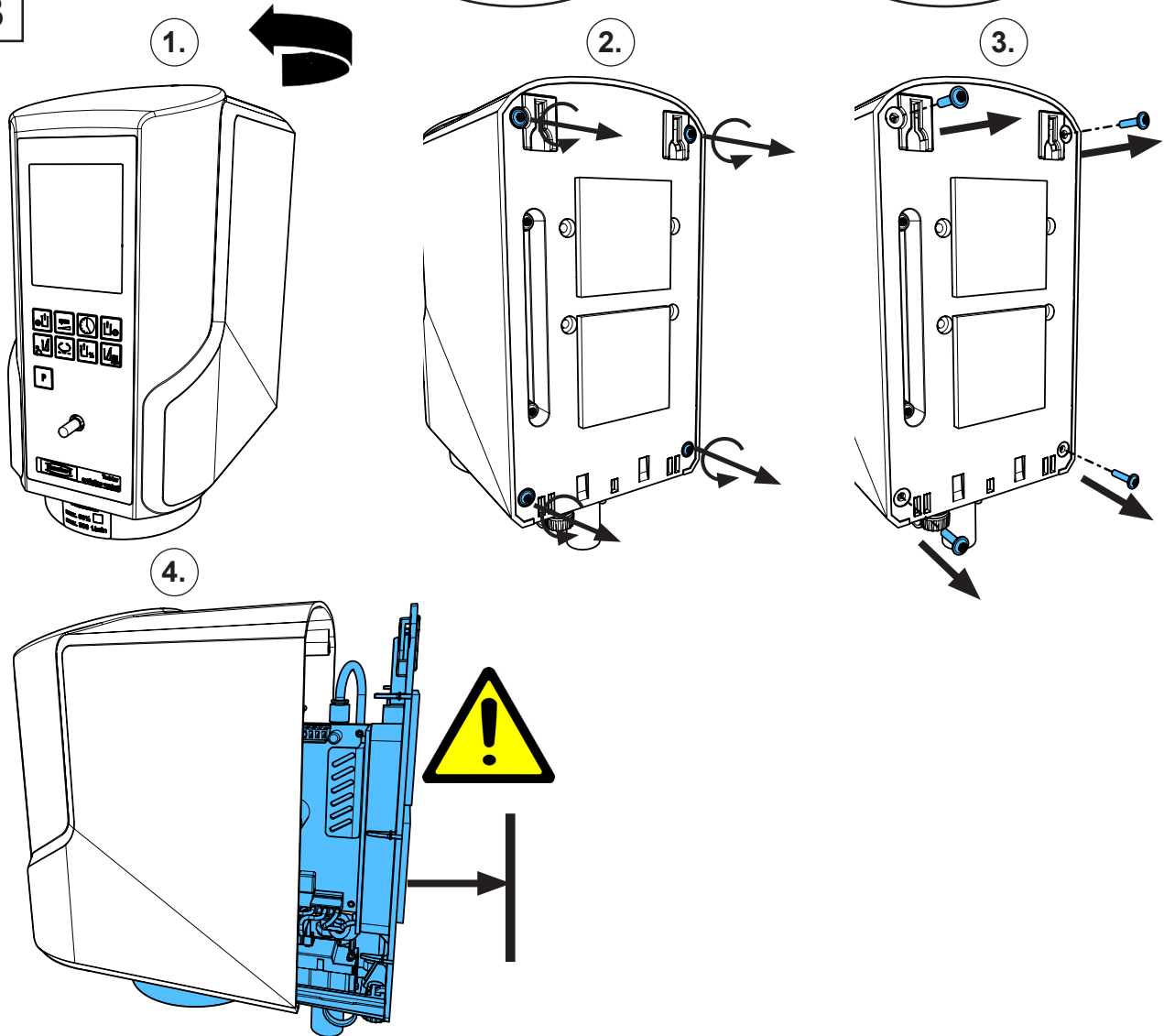





4

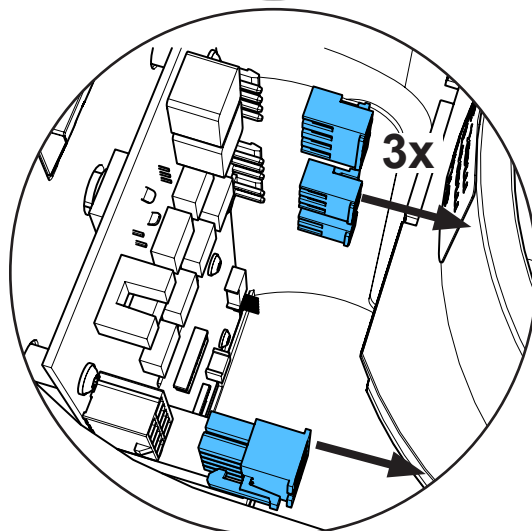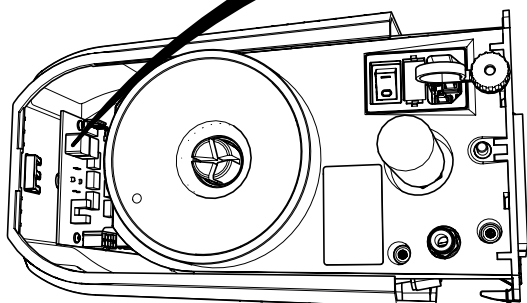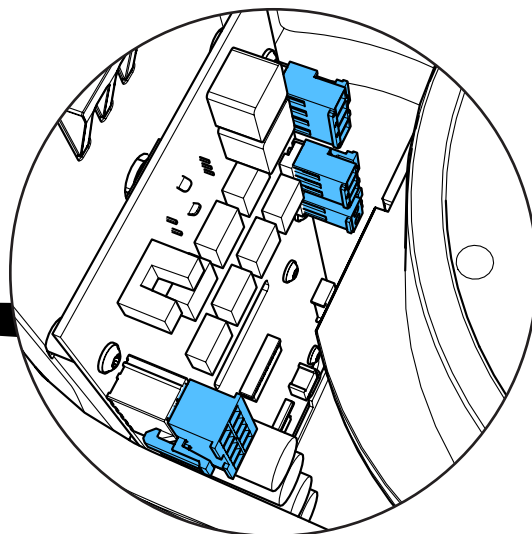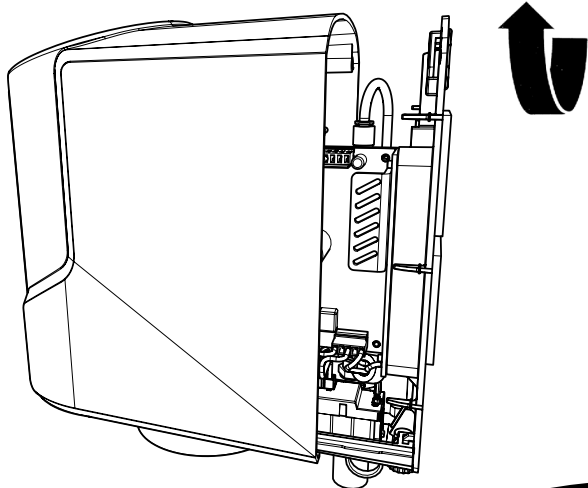


5



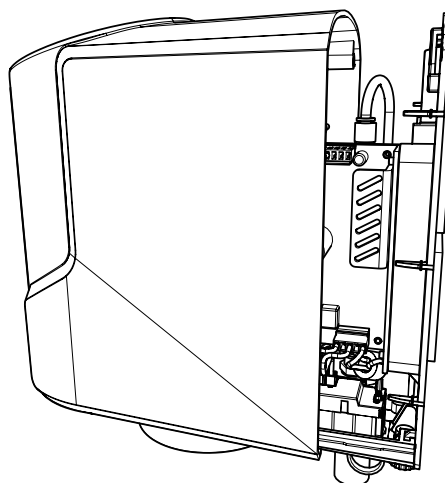
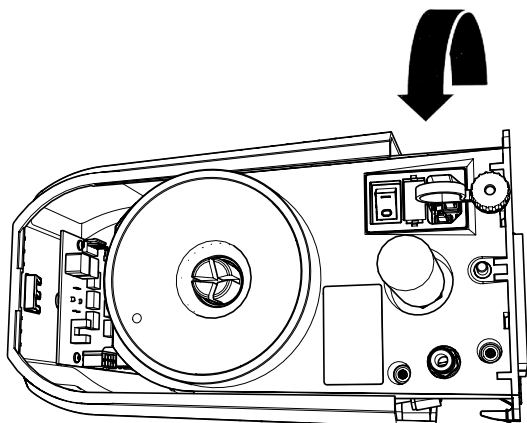


6





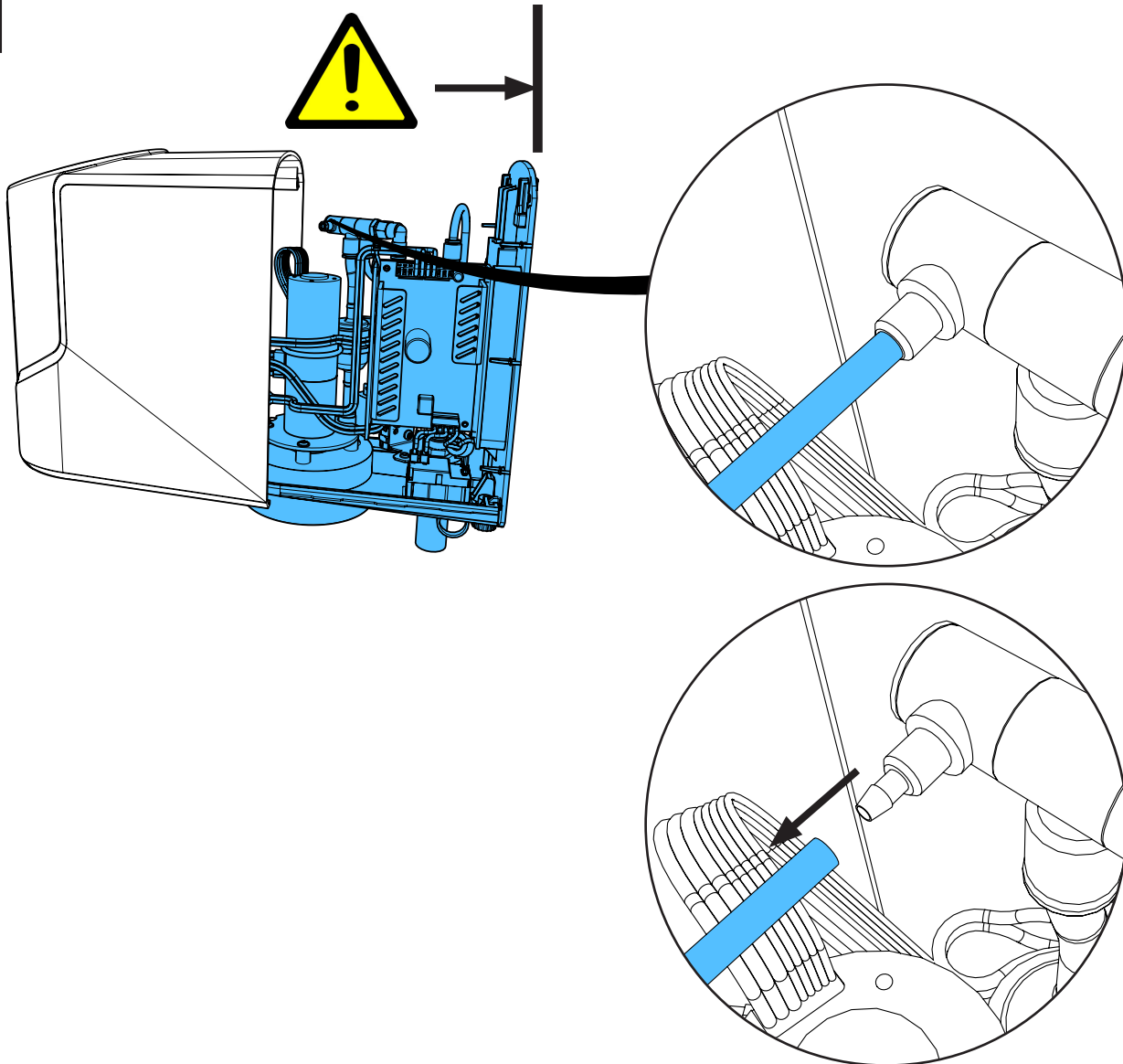


7

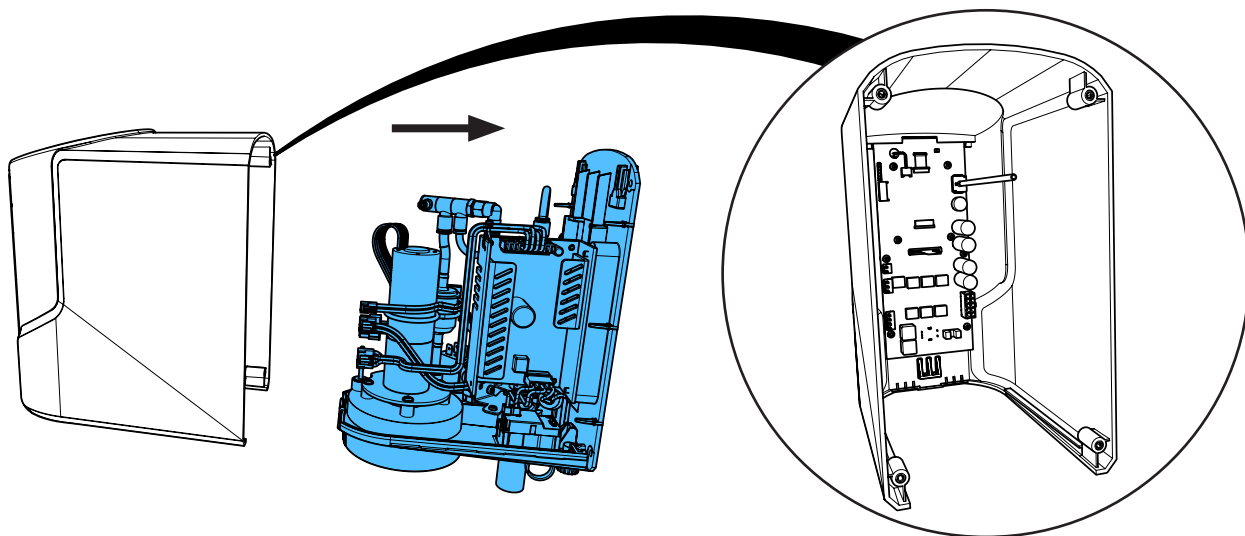


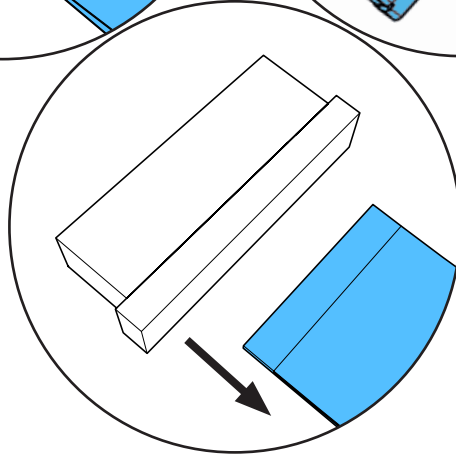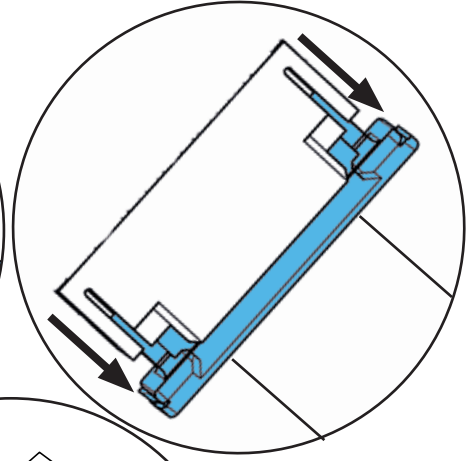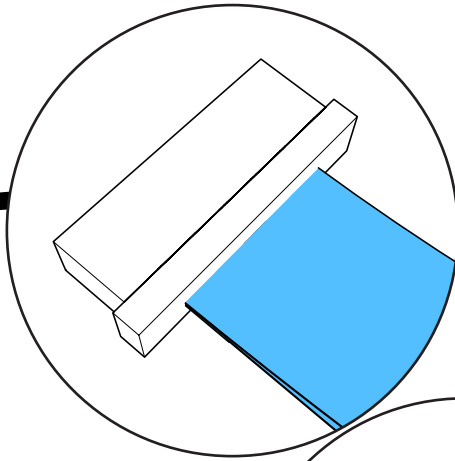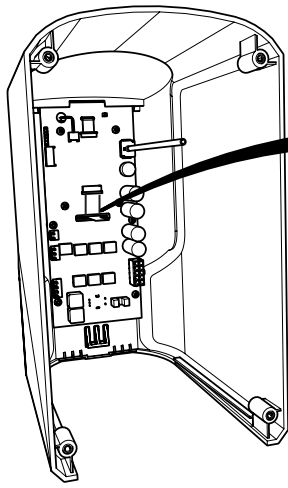






8

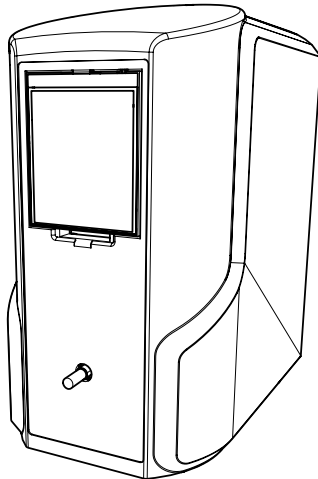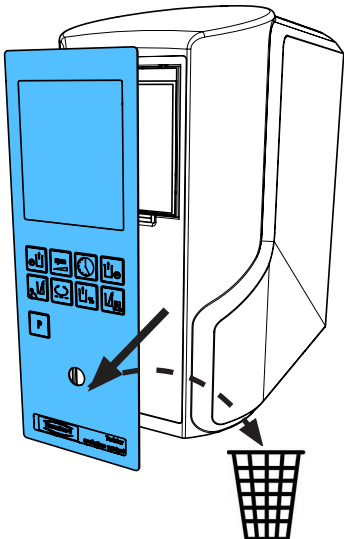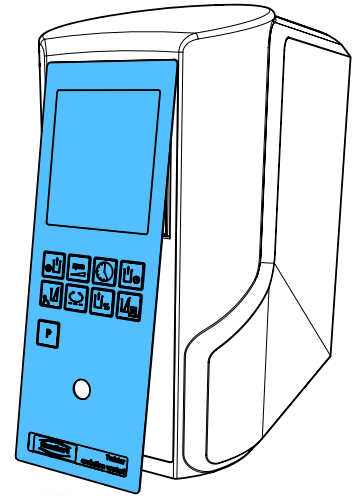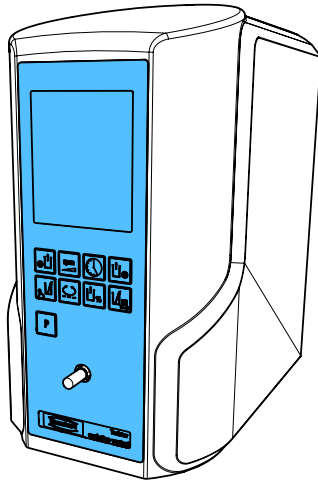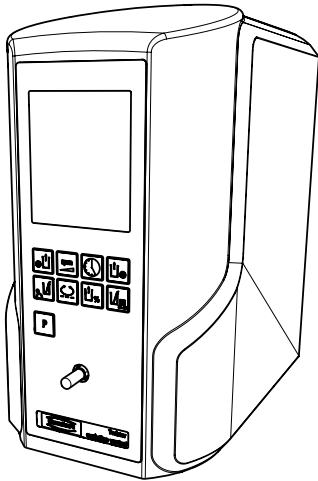



9

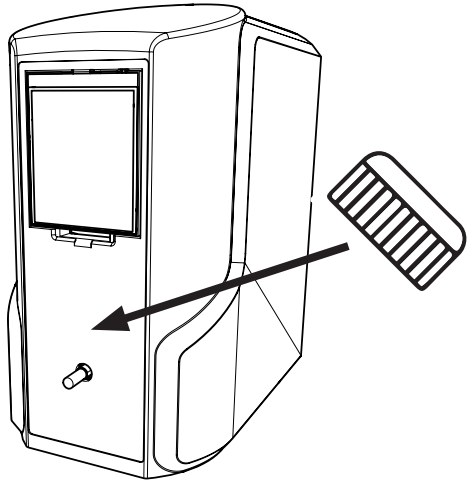







10

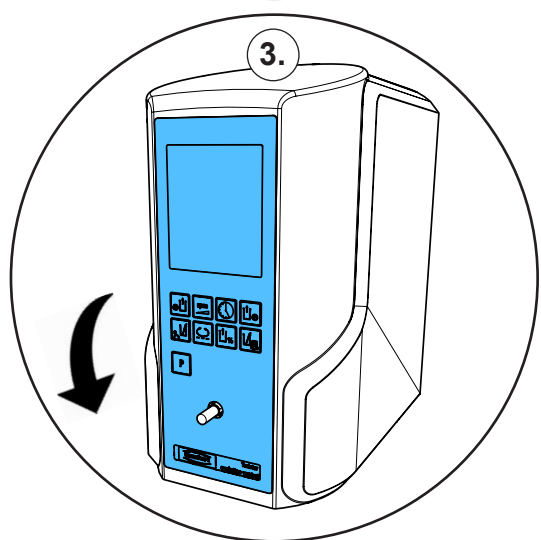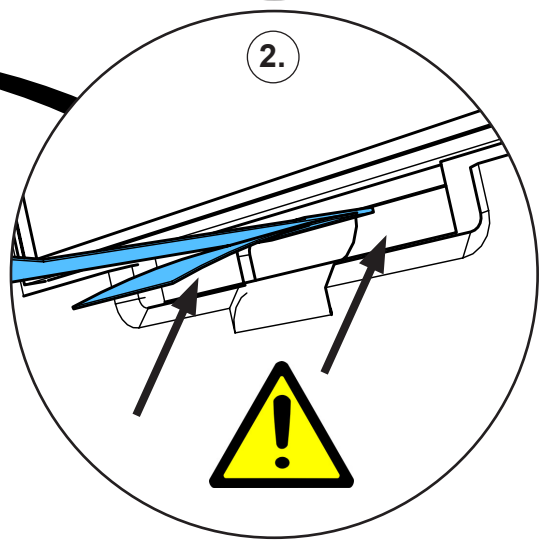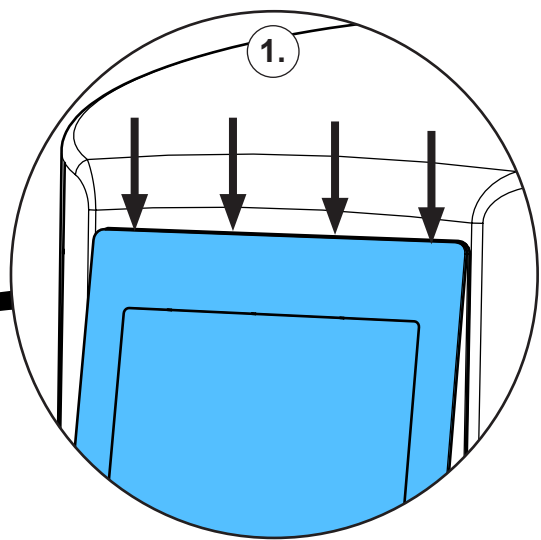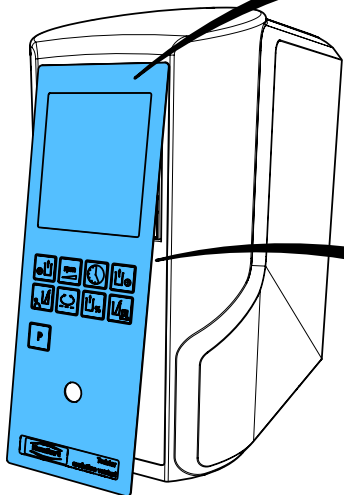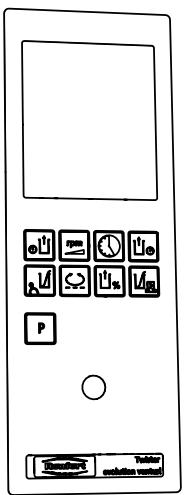




11

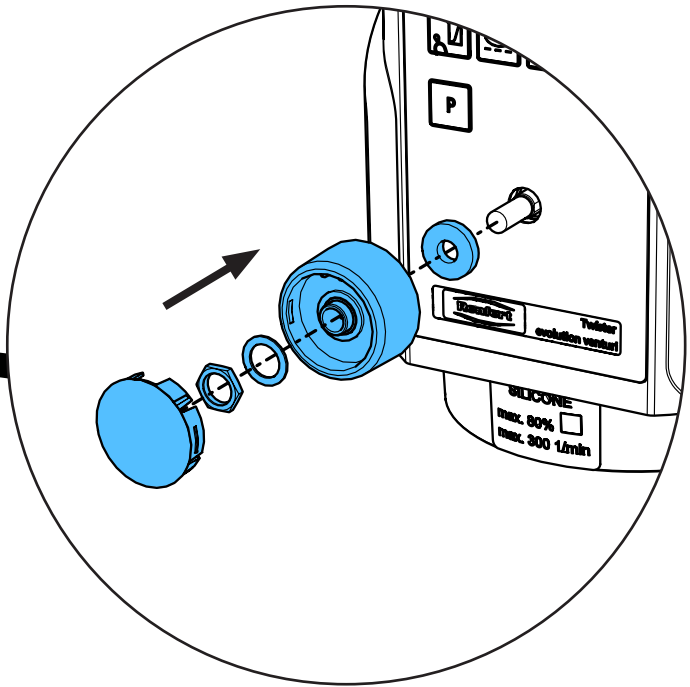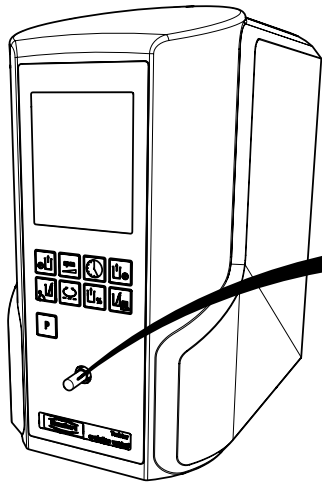



12

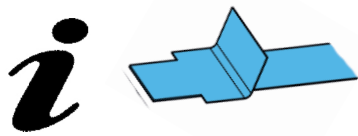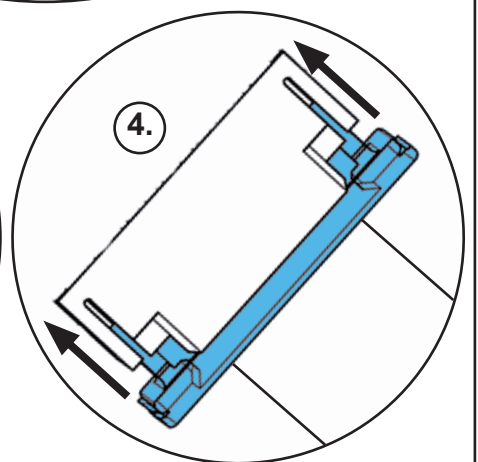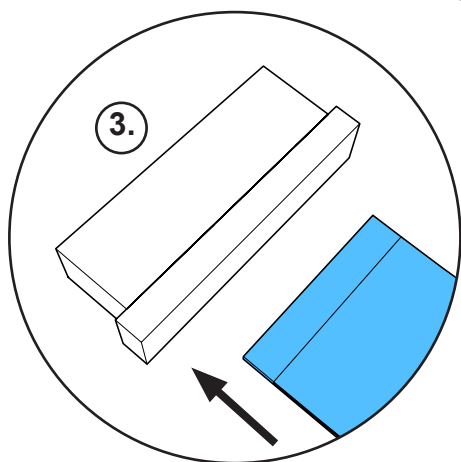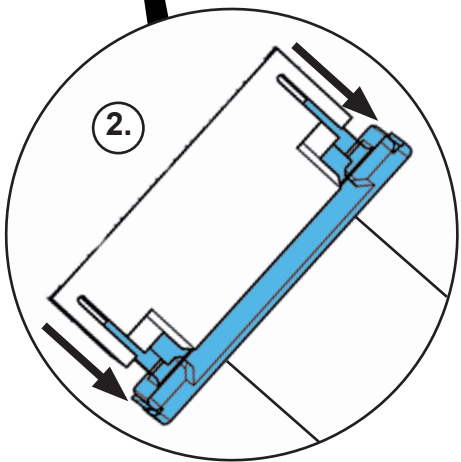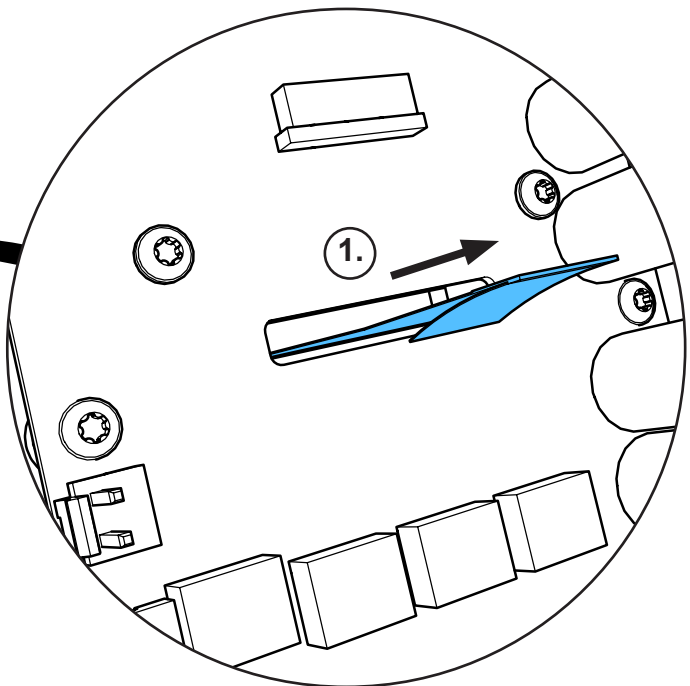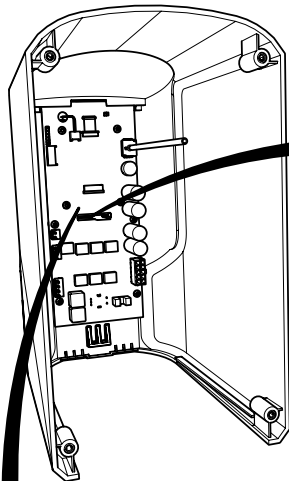


13

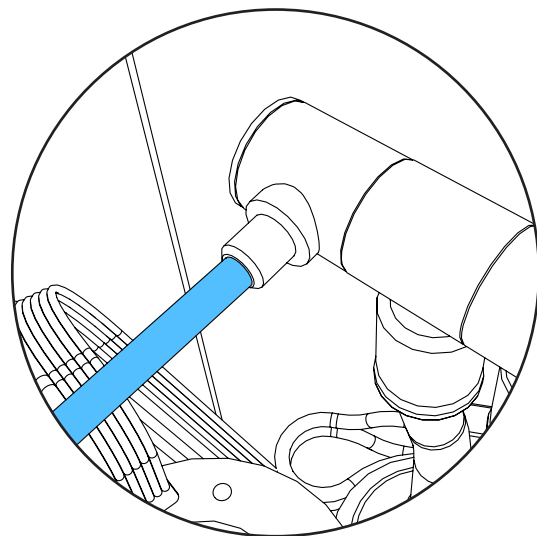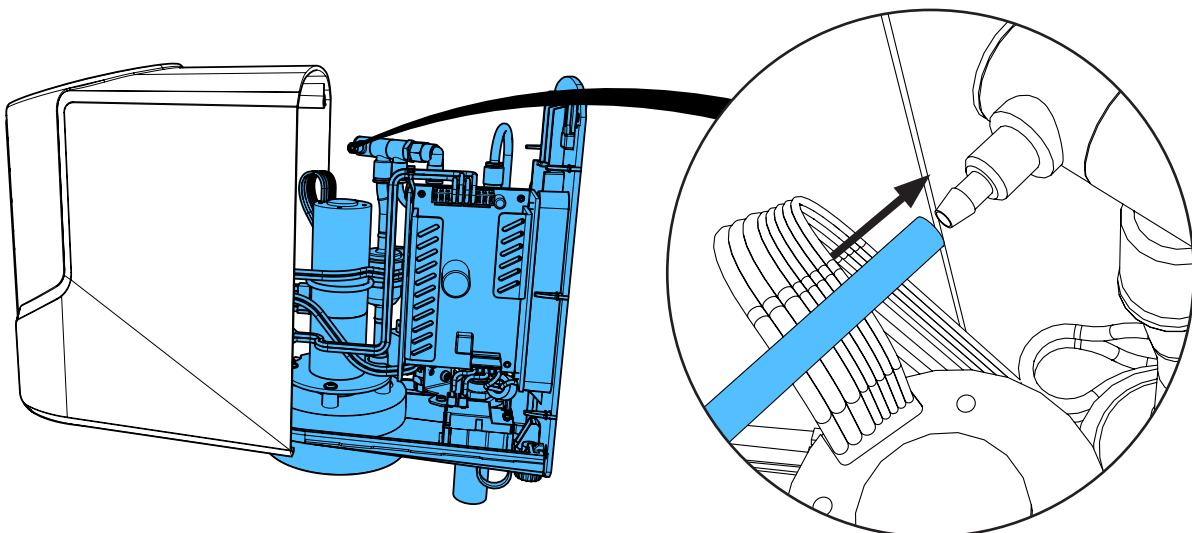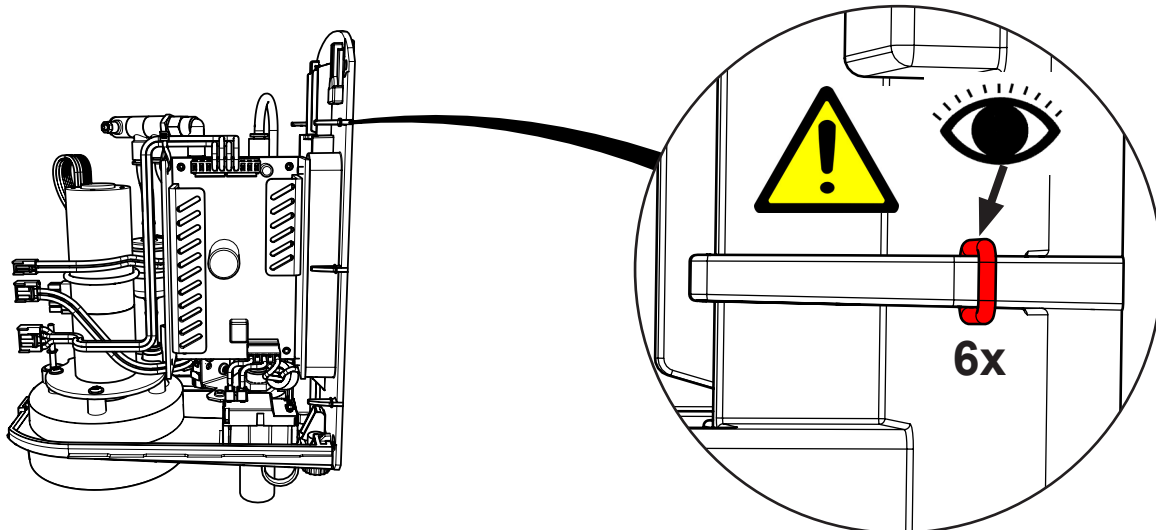


14

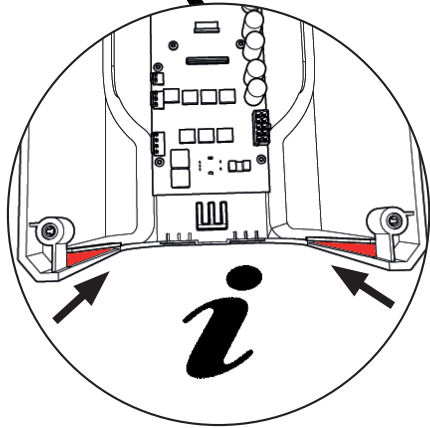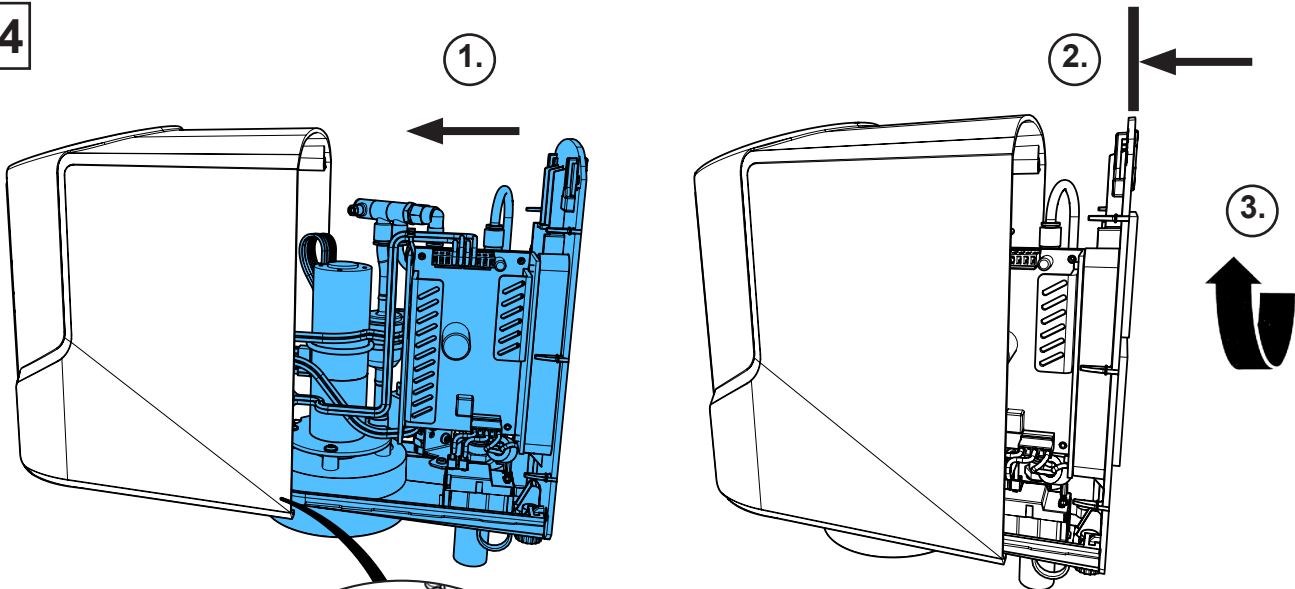


15

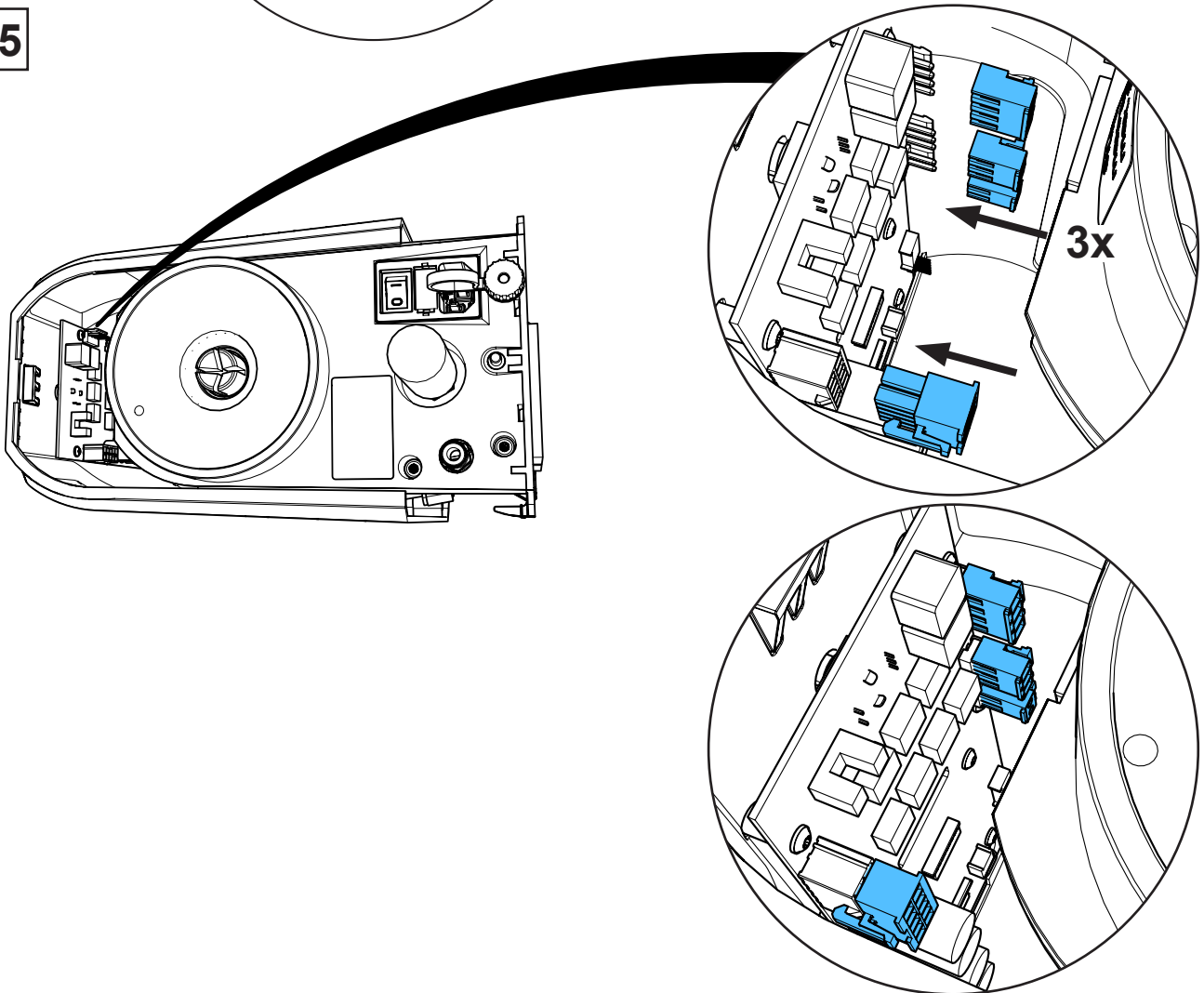


16

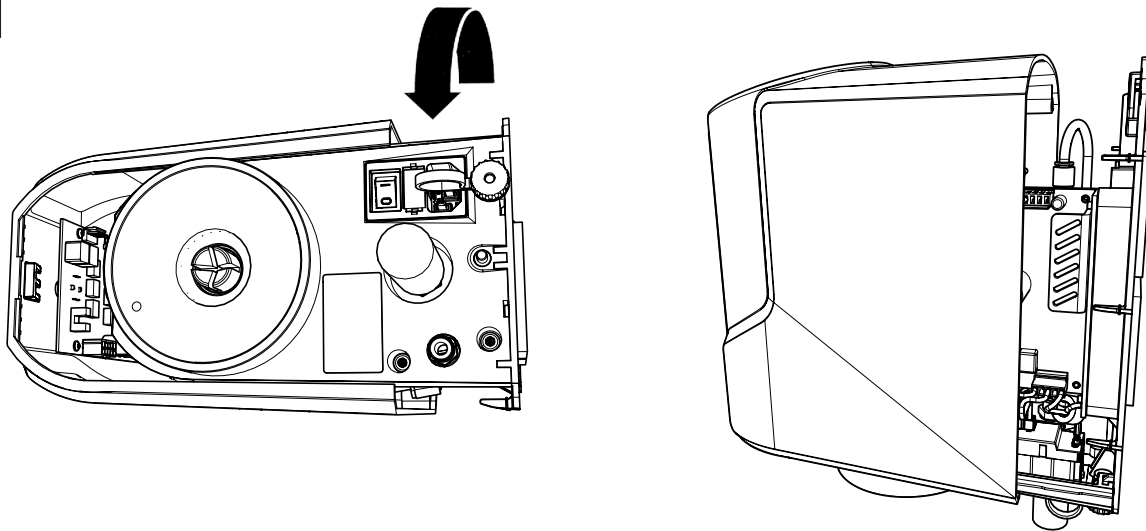

17

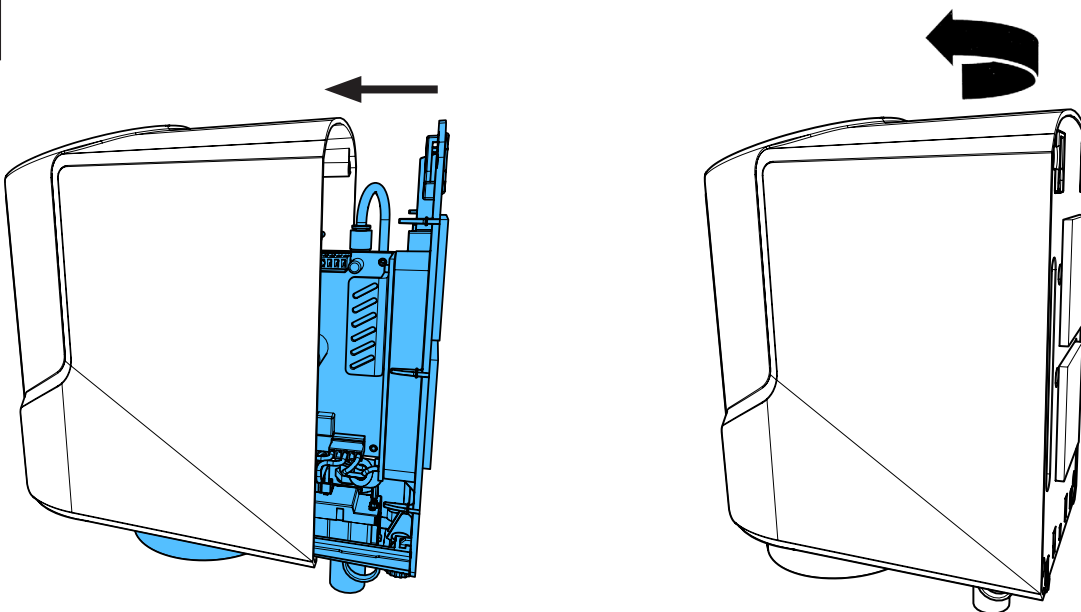




18

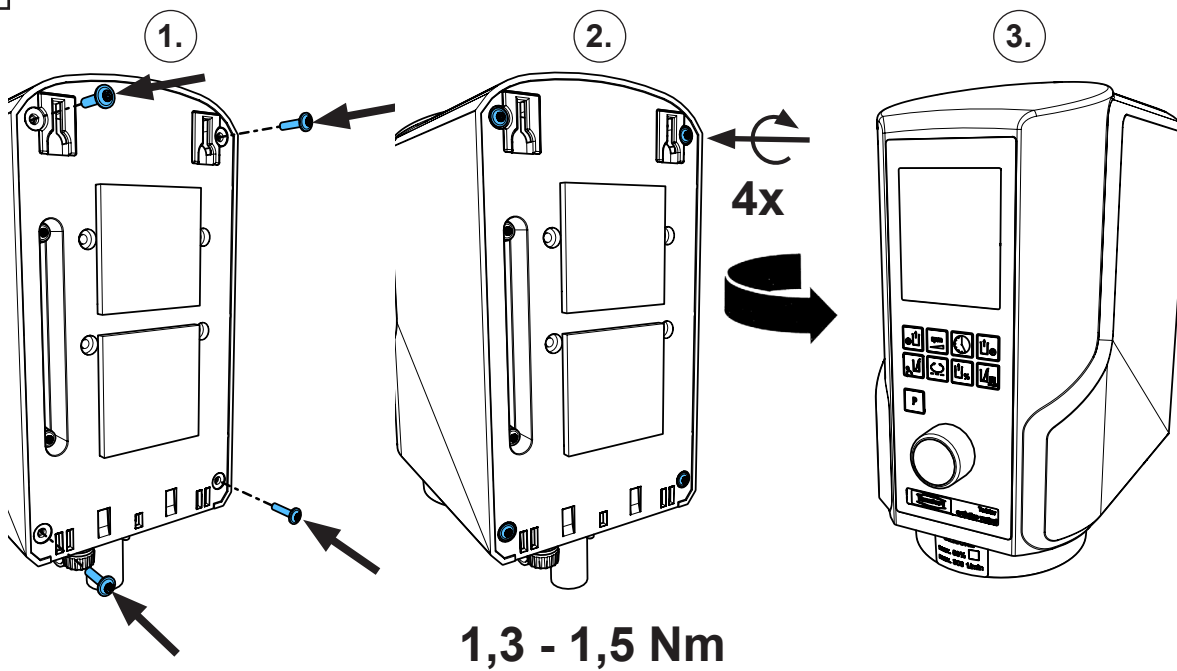




## 90013 4747


Leiterplatte kompl. 1827 • Printed circuit compl. 1827 • Carta imprimée compl. 1827  
Circuito stampato compl.1827 • Circuito impresso compl. 1827  
Печатная плата в комплекте 1827

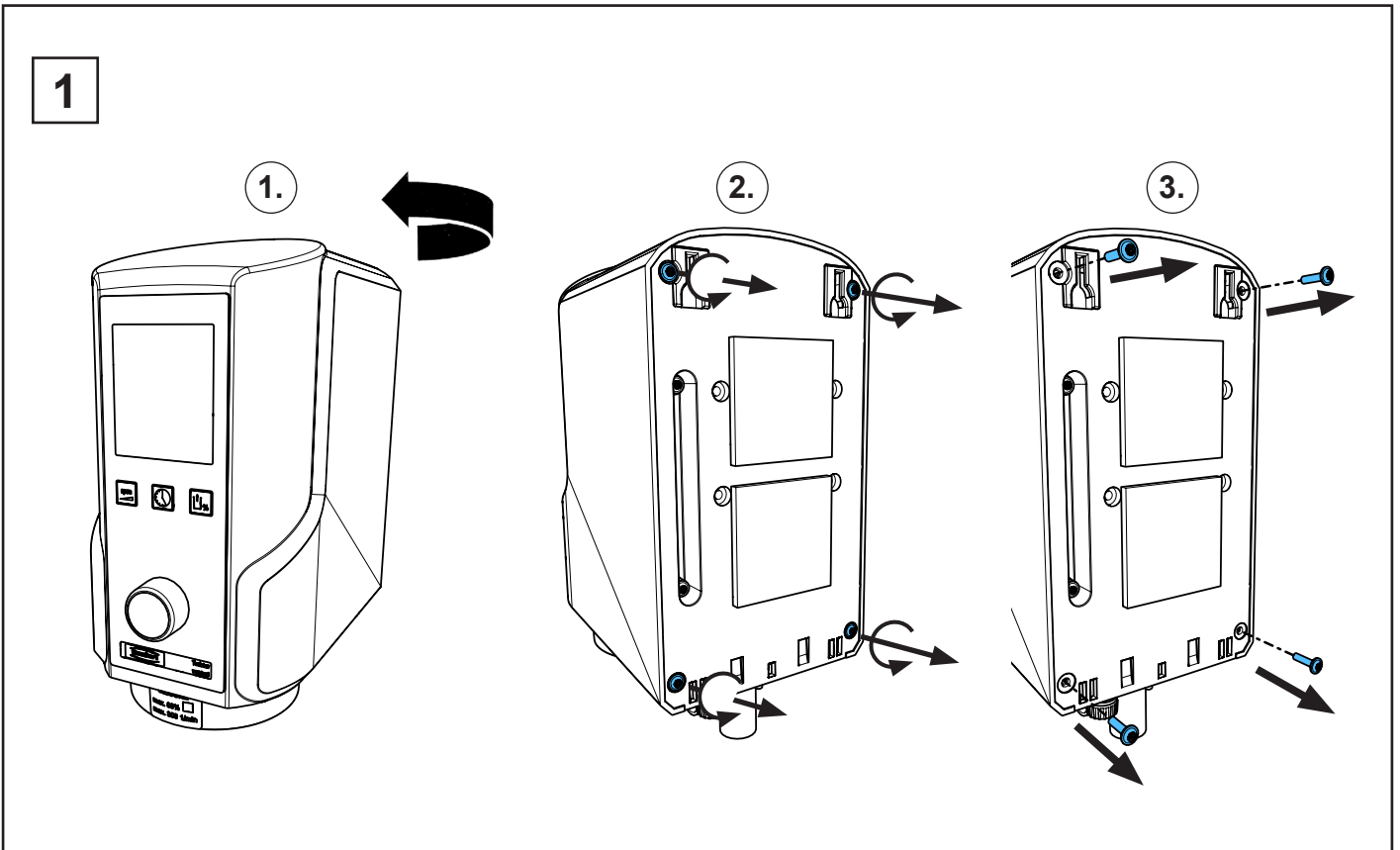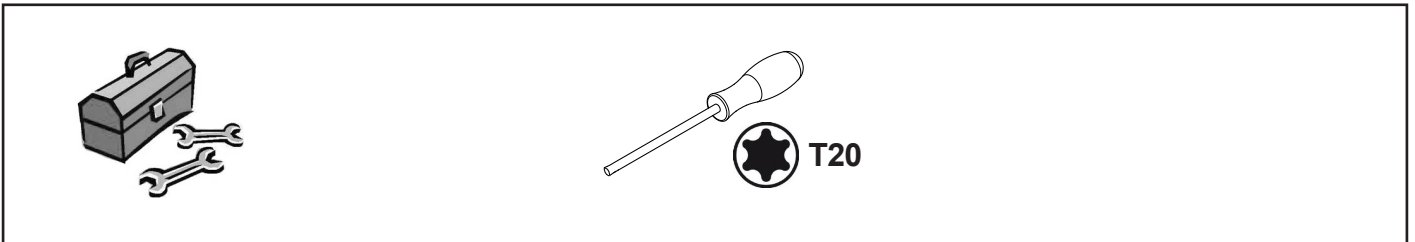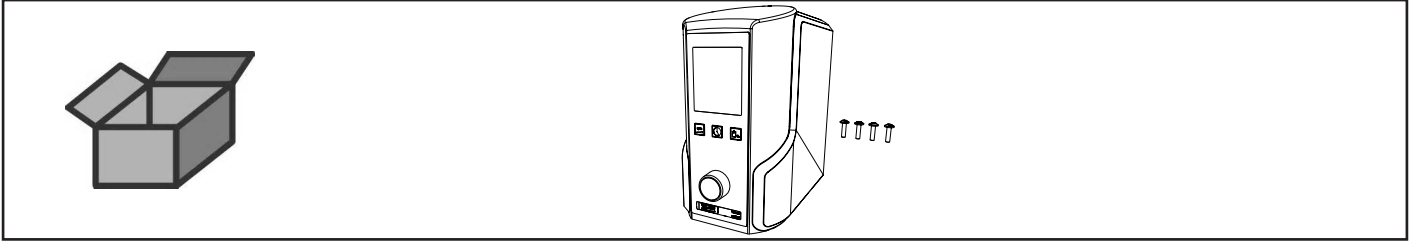


2

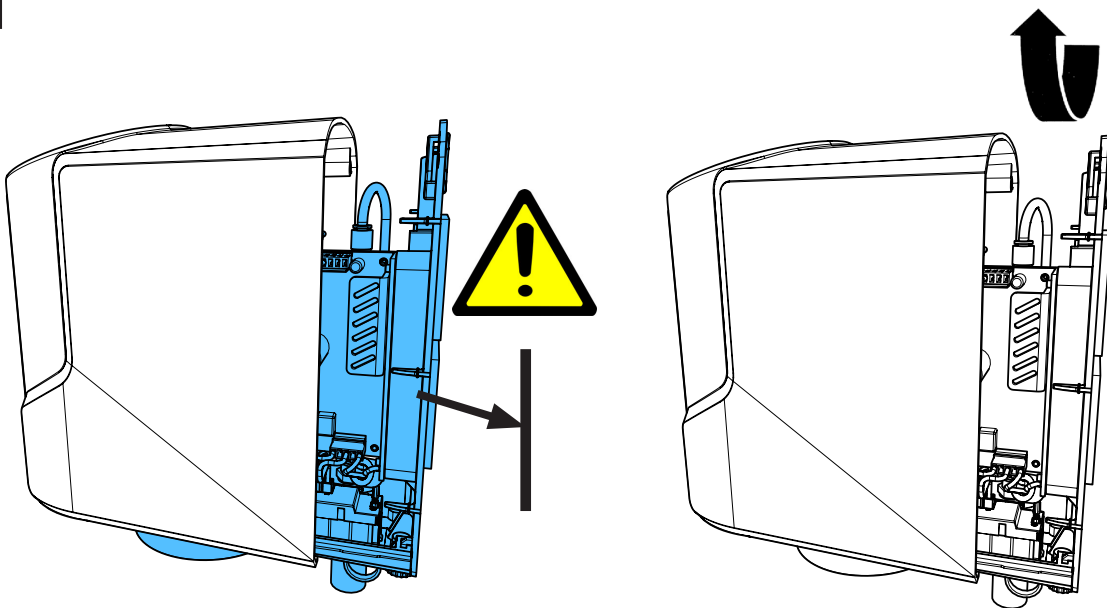


3

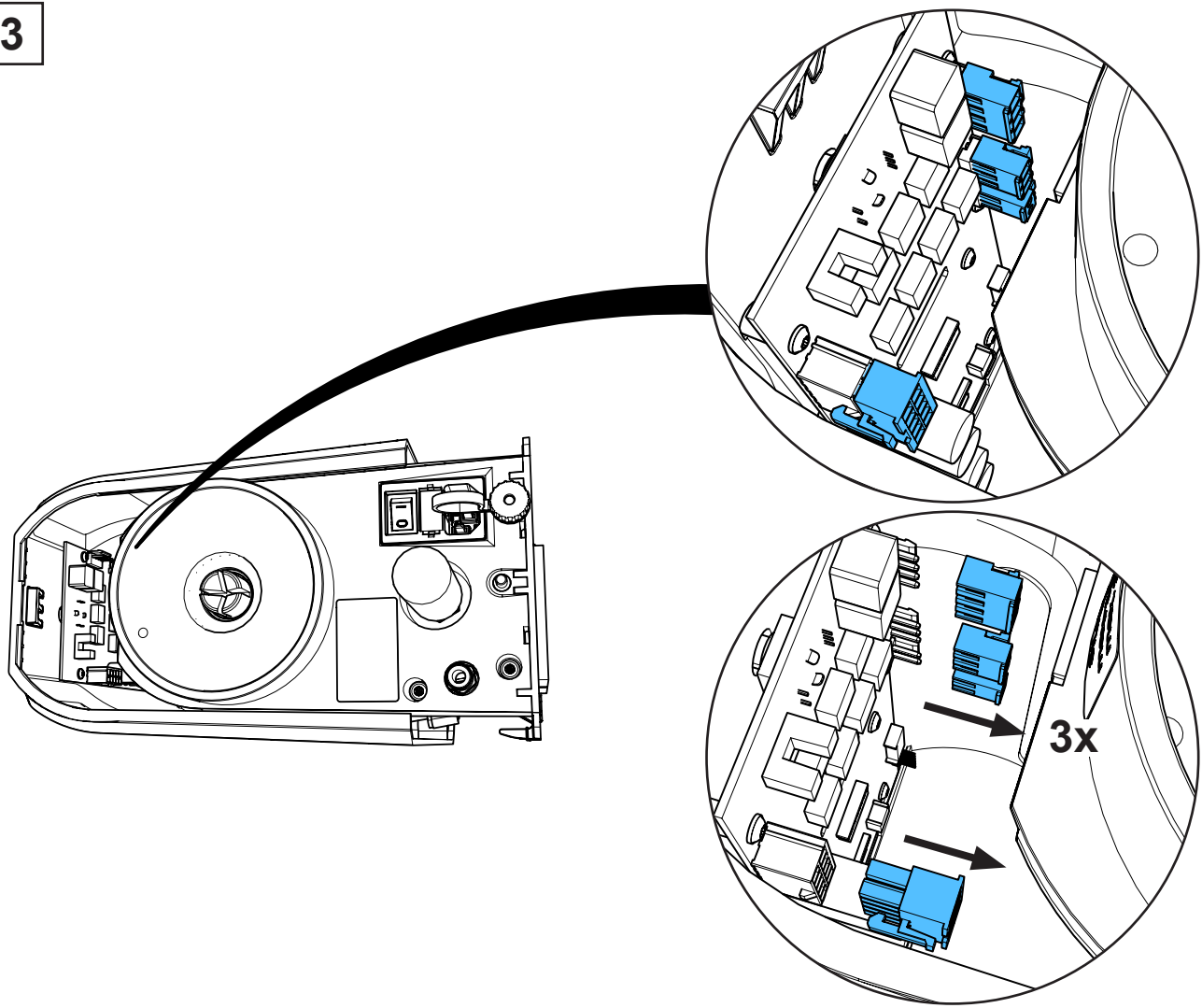


4

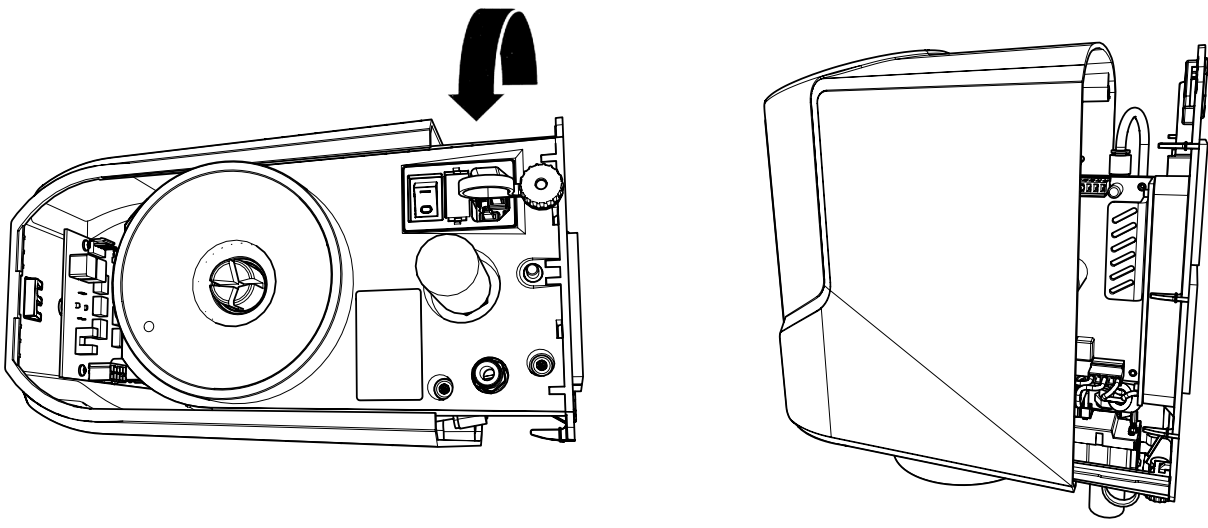


5

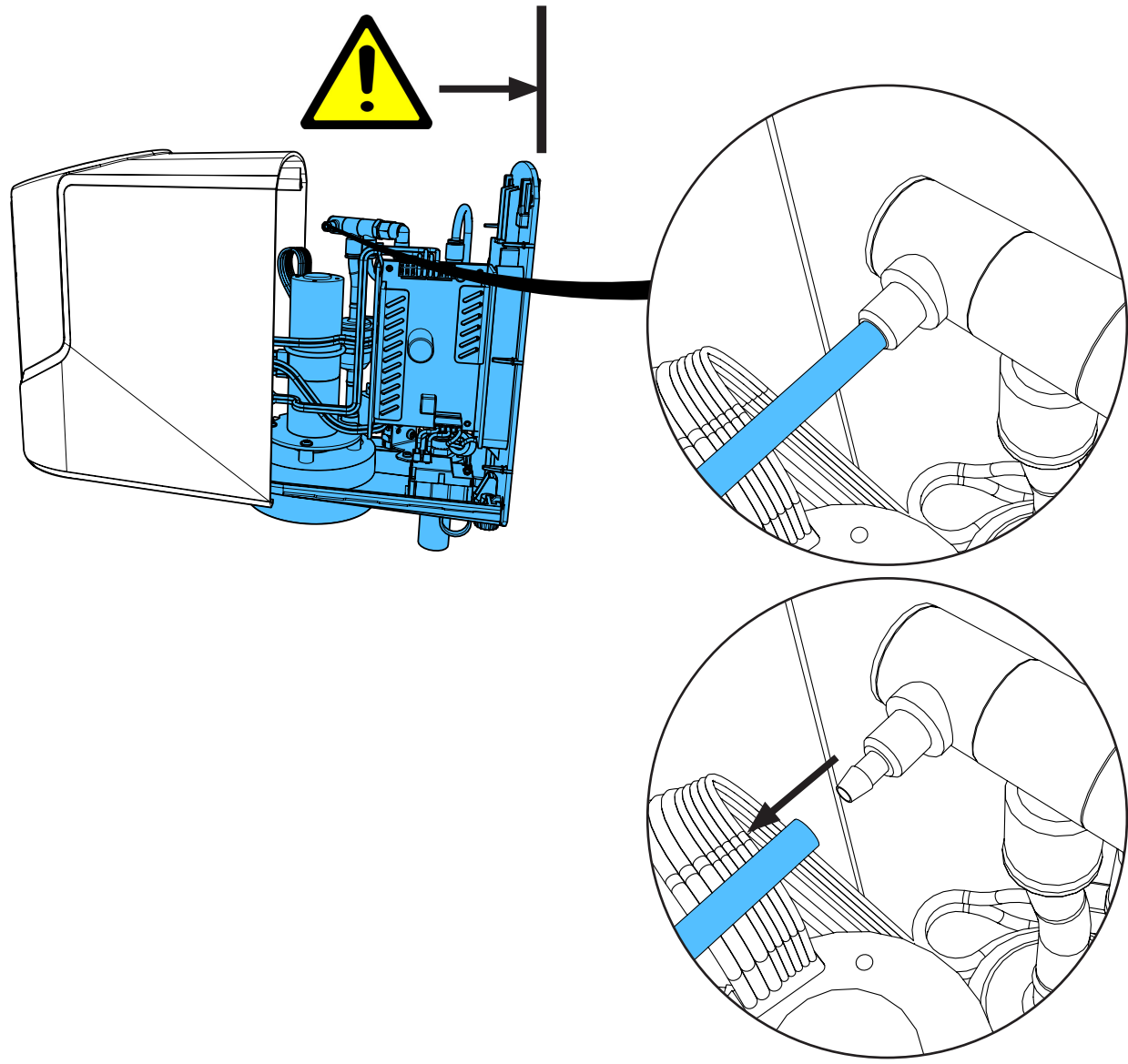


6

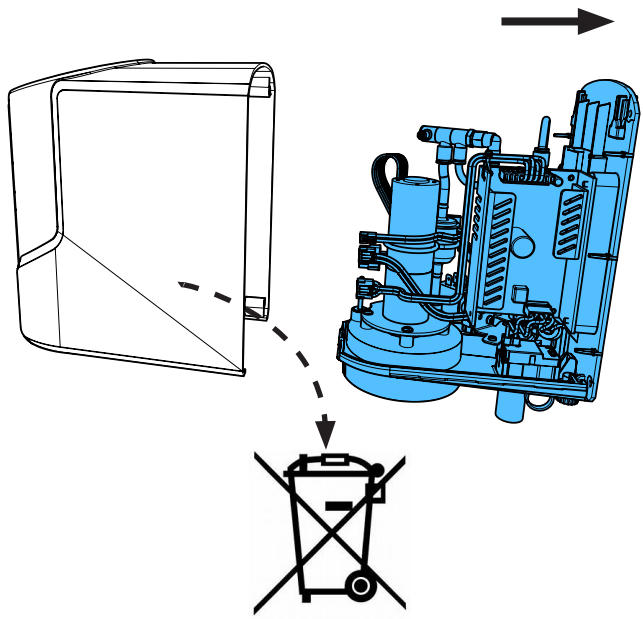


7

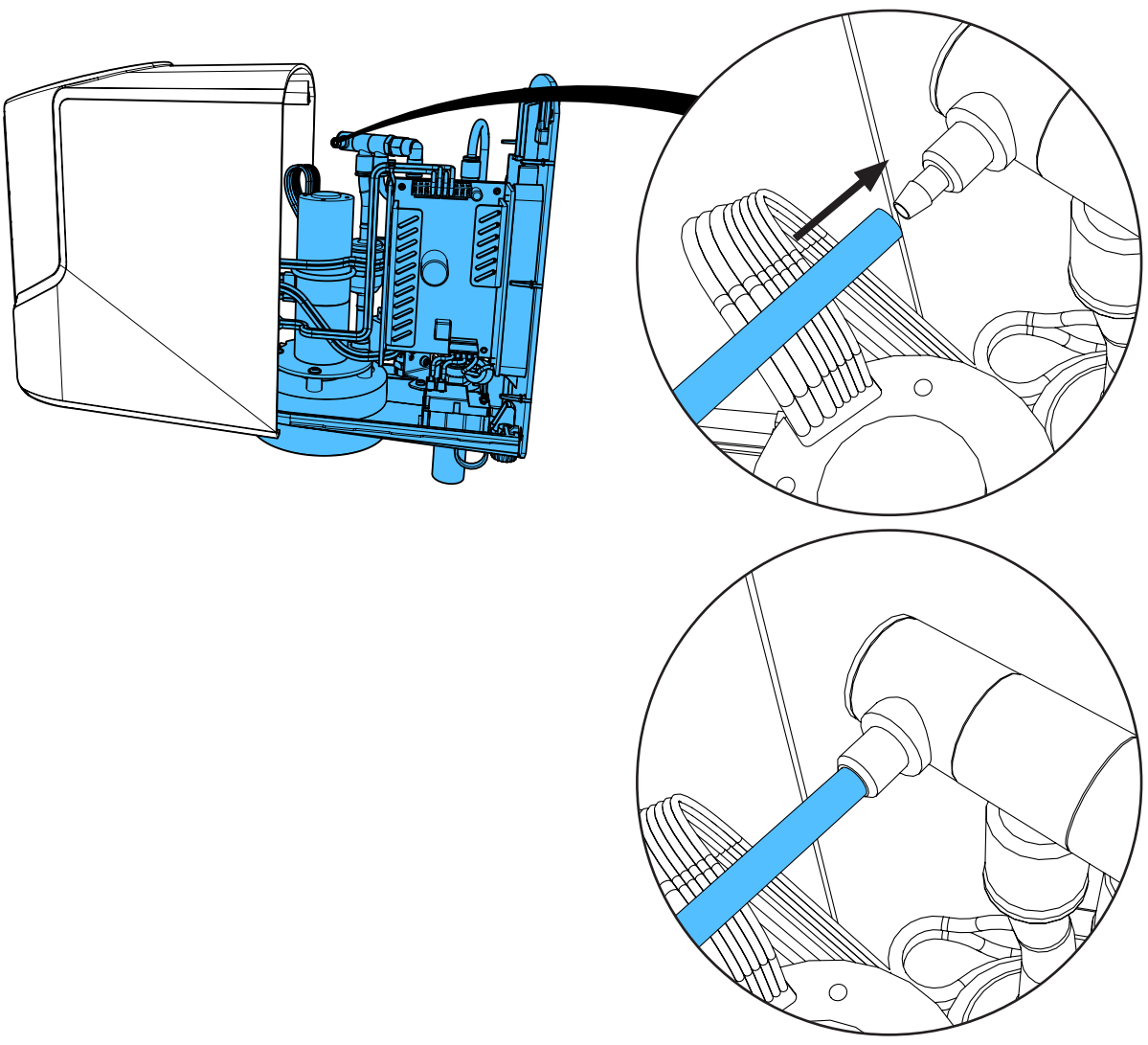


8

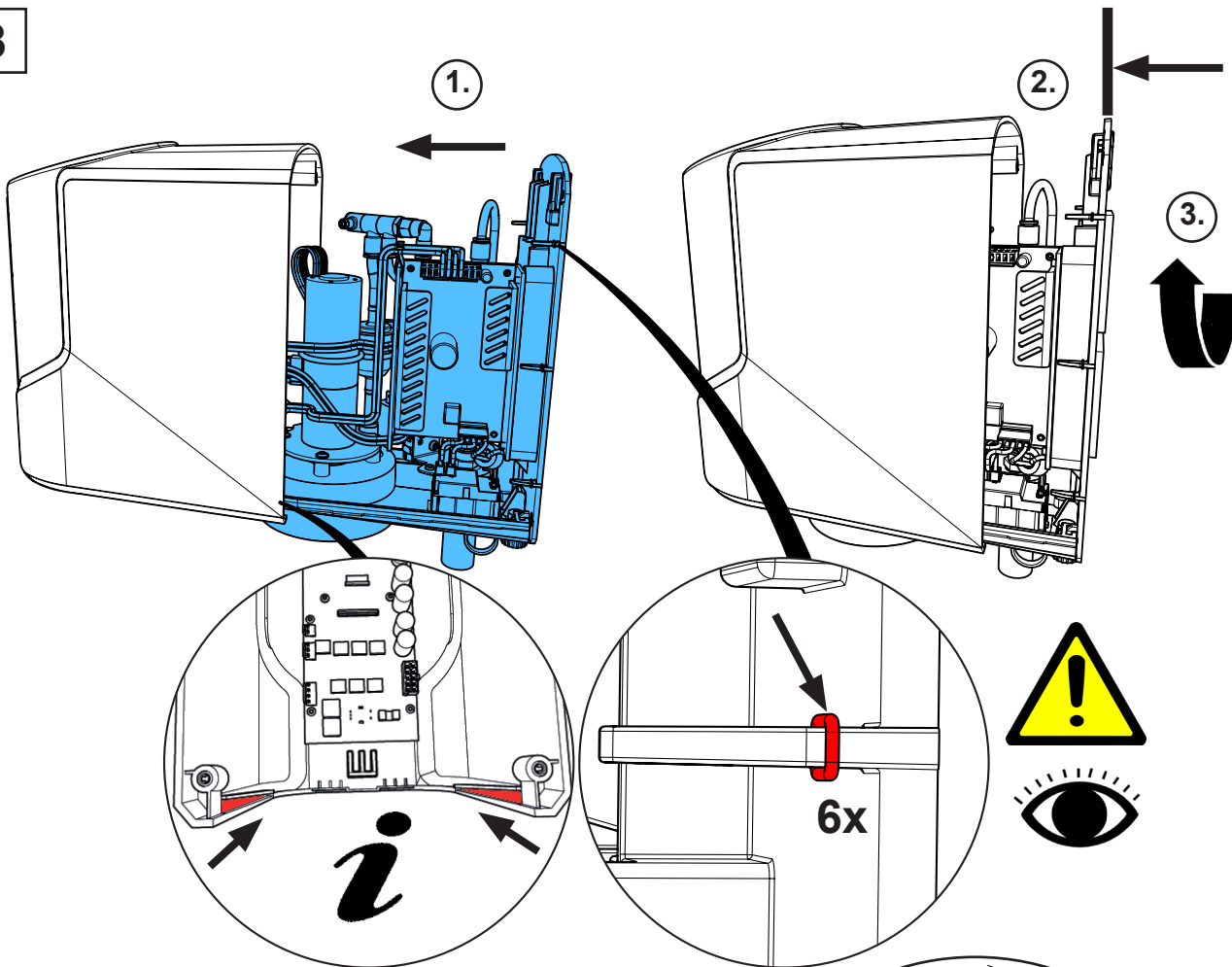


9

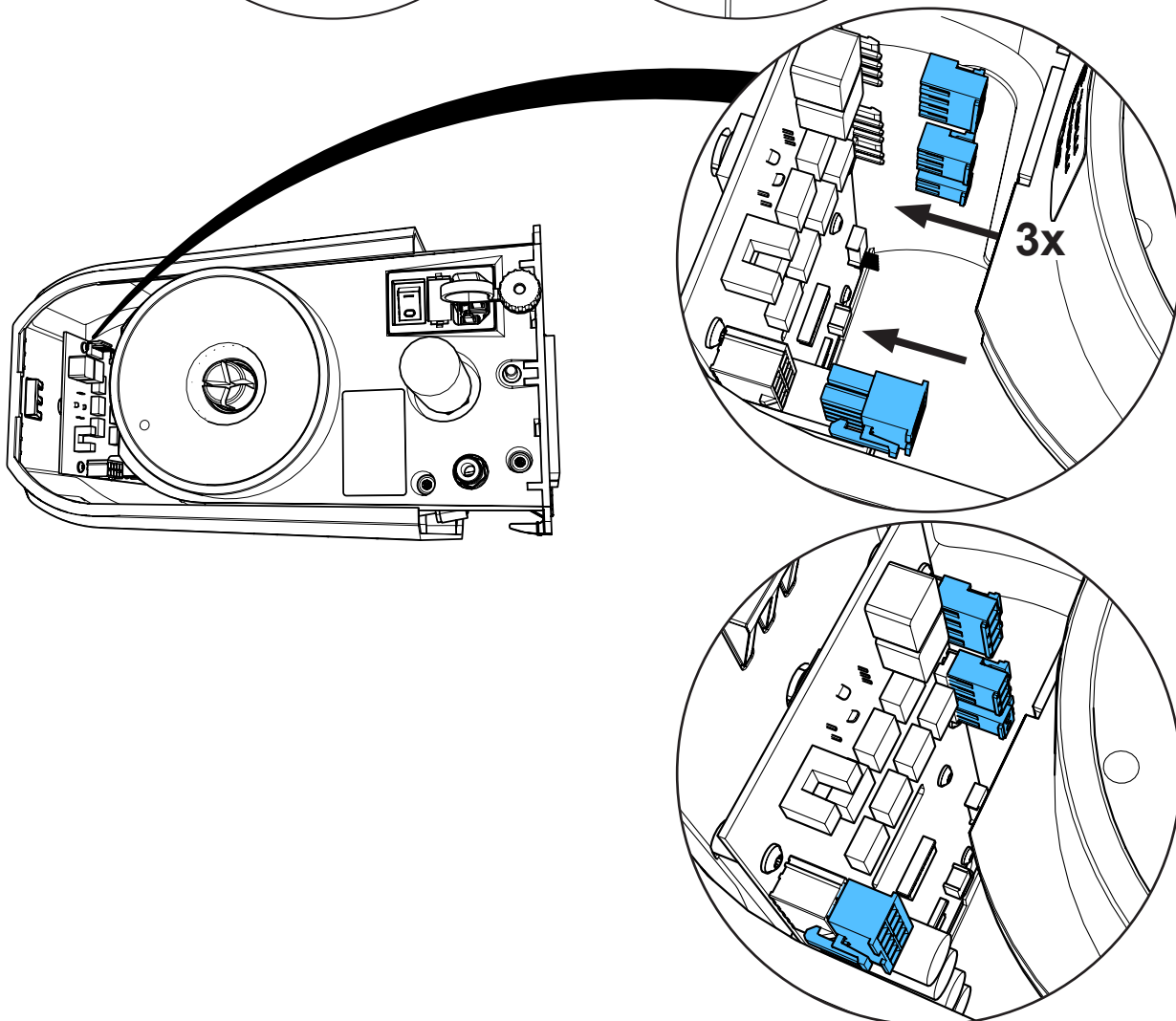


10

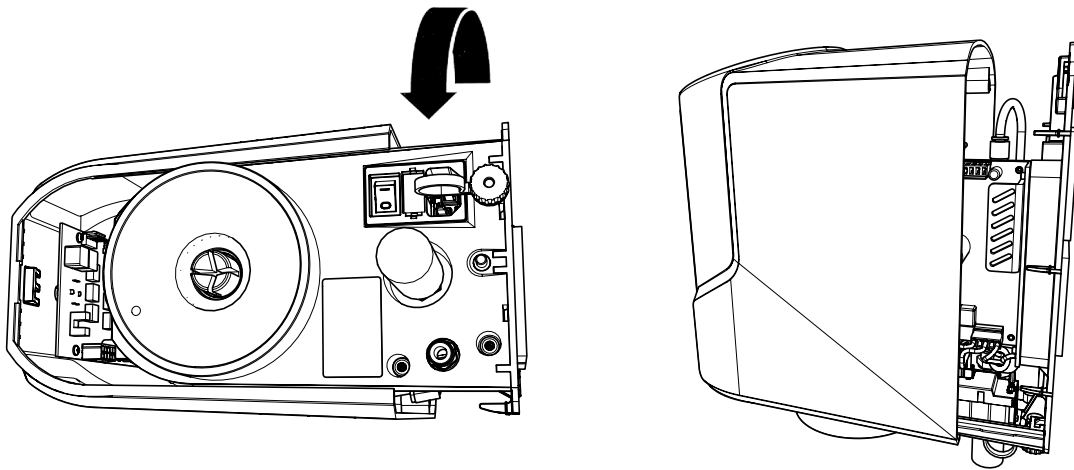

11



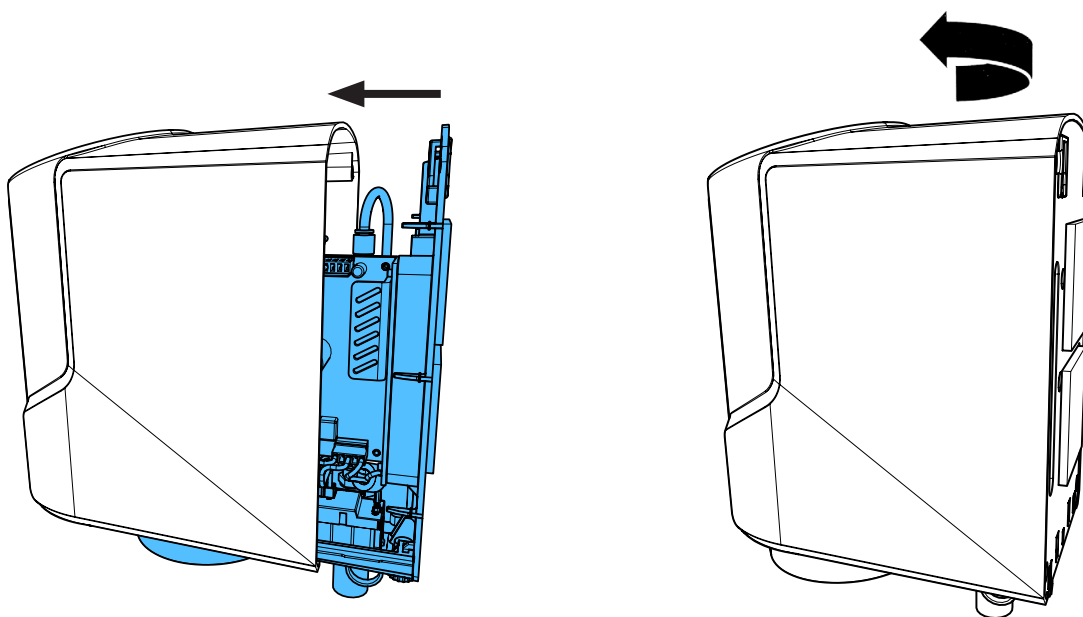


12

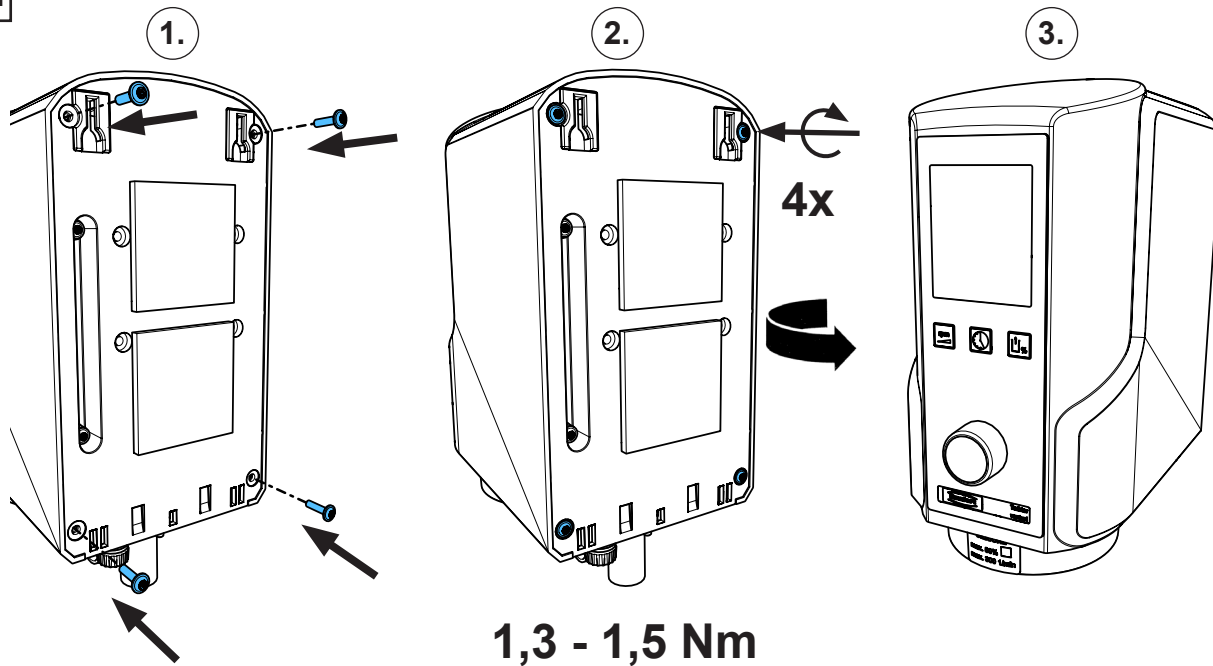


## 90033 4747


Leiterplatte kompl. 1829 • Printed circuit compl. 1829 • Carta imprimée compl. 1829  
Circuito stampato compl.1829 • Circuito impresso compl. 1829  
Печатная плата в комплекте 1829

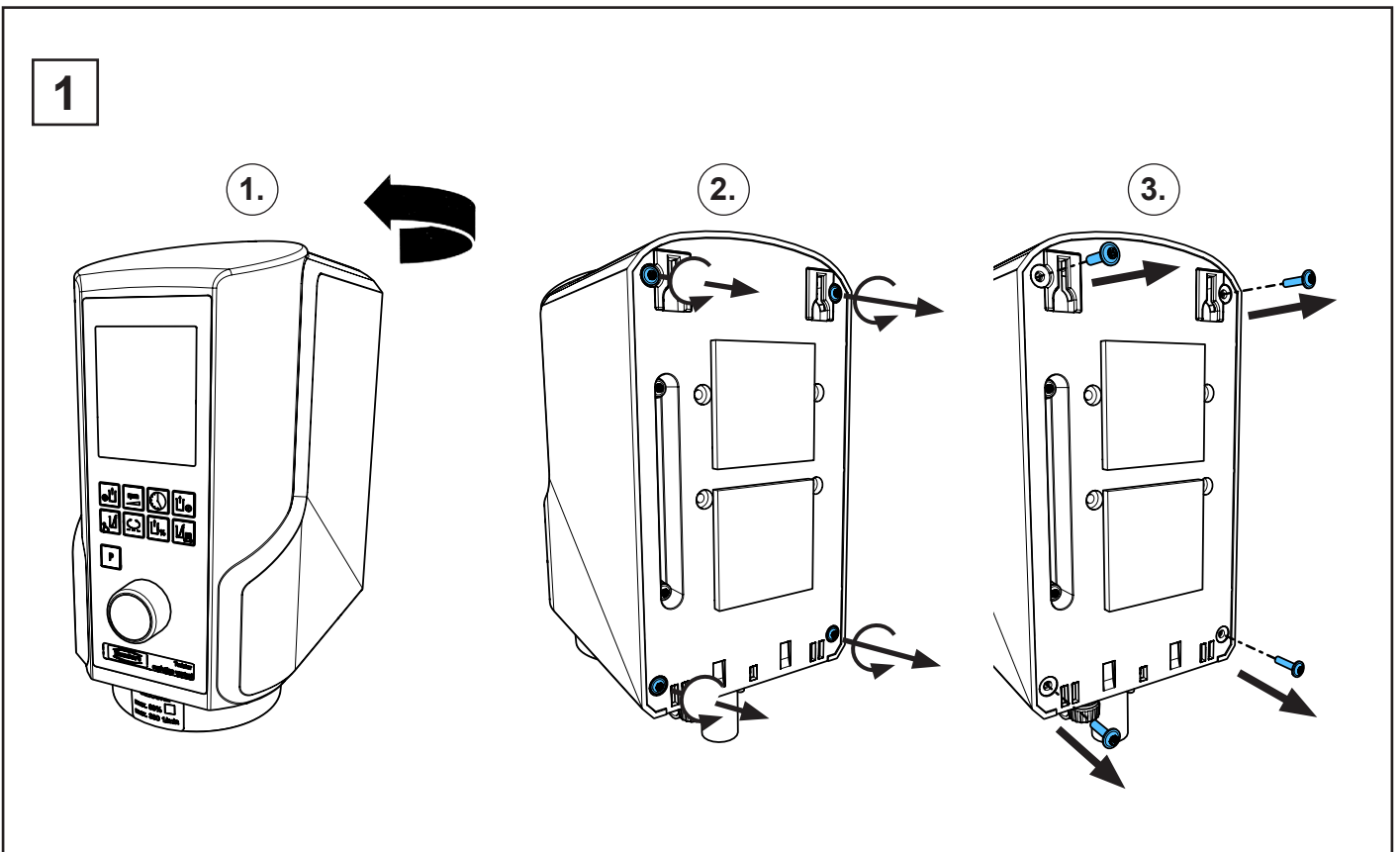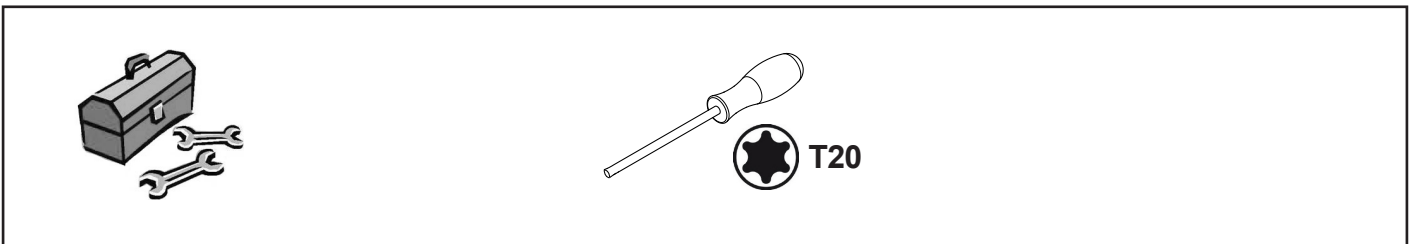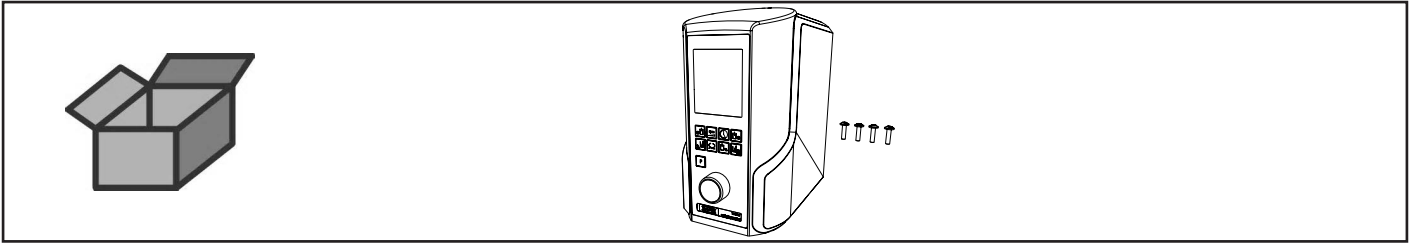

2

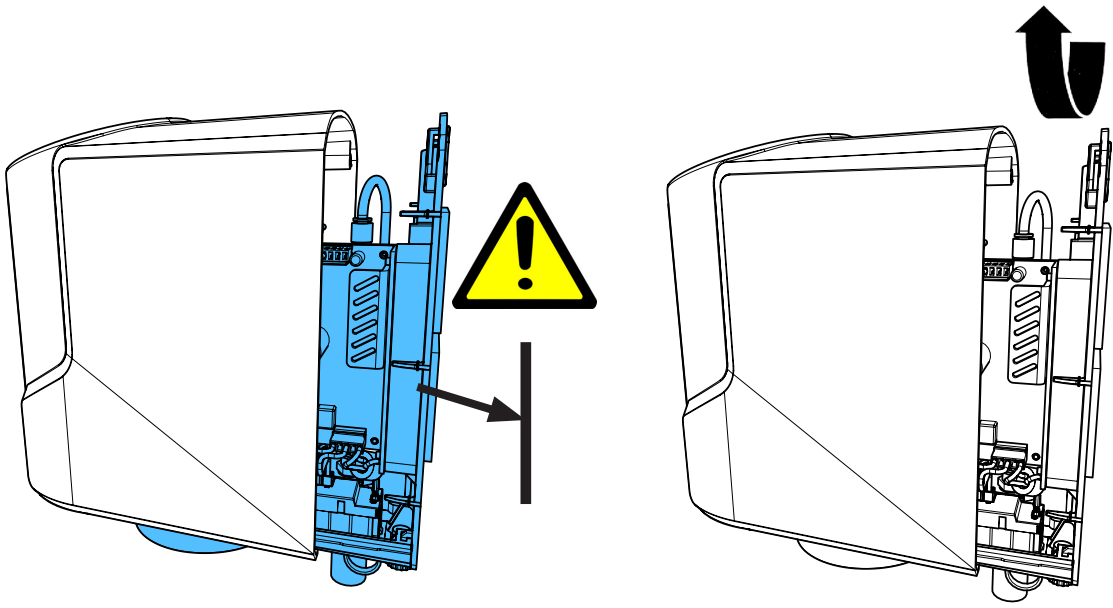


3

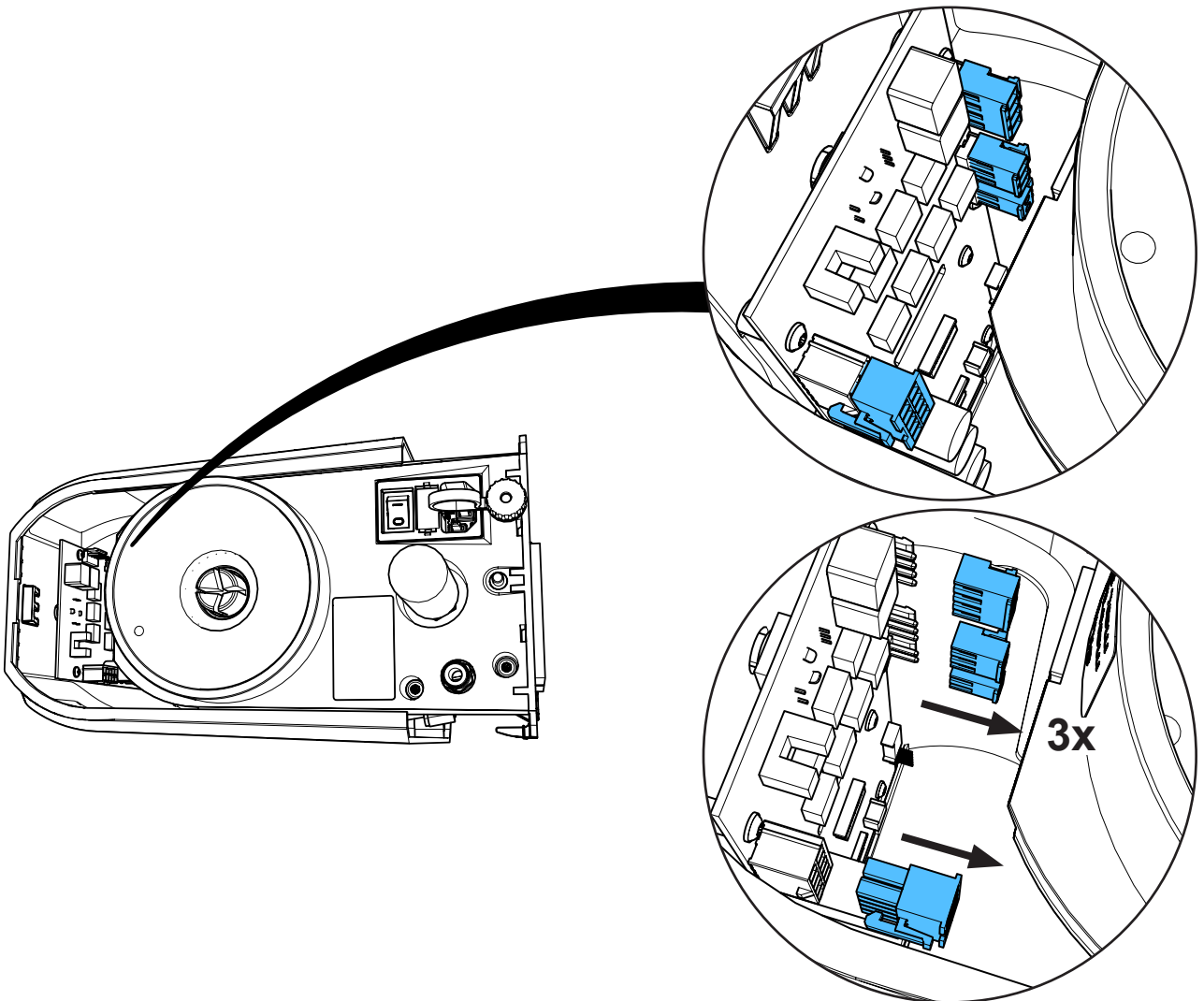





4

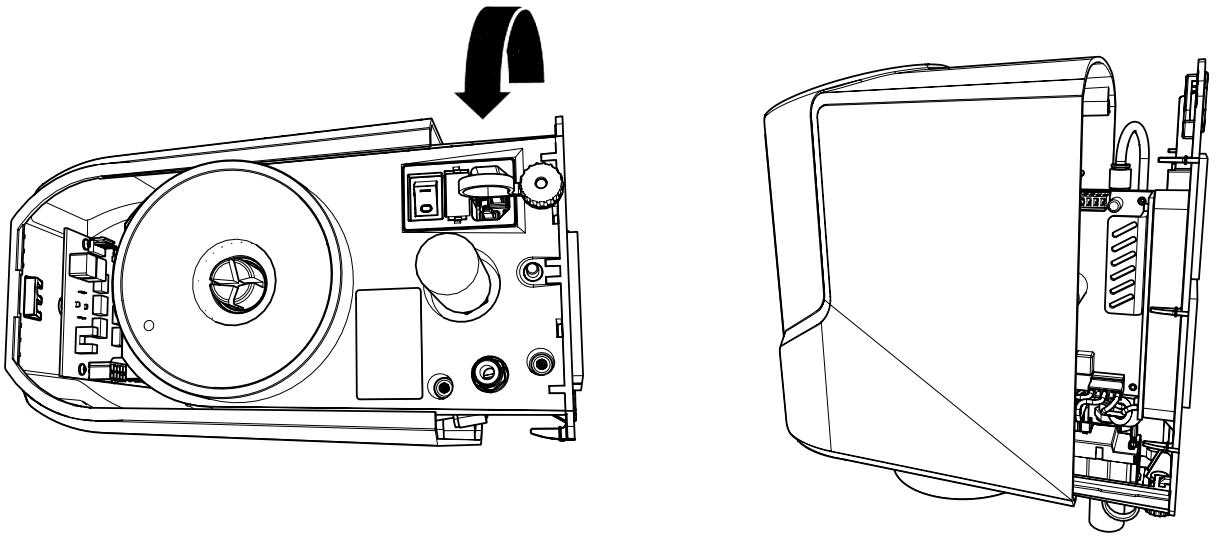


5

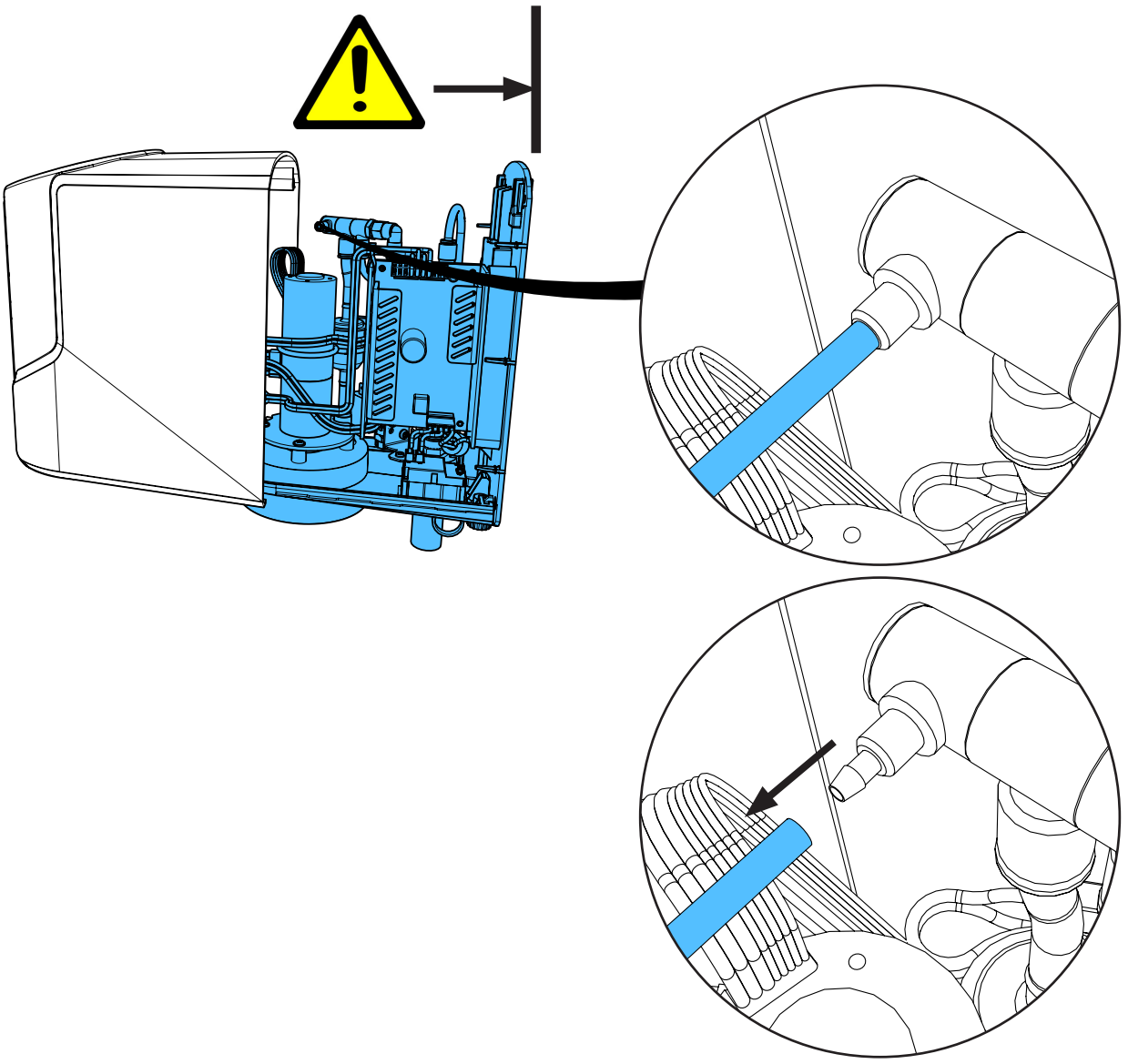


6

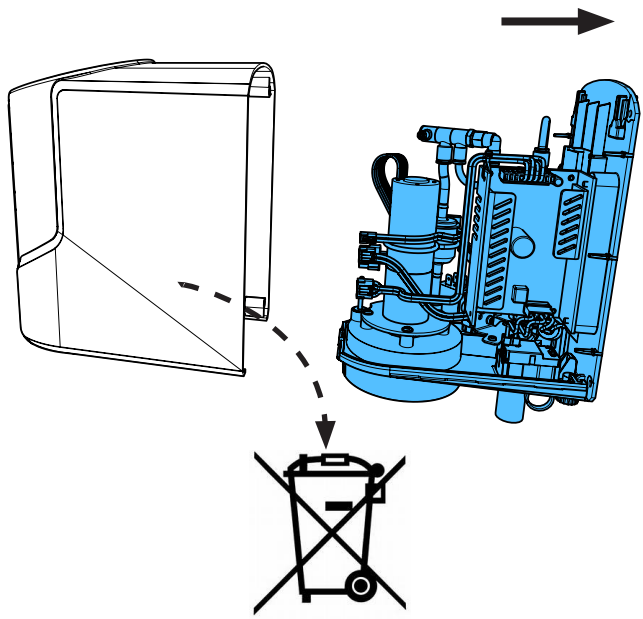


7

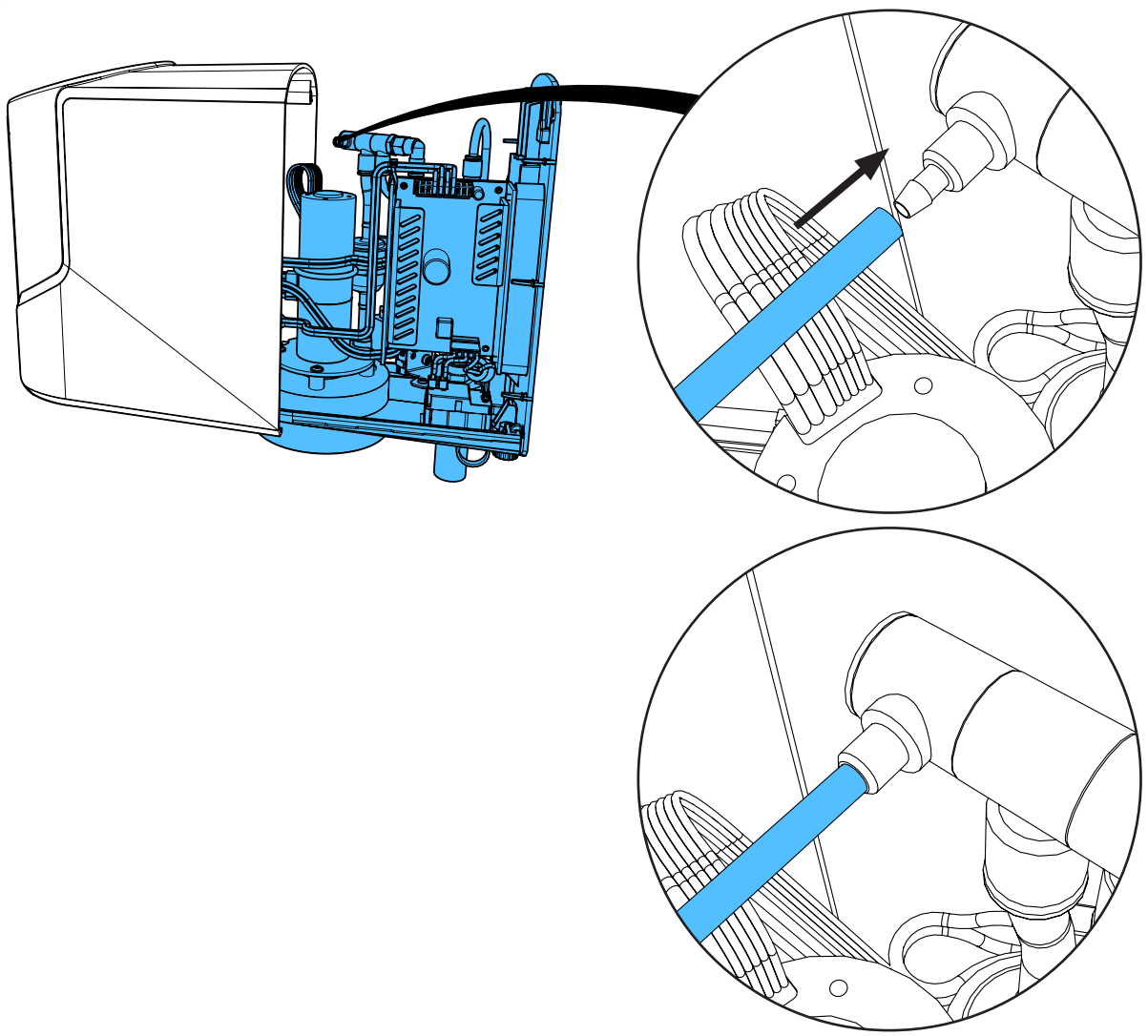


8

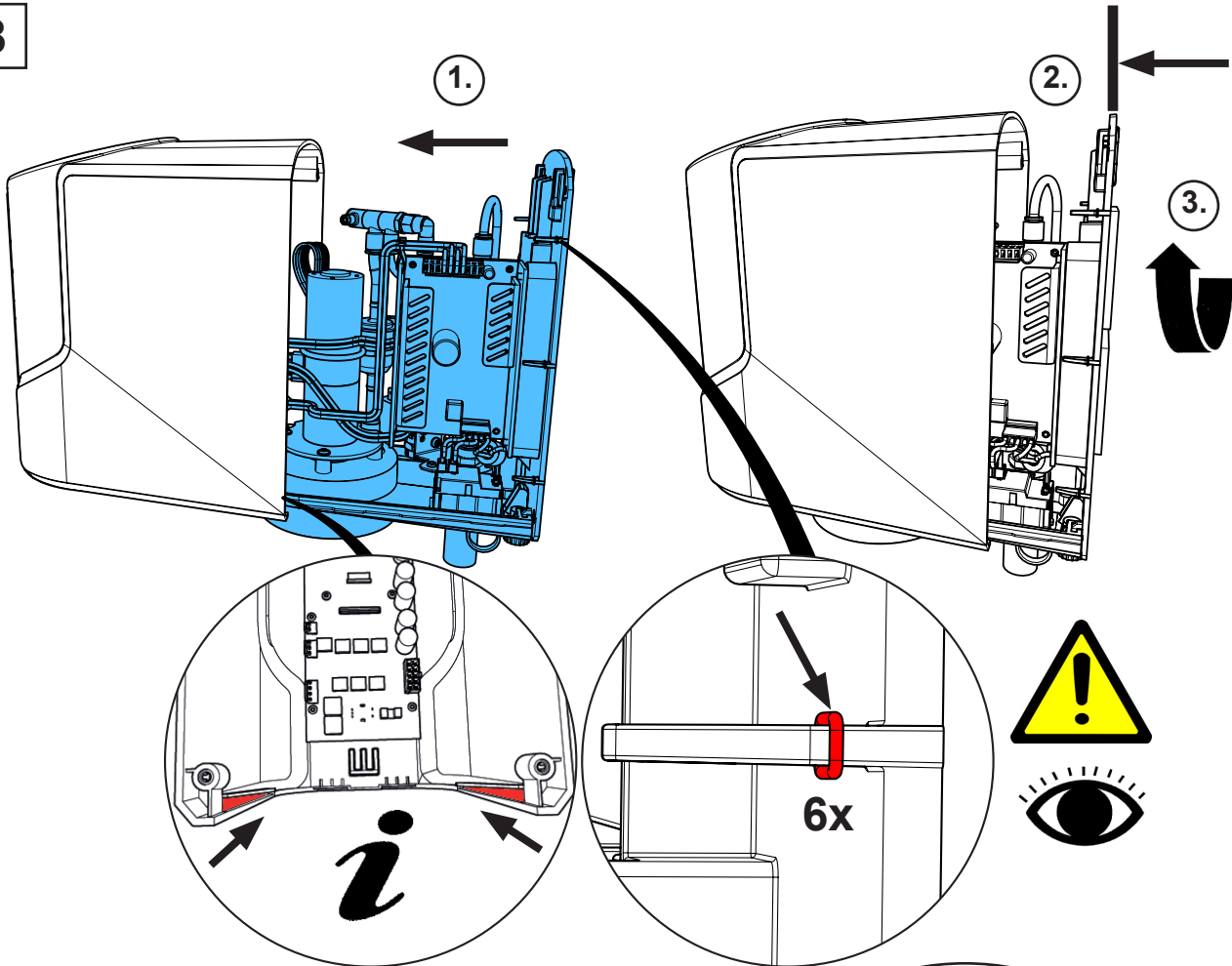


9

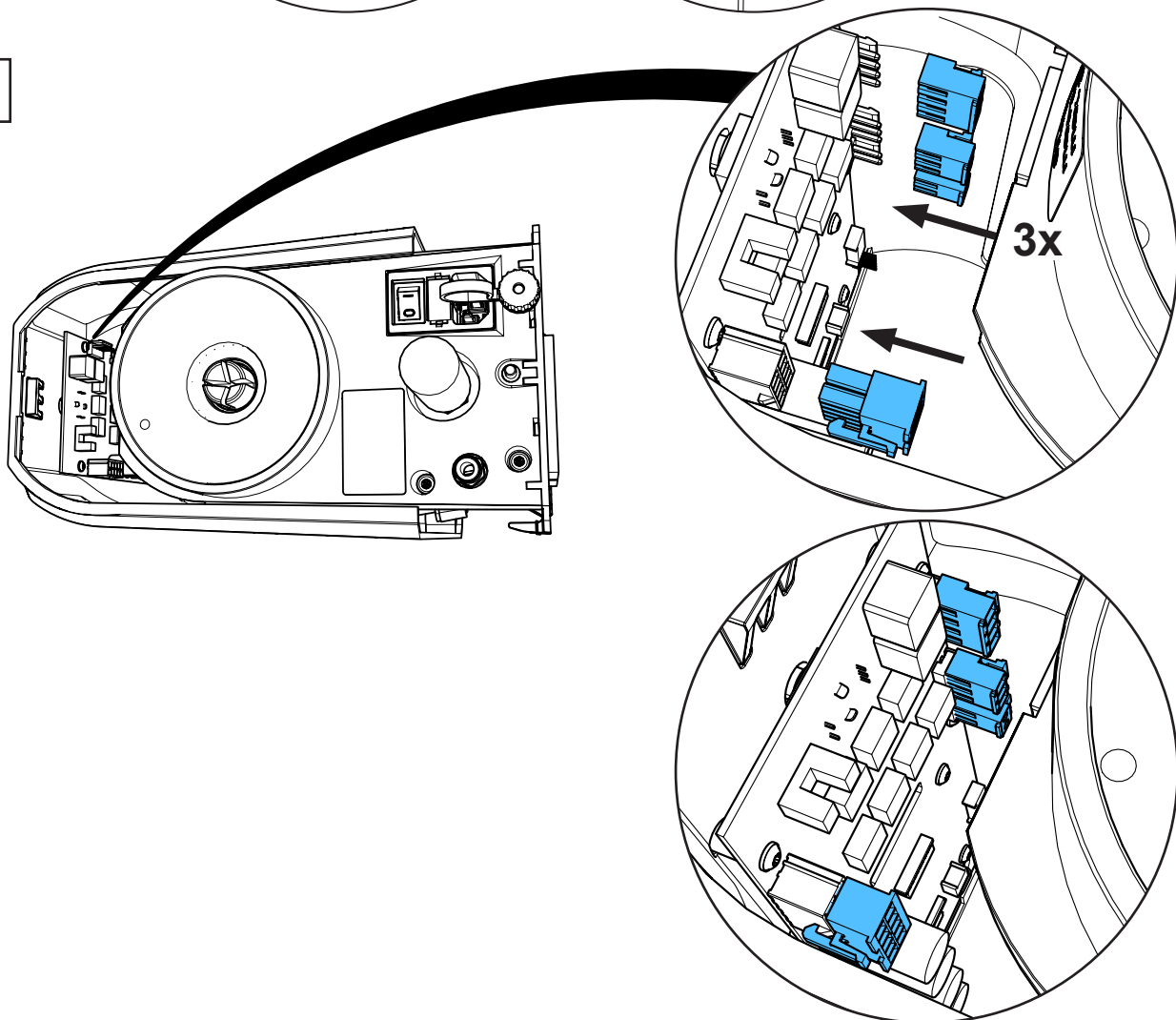


10

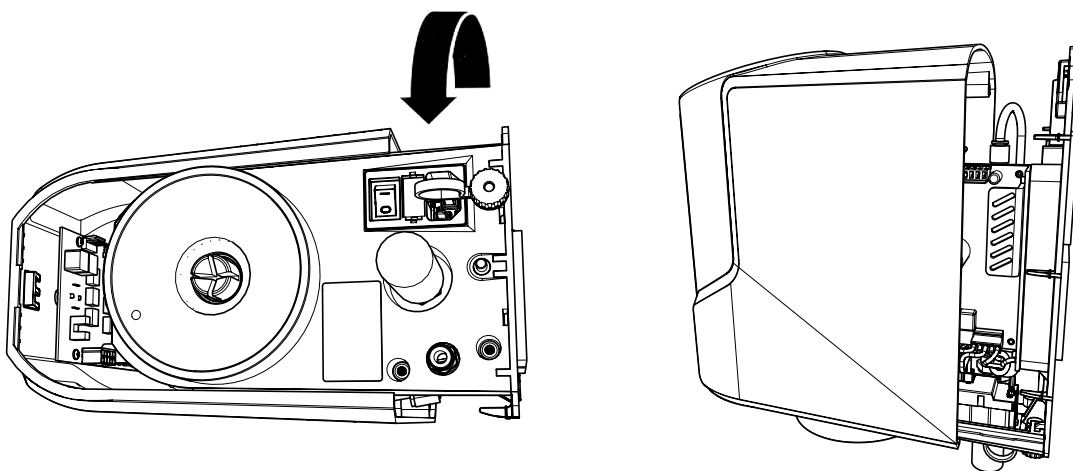

11



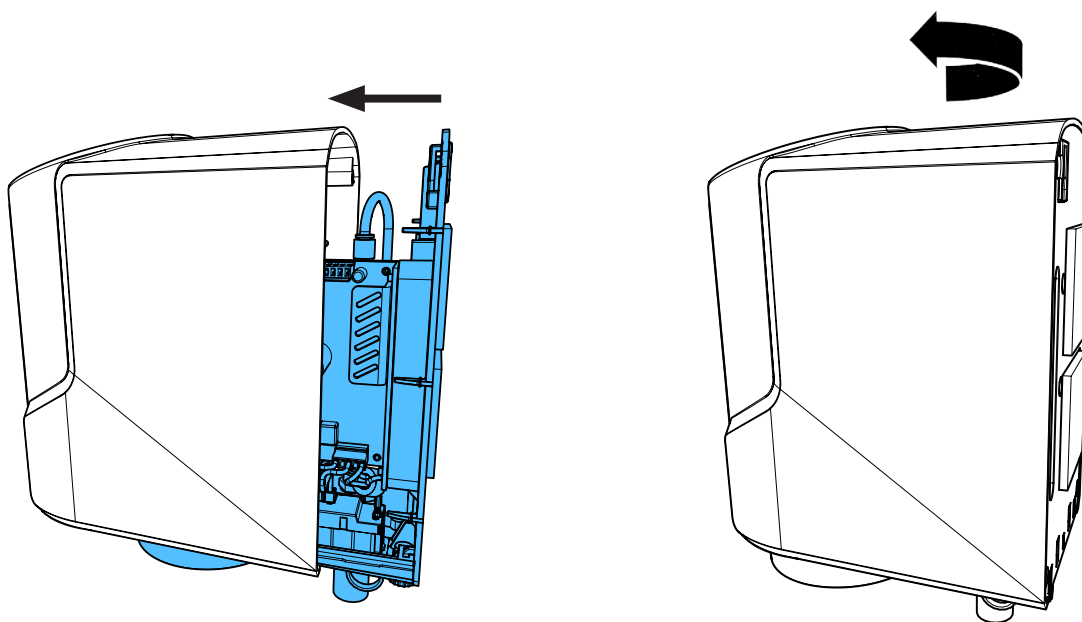


12

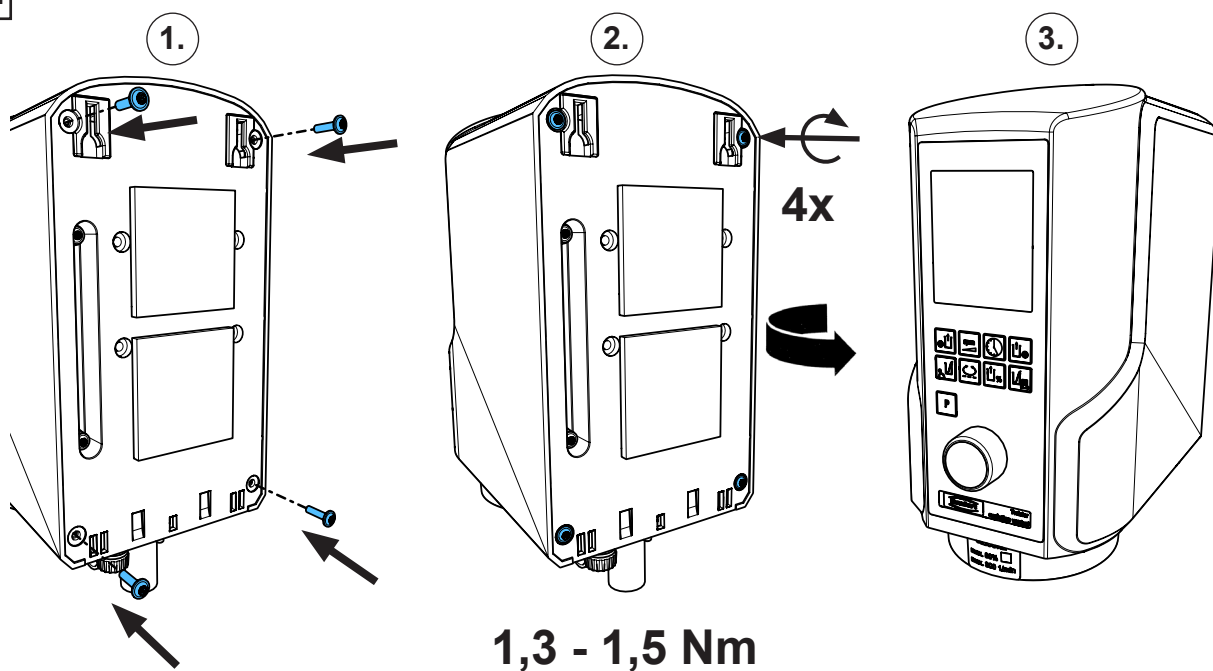


## 90003 4745


Getriebemotor • Geared motor • Moto-réducteur  
Motoriduttore • Motor reductor • Редукторный двигатель

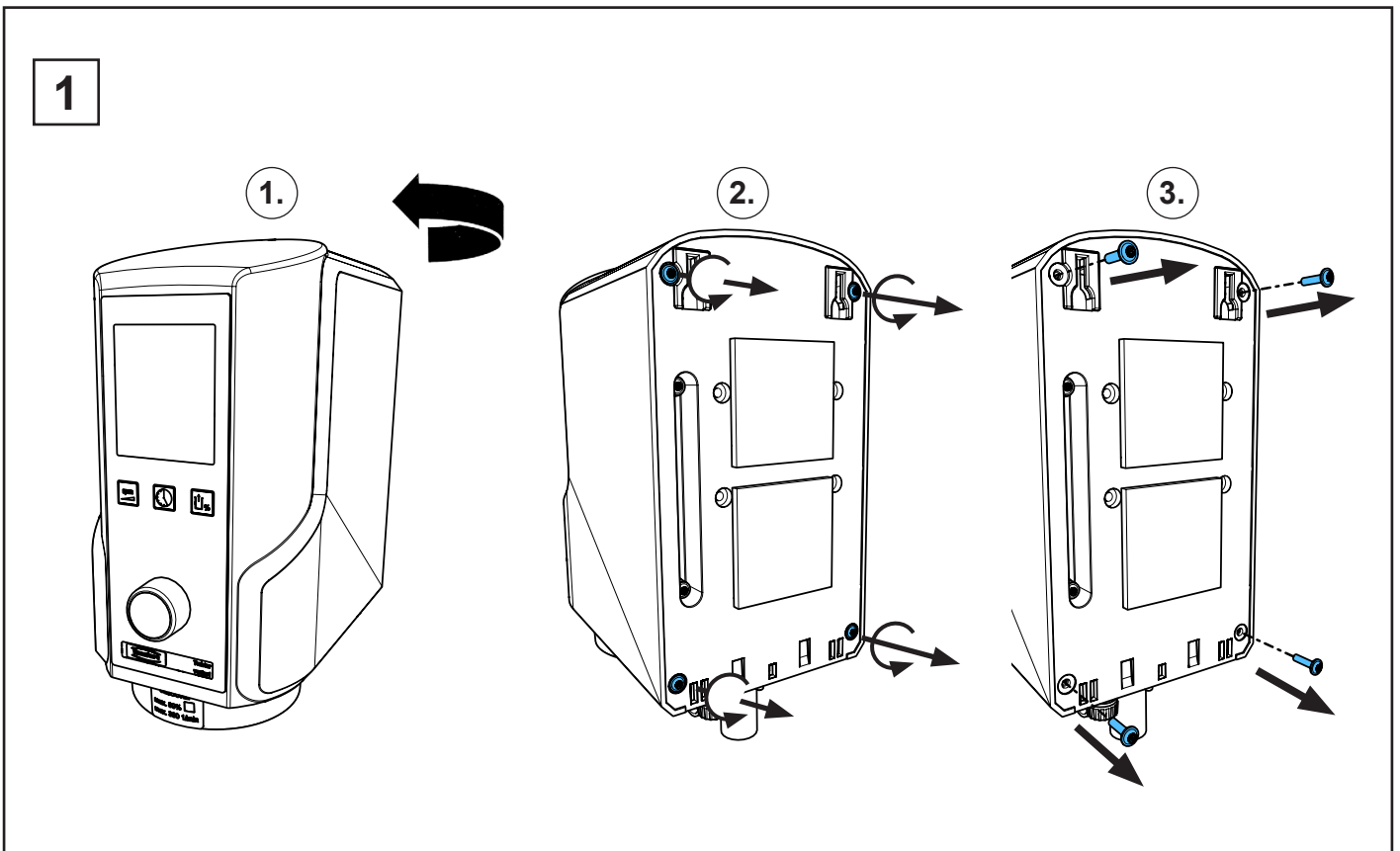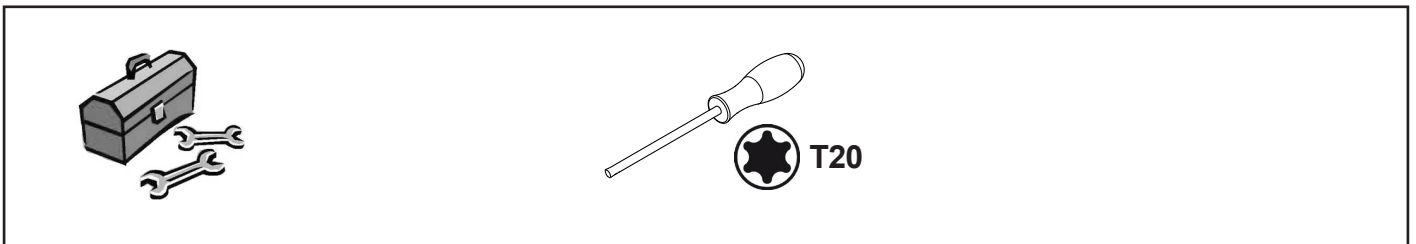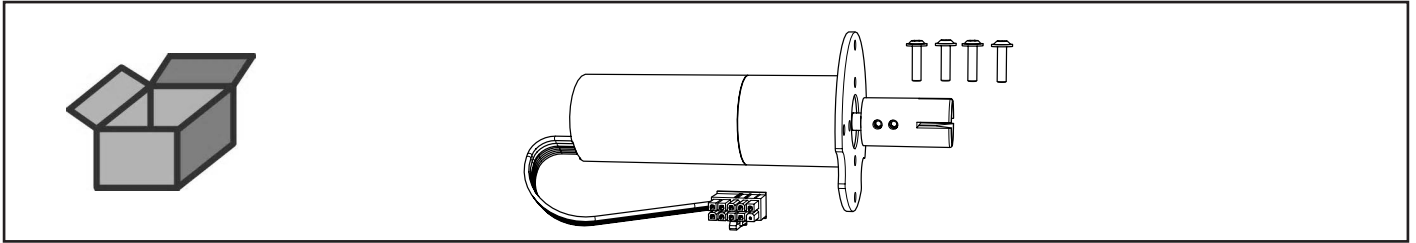


2

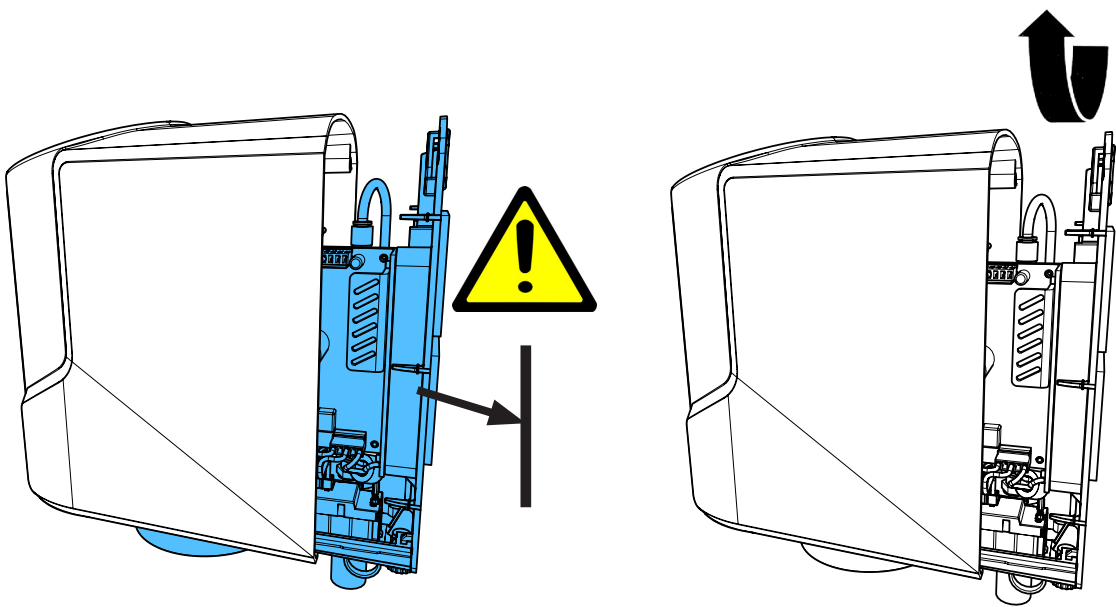


3

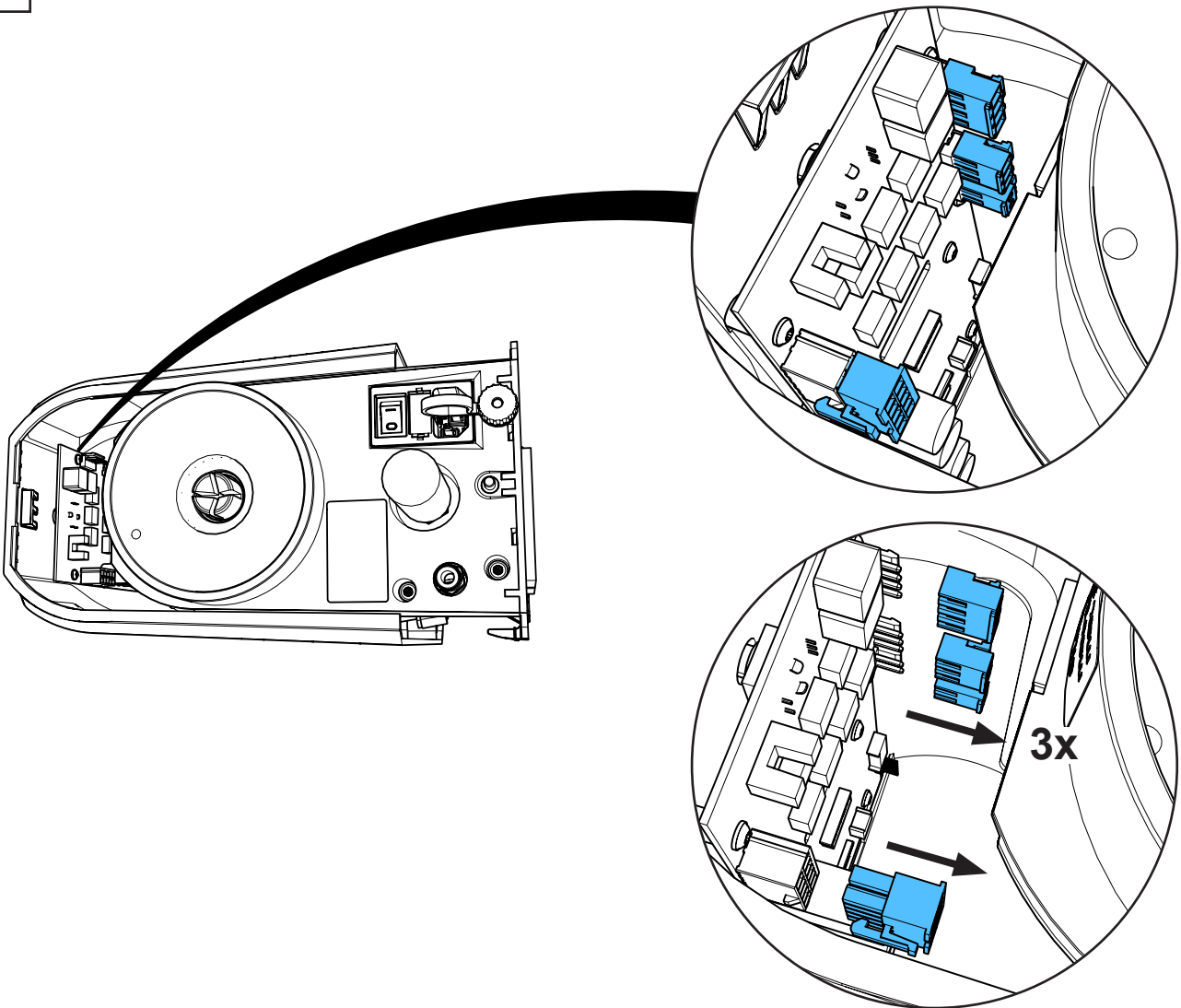


4

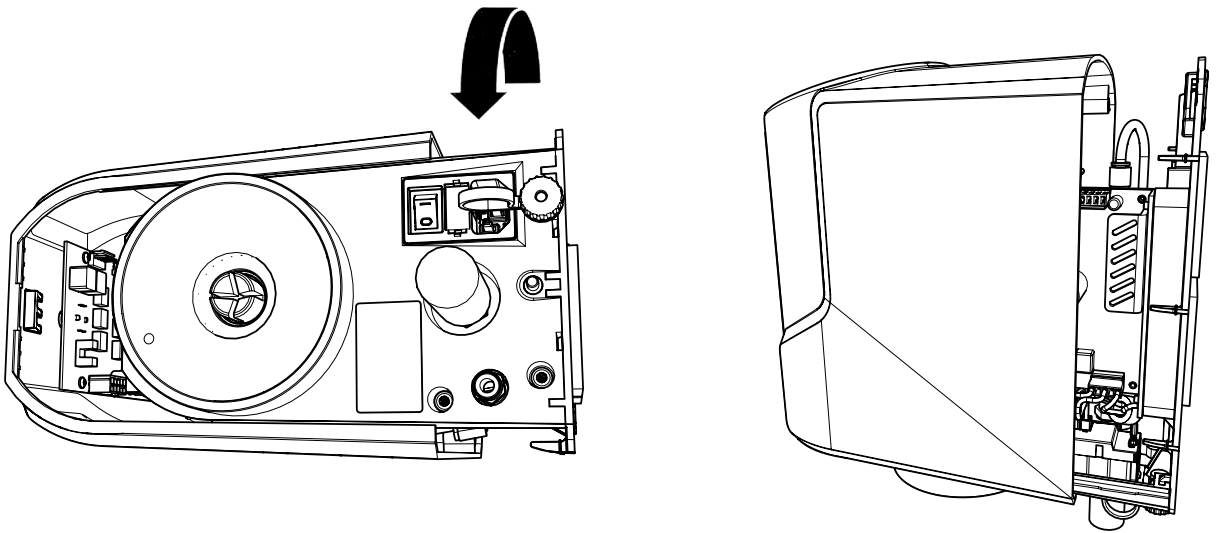


5

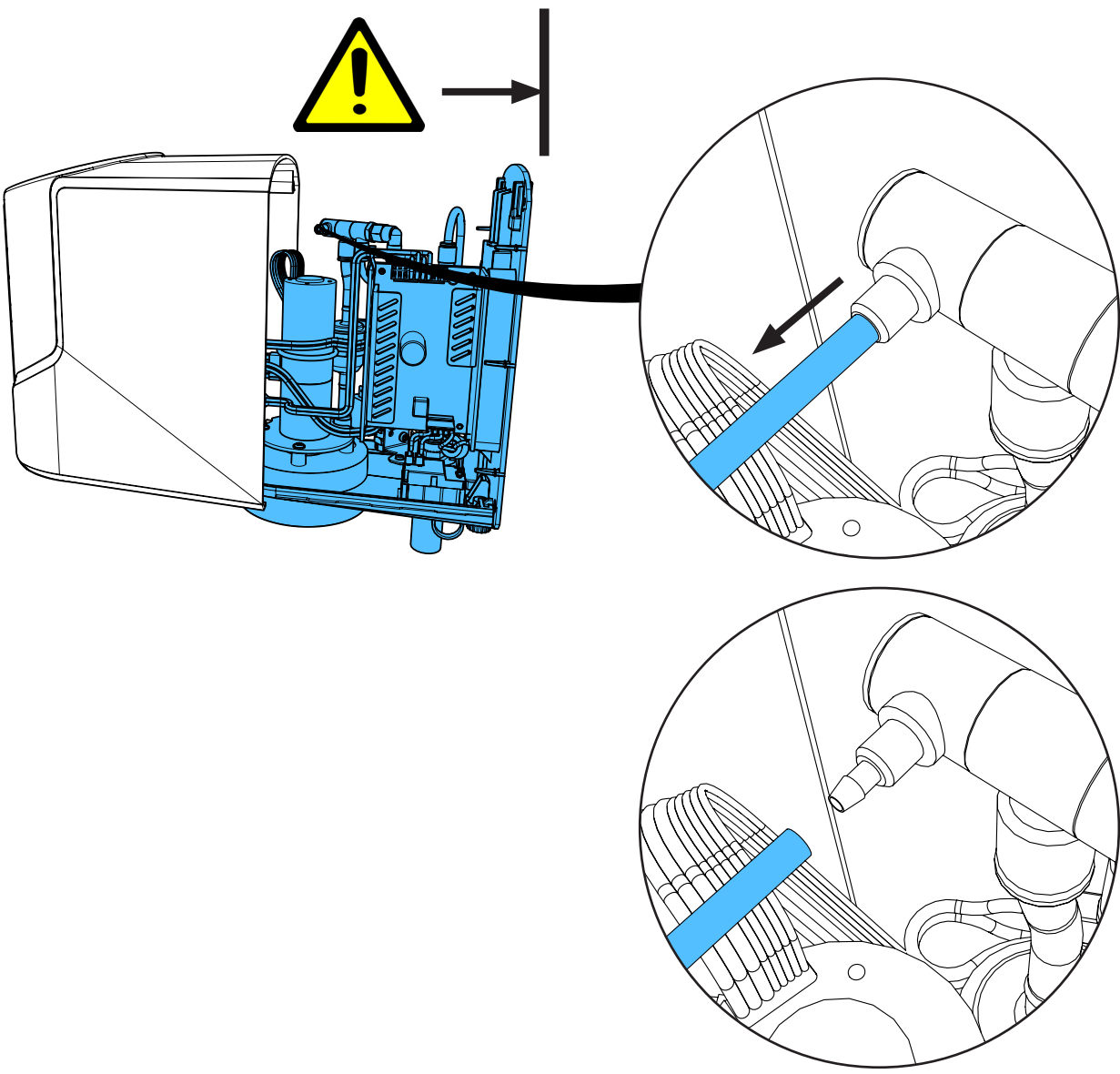


6

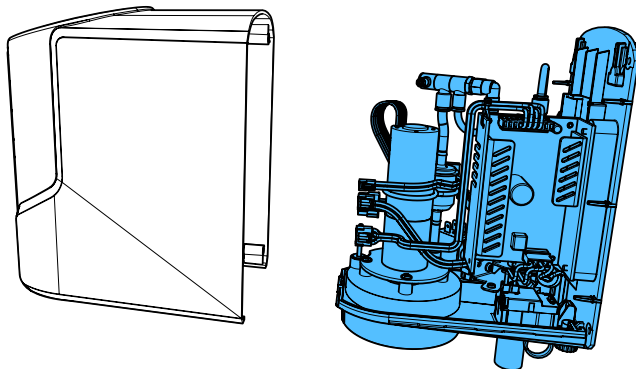

7

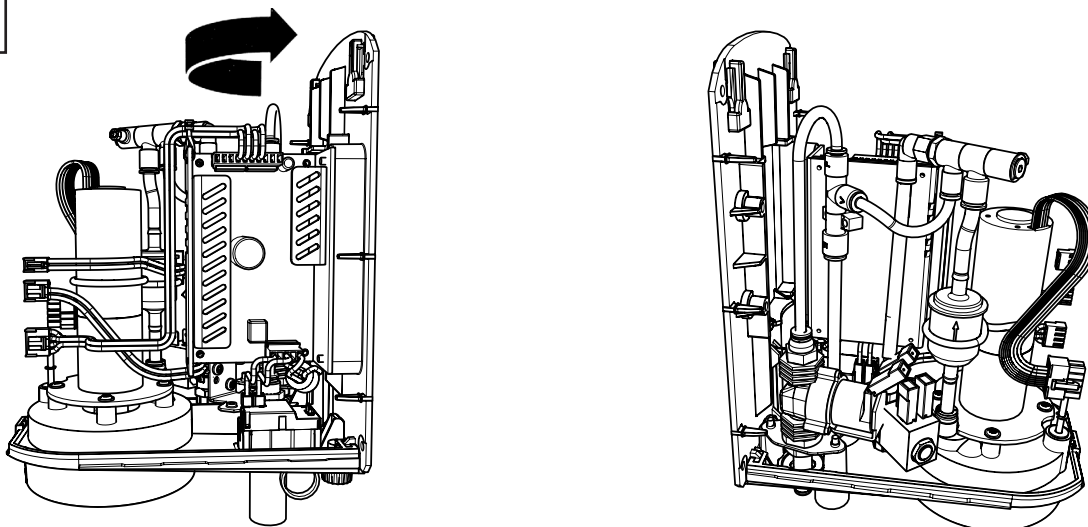


8

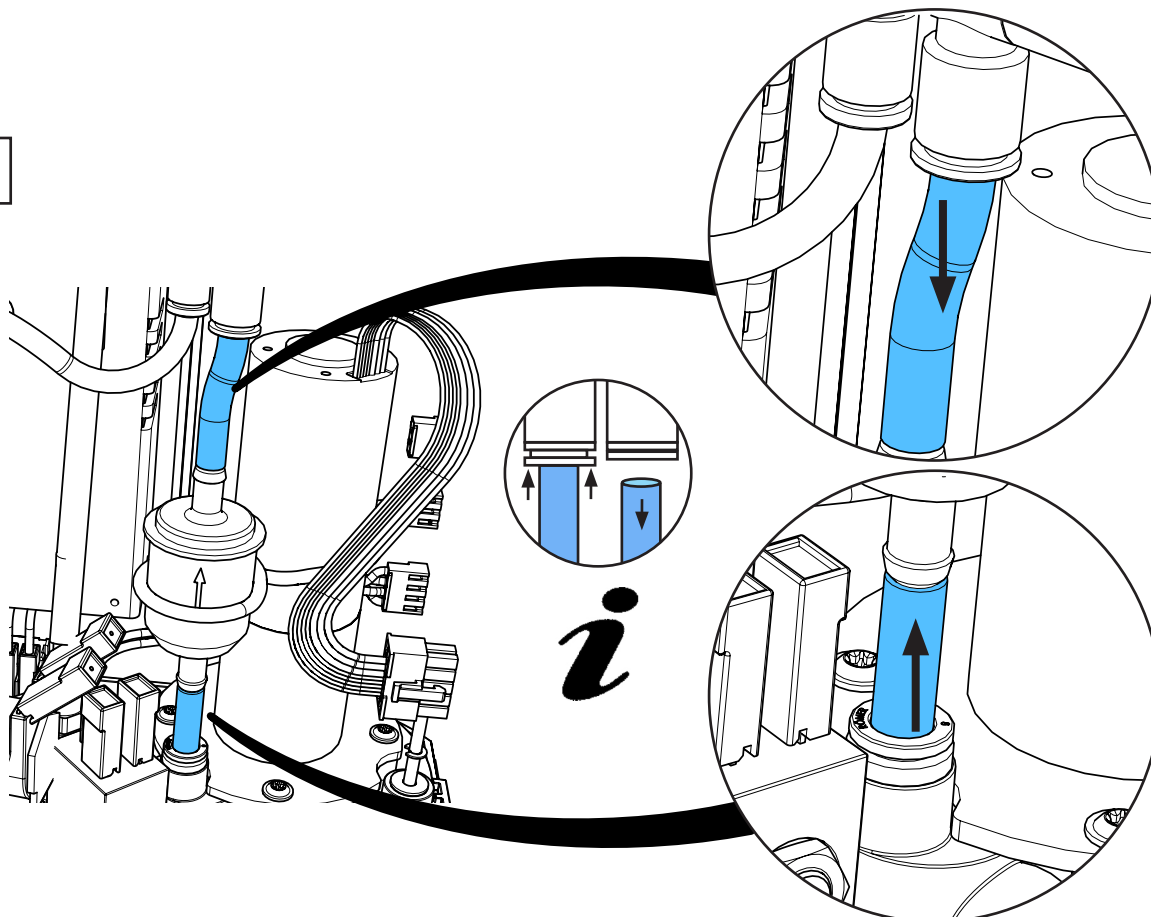





9

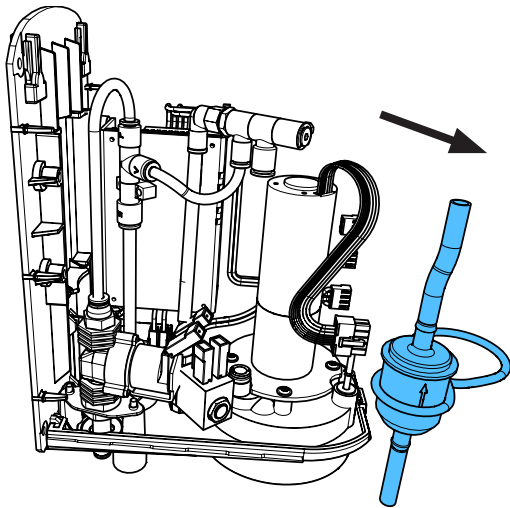




10

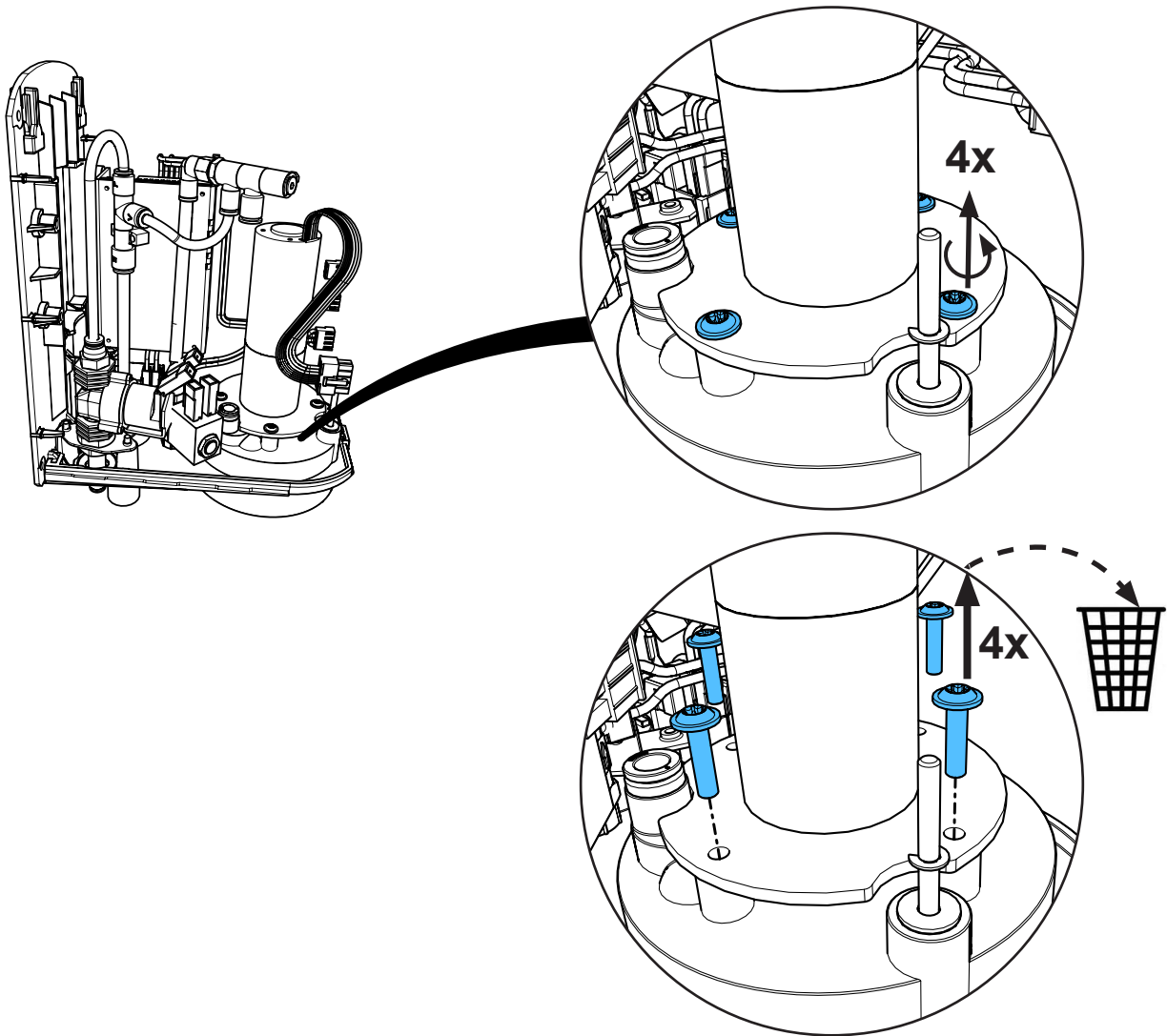

11




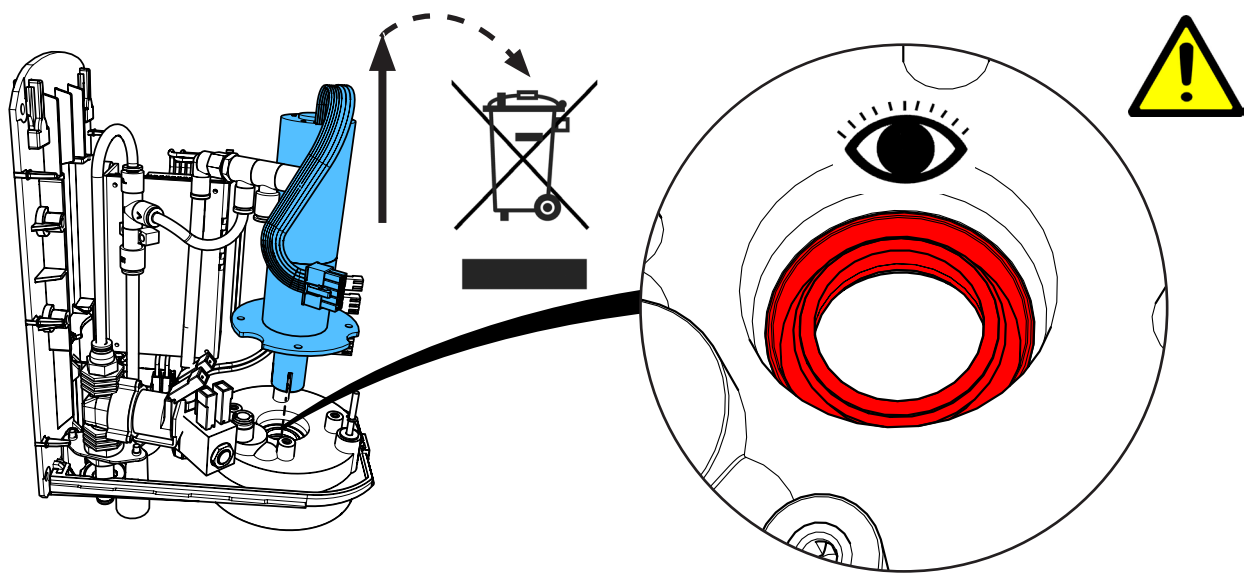


12


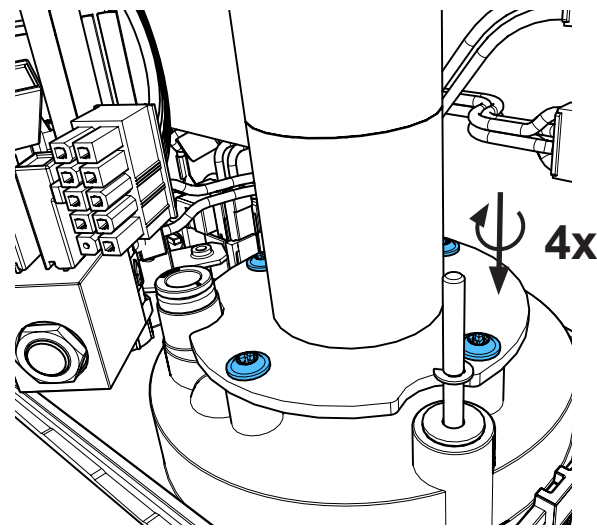
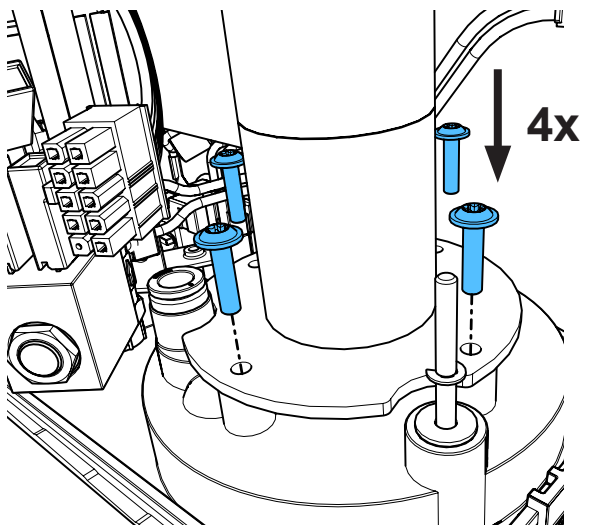
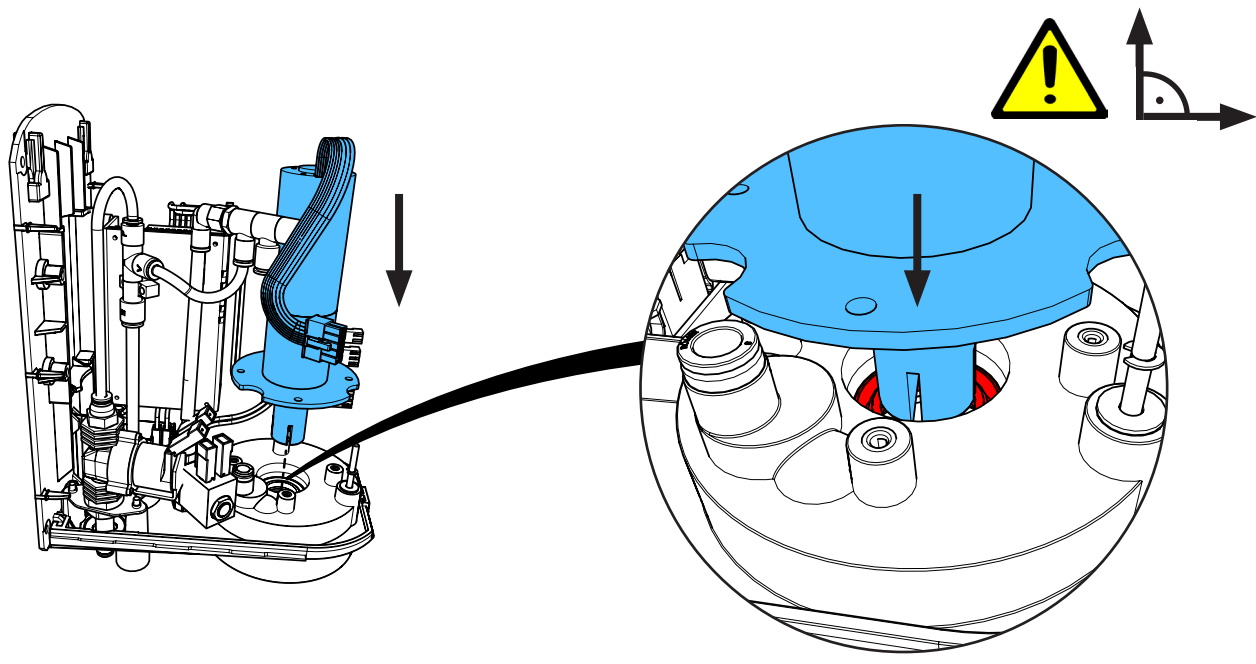


2,4 - 2,6 Nm

13

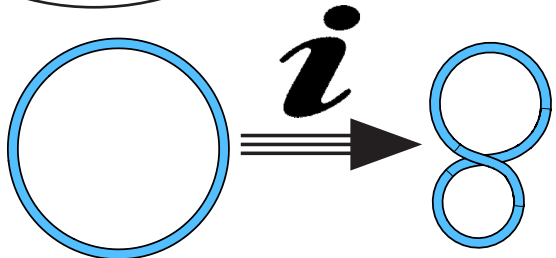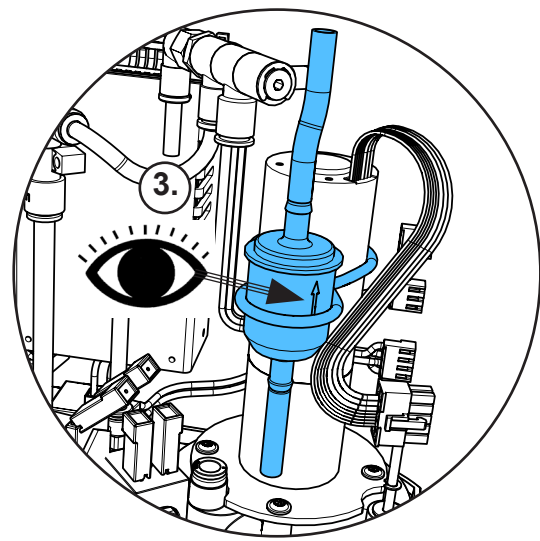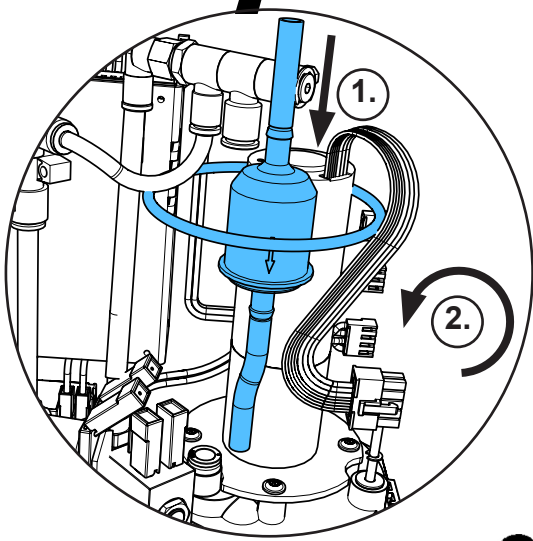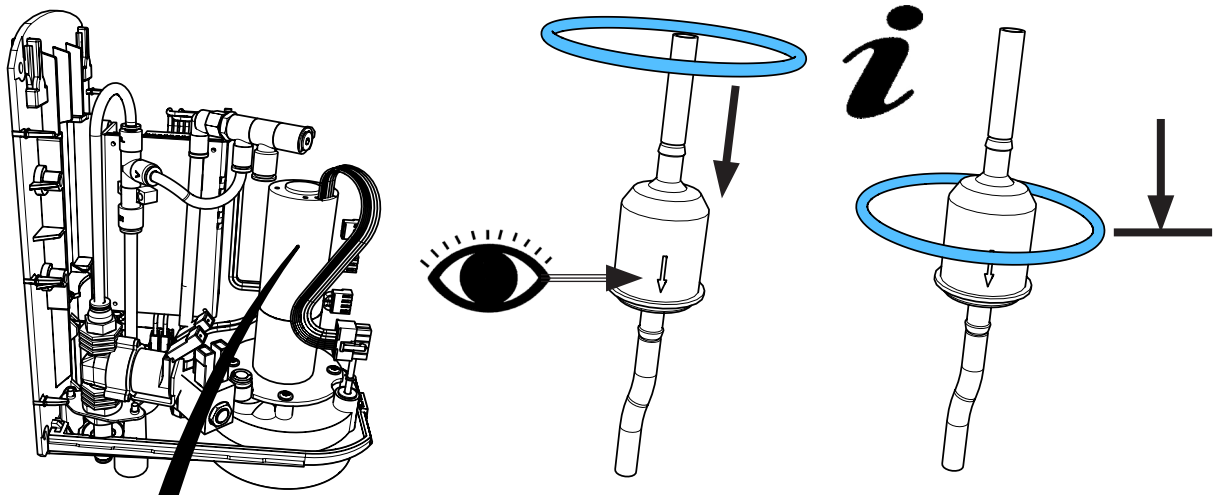


14

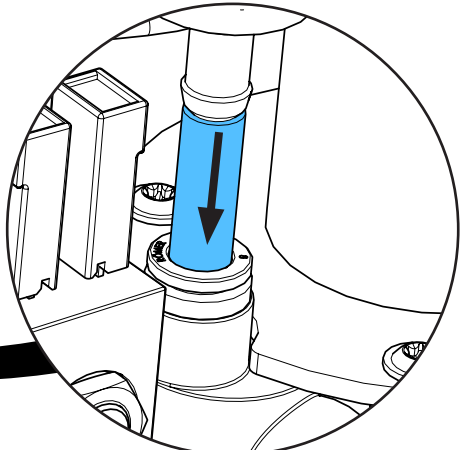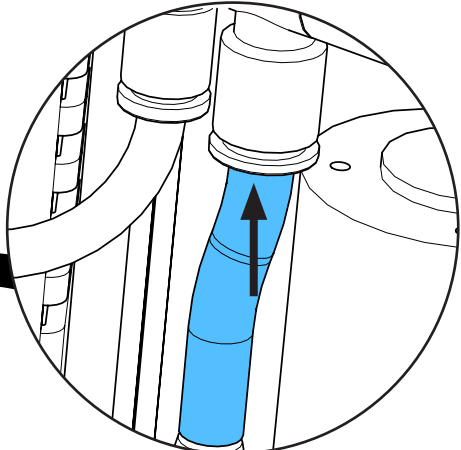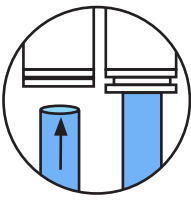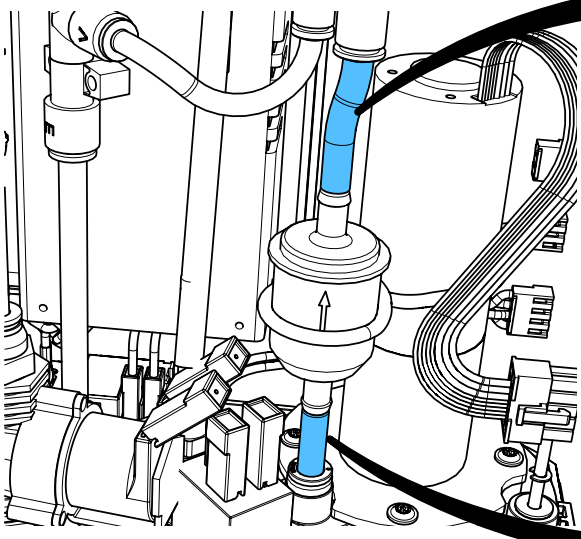


*i*


15

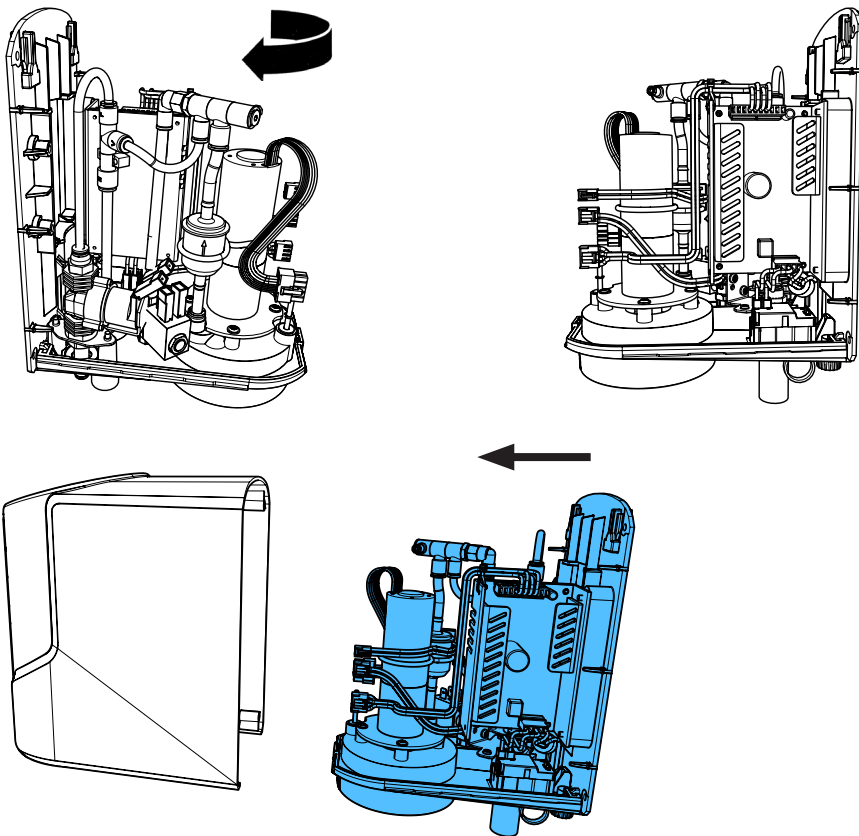


16

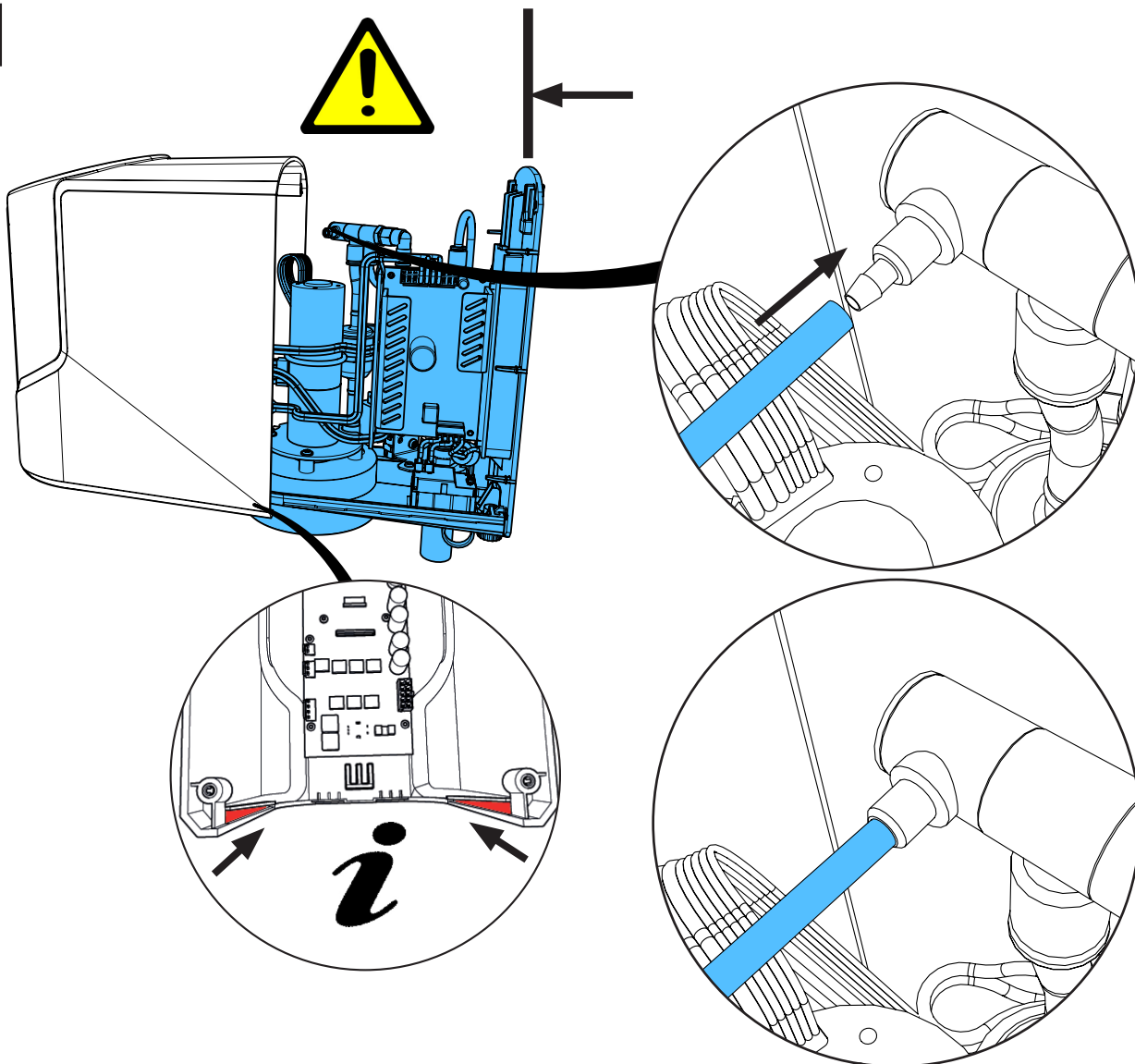


17

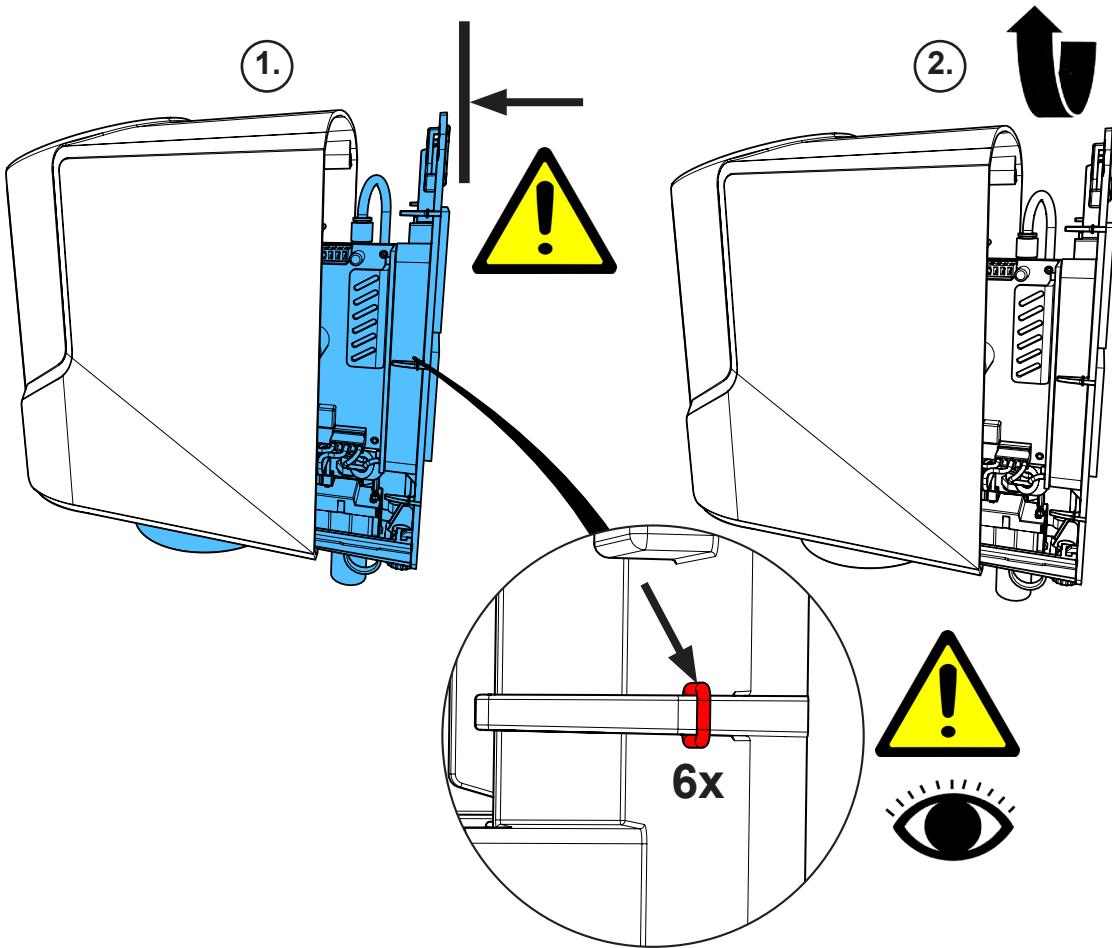


18

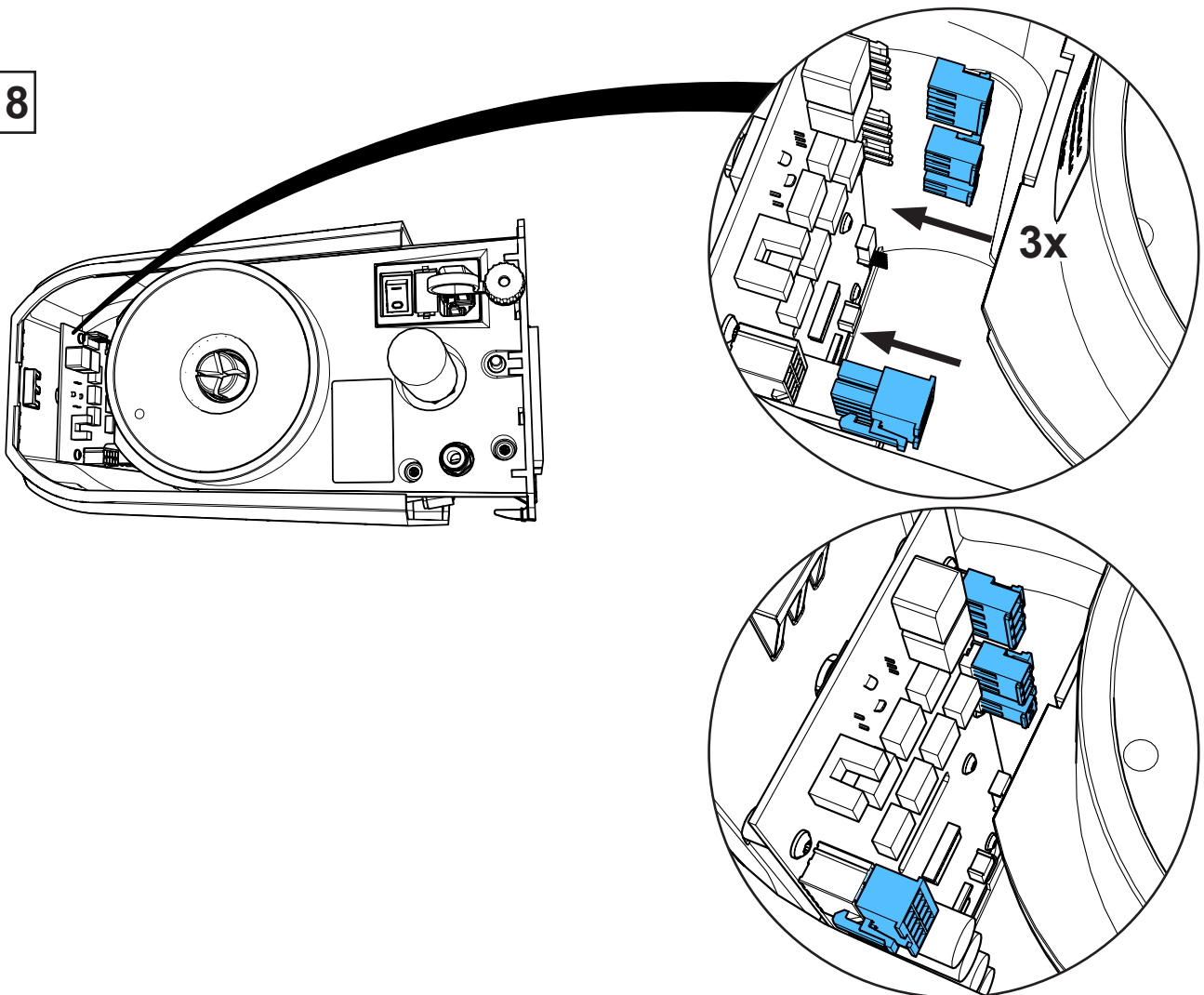


19

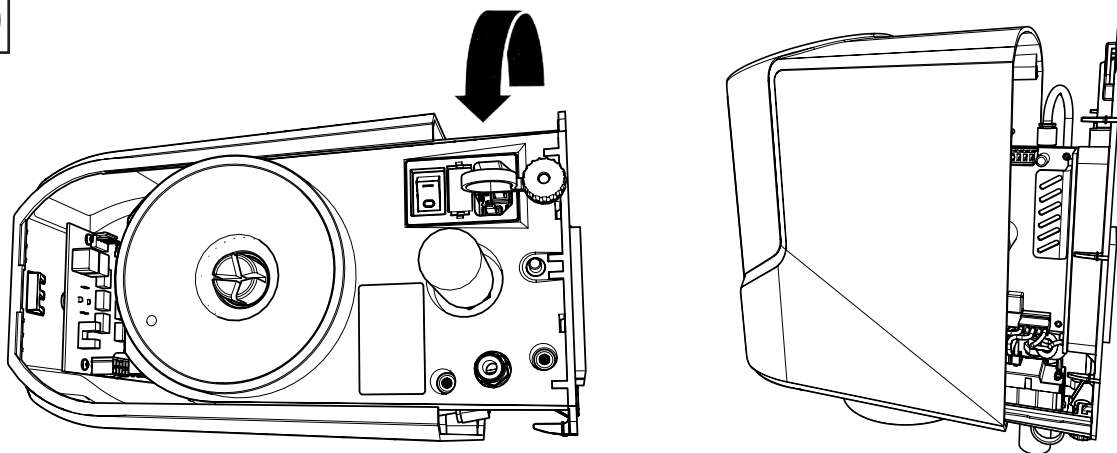

20



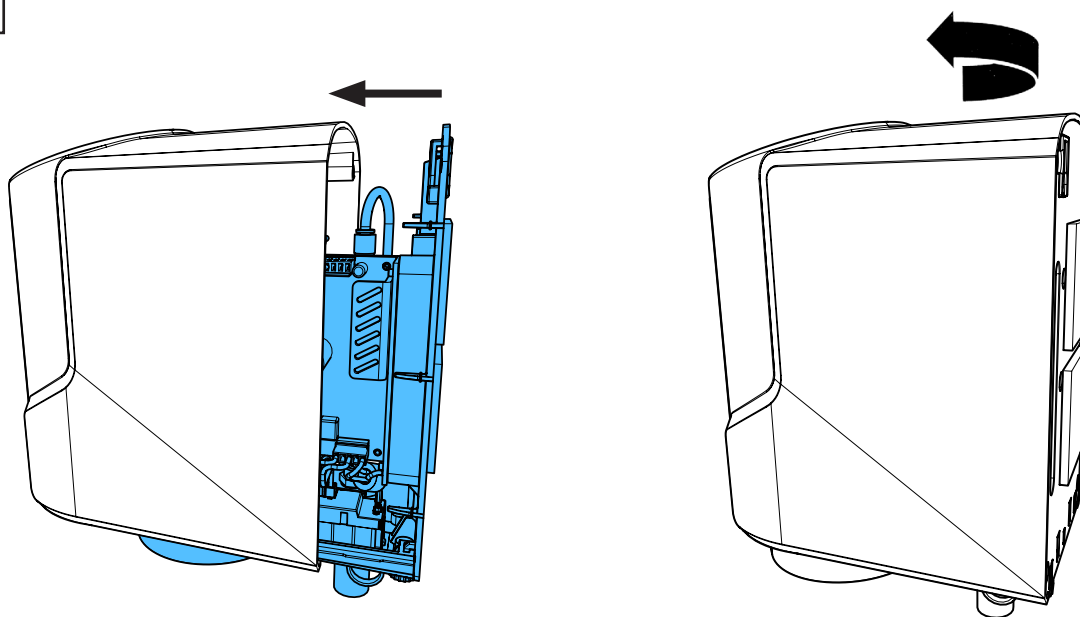


21

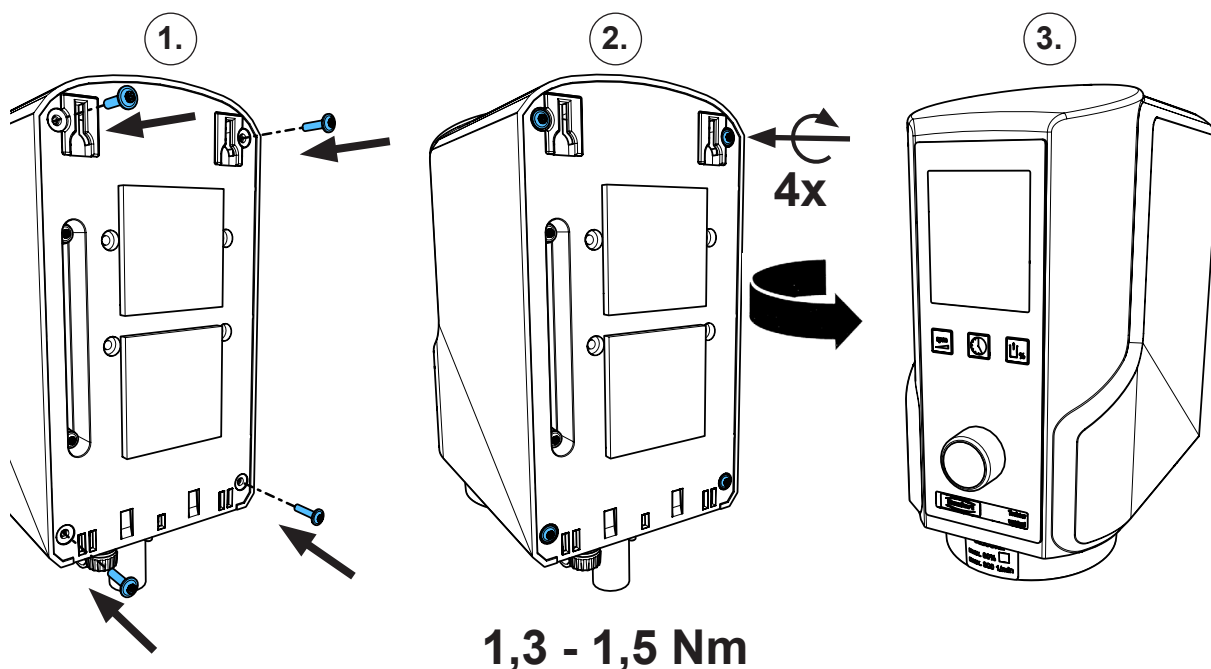


## 90013 4162


Dichtung Motorwelle • Seal for motor shaft • Joint de l'arbre du moteur  
Guarnizione per albero motore • Obturación para árbol del motor  
Уплотнитель вала двигателя

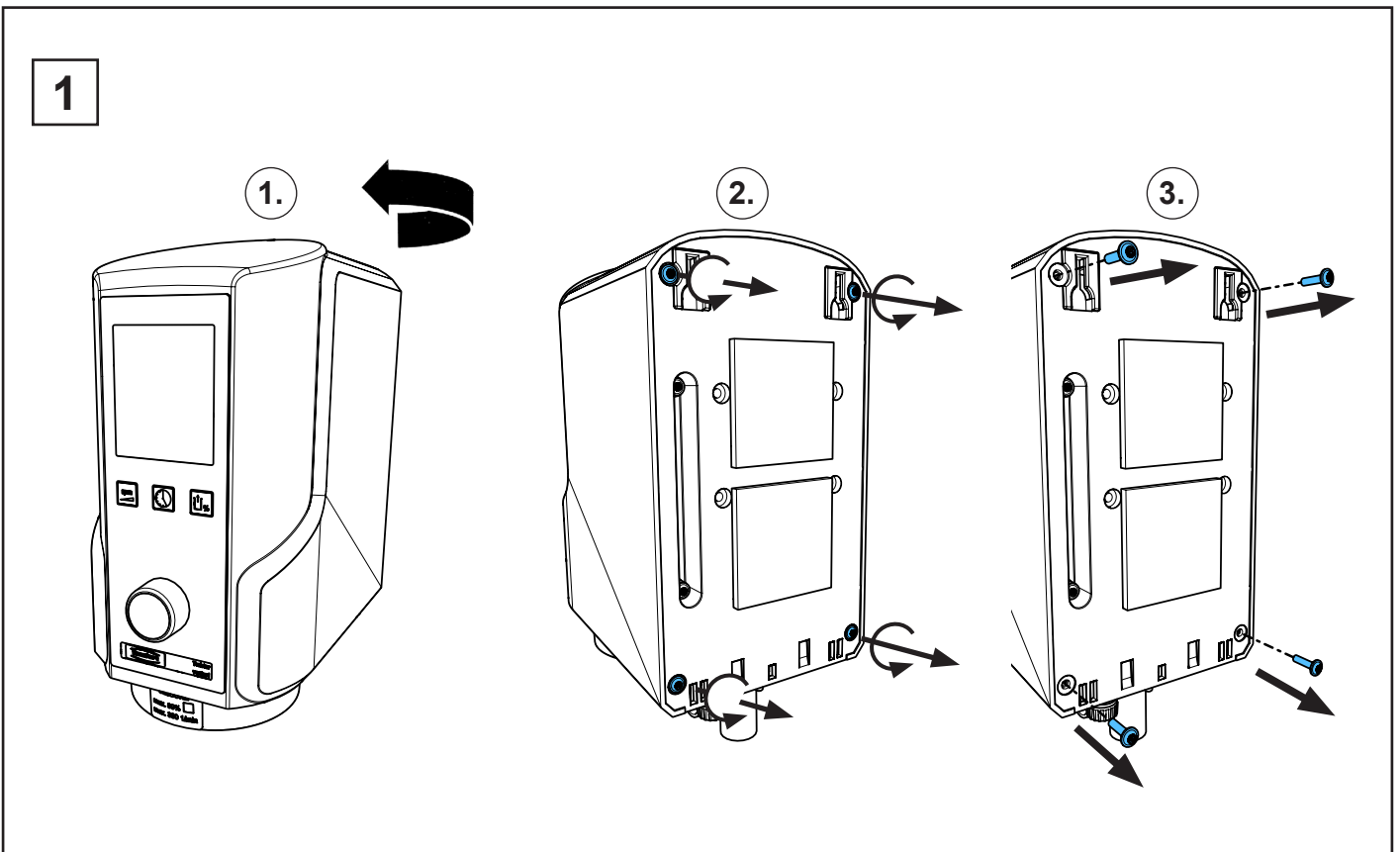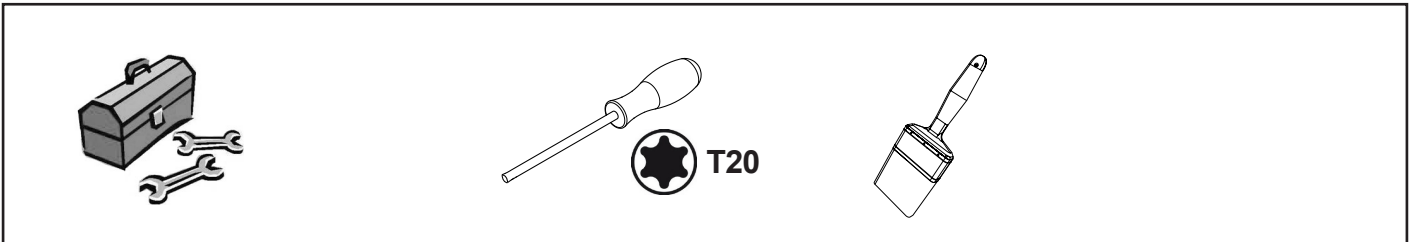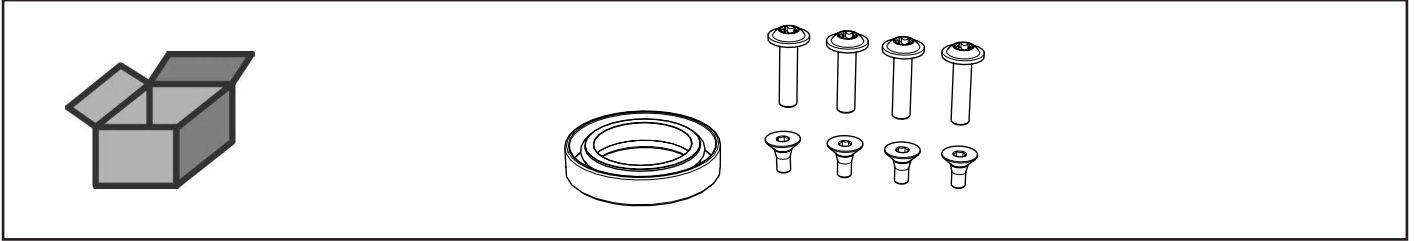

2

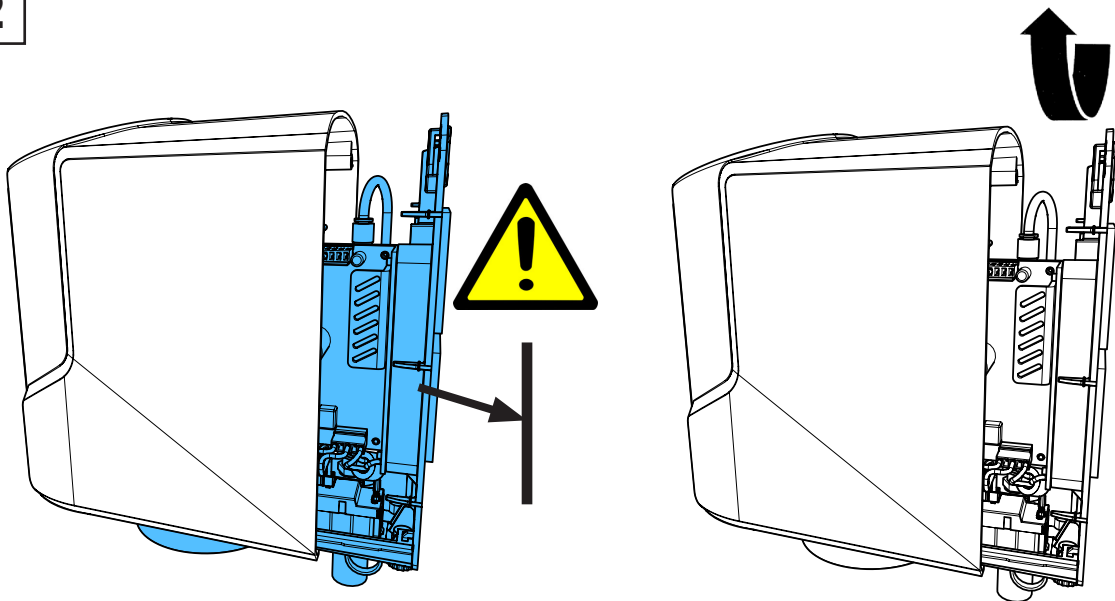


3

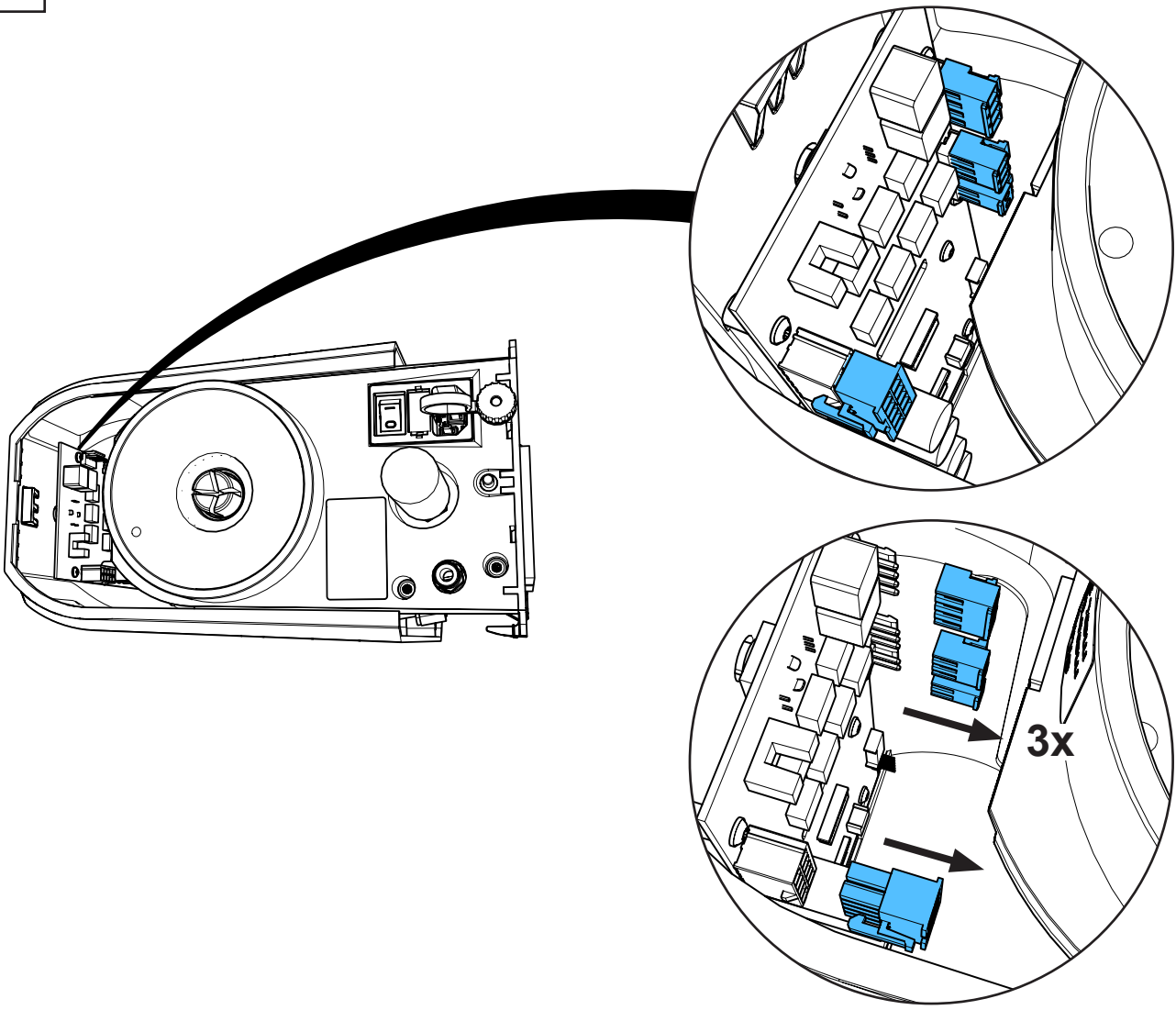





4

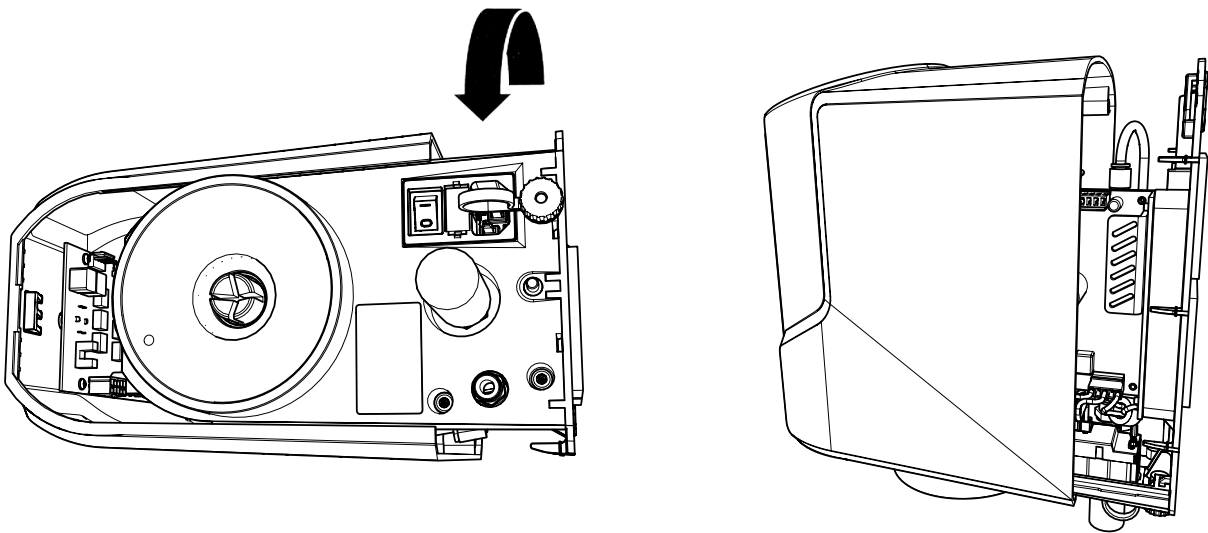


5

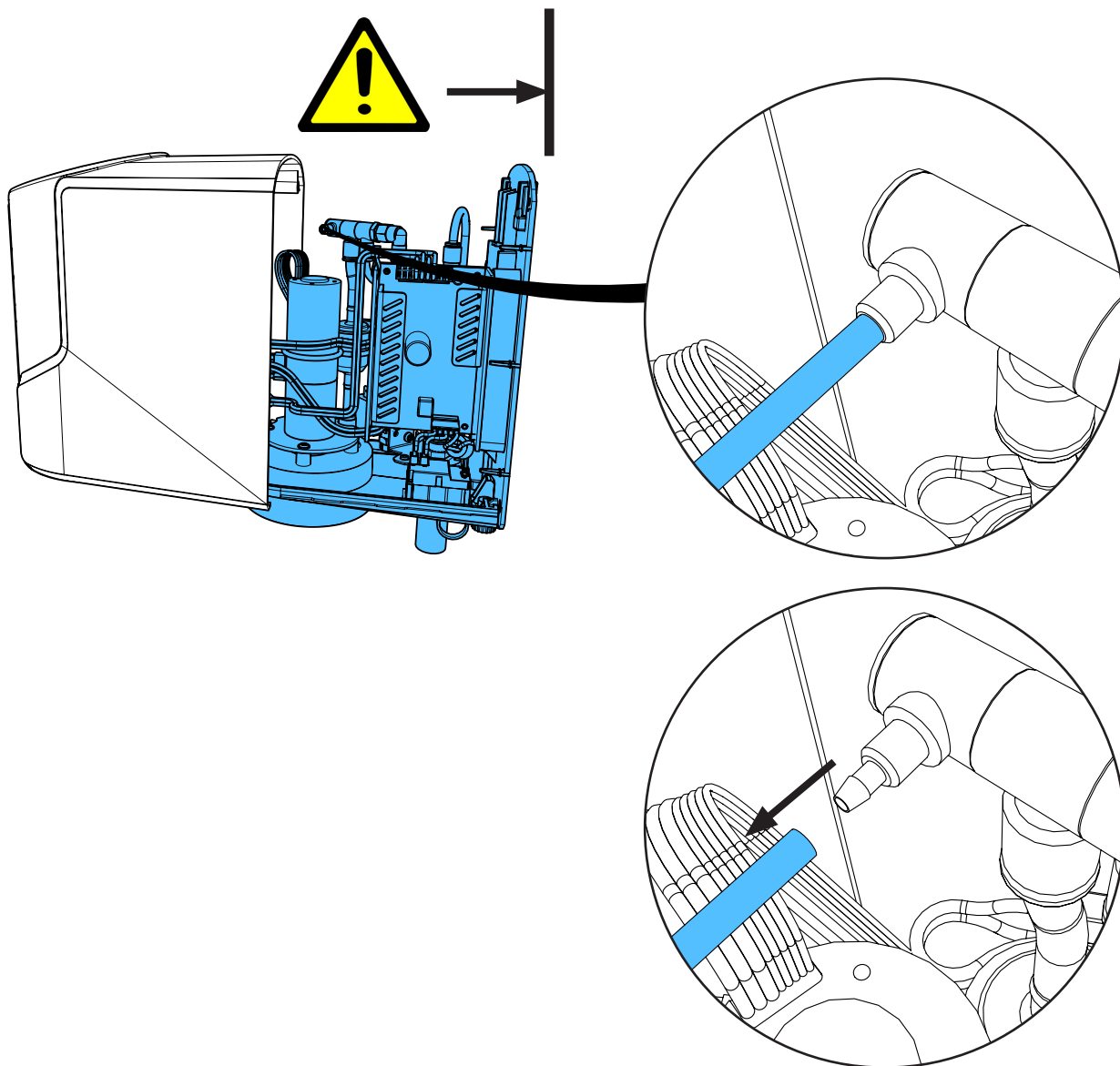


6

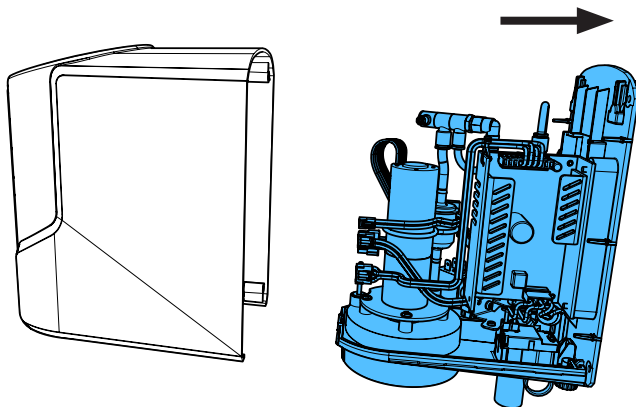


7

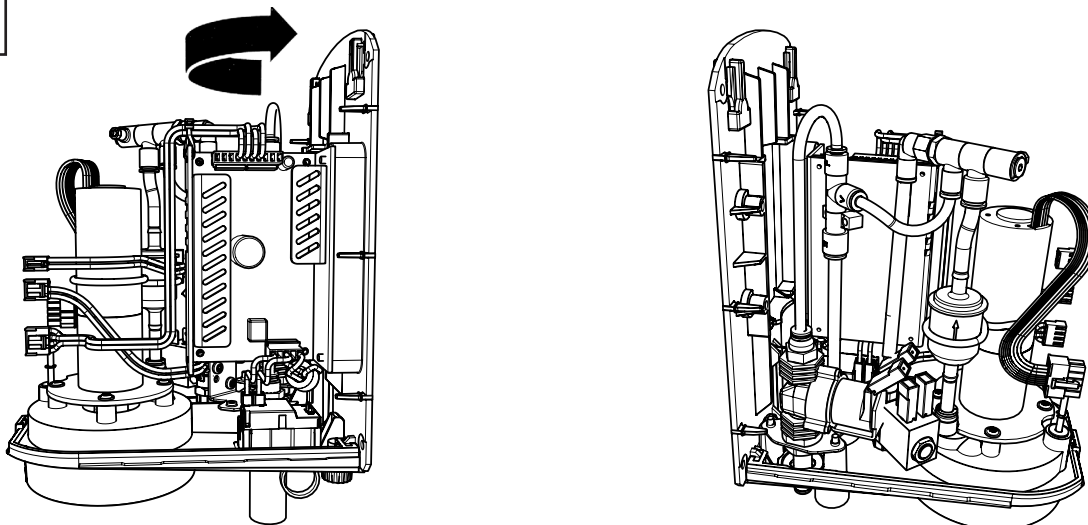



8

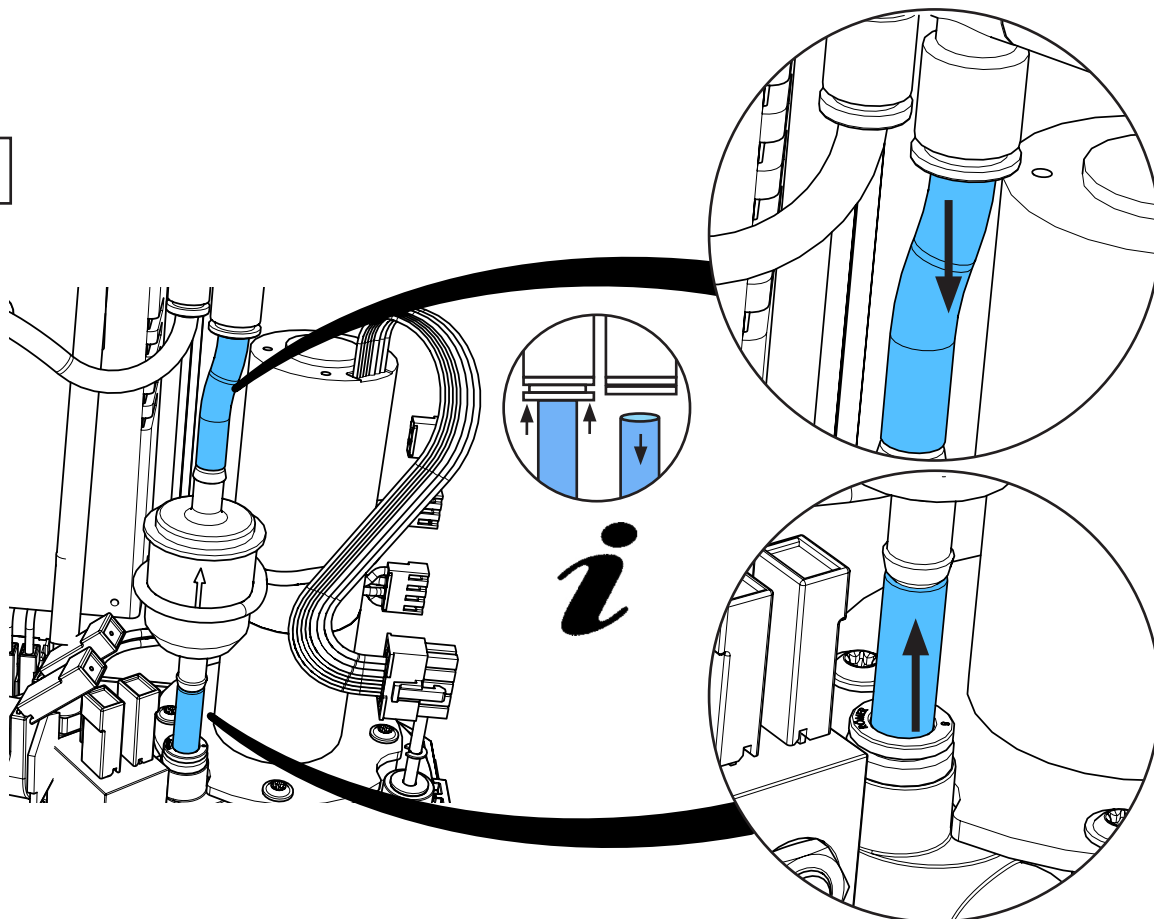


9

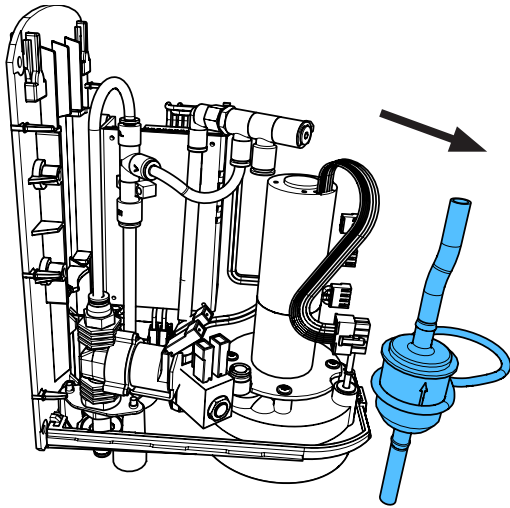


10

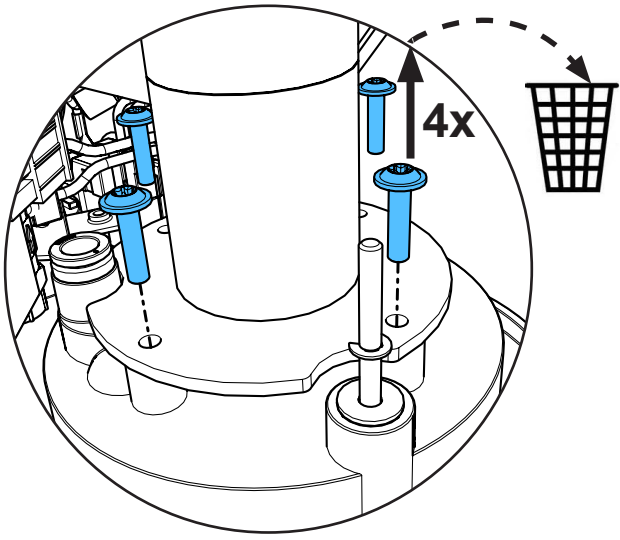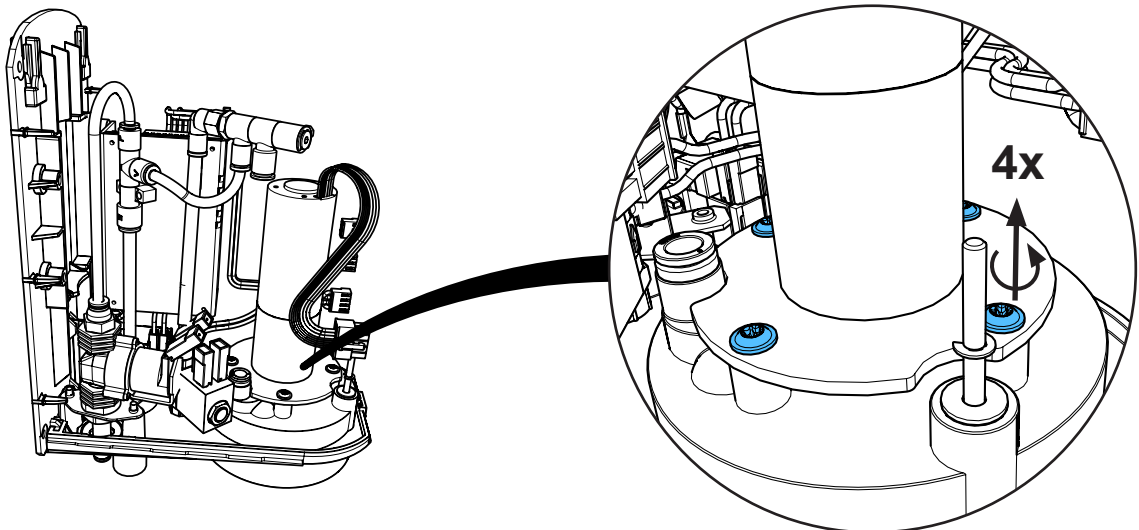





11

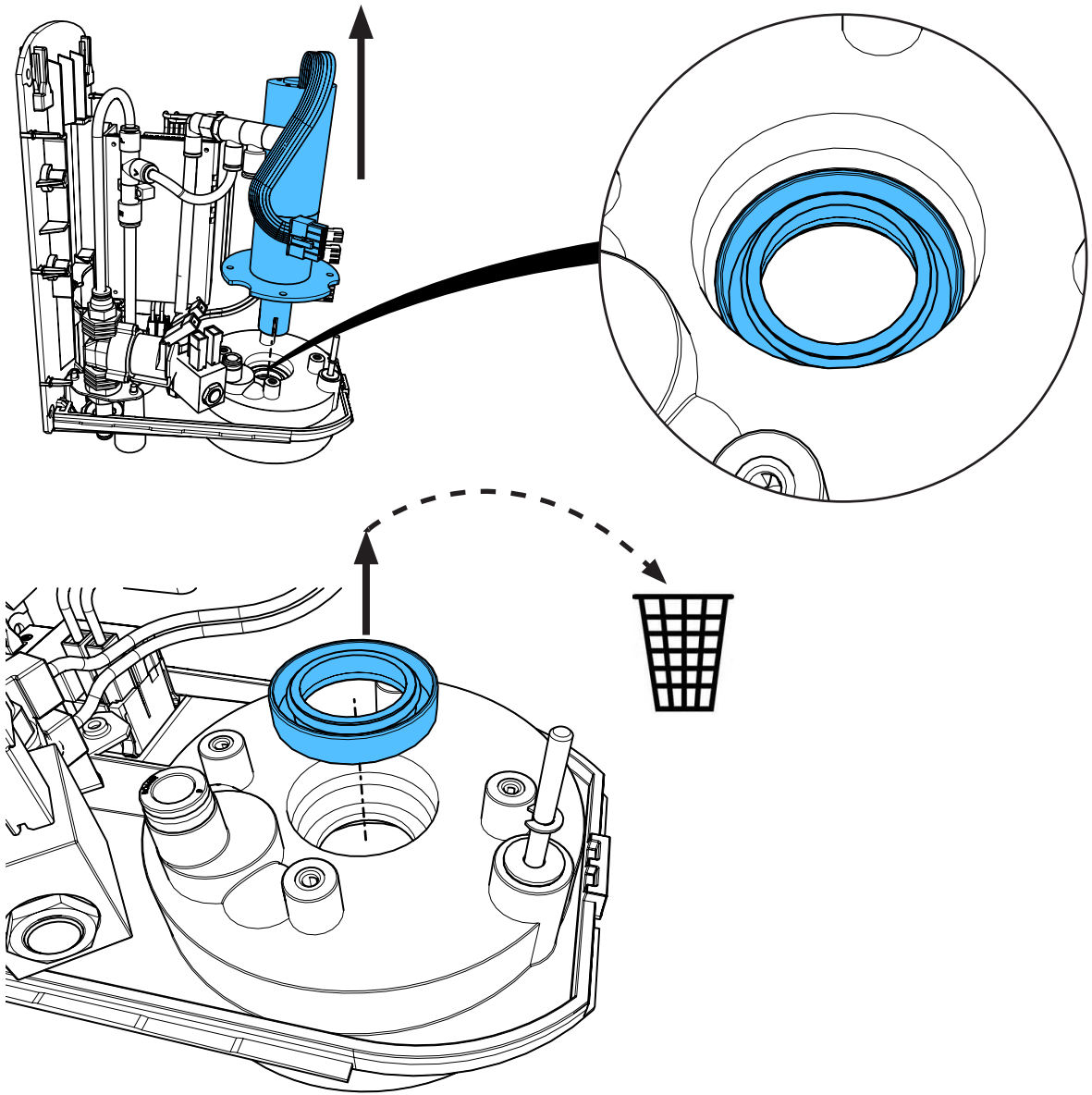



12


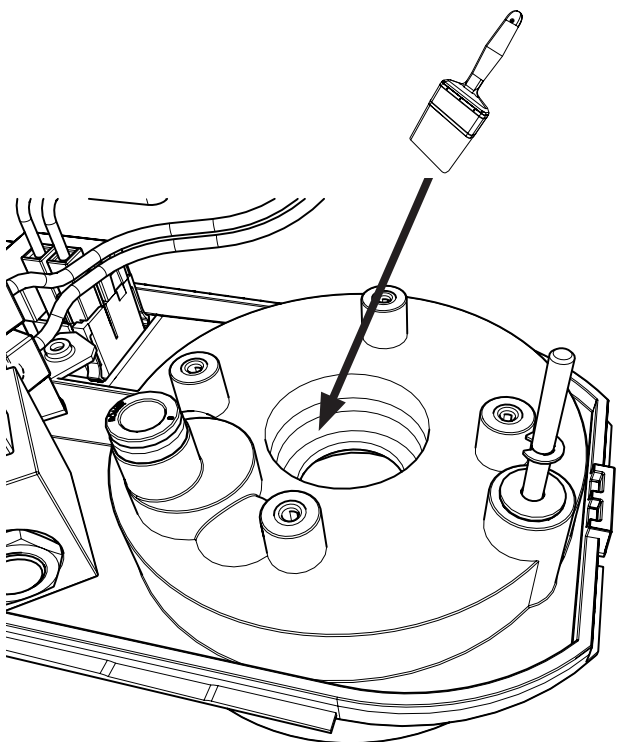


13

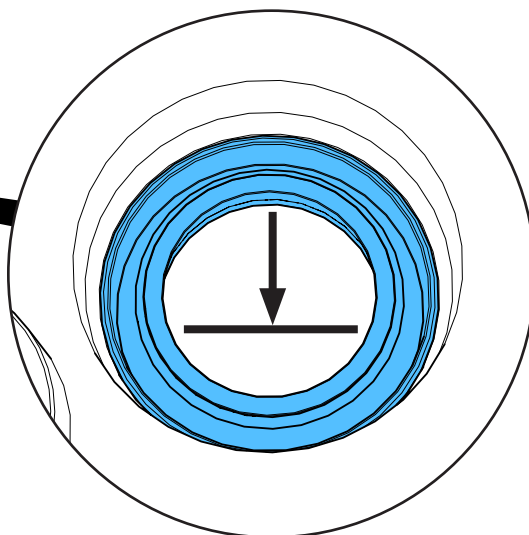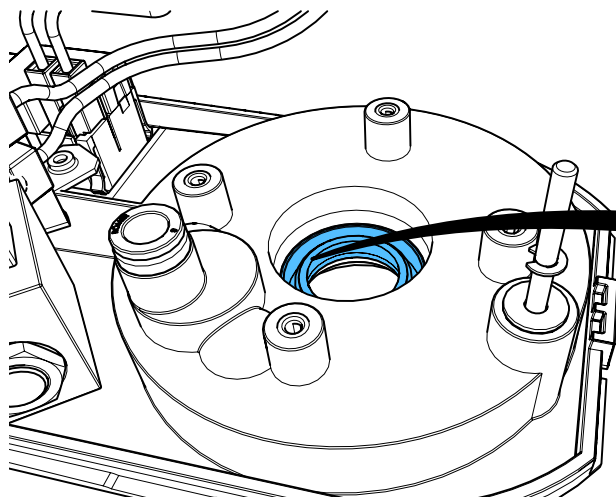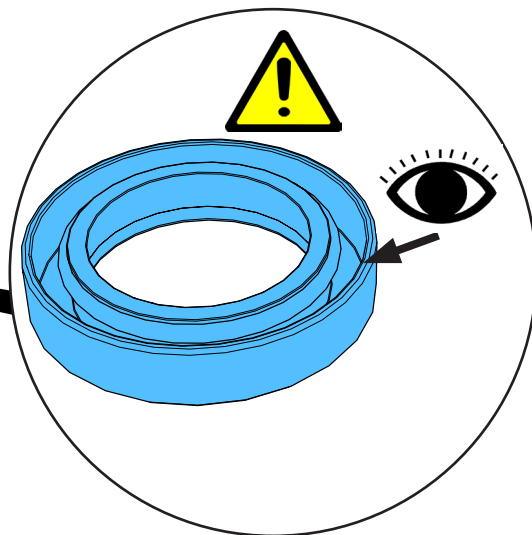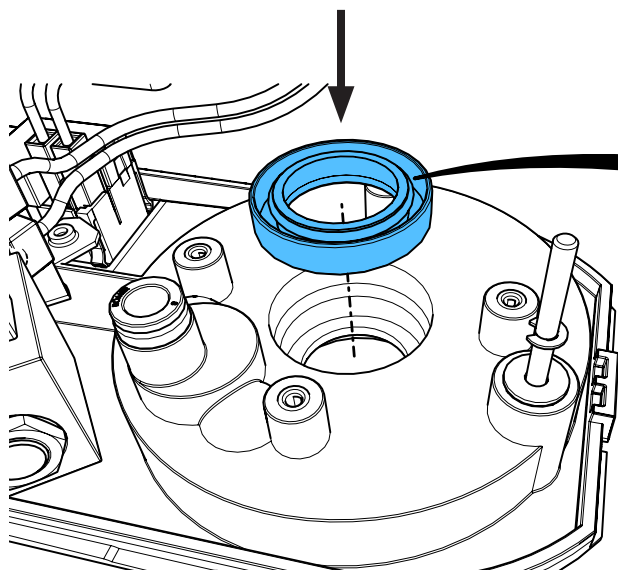






14

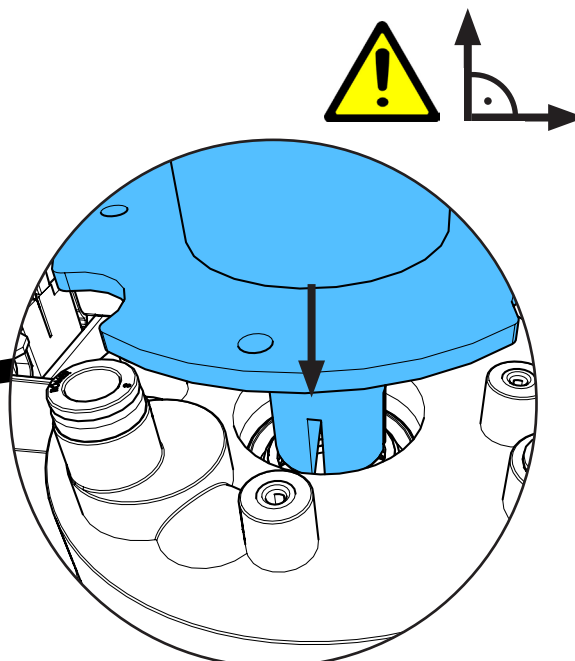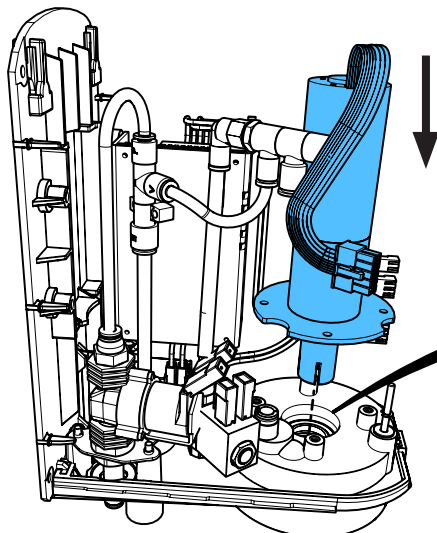

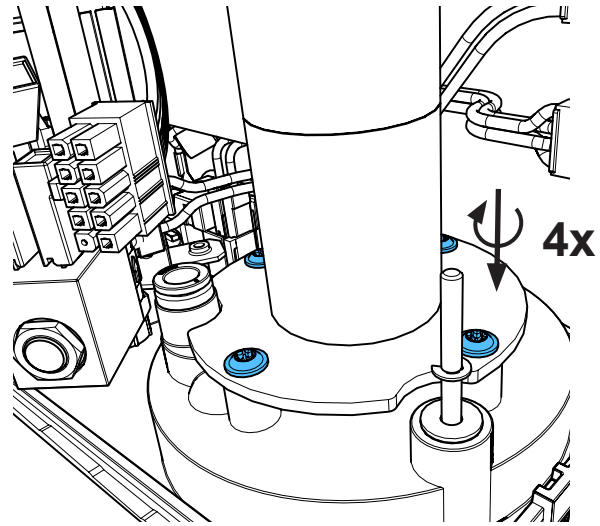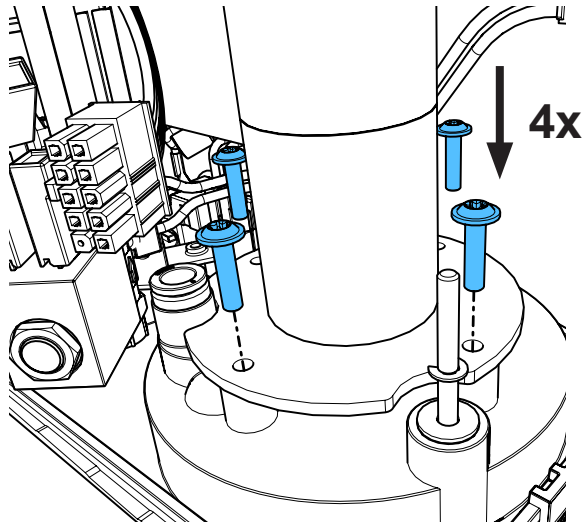


2,4 - 2,6 Nm


15

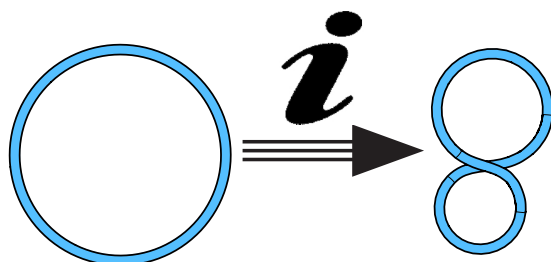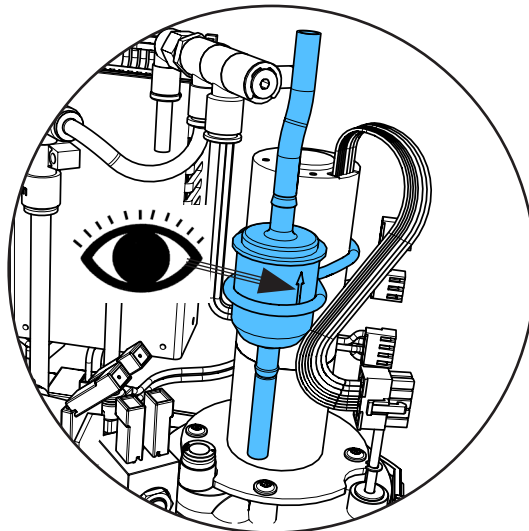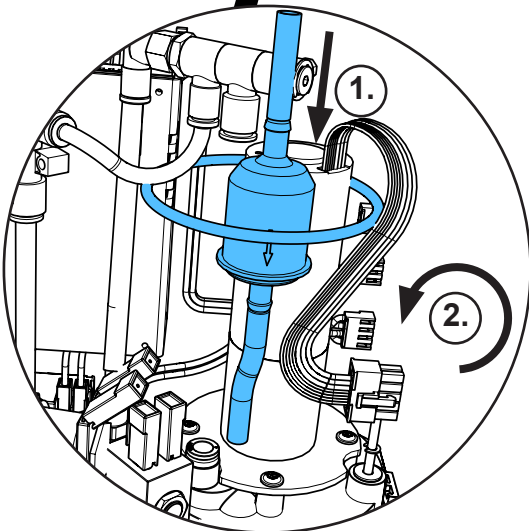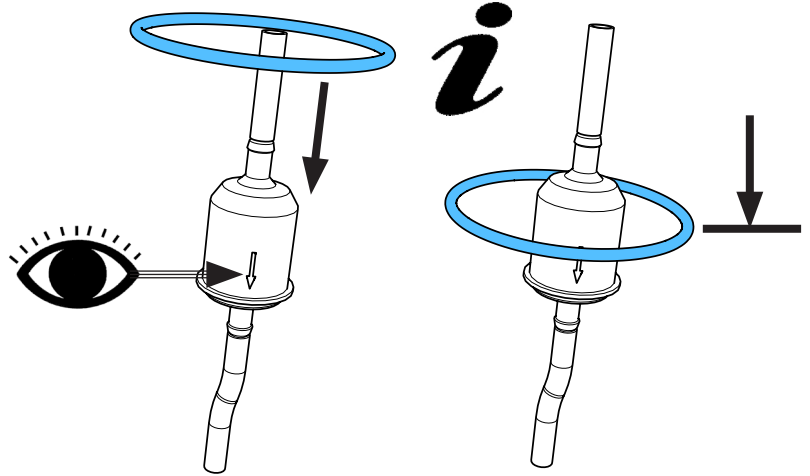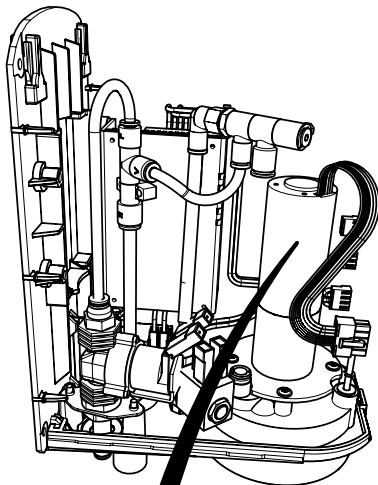


16

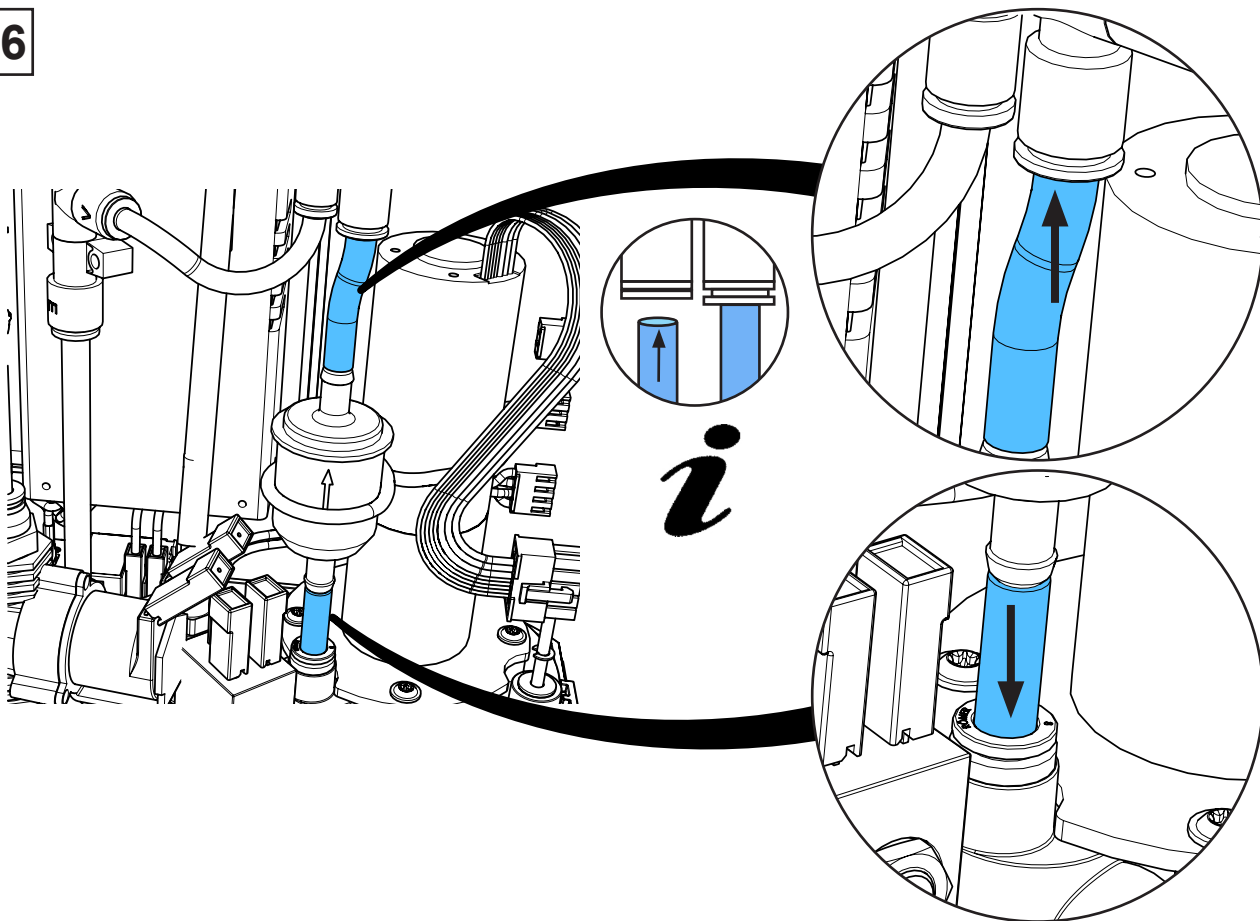


17

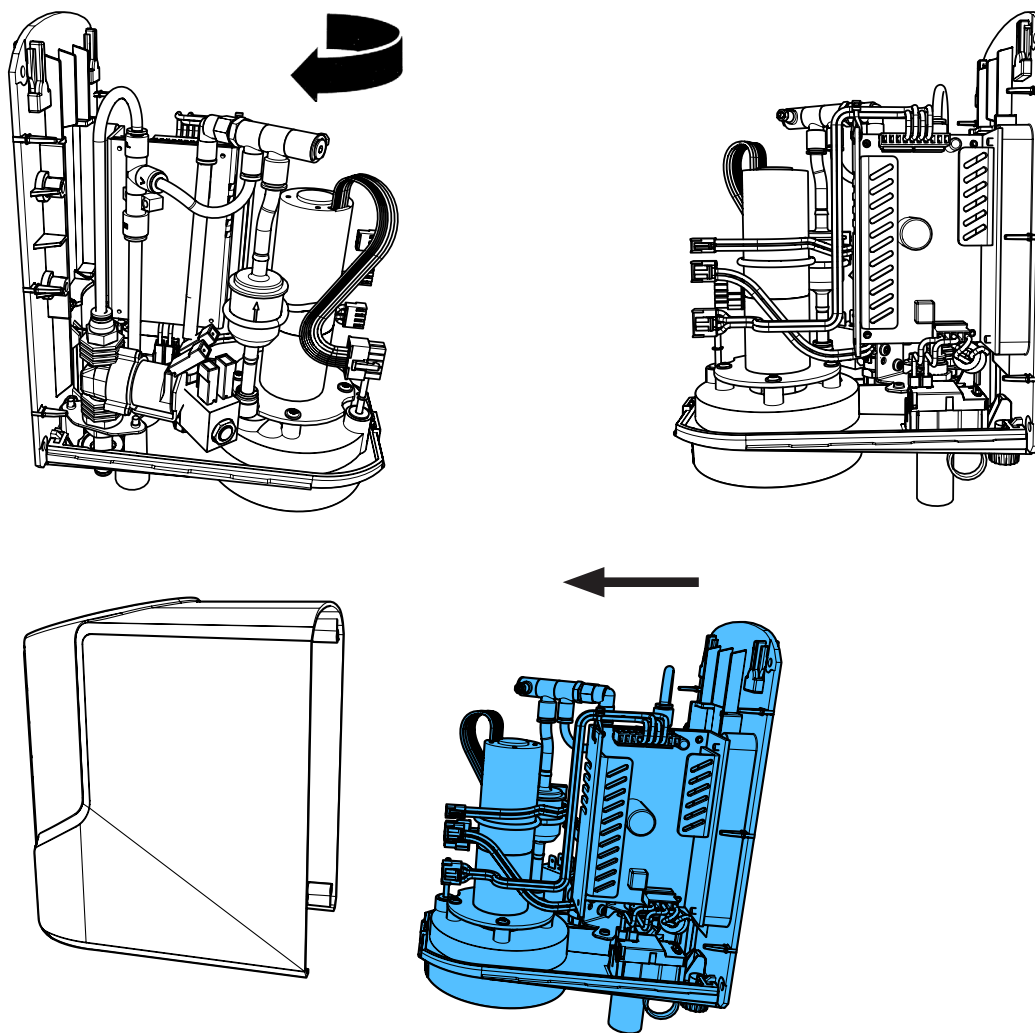

18

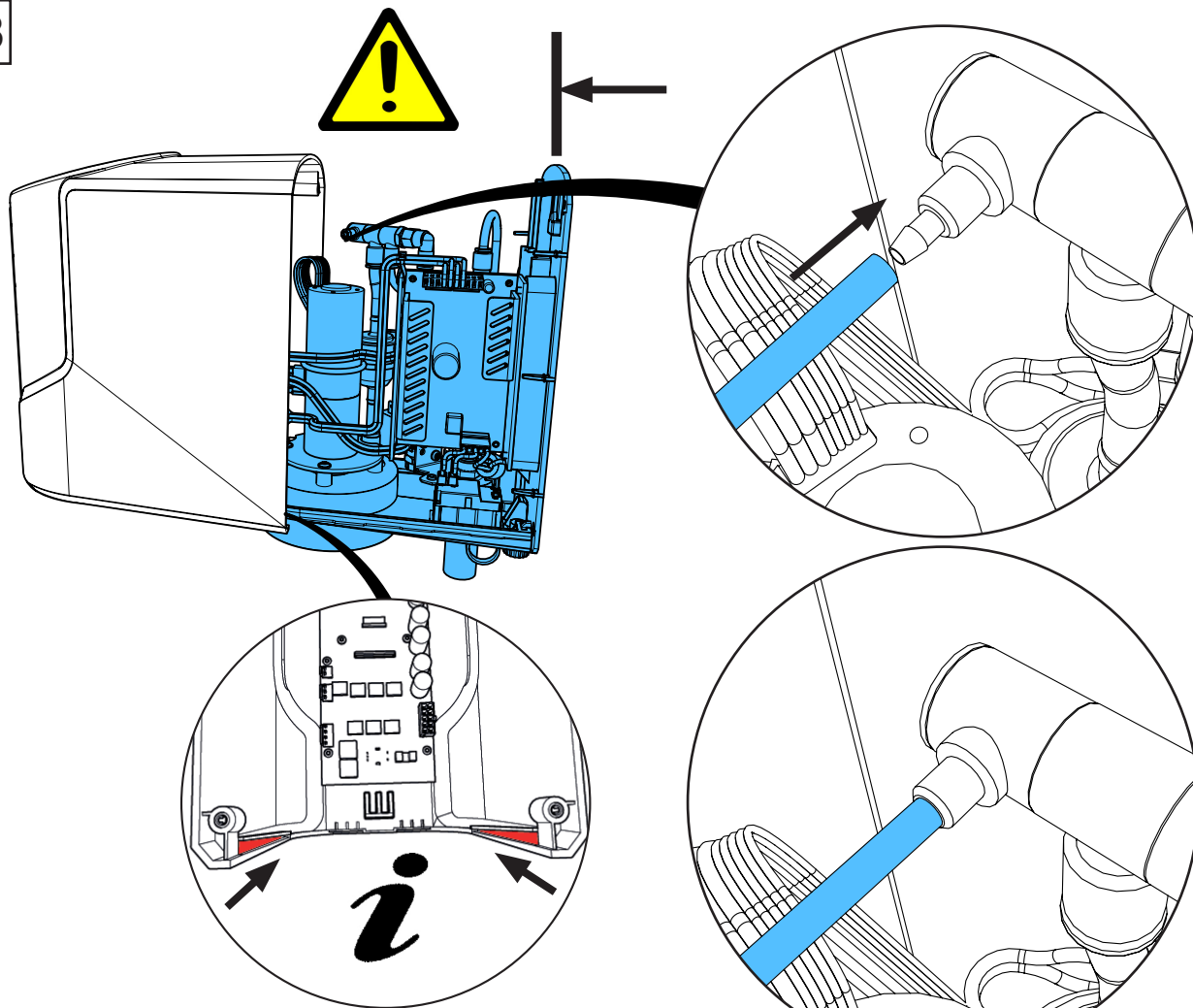


19

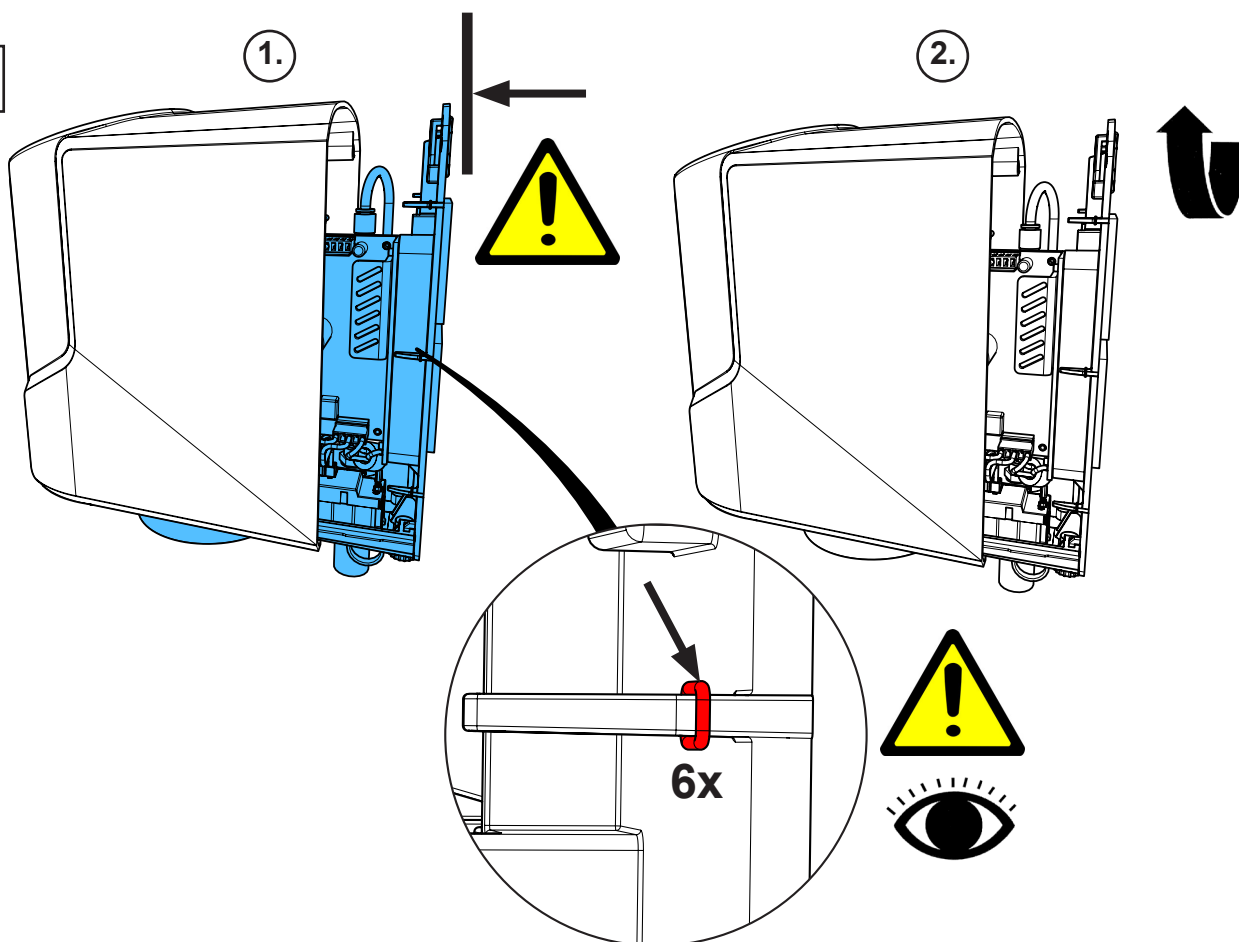





20

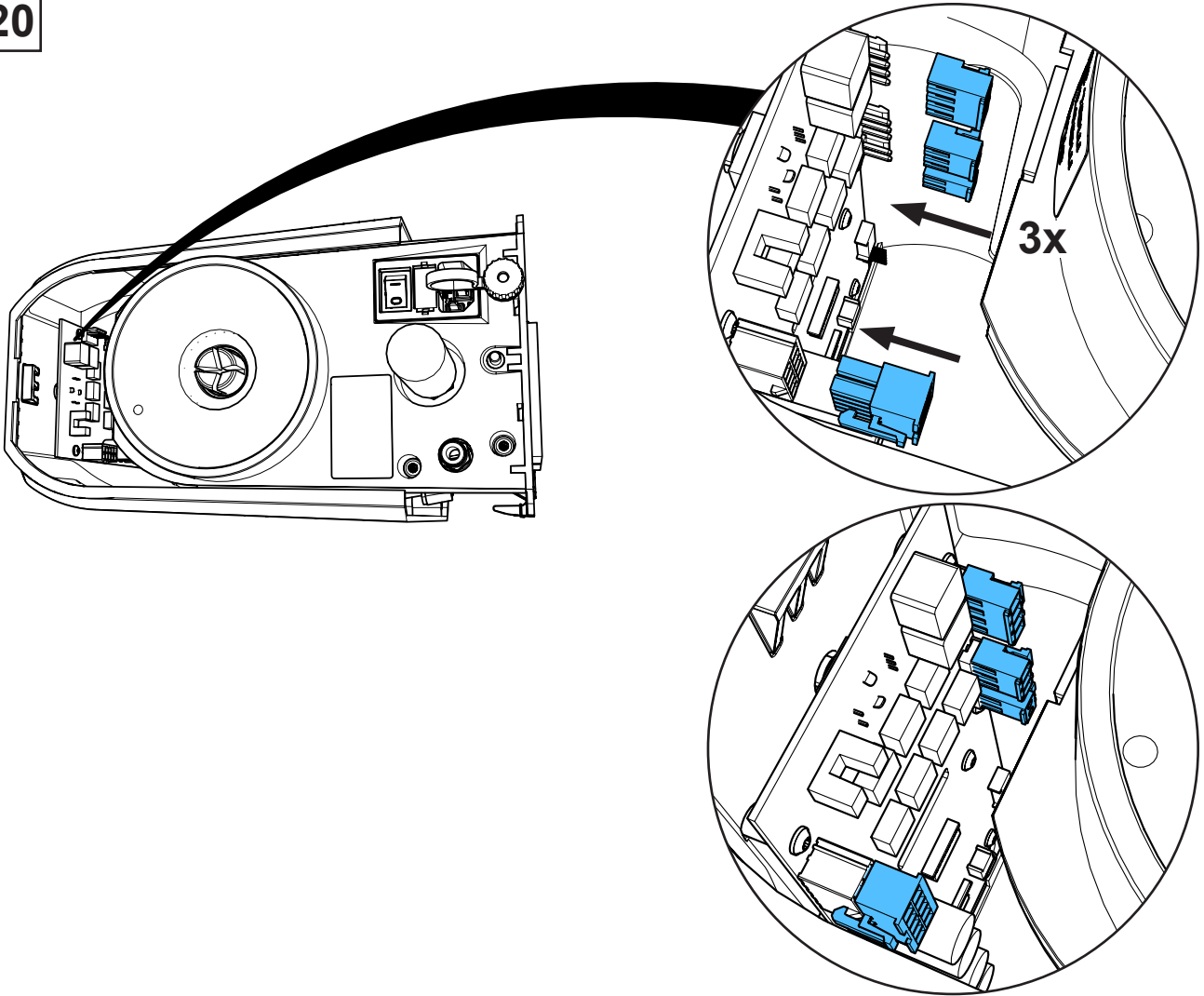


21

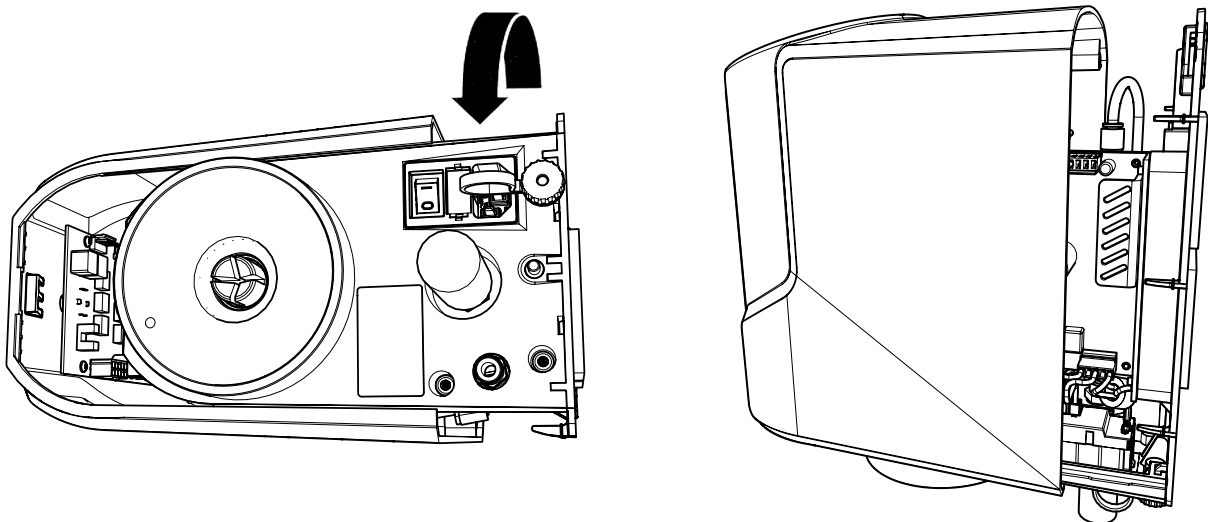

22



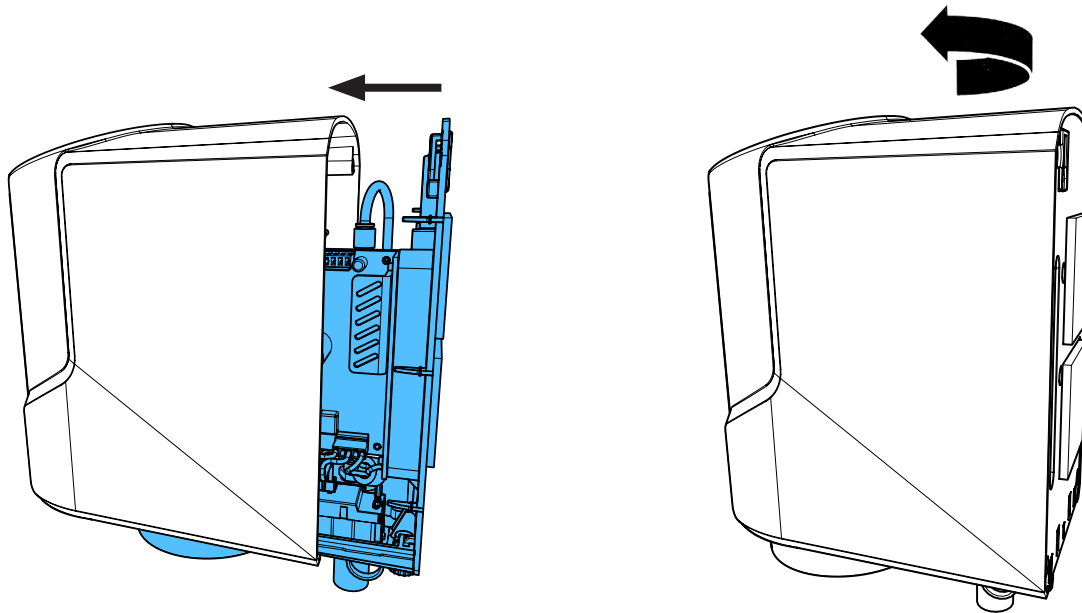


23

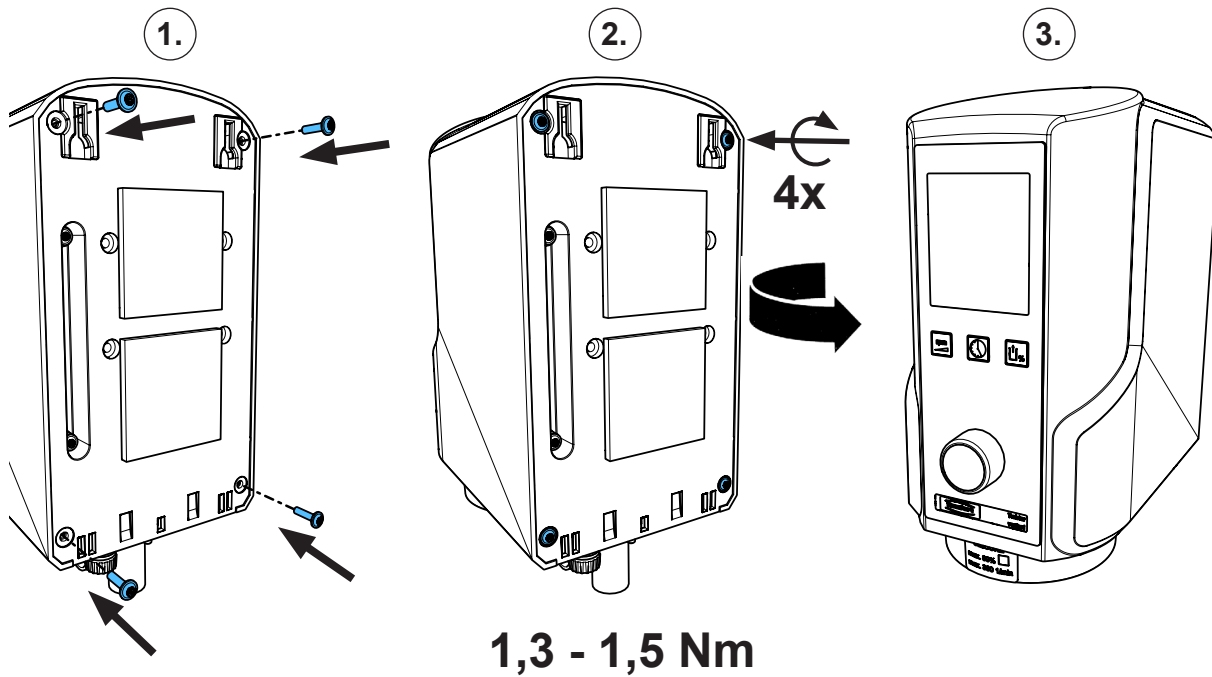


90003 4719


Feinfilter • Micro filter • Filtre fin • Microfiltro • Filtro fino • Фильтр тонкой очистки

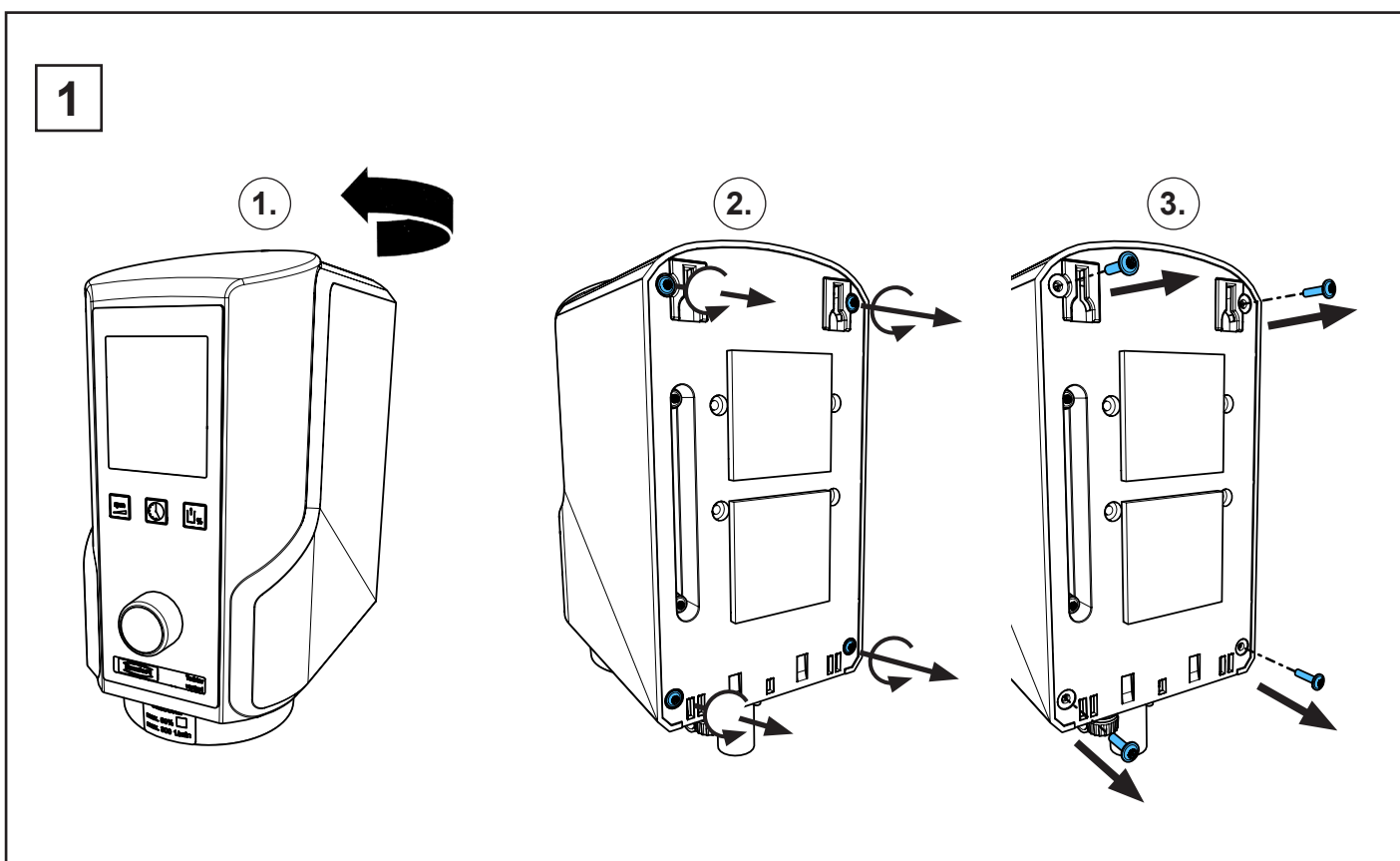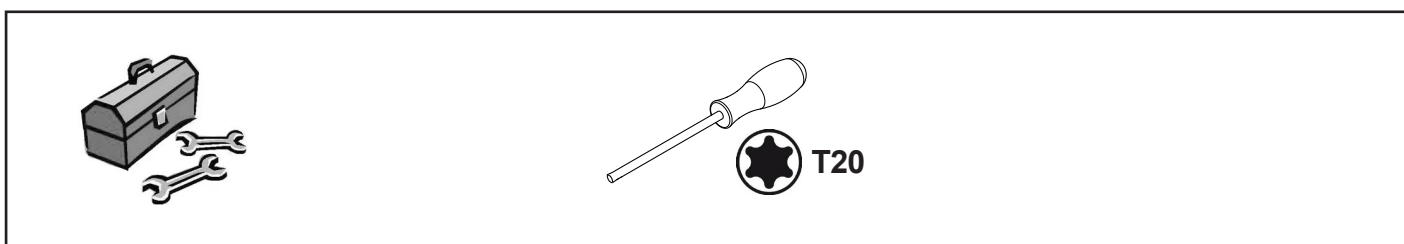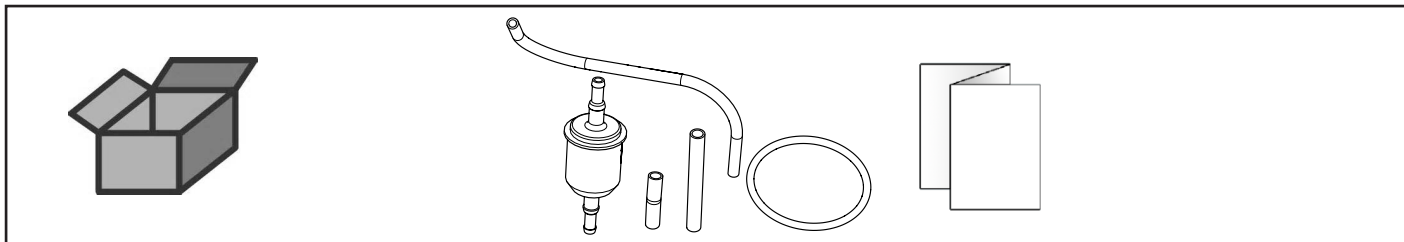


2

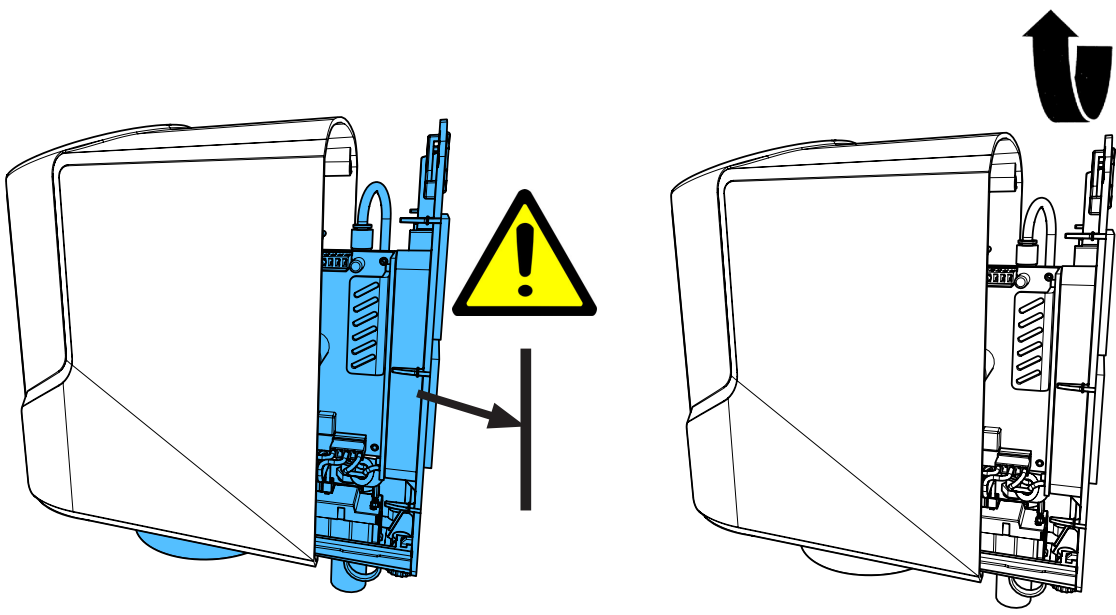


3

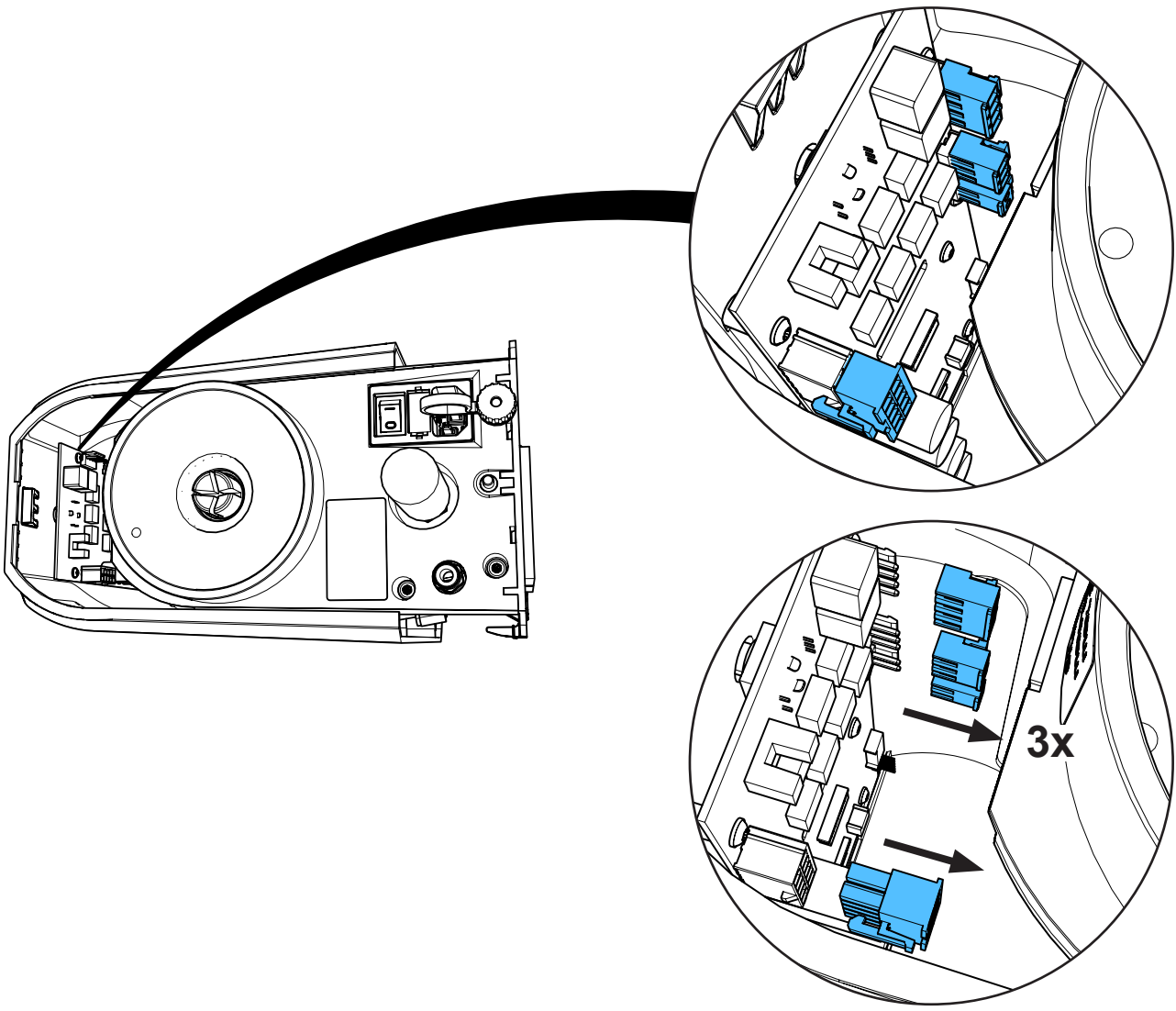


4

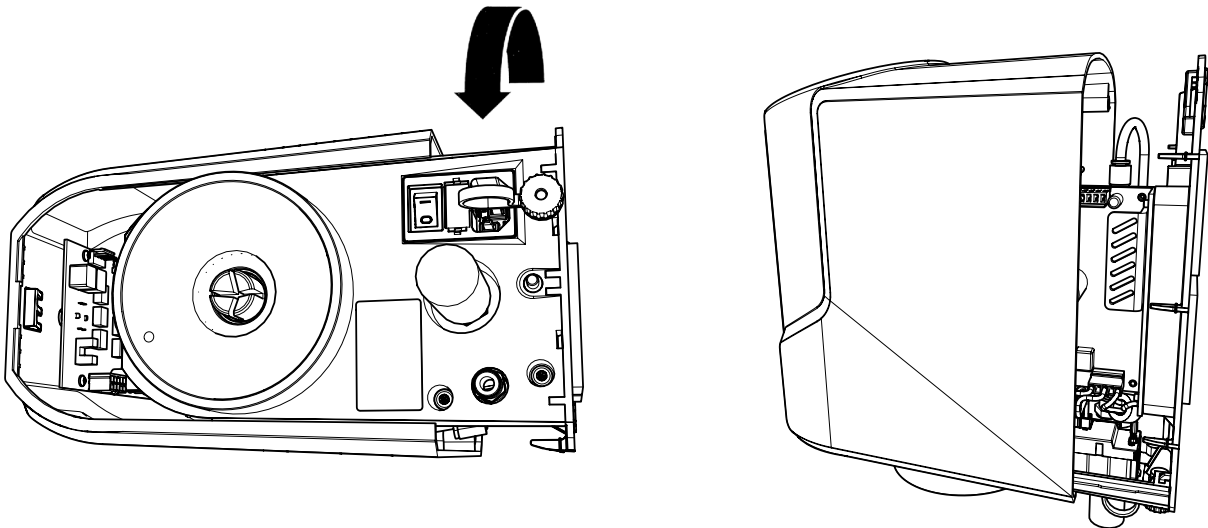


5

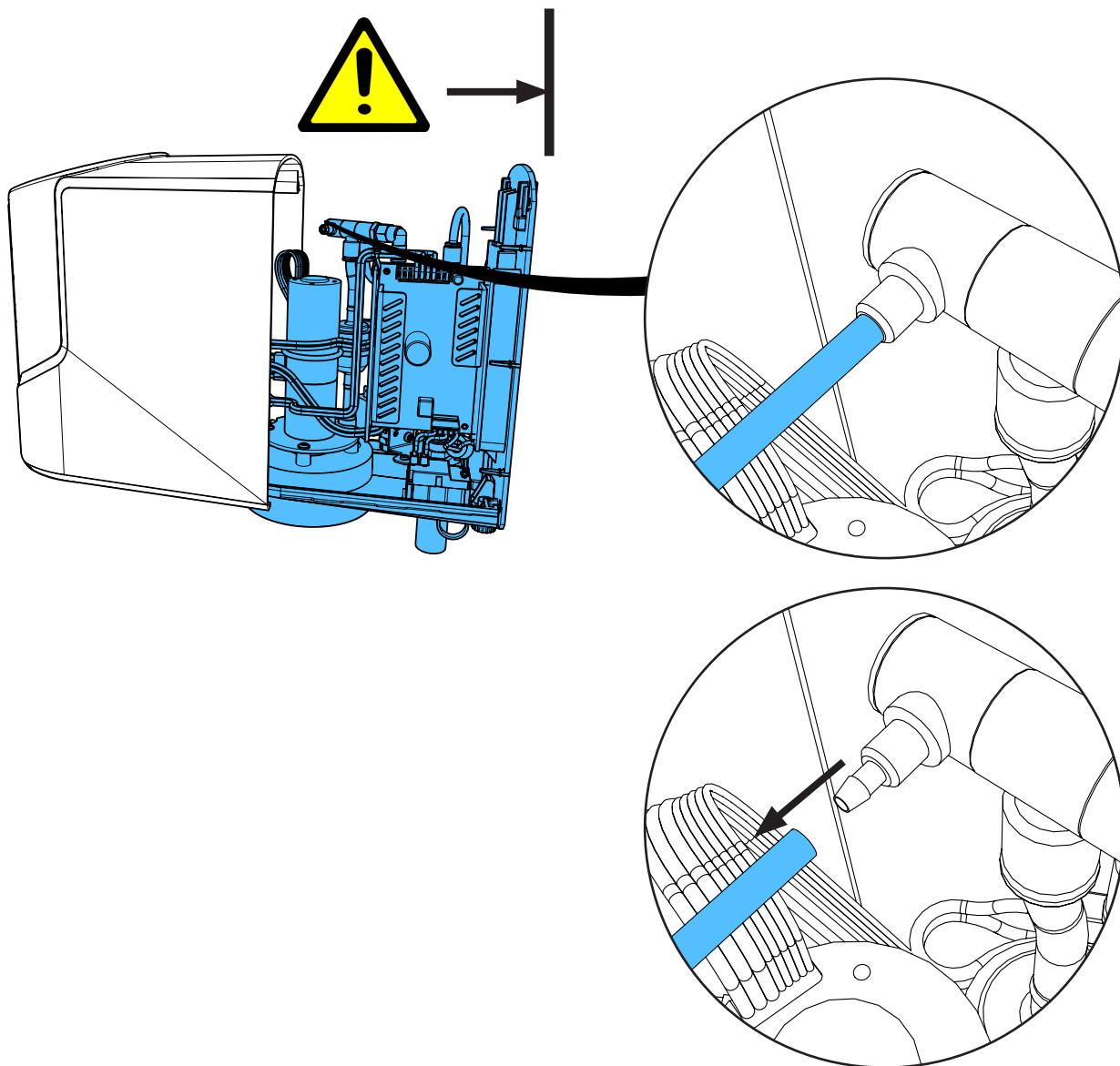


6

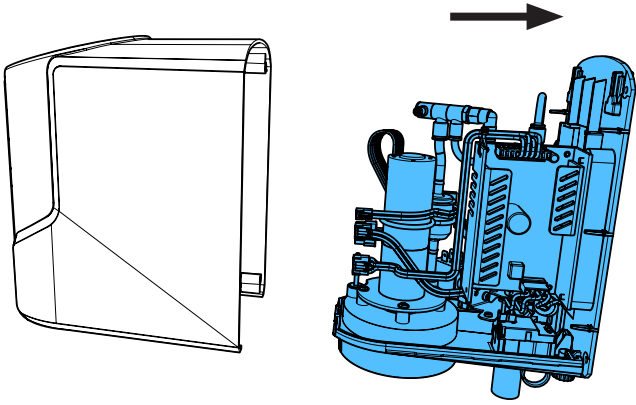


7

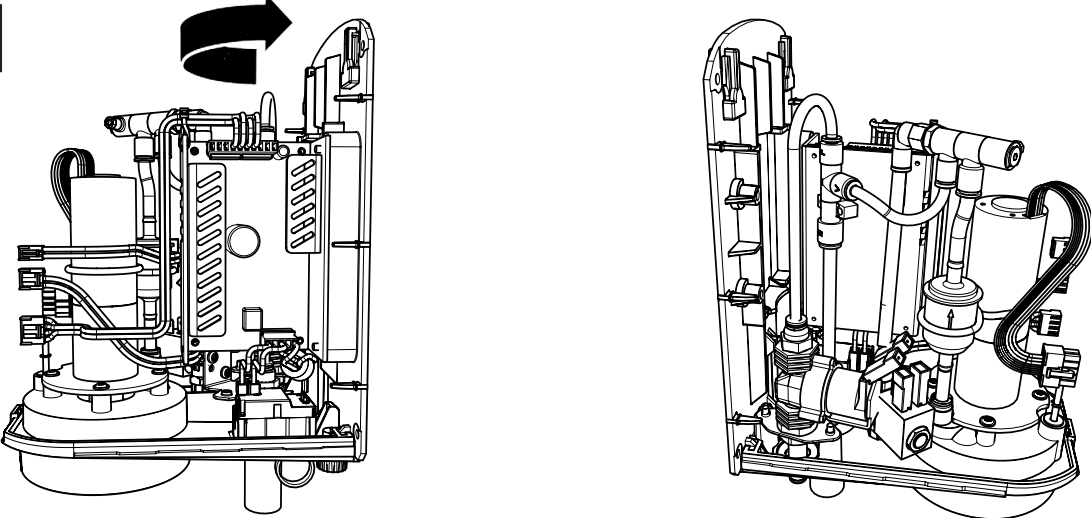


8

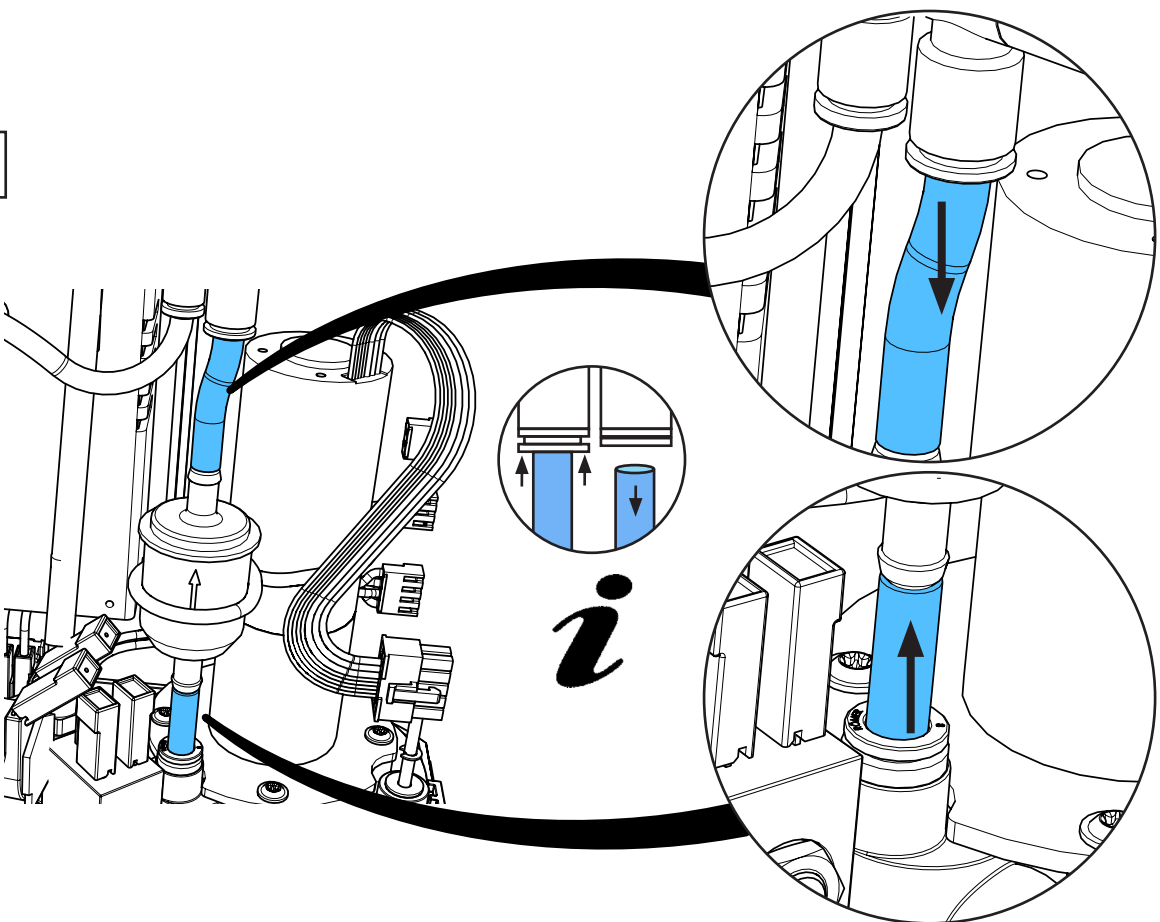


9

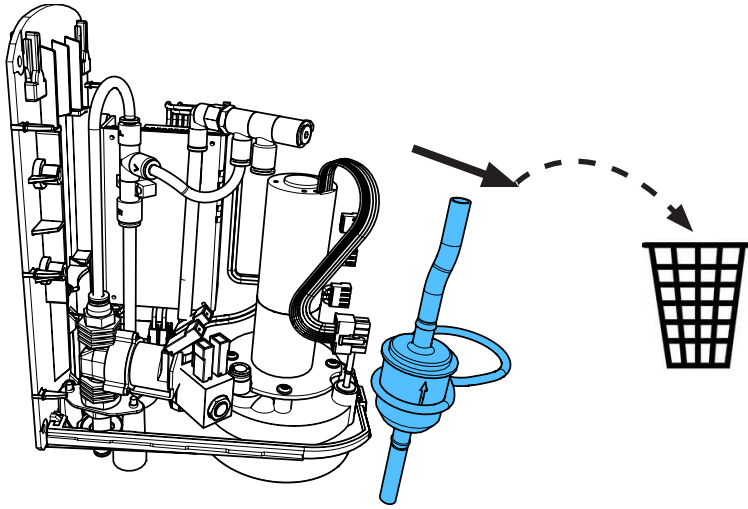




10

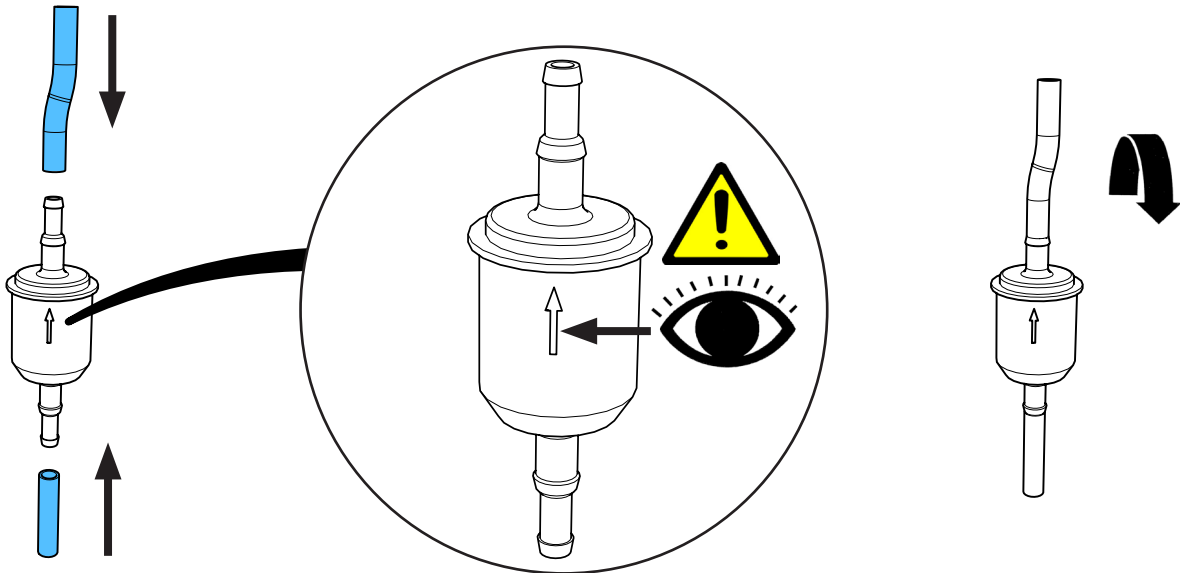


11

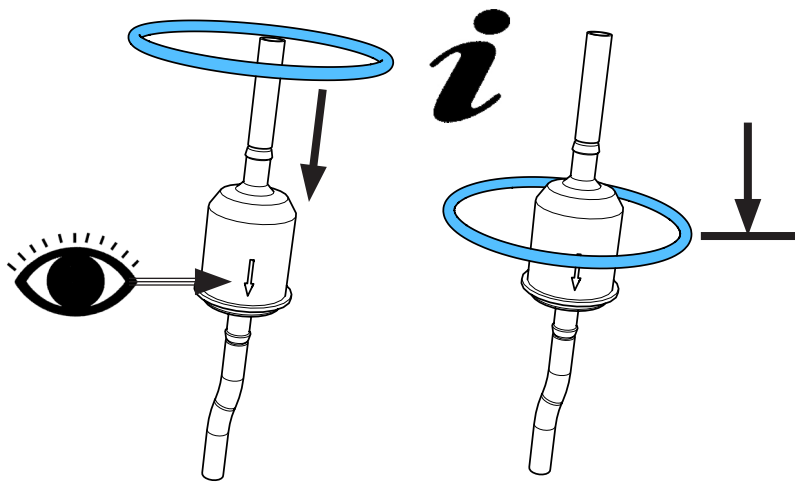

12

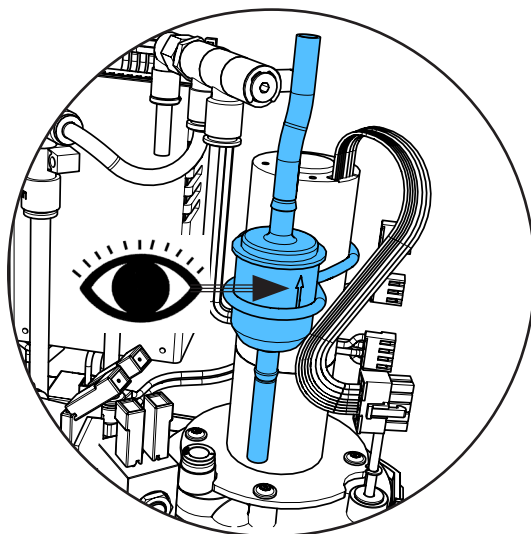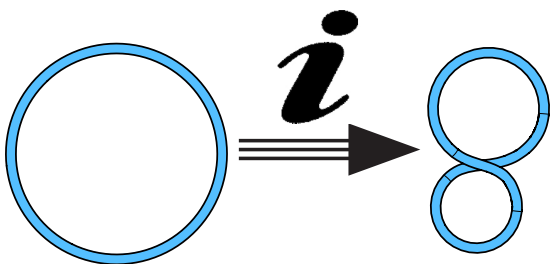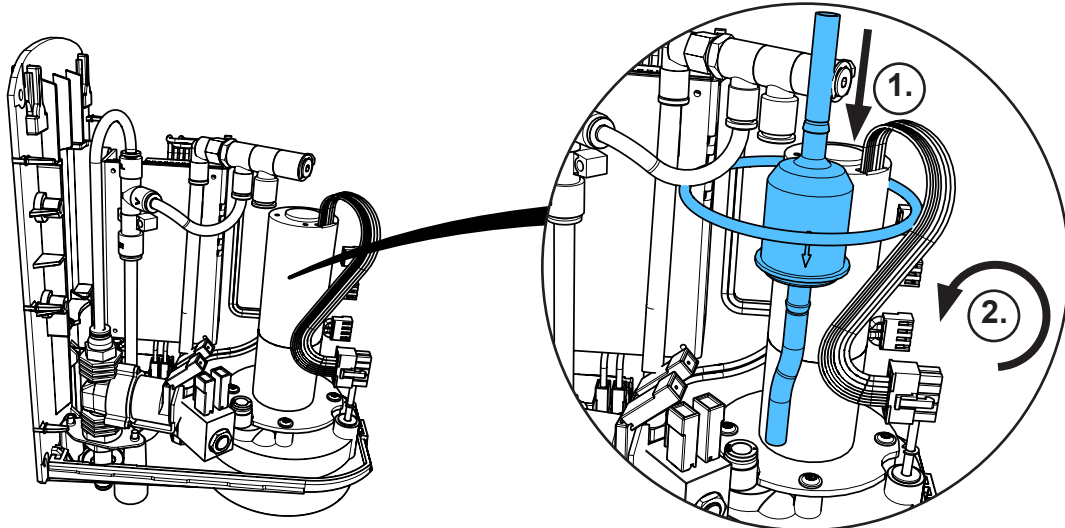


13

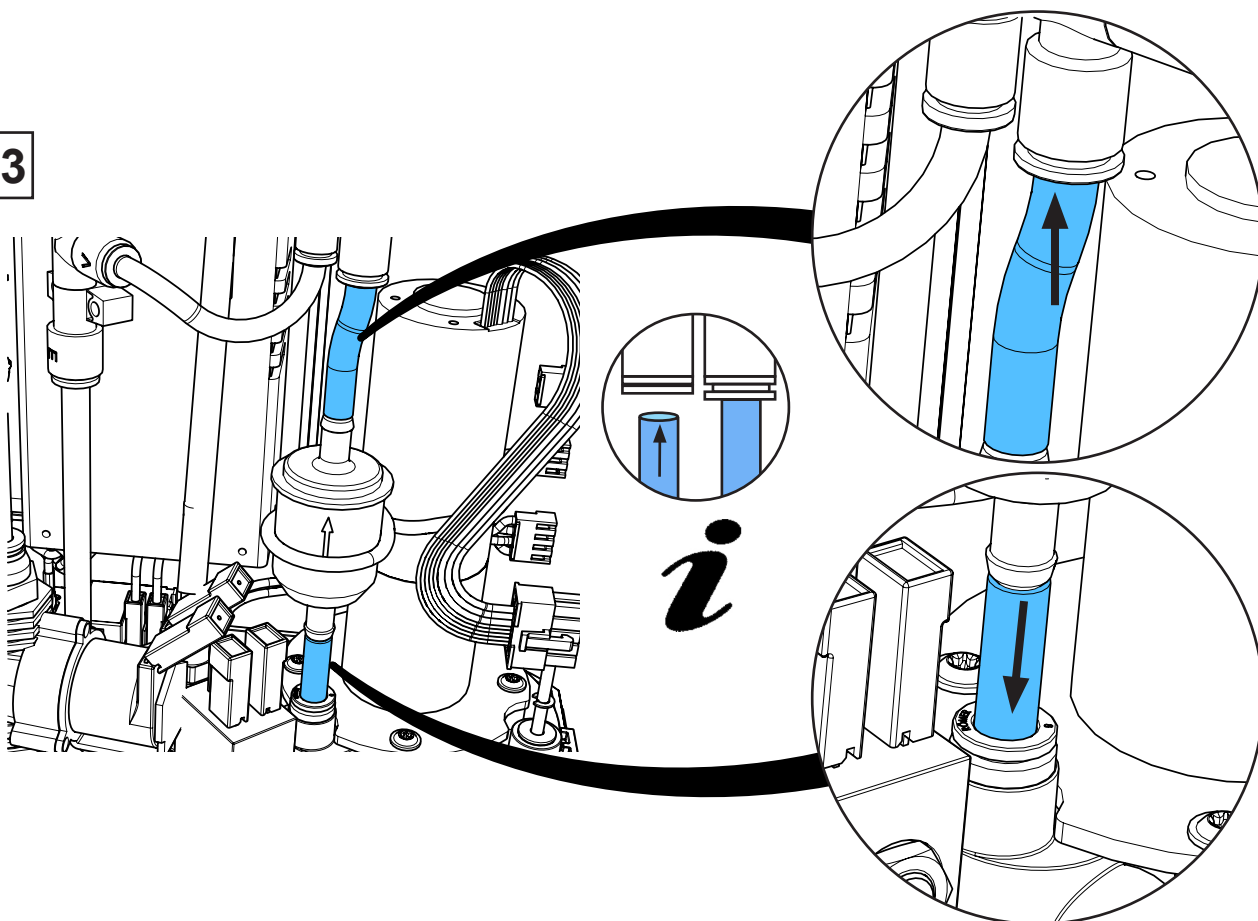





14

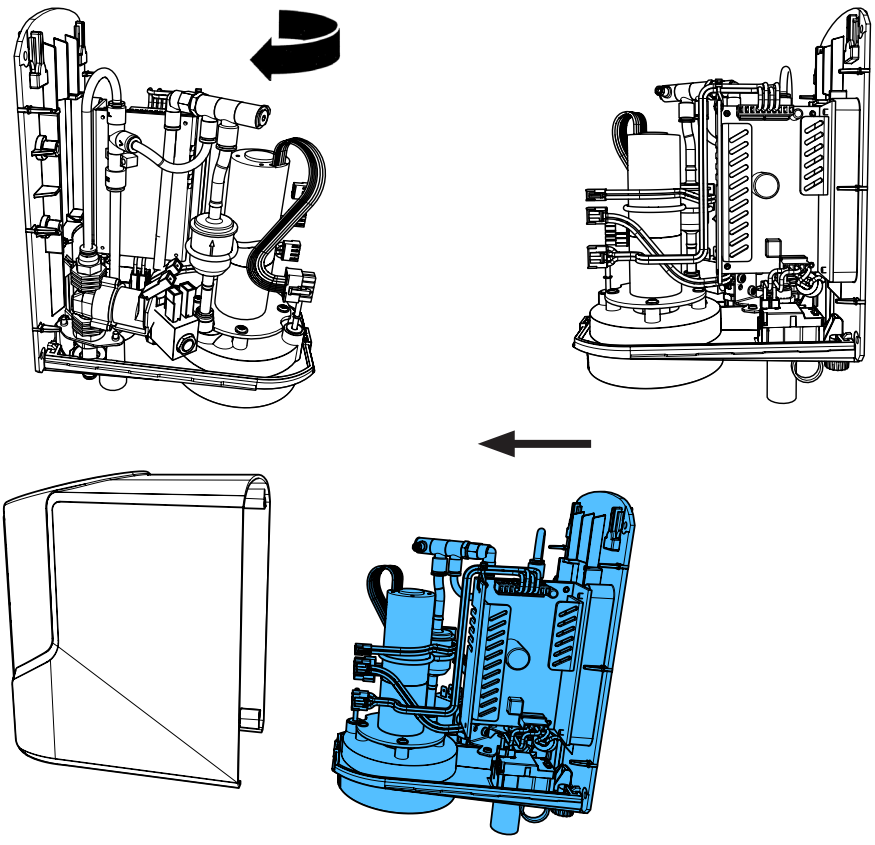


15

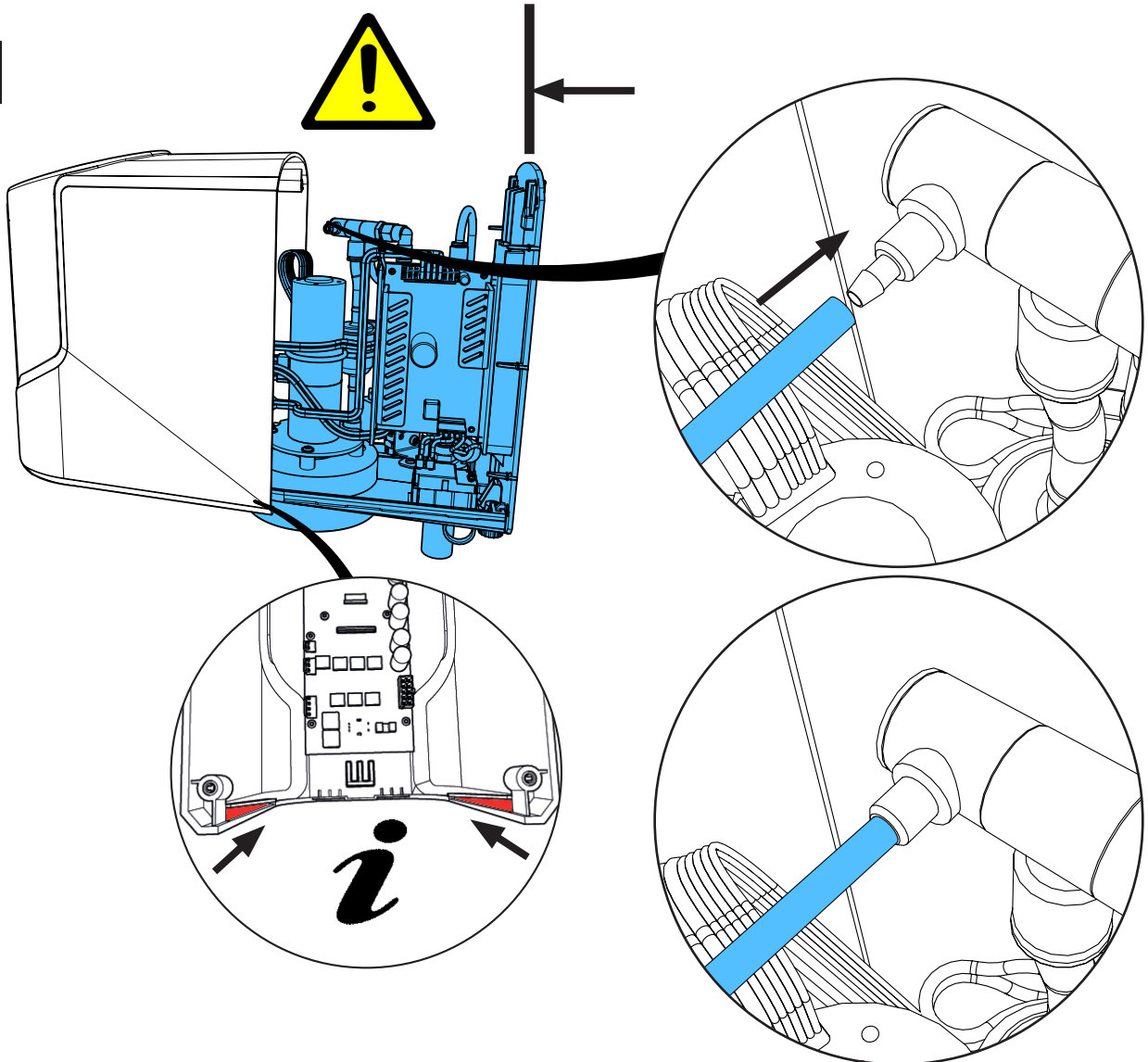


16

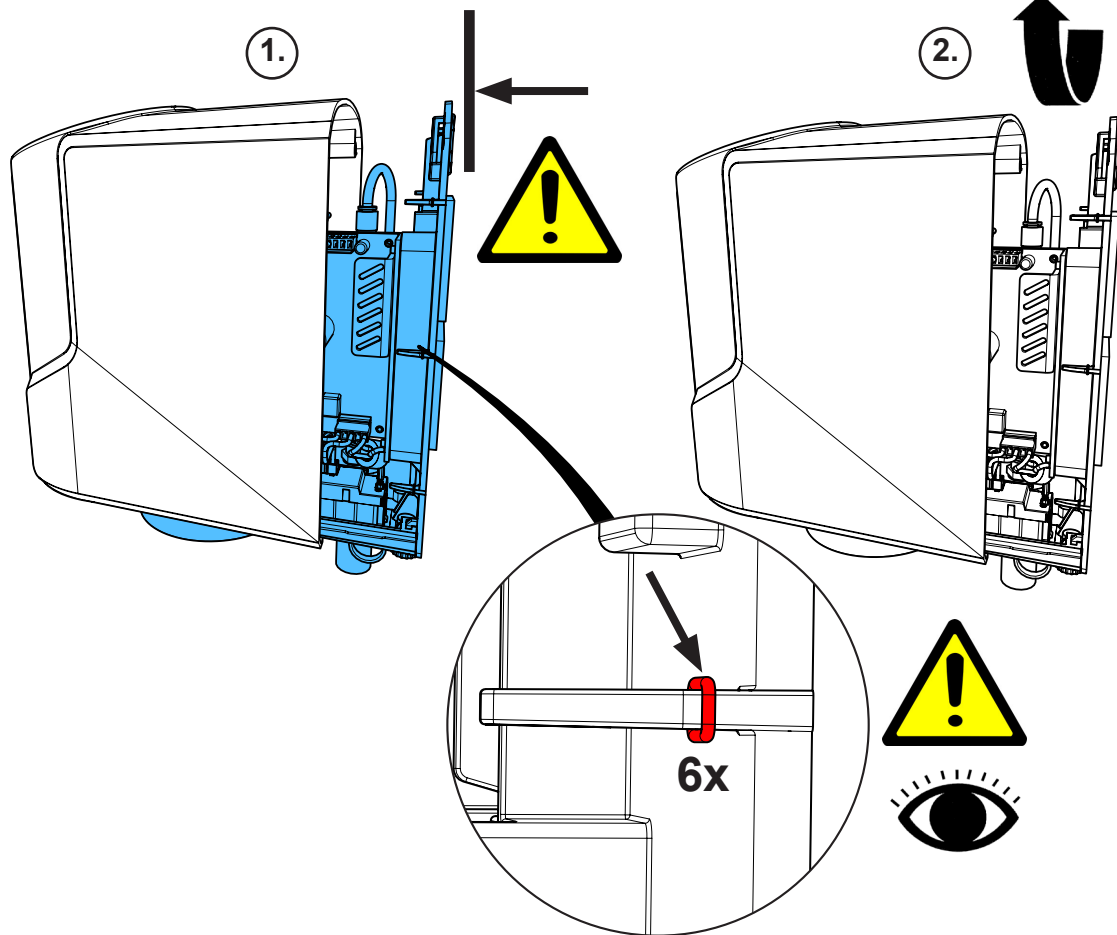


17

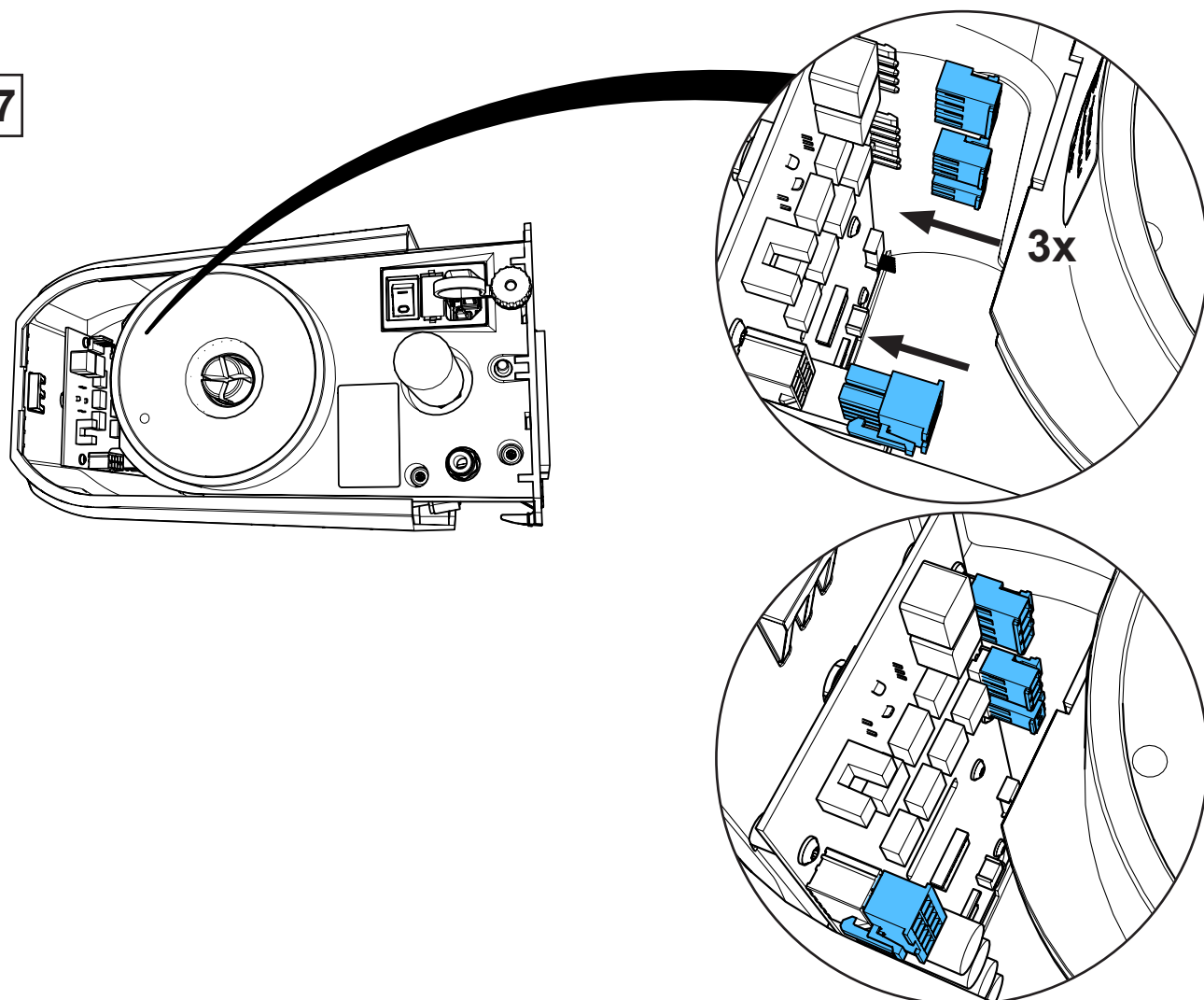


18

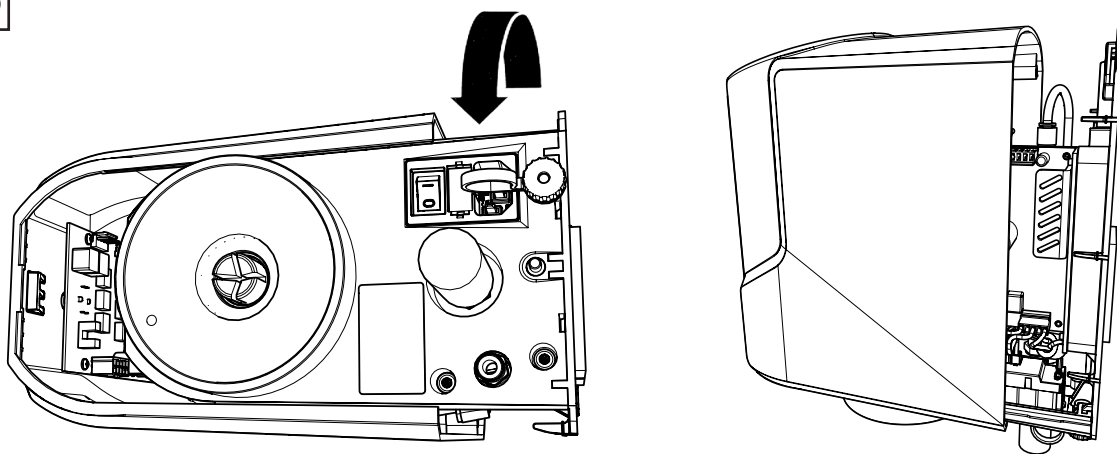

19

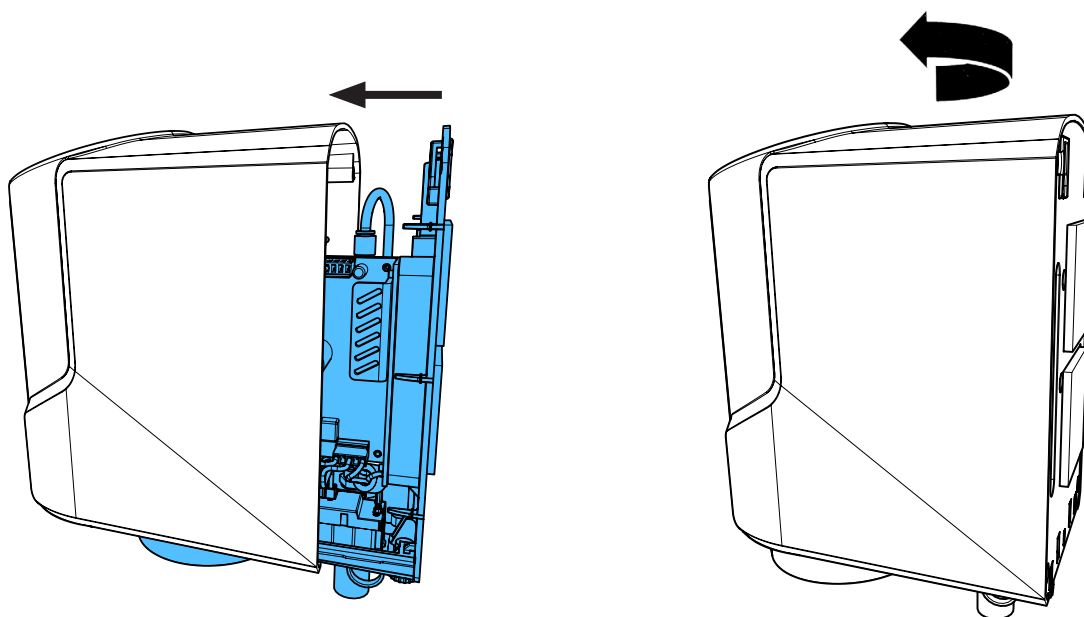




20

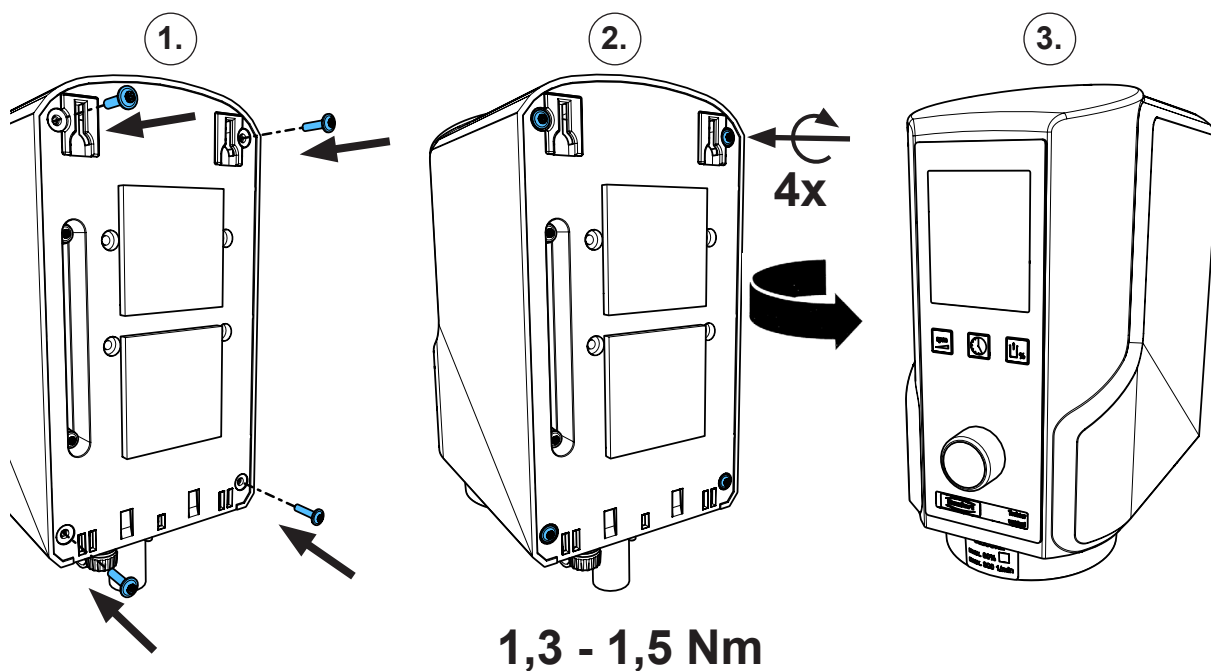




## 90003 4690


Schaltnetzteil • Power supply • Alimentation sur secteur • Alimentatore di rete  
Fuente de alimentación • Сетевой переключатель

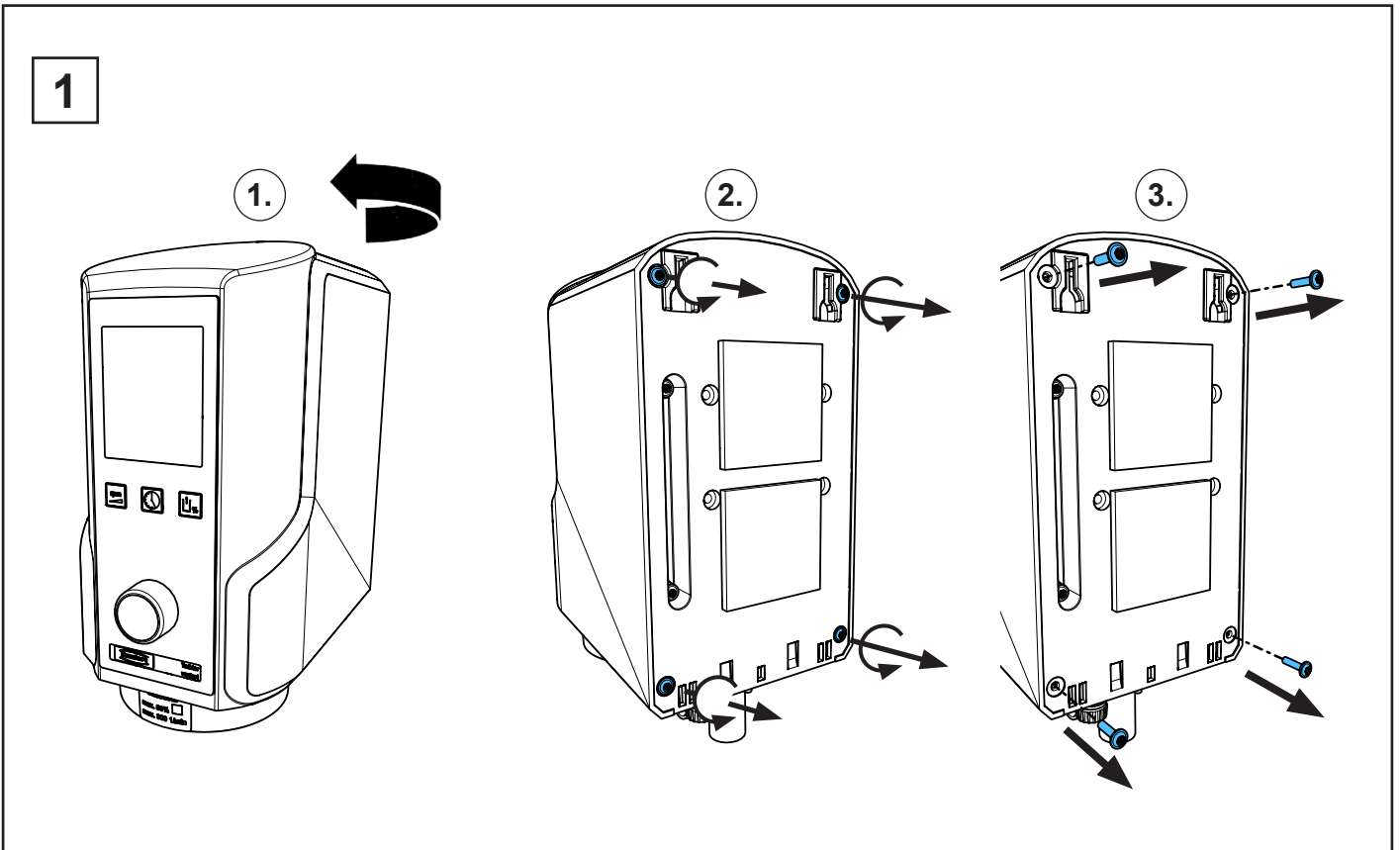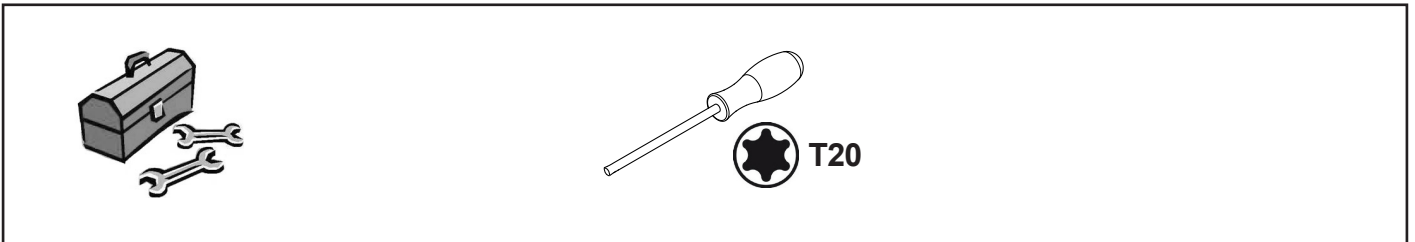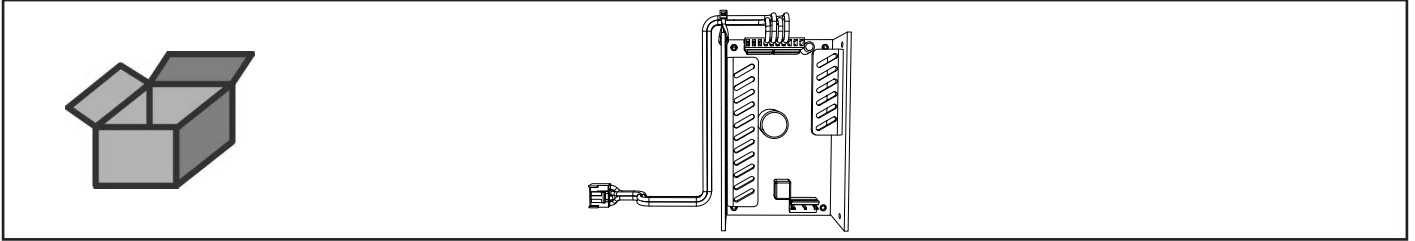


2

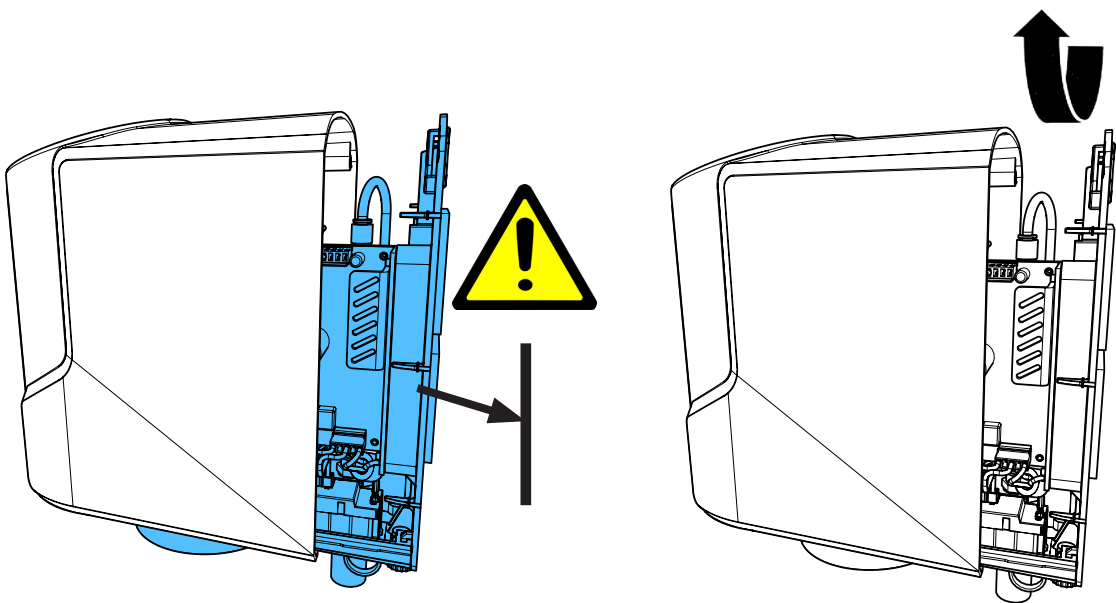


3

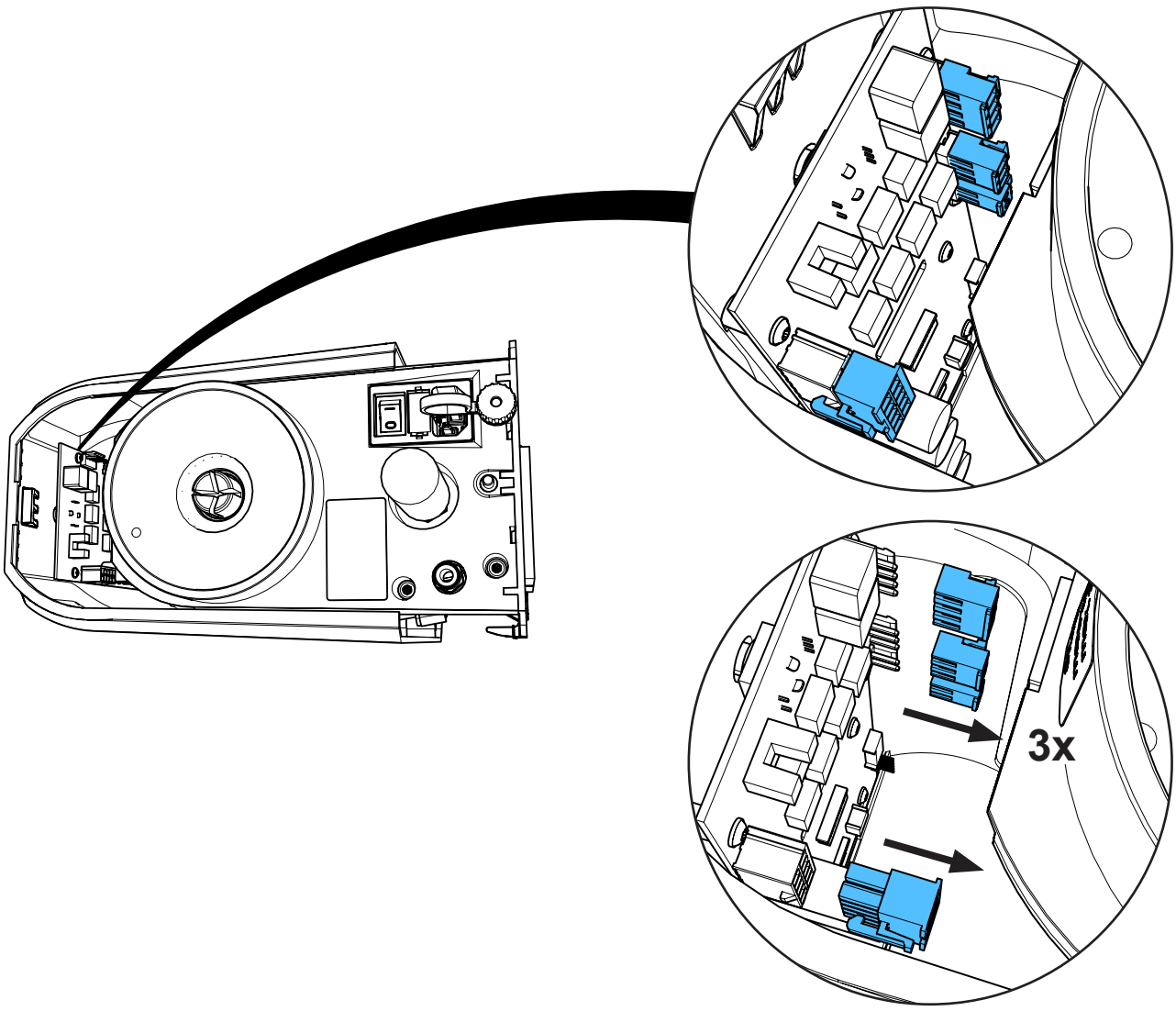


4

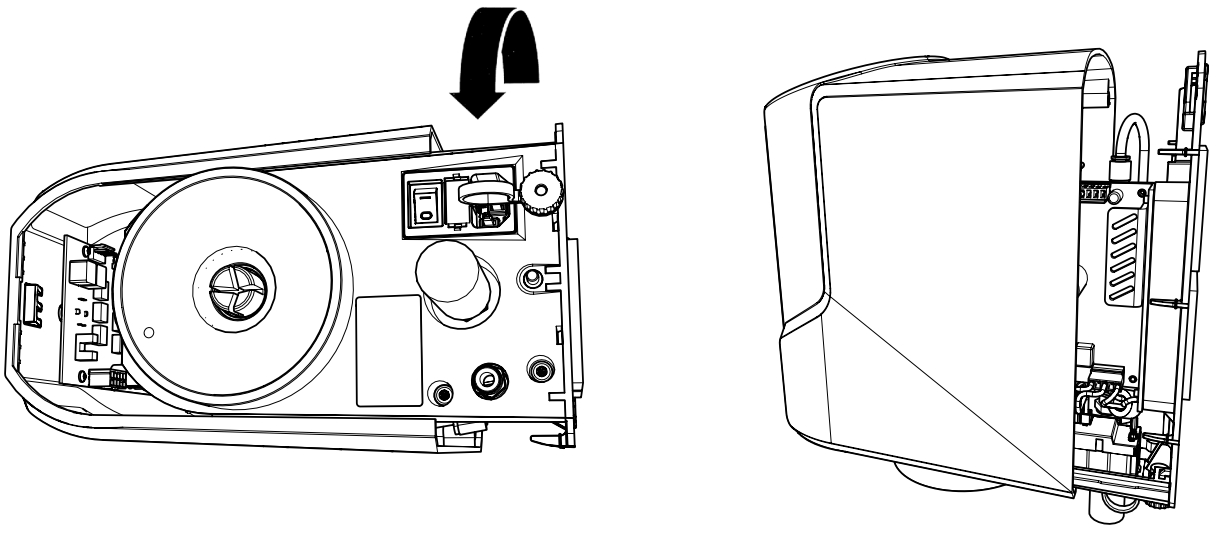




5

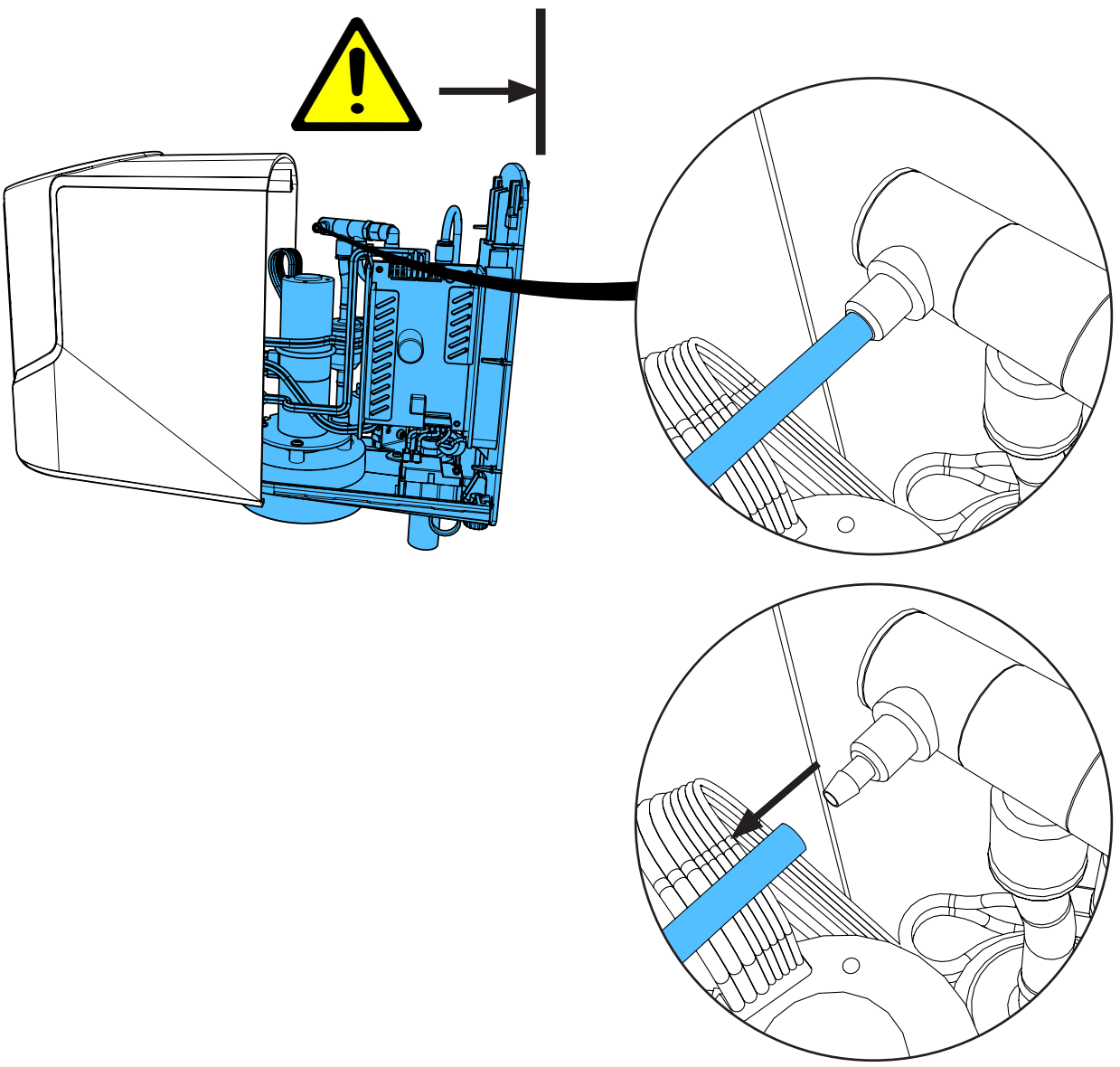



6

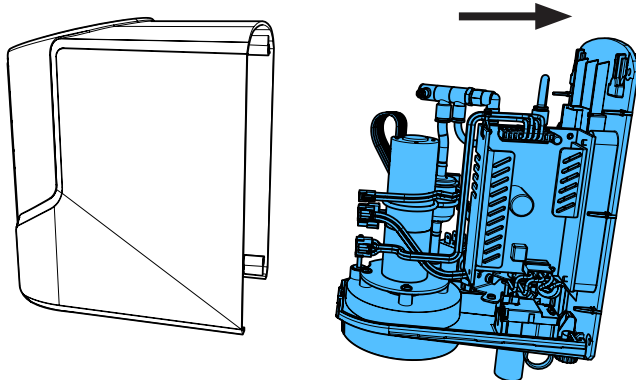

7

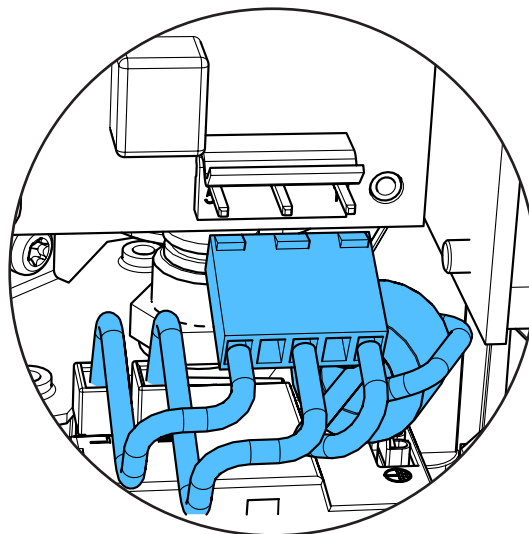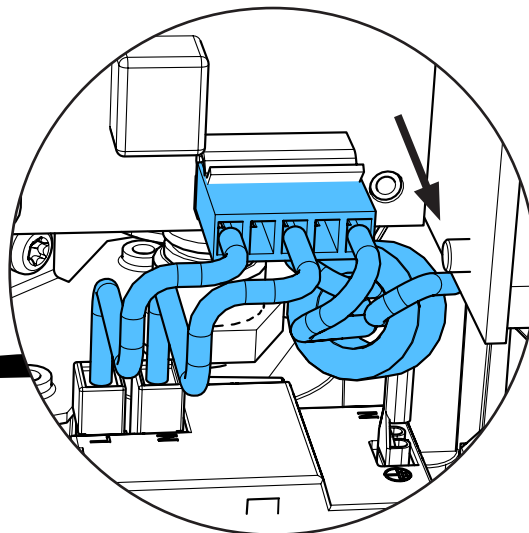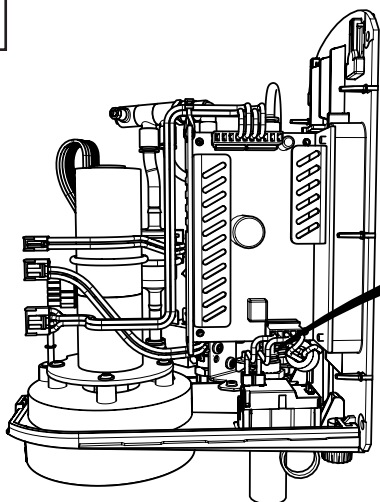


8

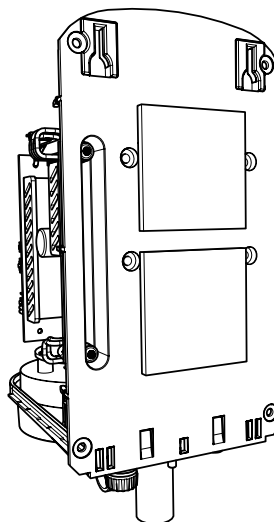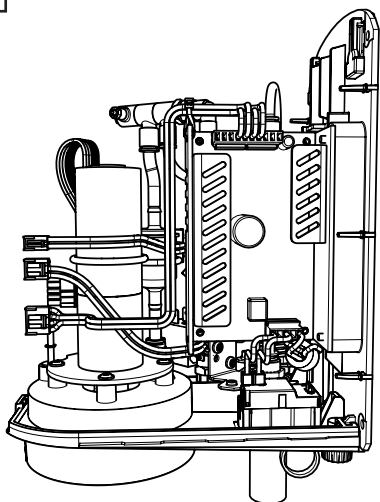





9

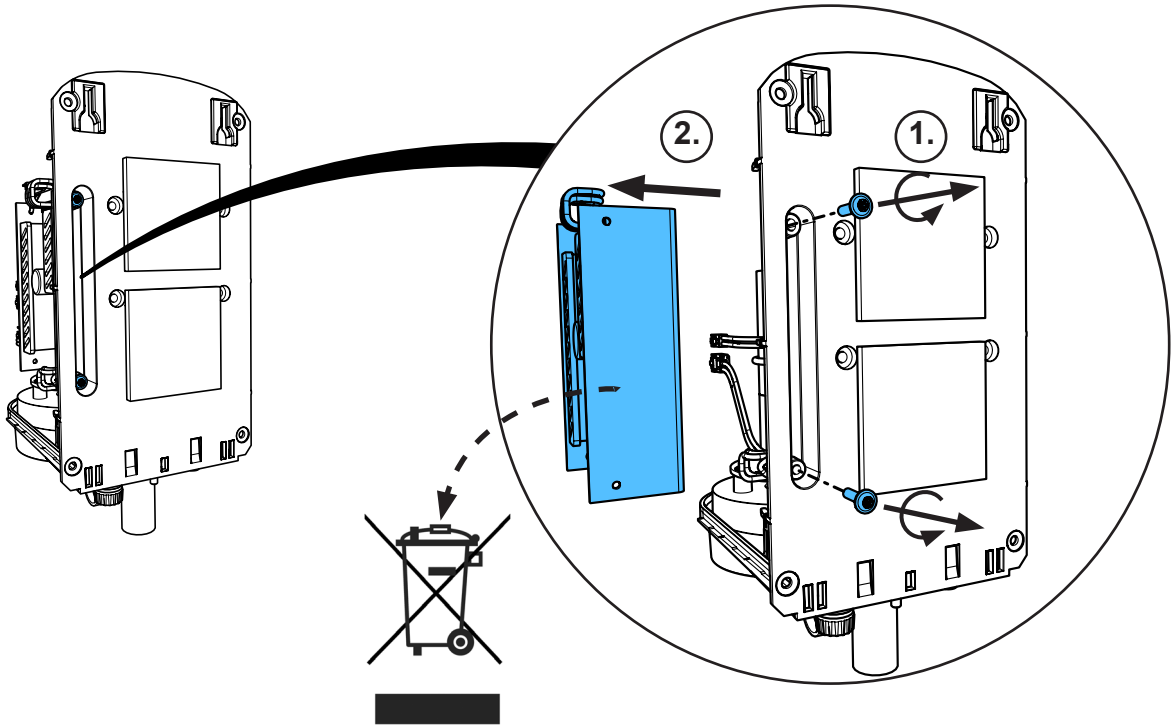


10

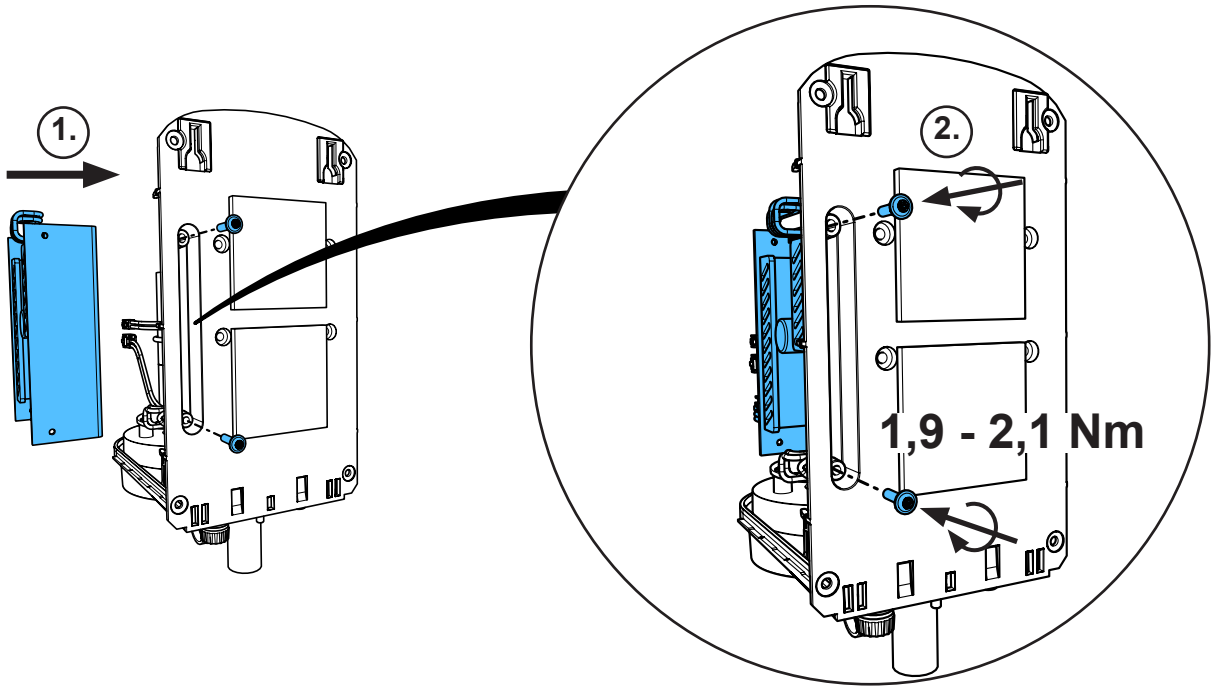


11

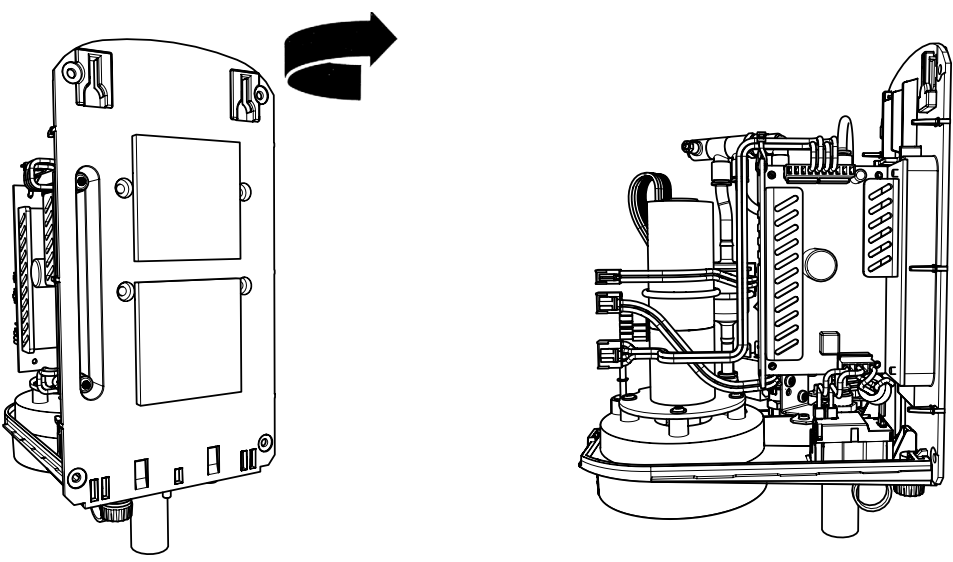


12

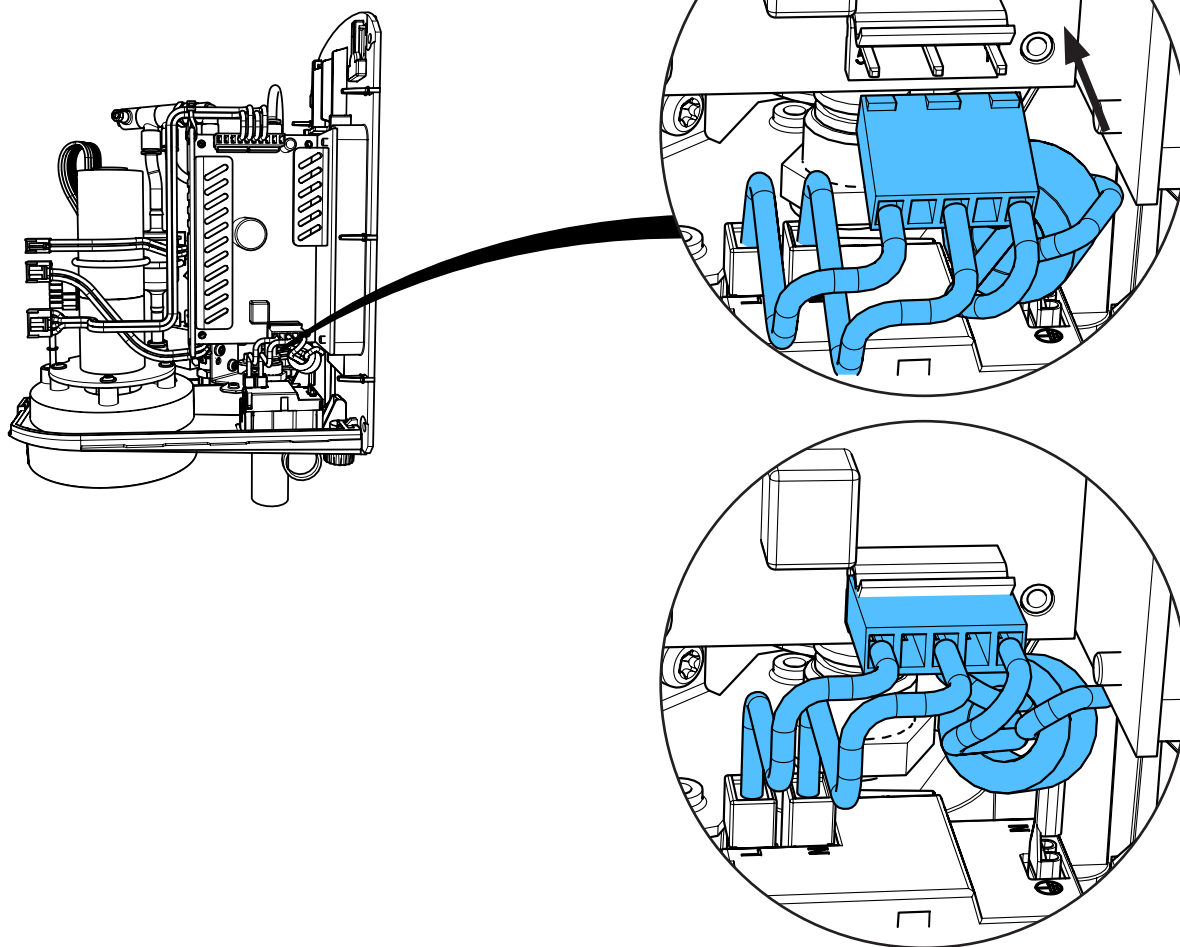


13

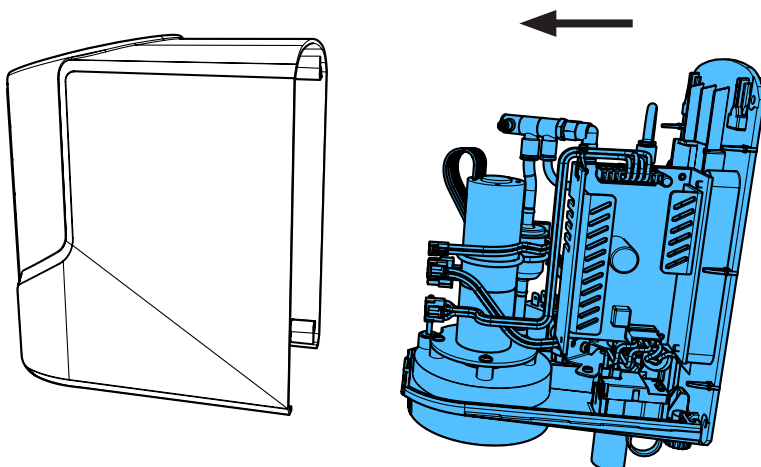


14

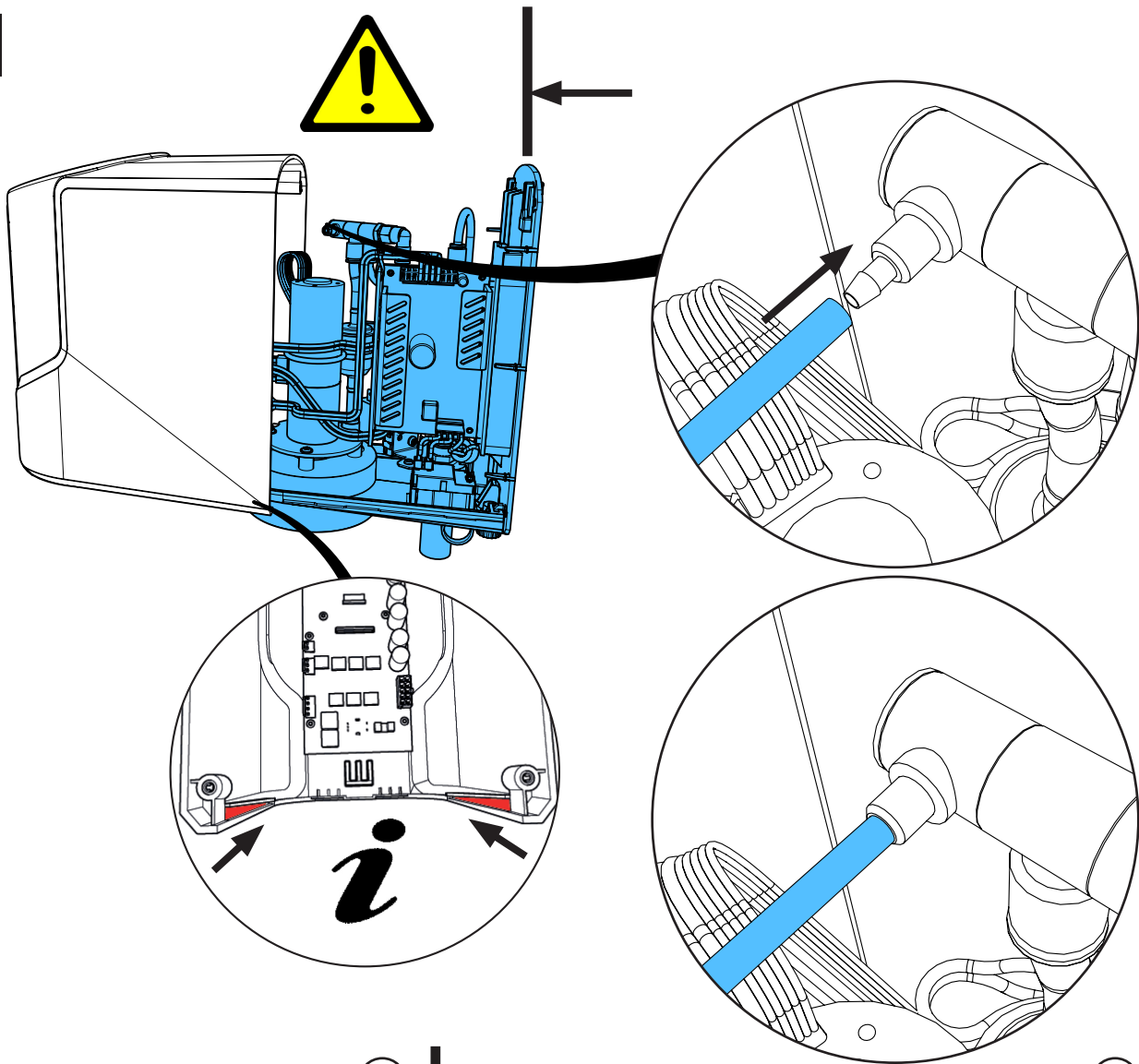


15

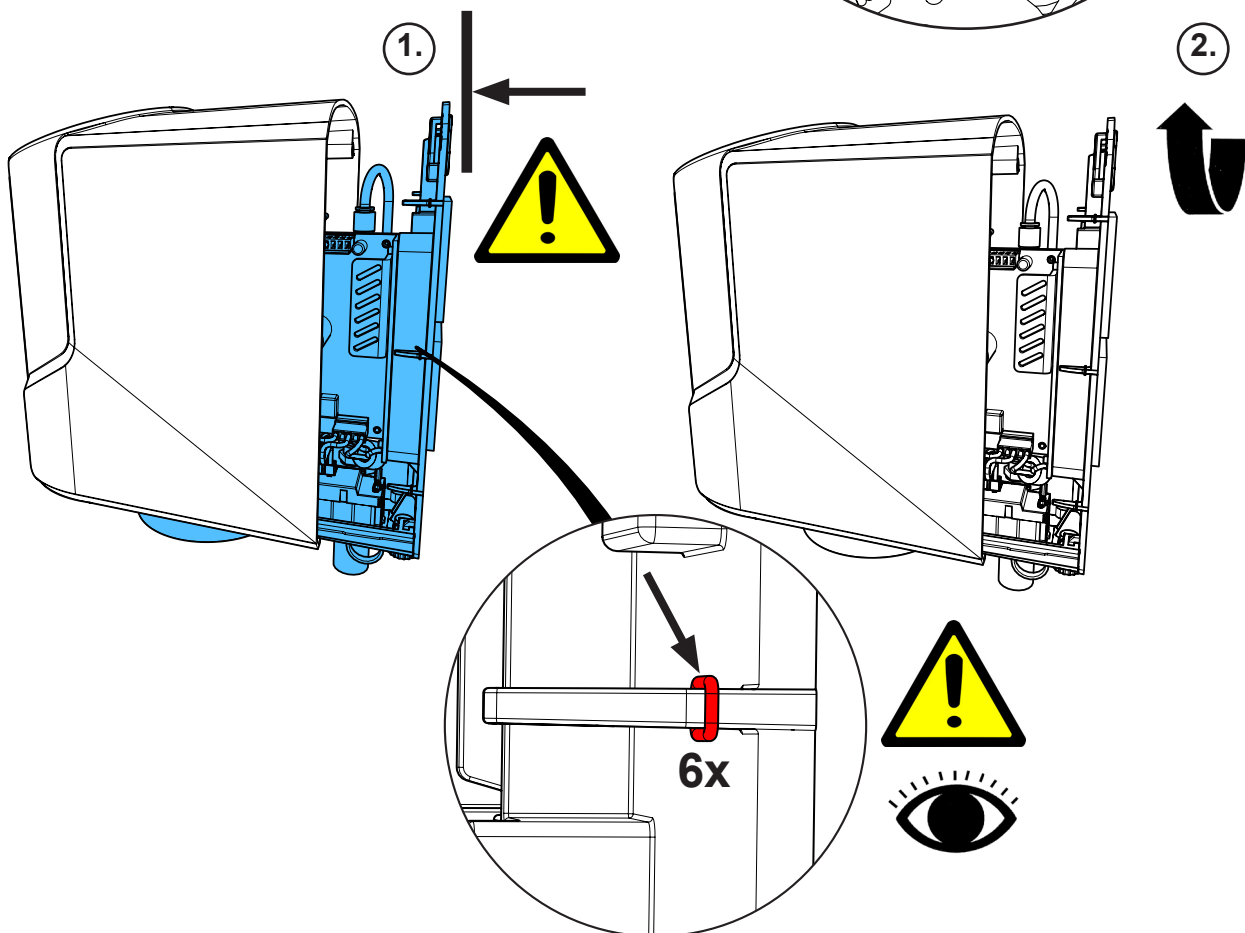


16

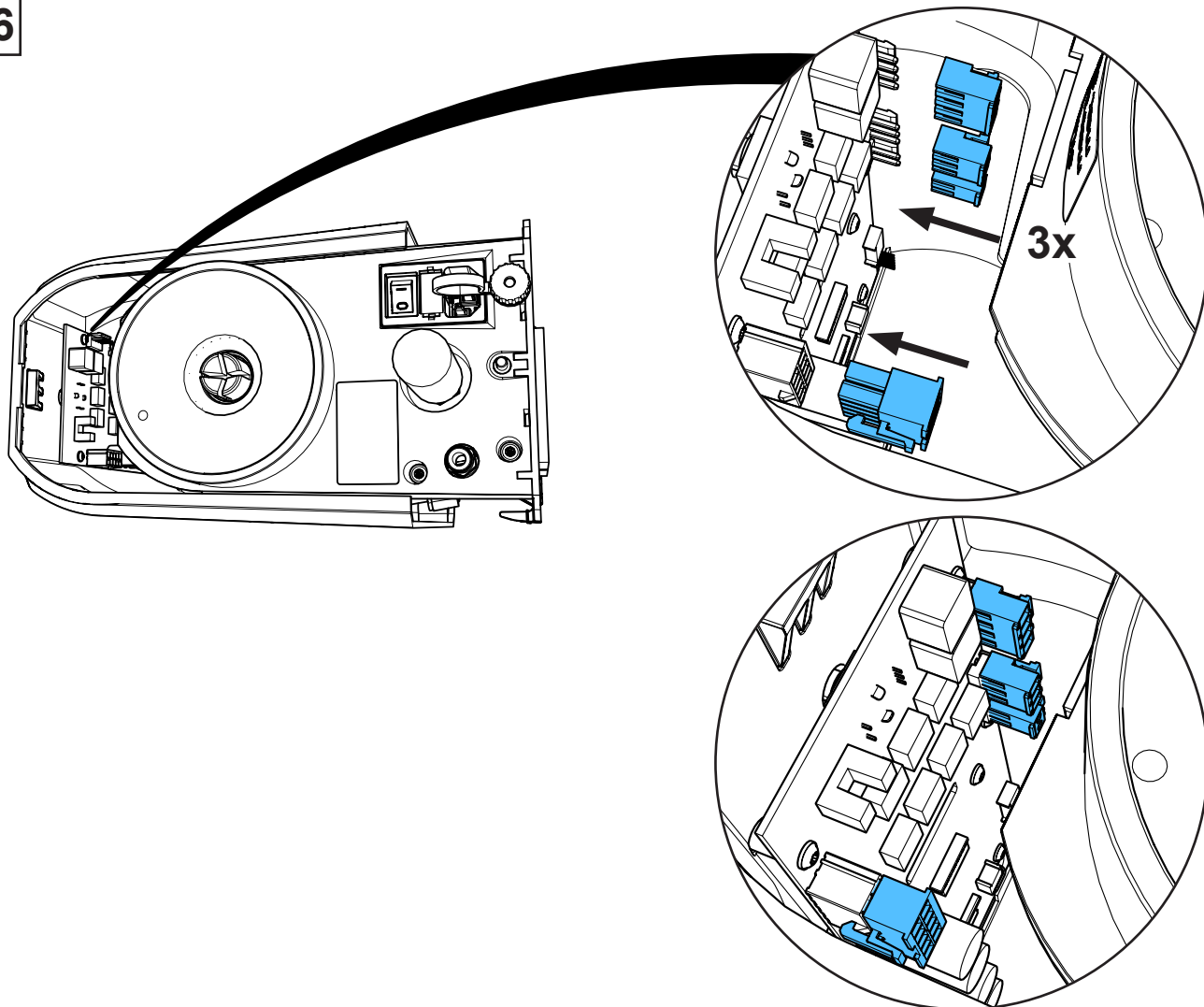


17

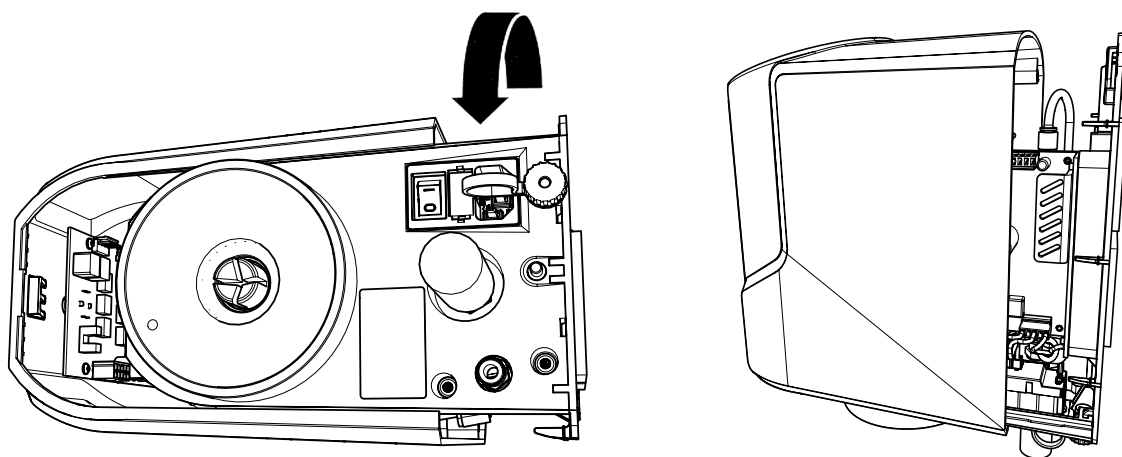

18

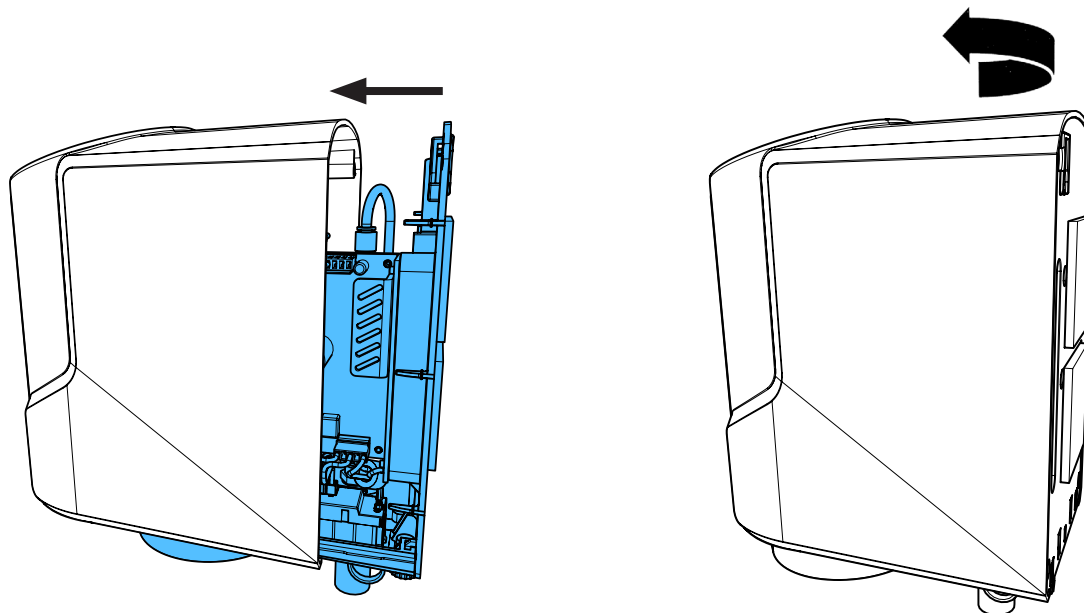




19

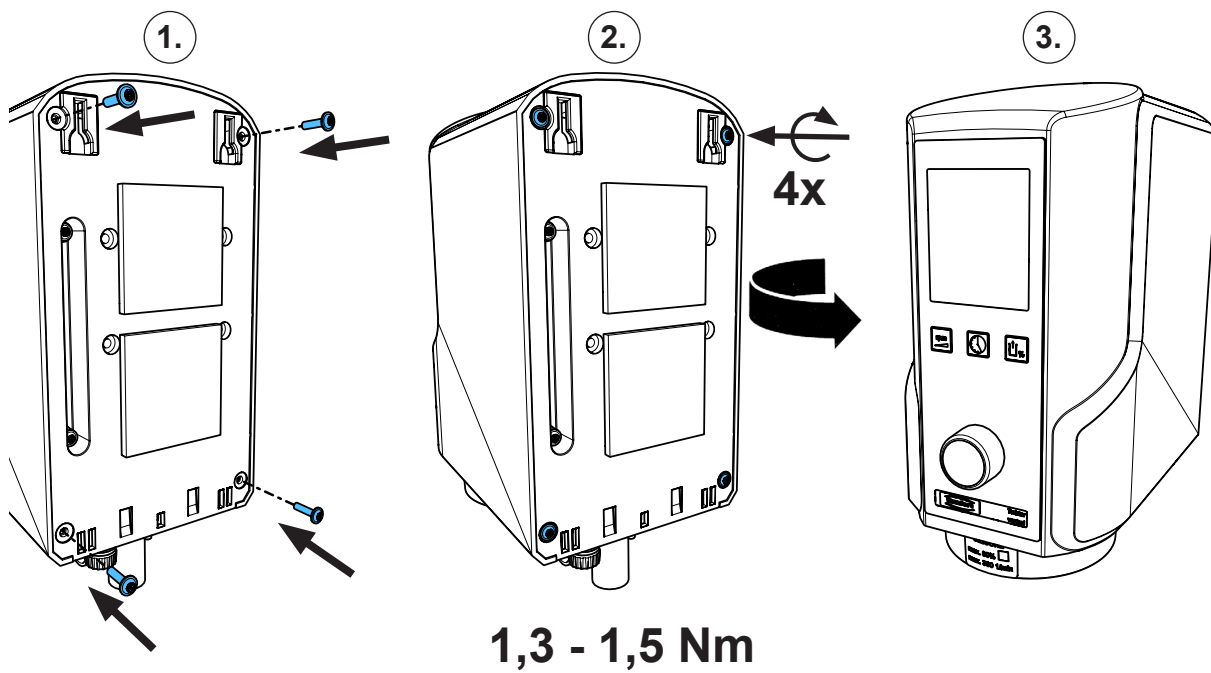




**90003 5989**


Netzschalter • Mains switch • Interrupteur de réseau • Interruttore di rete  
Interruptor de red • Сетевой выключатель

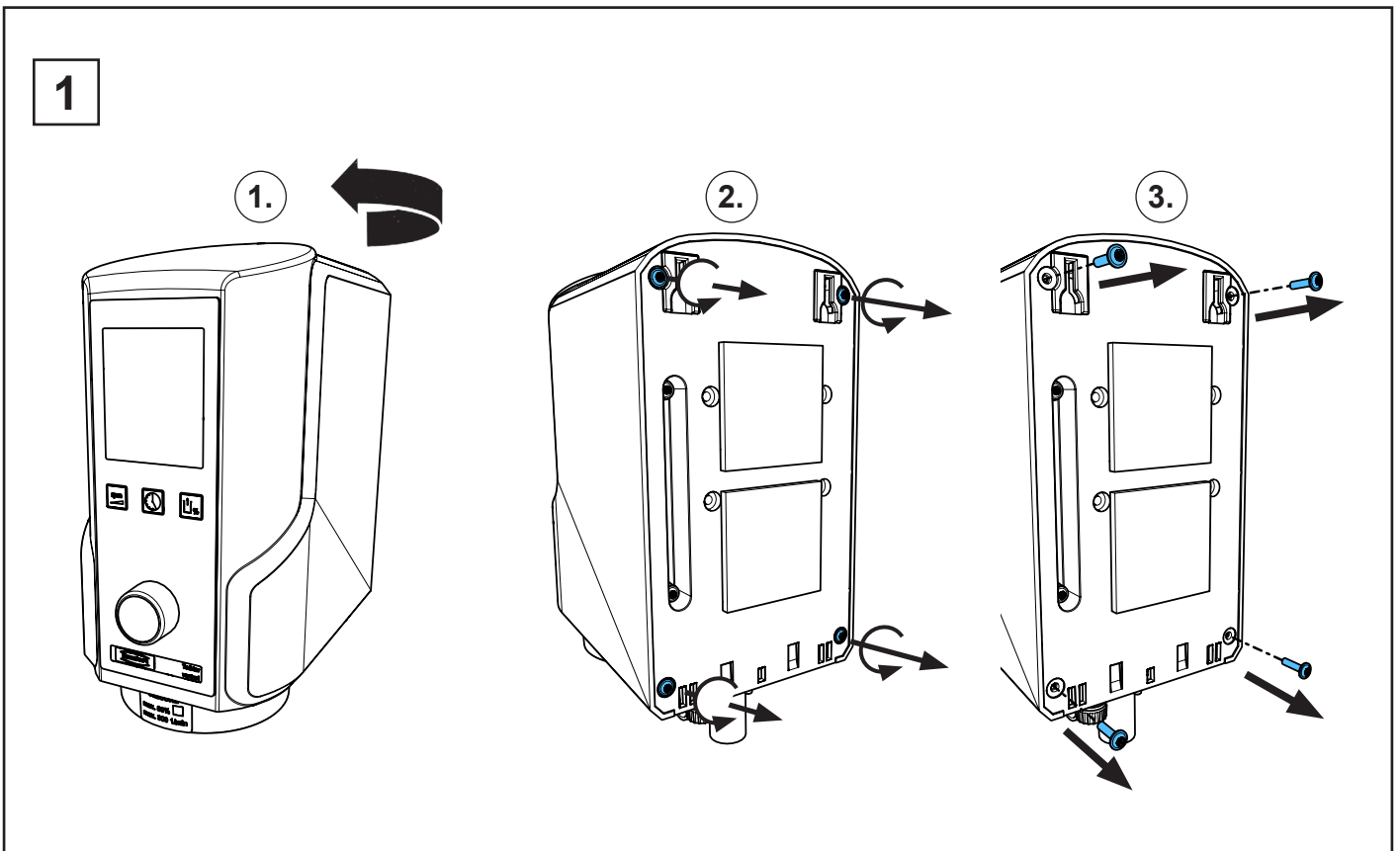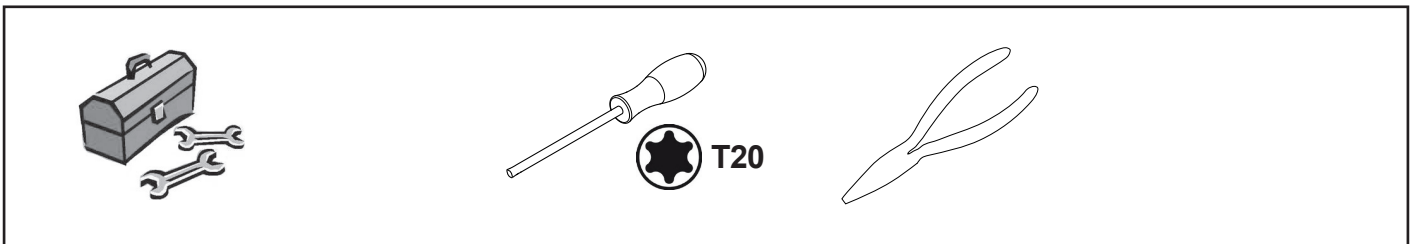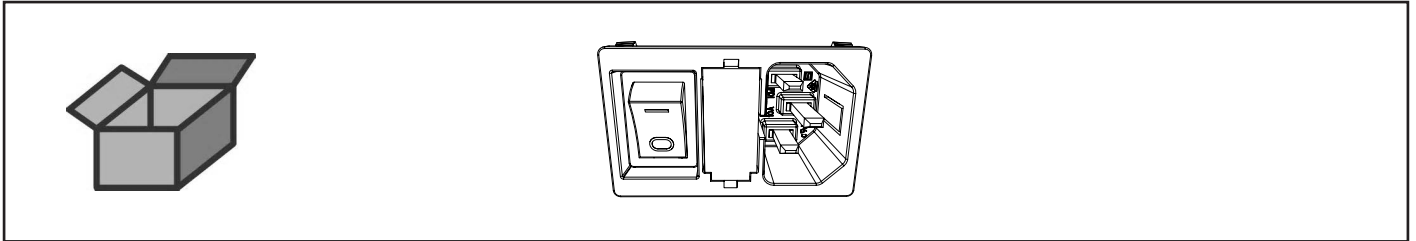

2

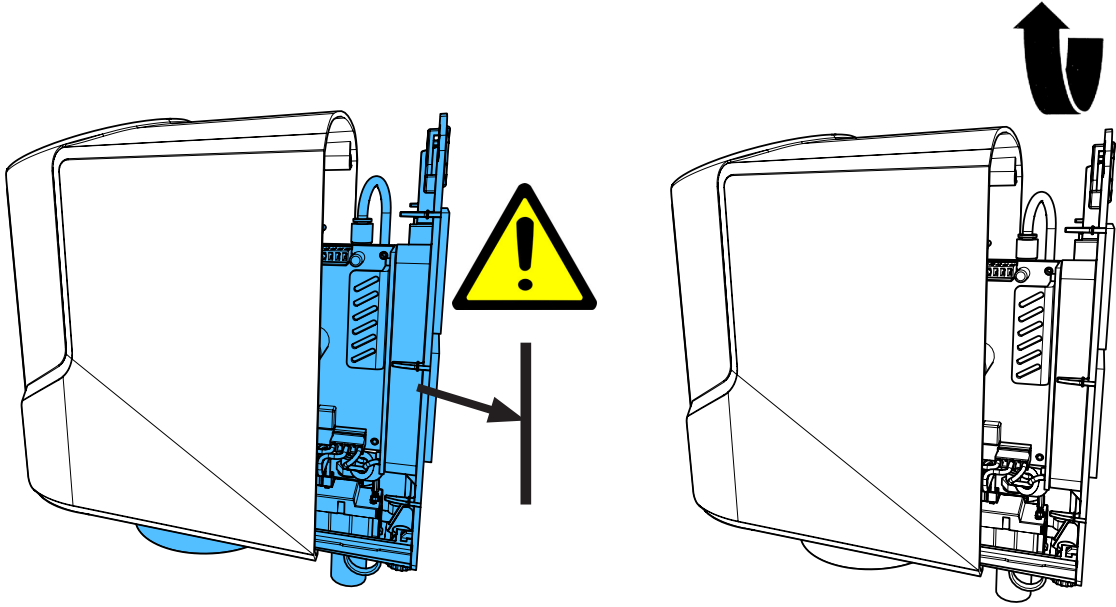


3

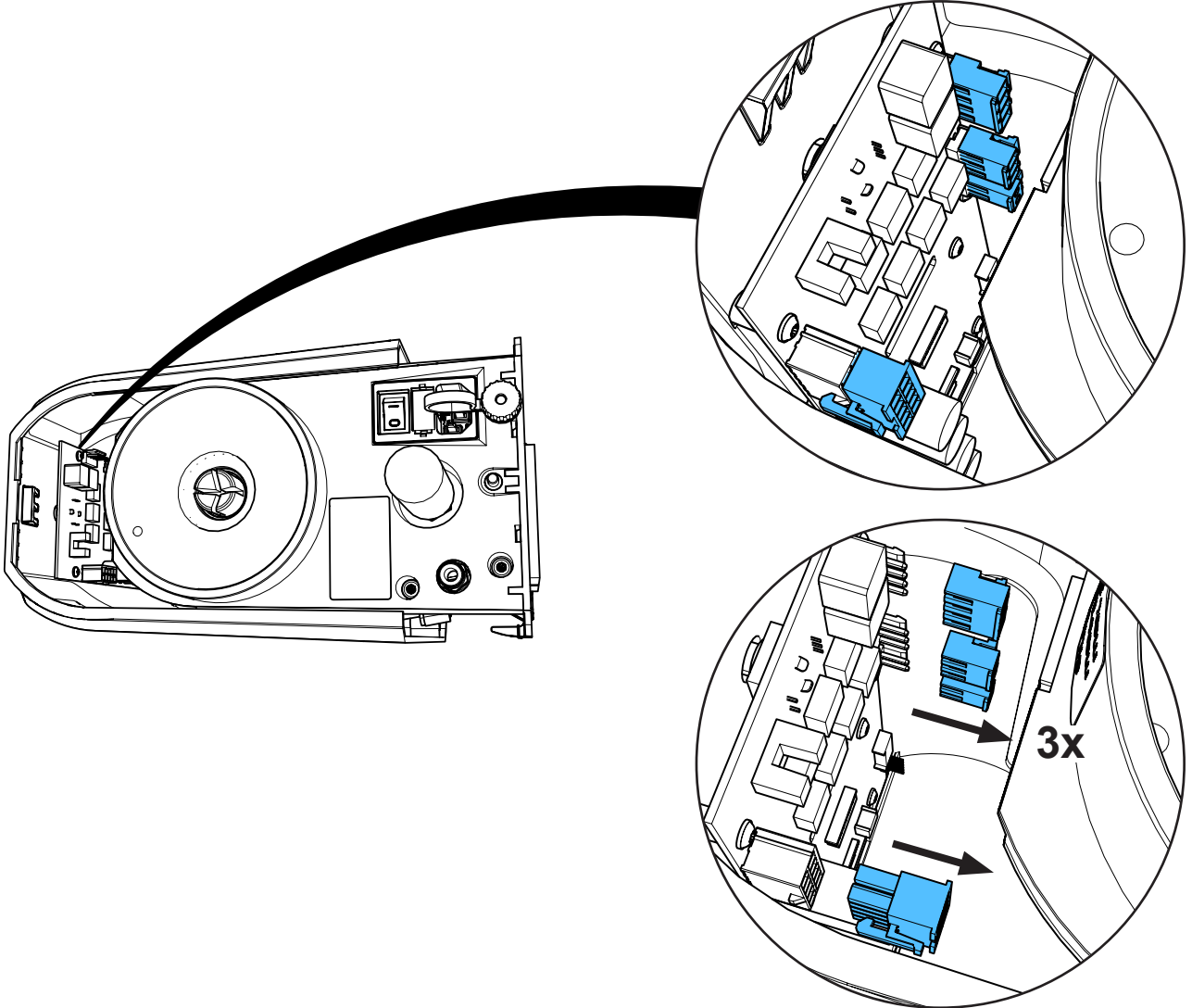





4

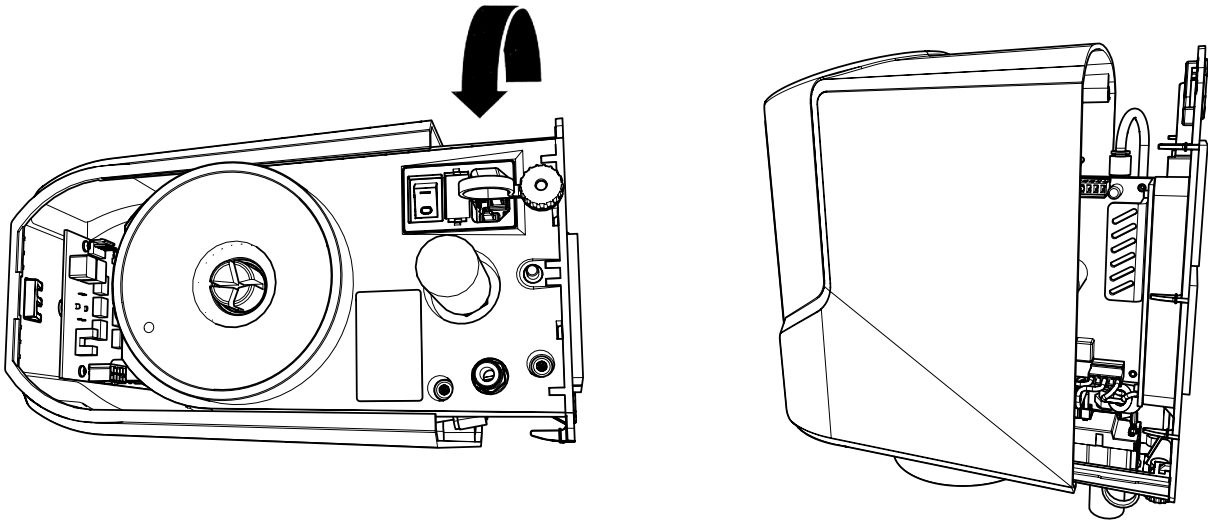


5

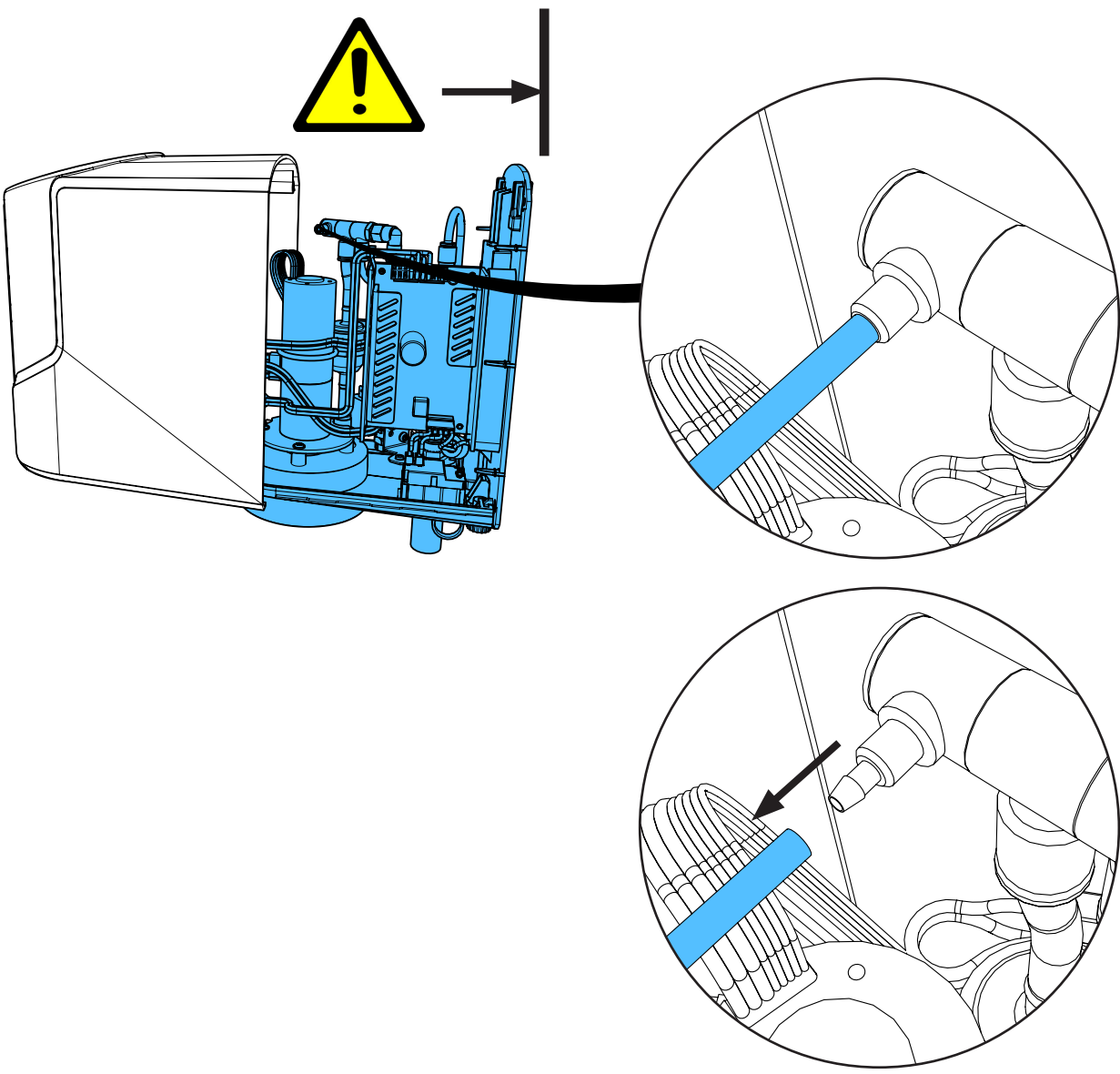


6

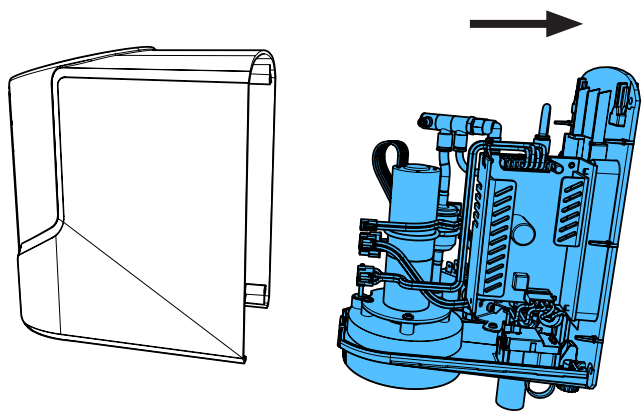


7

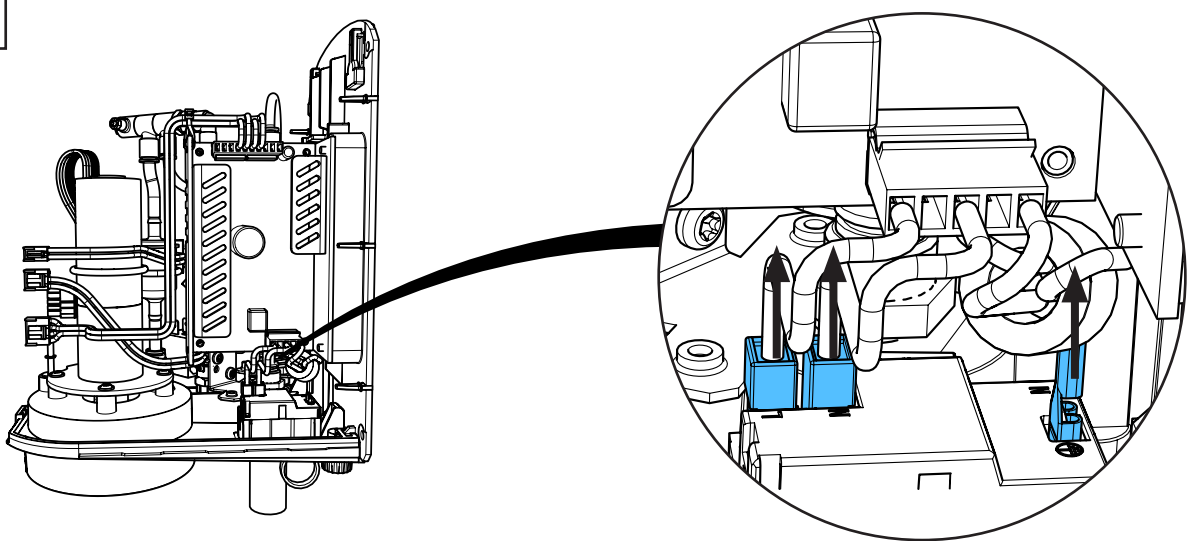


8

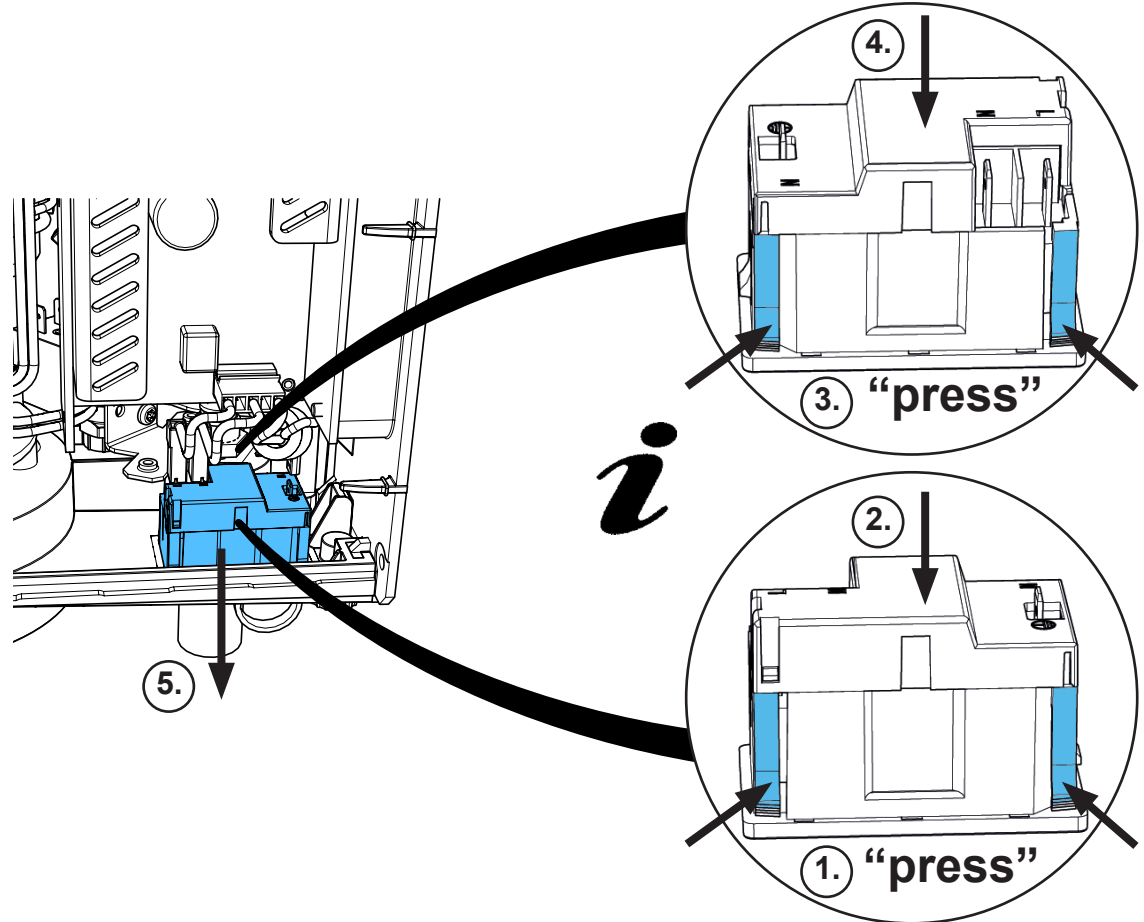


9

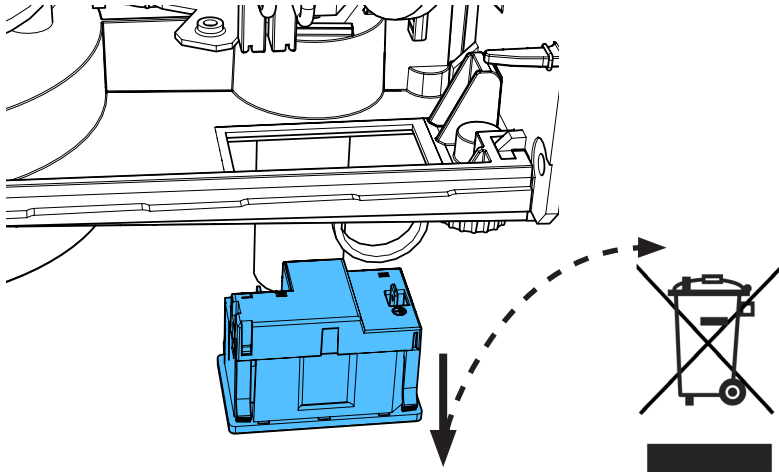


10

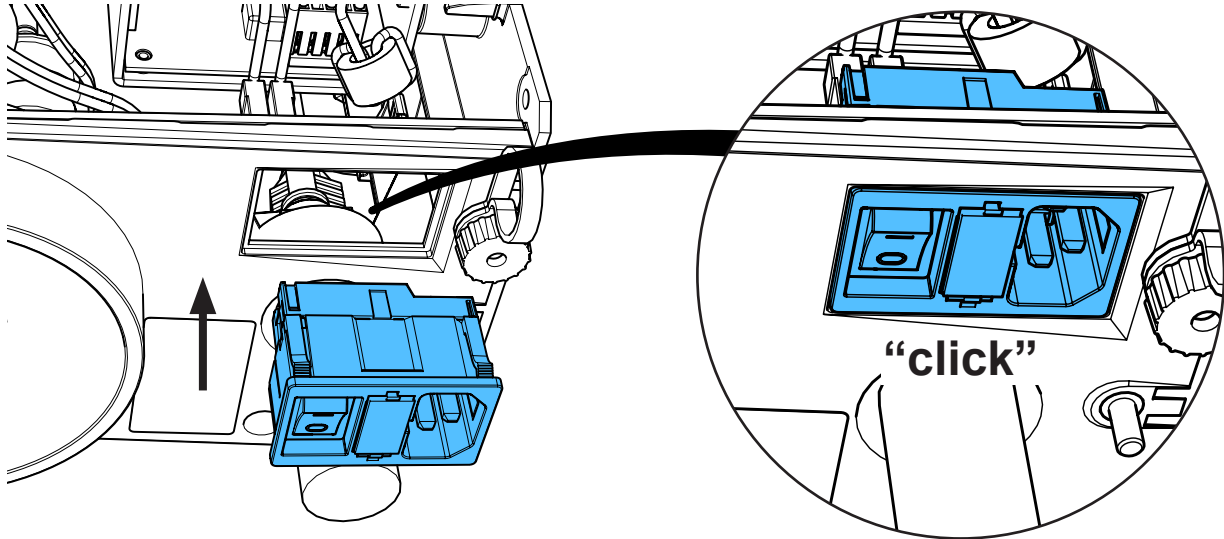


11

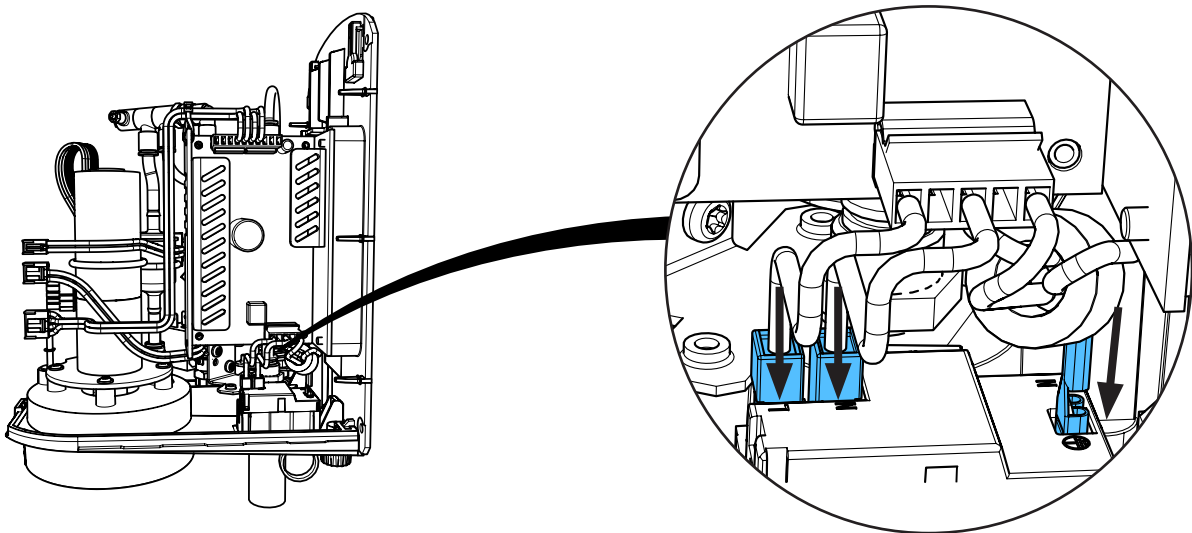


12

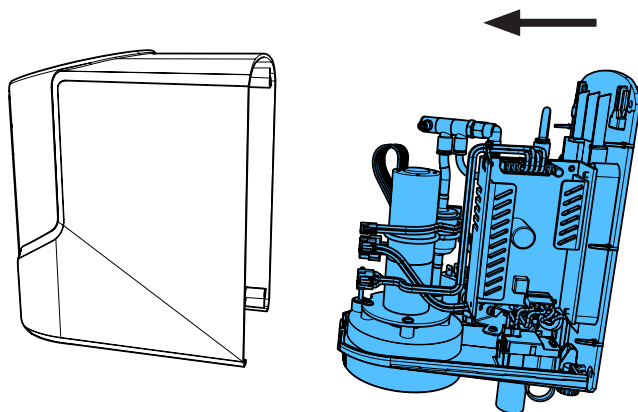


13

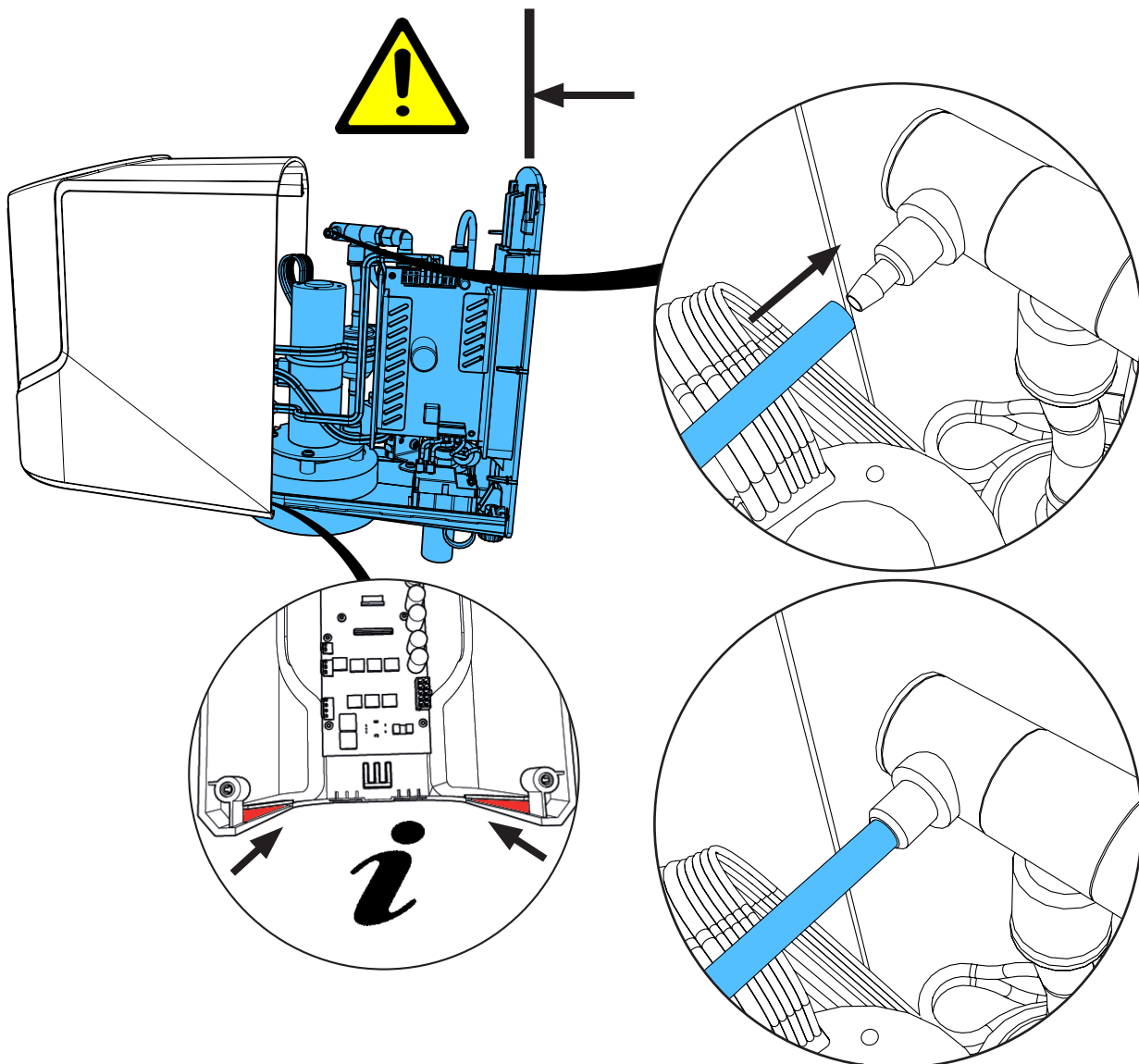


14

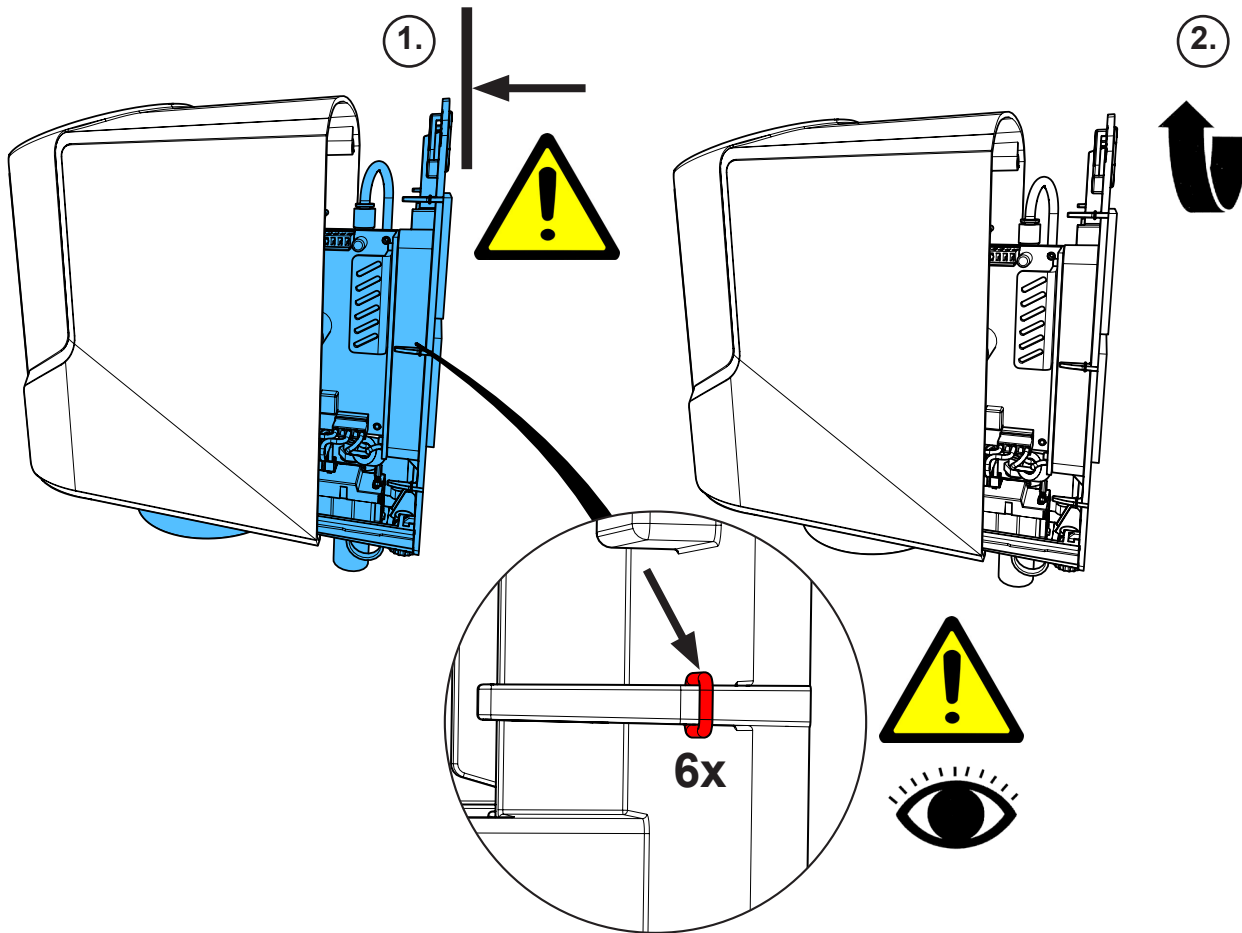


15

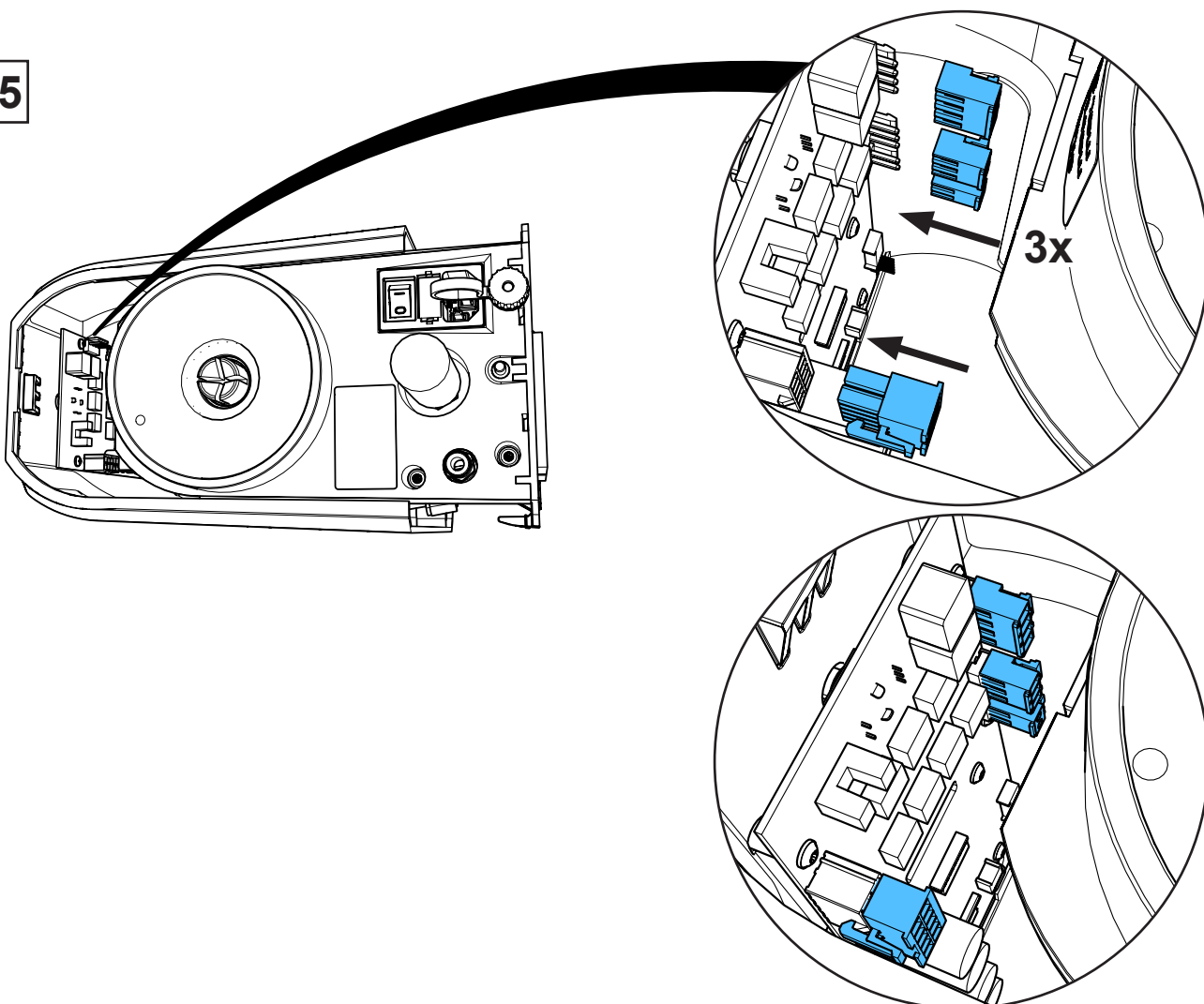


16

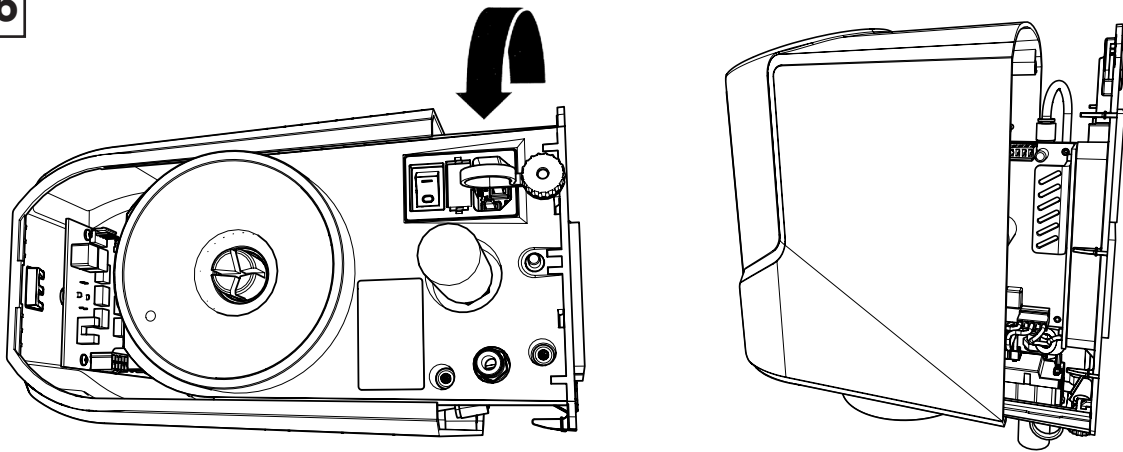

17



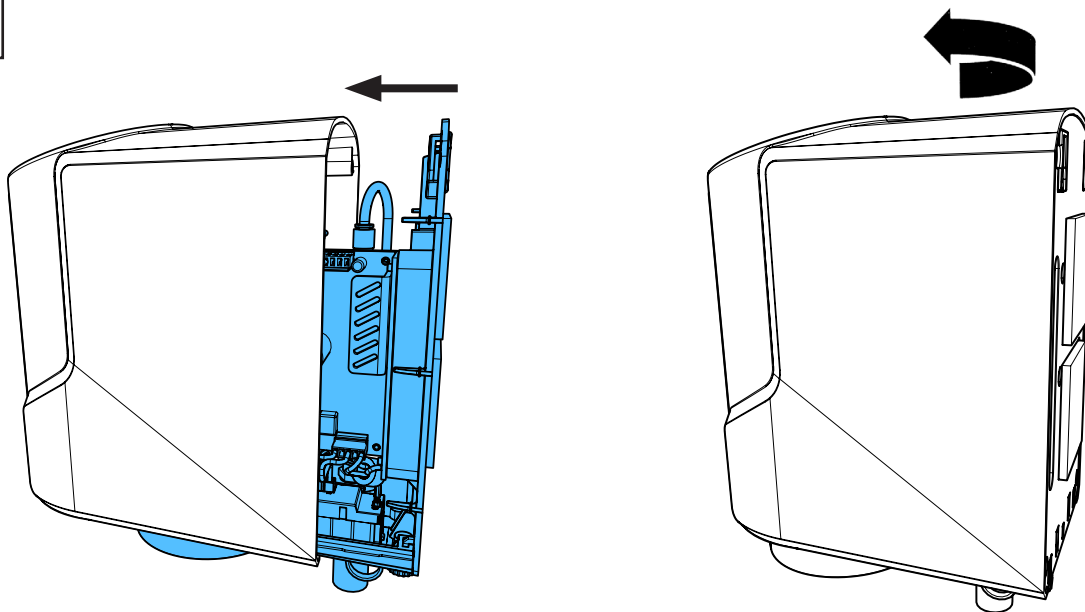


18

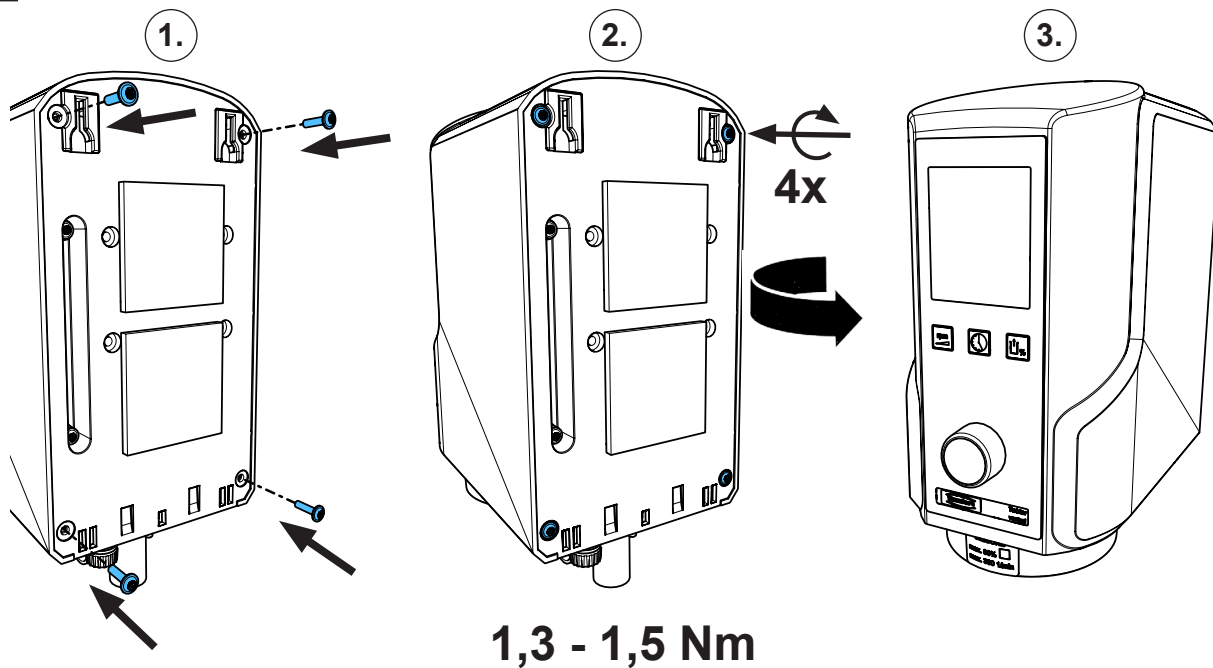


## 90002 1527


Magnetventil • Solenoid valve • Electrovanne • Valvola elettromagnetica  
Válvula magnética • Электромагнитный клапан

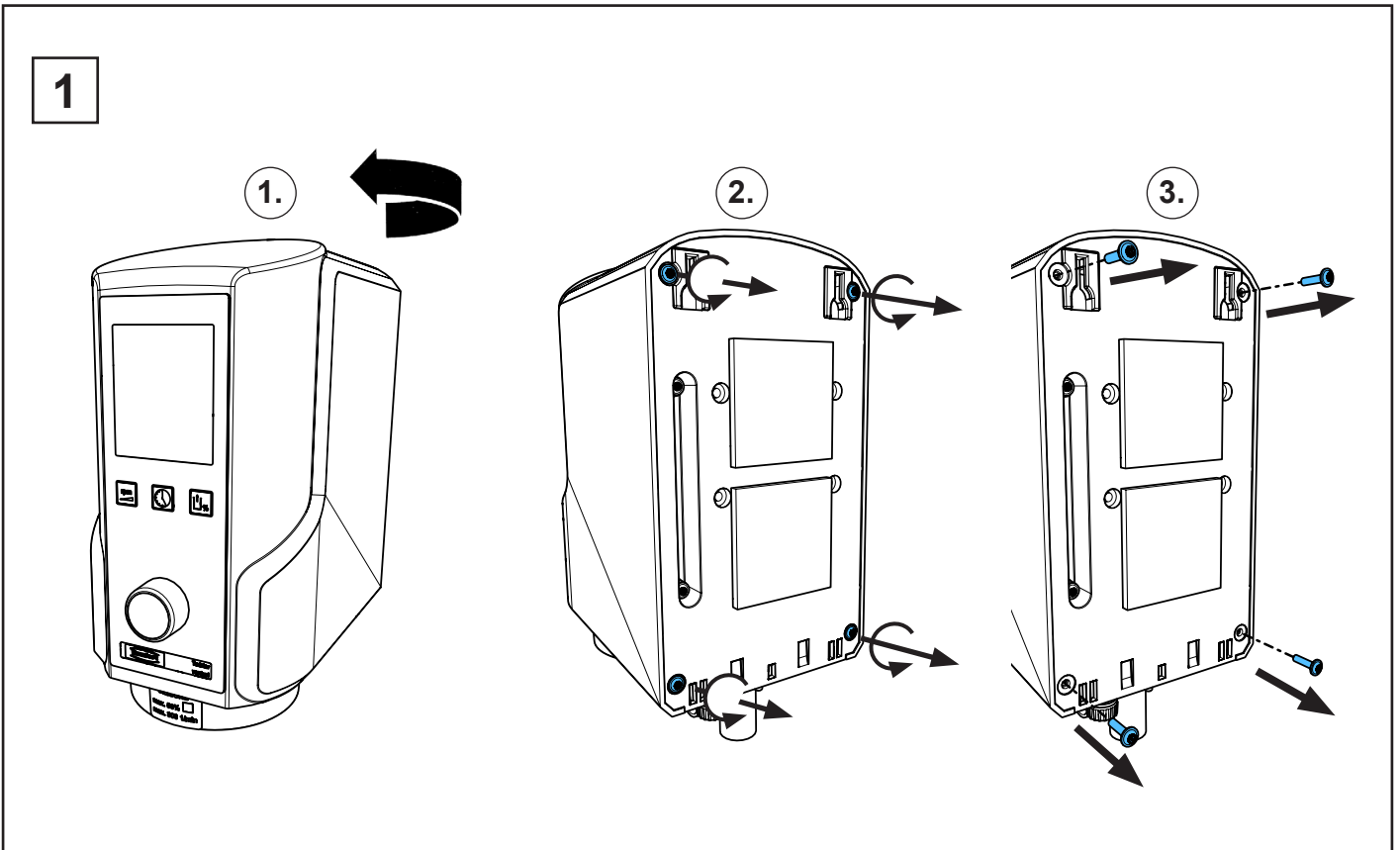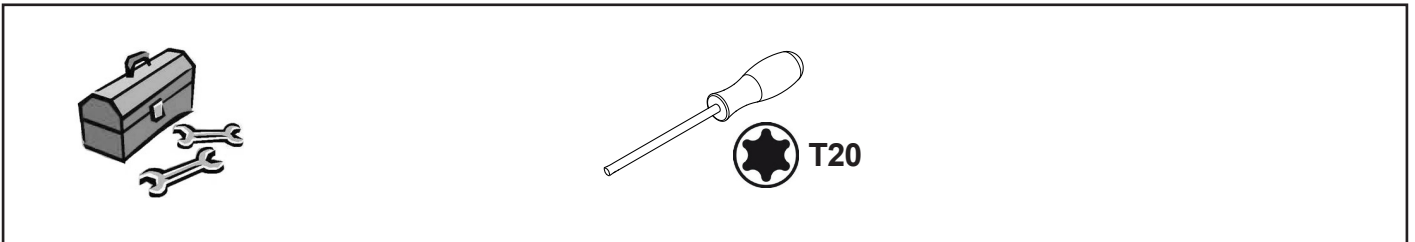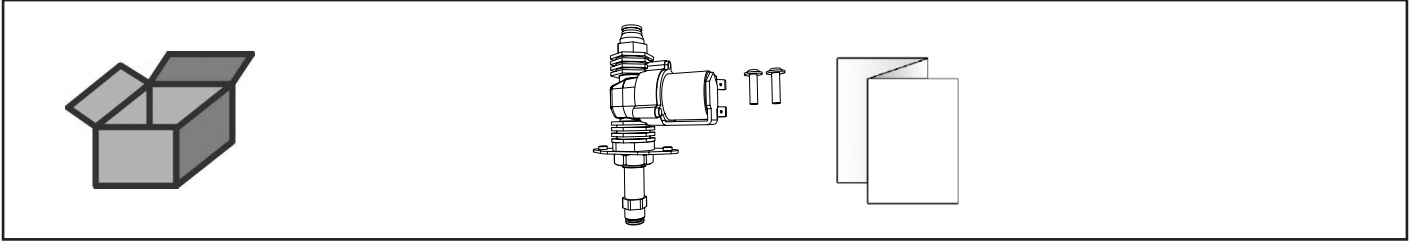

2

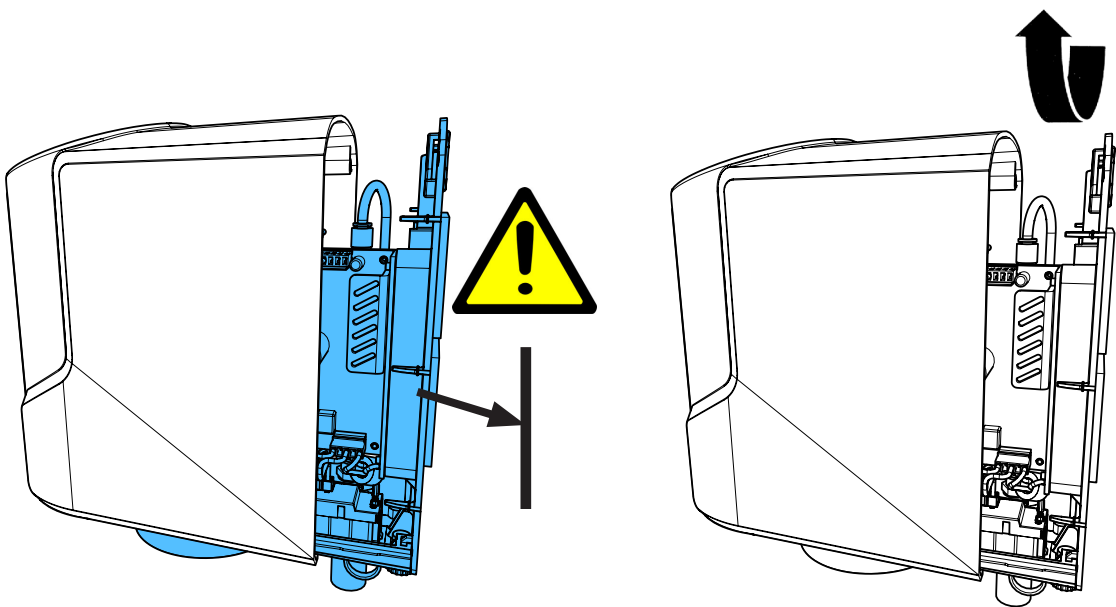


3

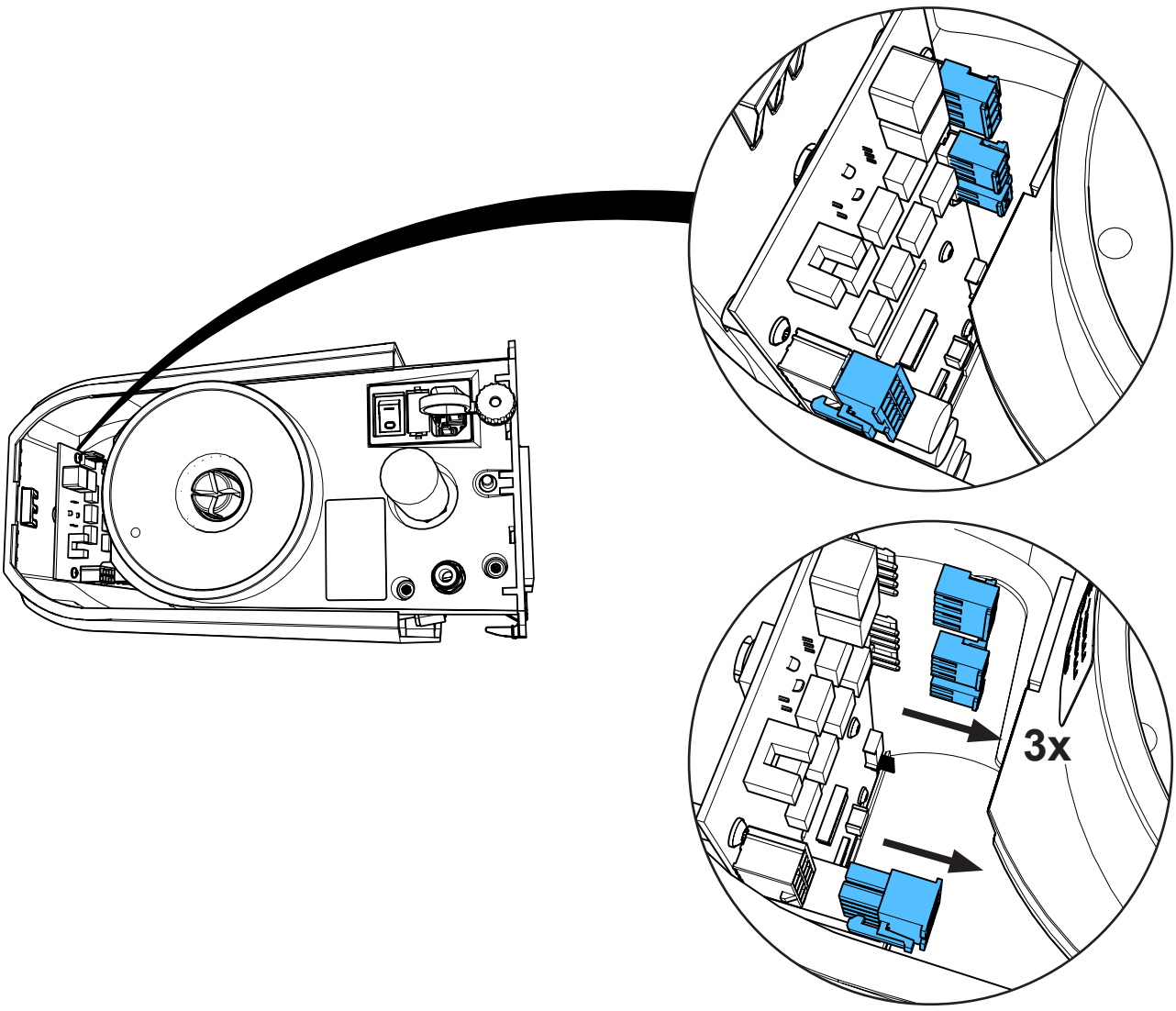





4

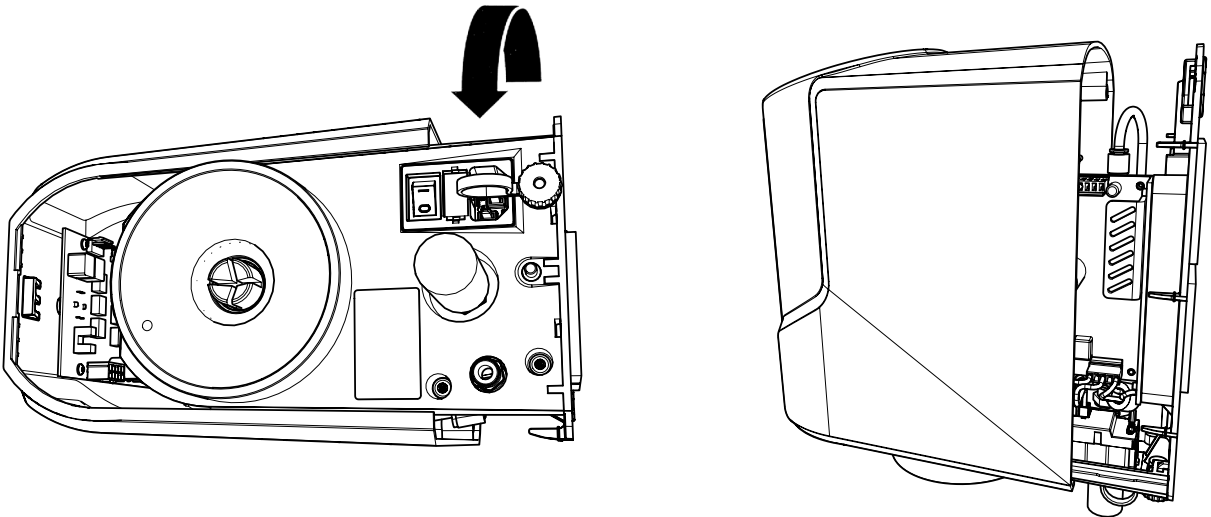


5

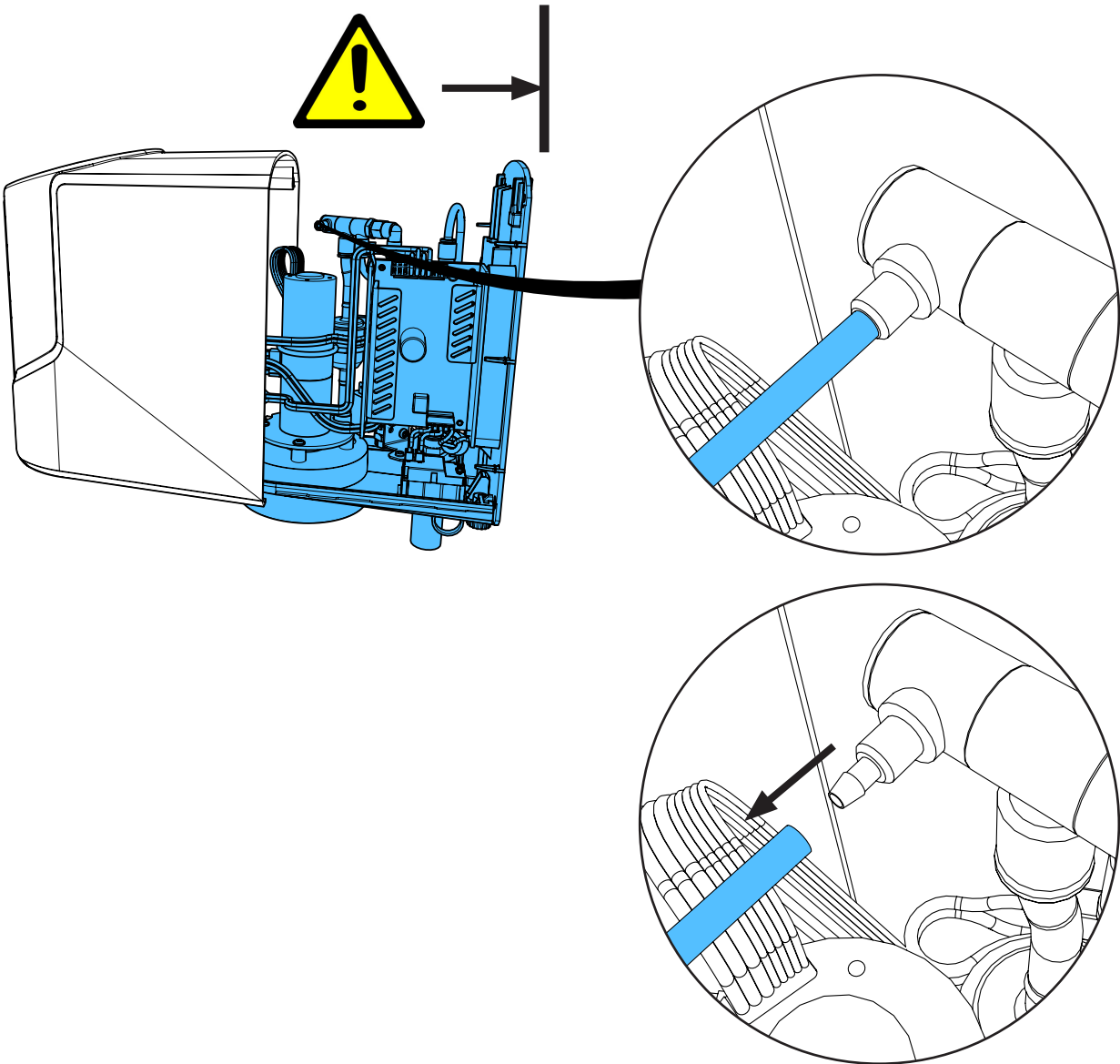


6

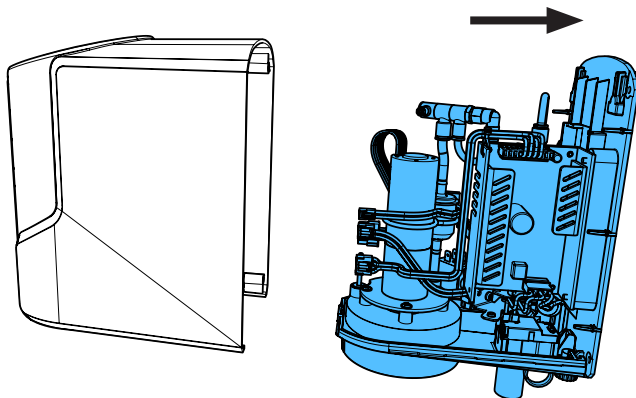


7

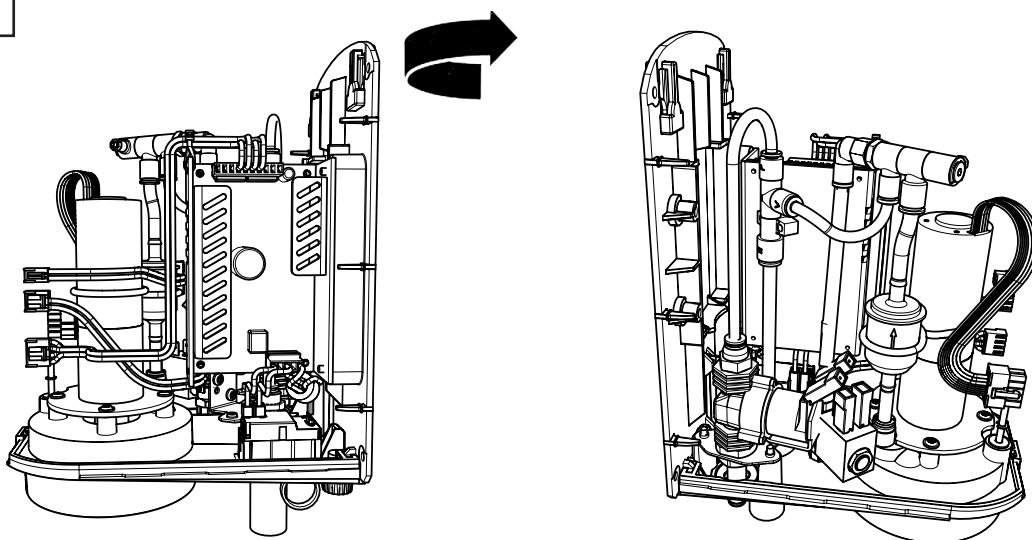





8

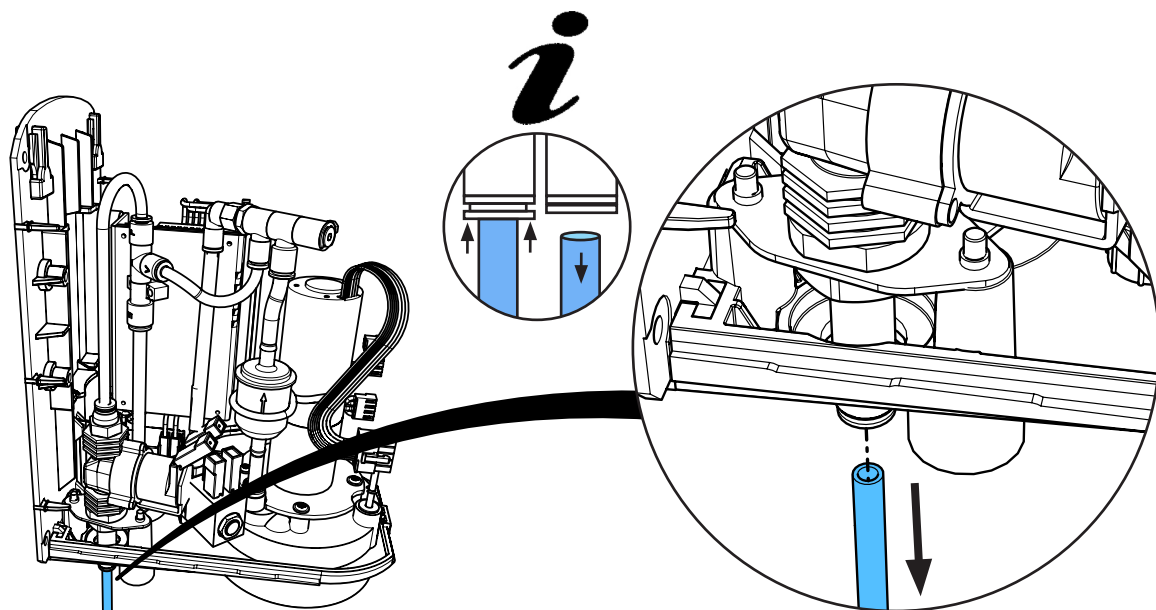



9

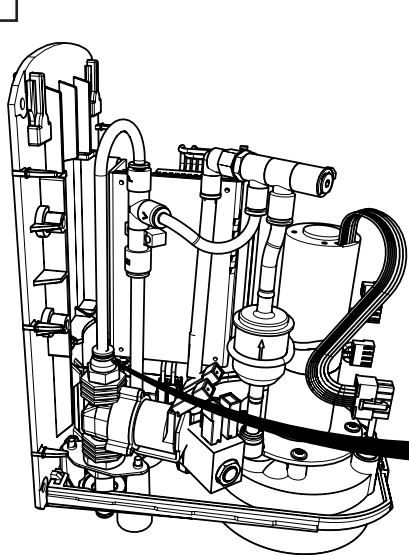



*i*

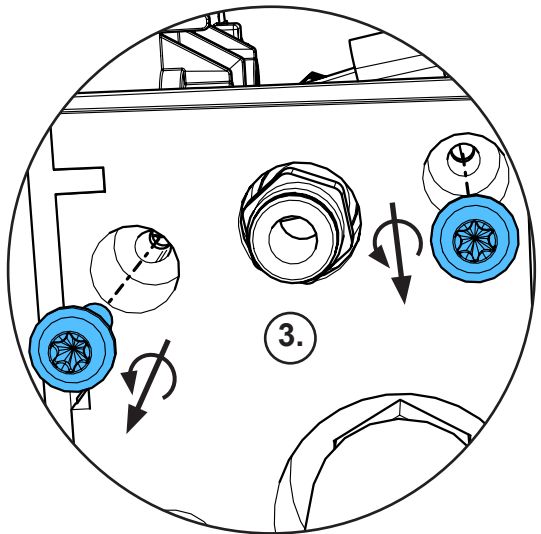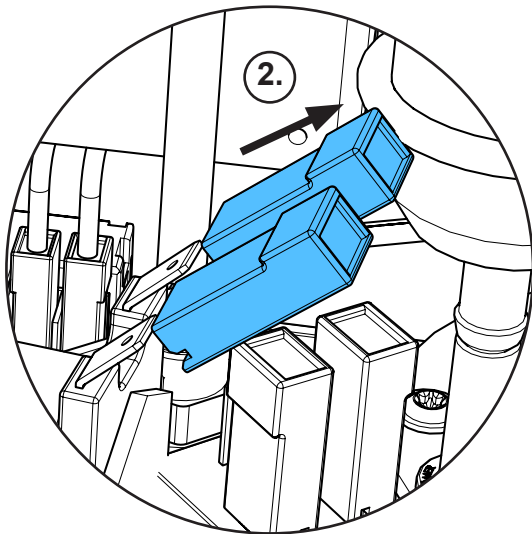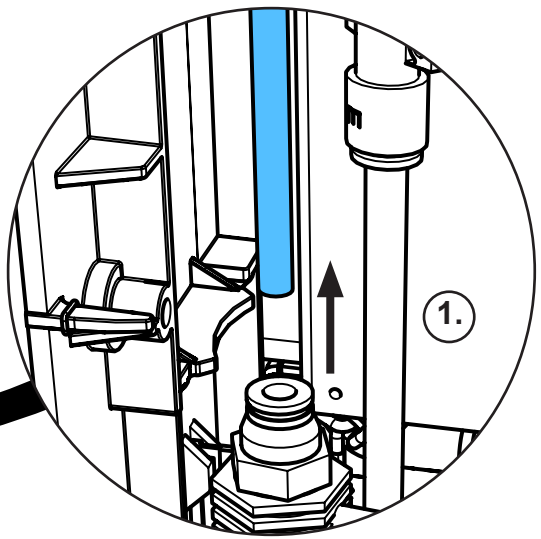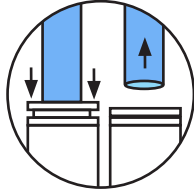






10

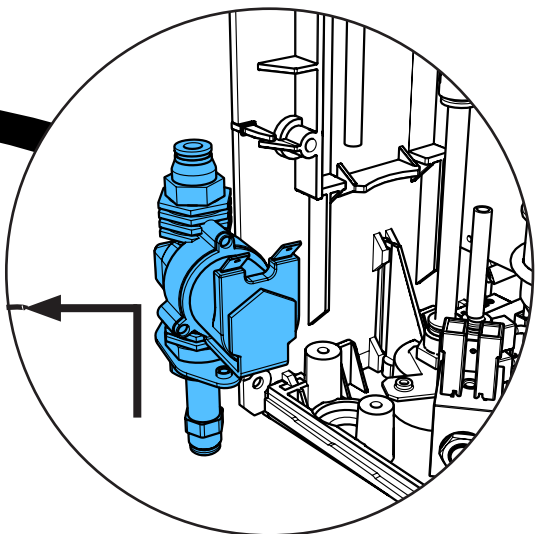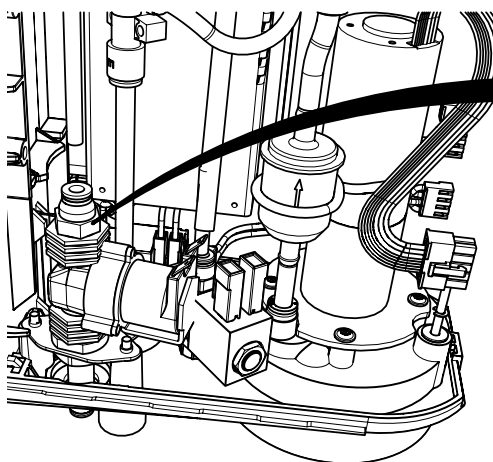


11

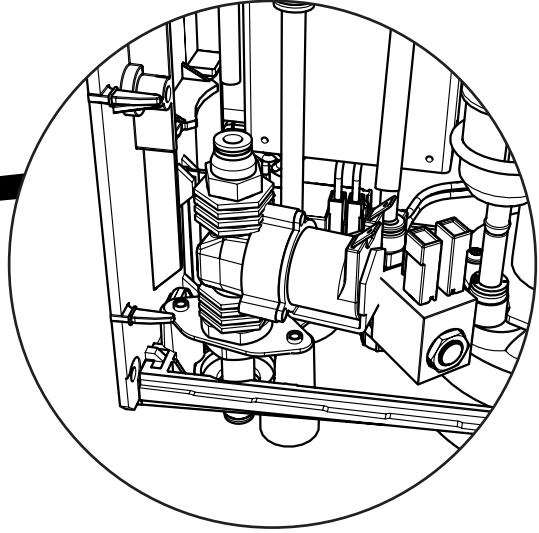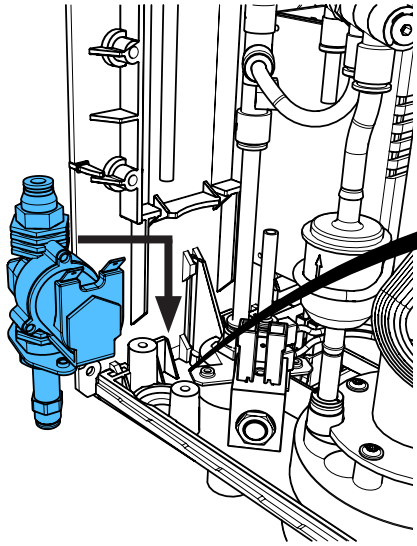


12

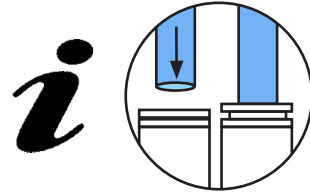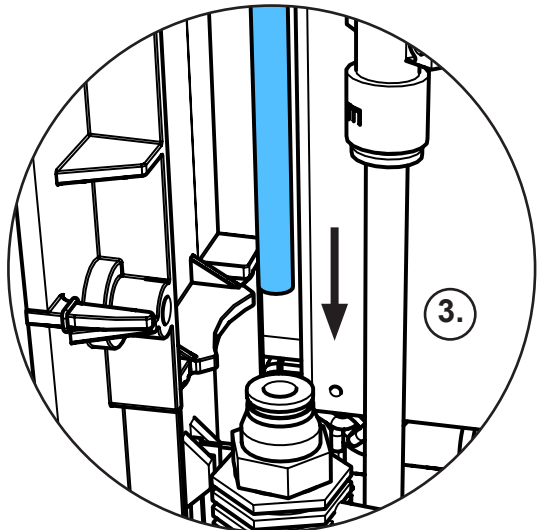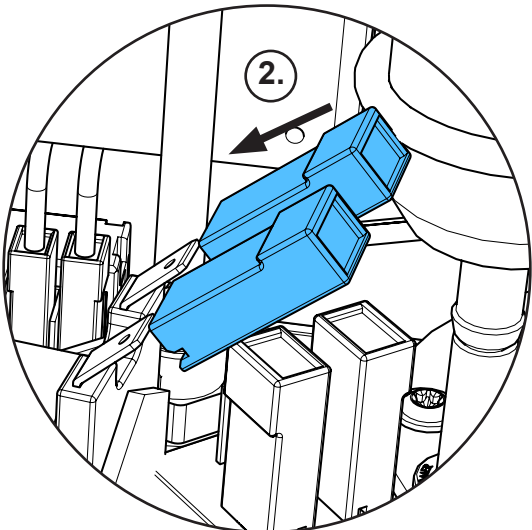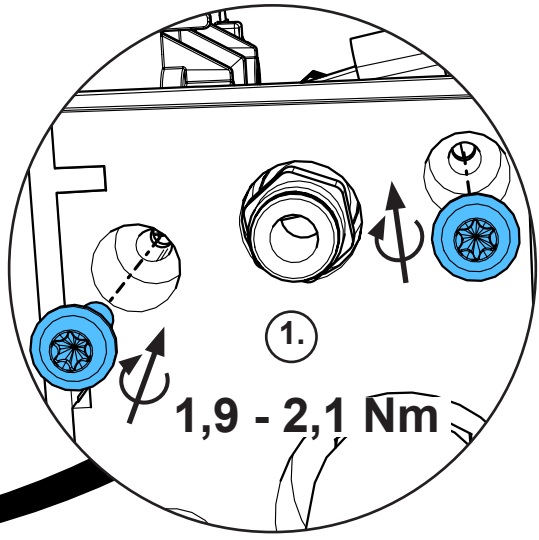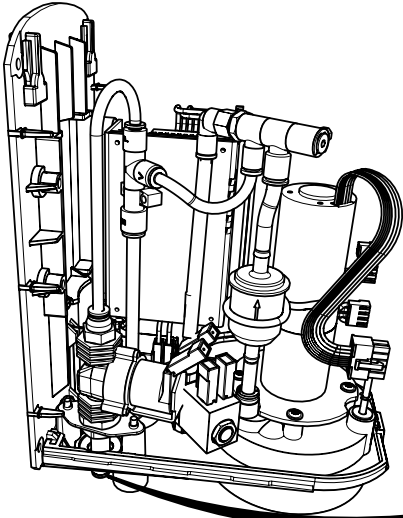


13

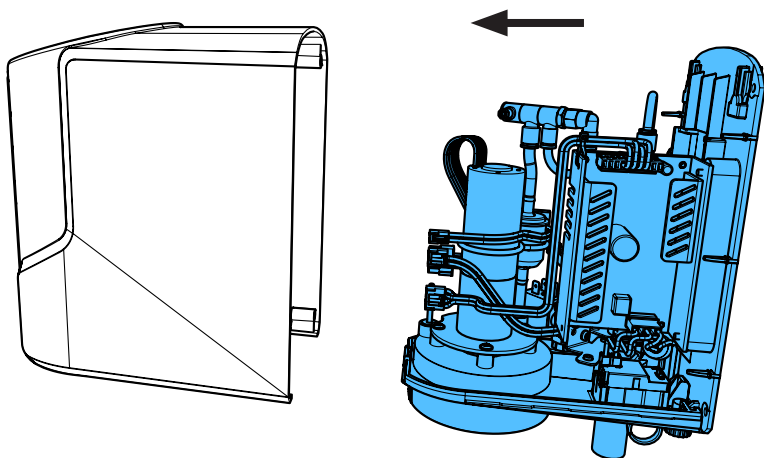


14

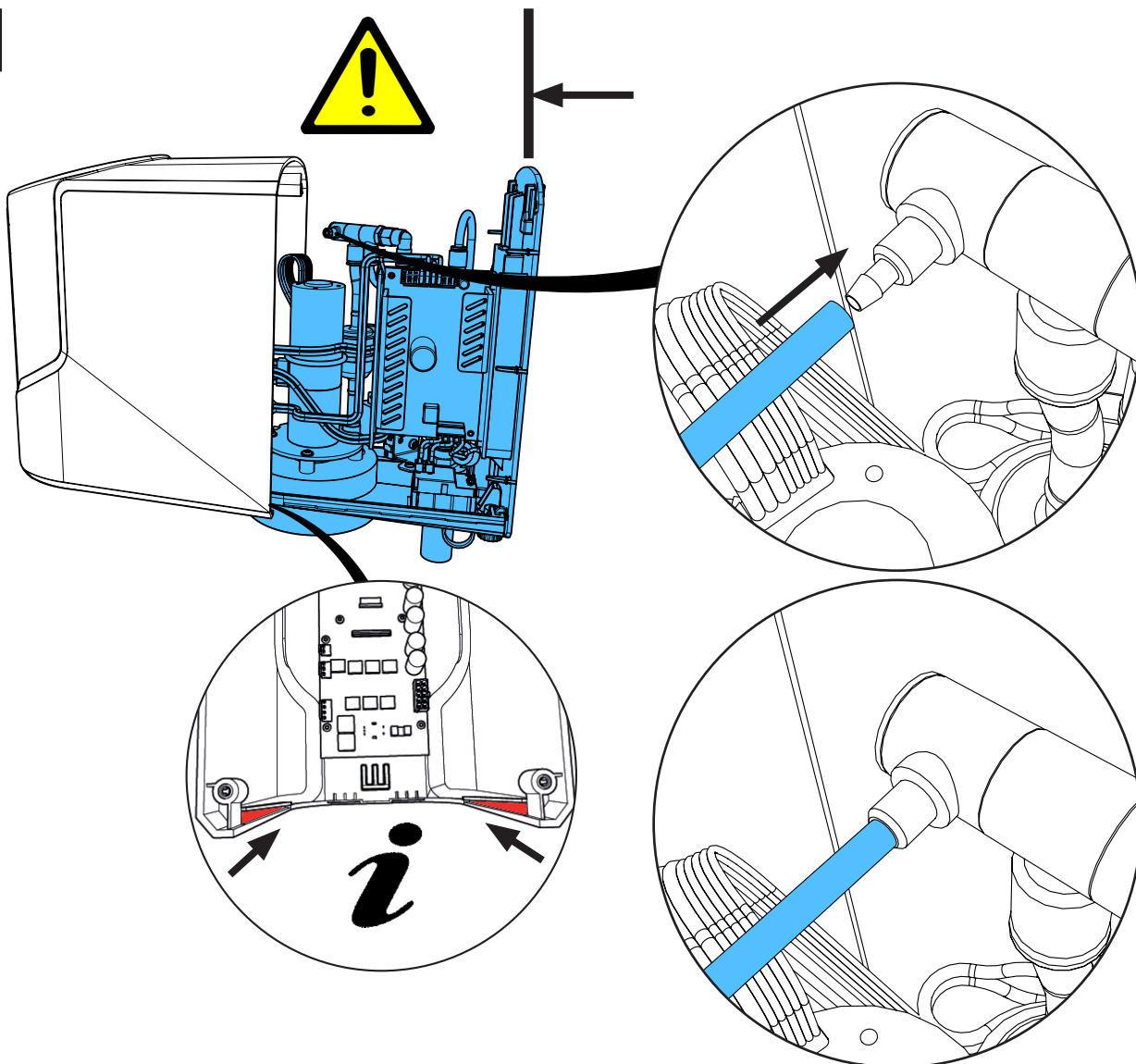


15

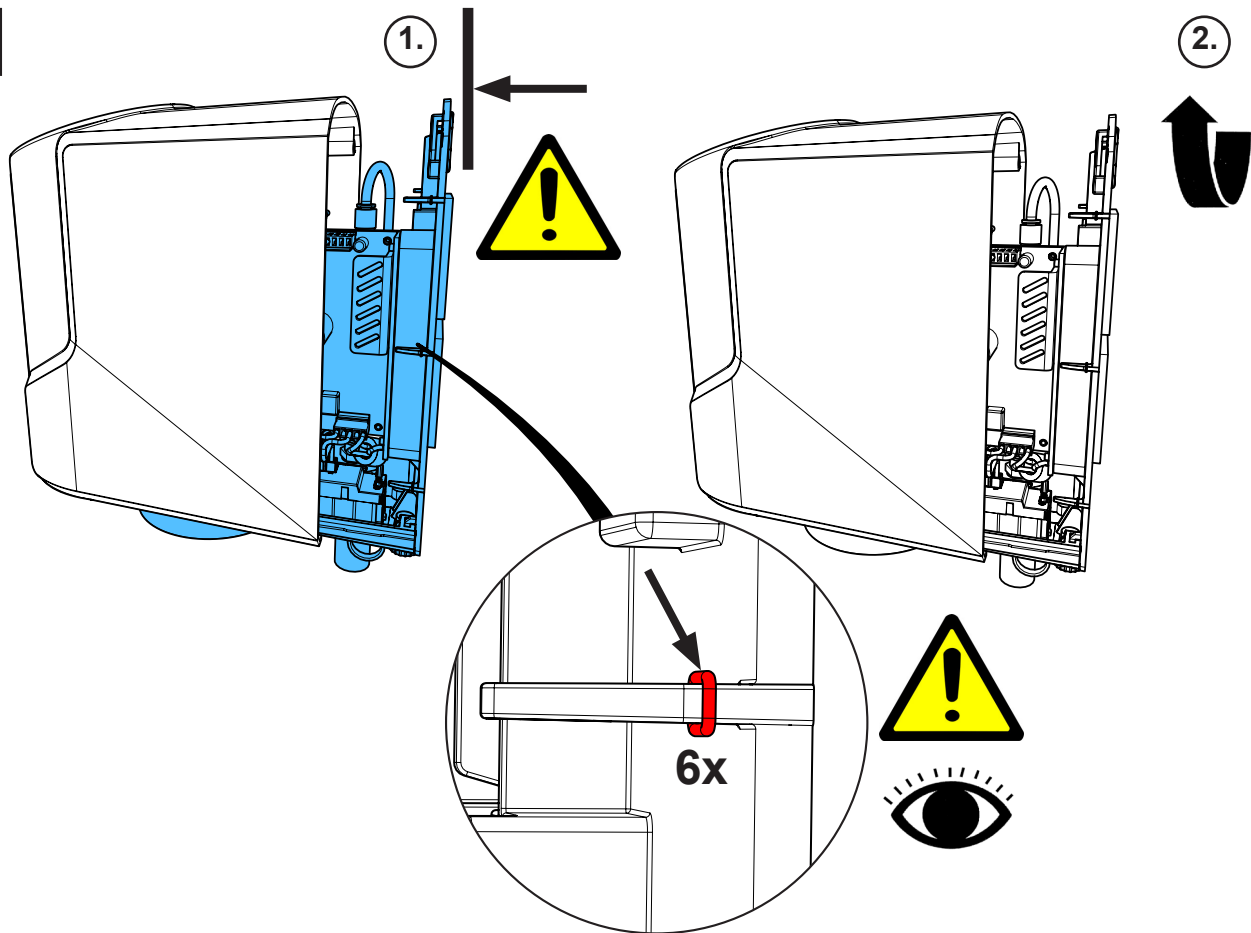


16

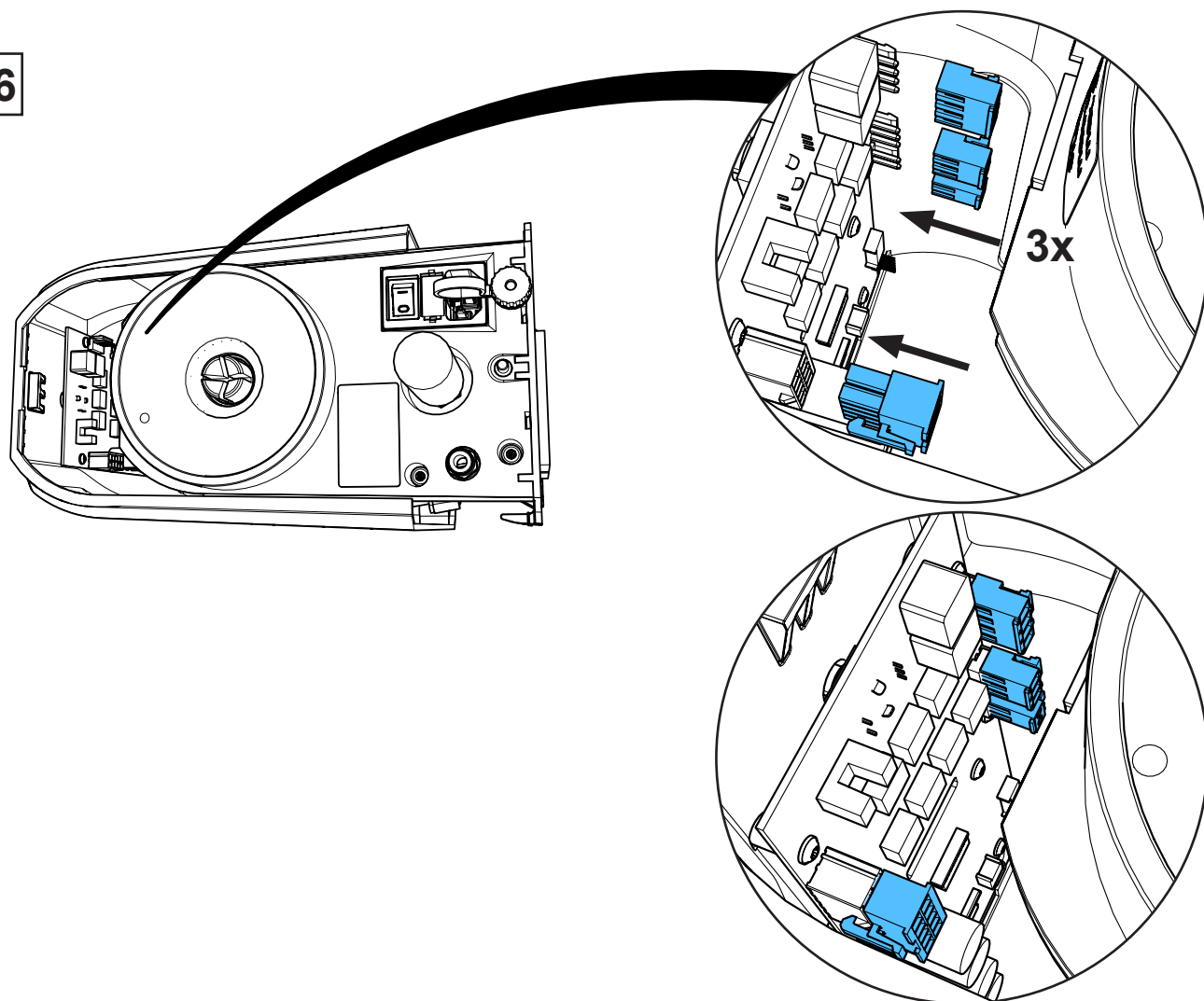


17

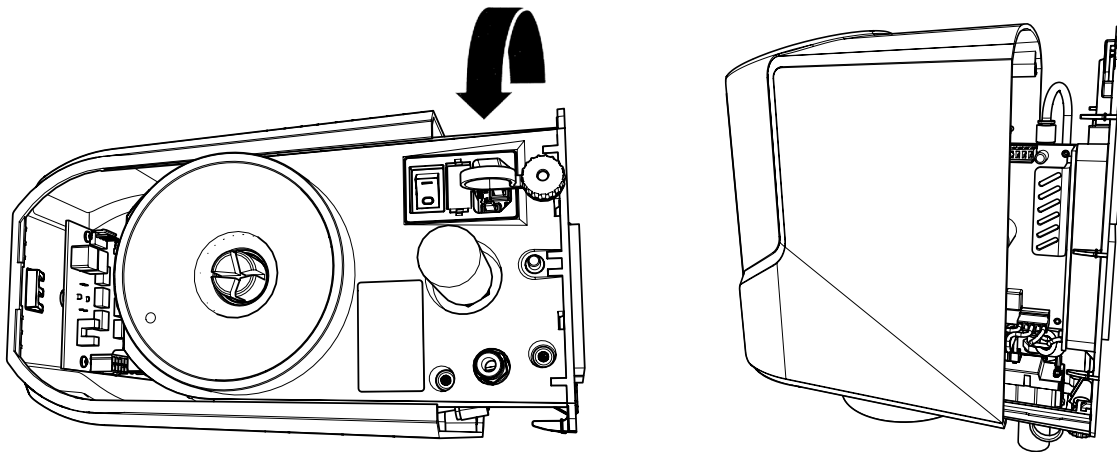

18

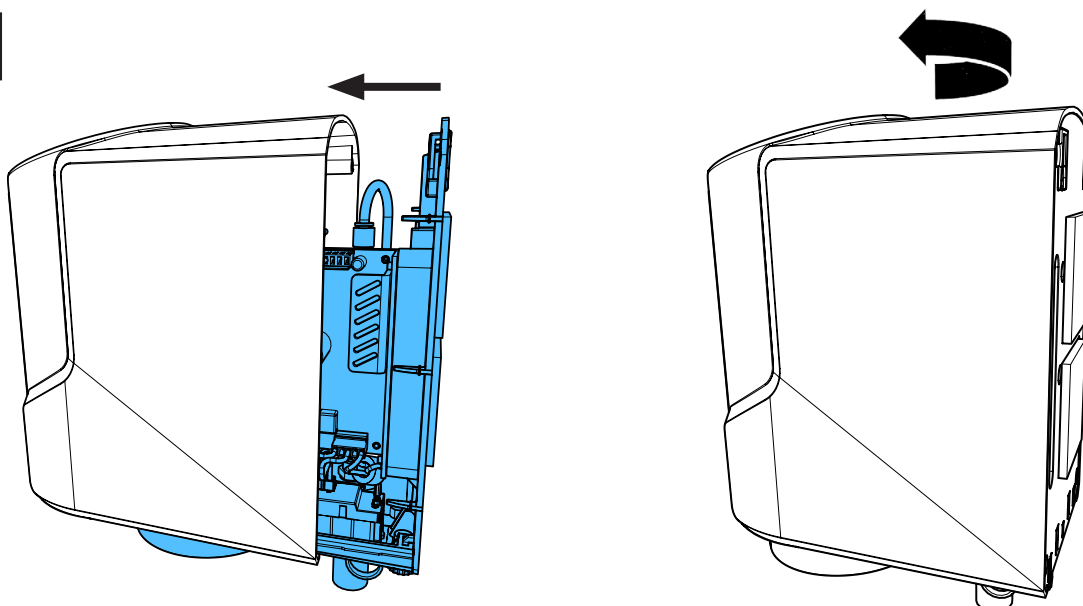

19



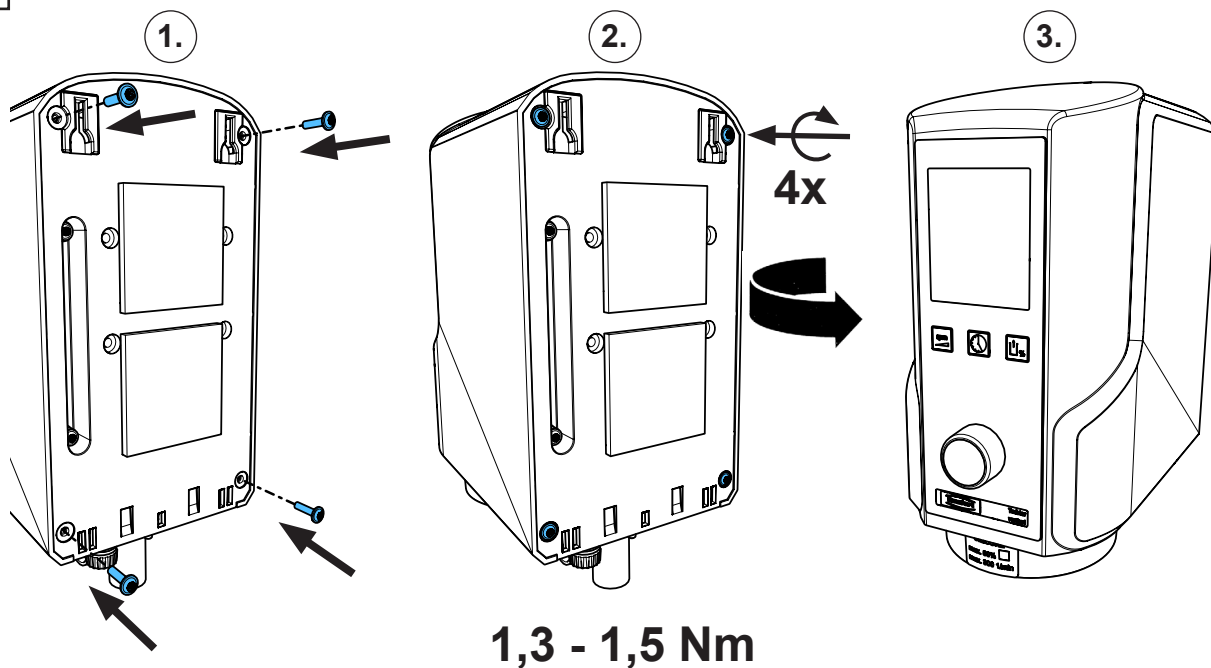







**90002 1586**


Belüftungsventil • Air valve • Soupape d'aération • Valvola di aerazione  
Válvula de aeración • Клапан вентиляции

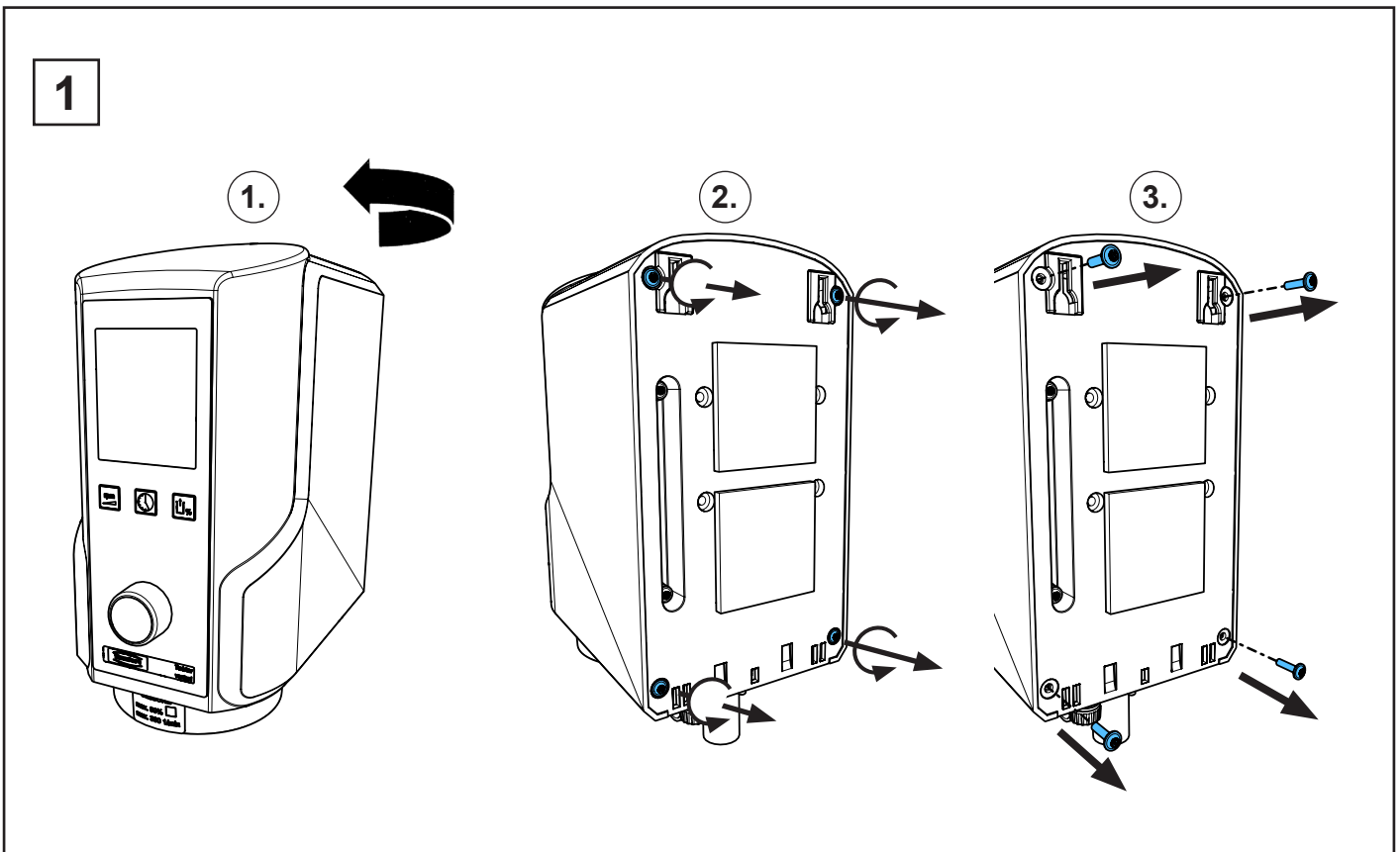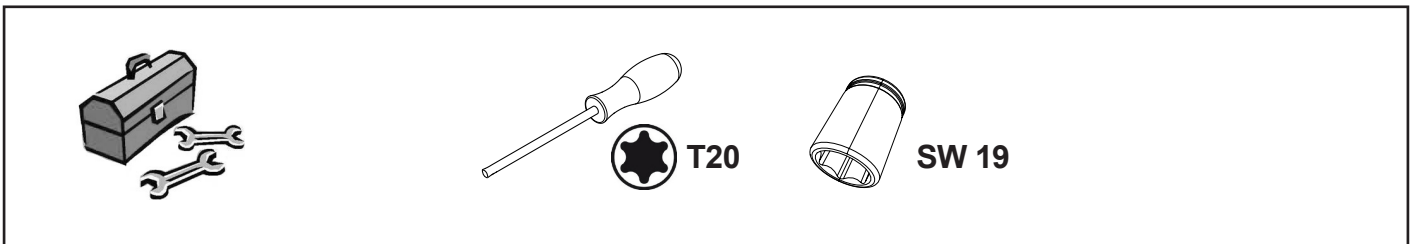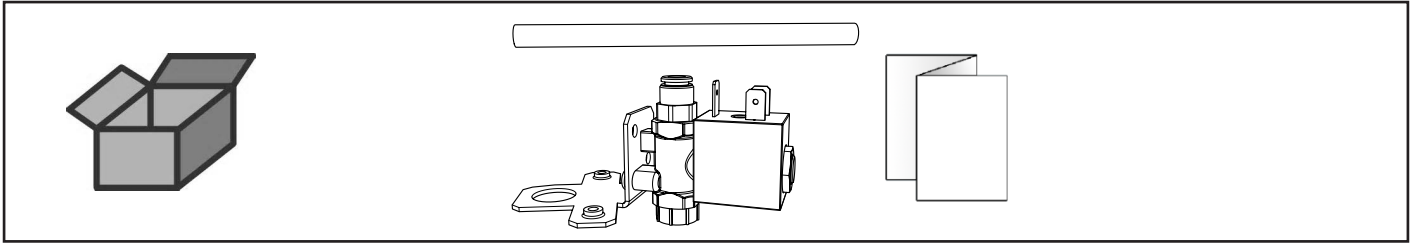


2

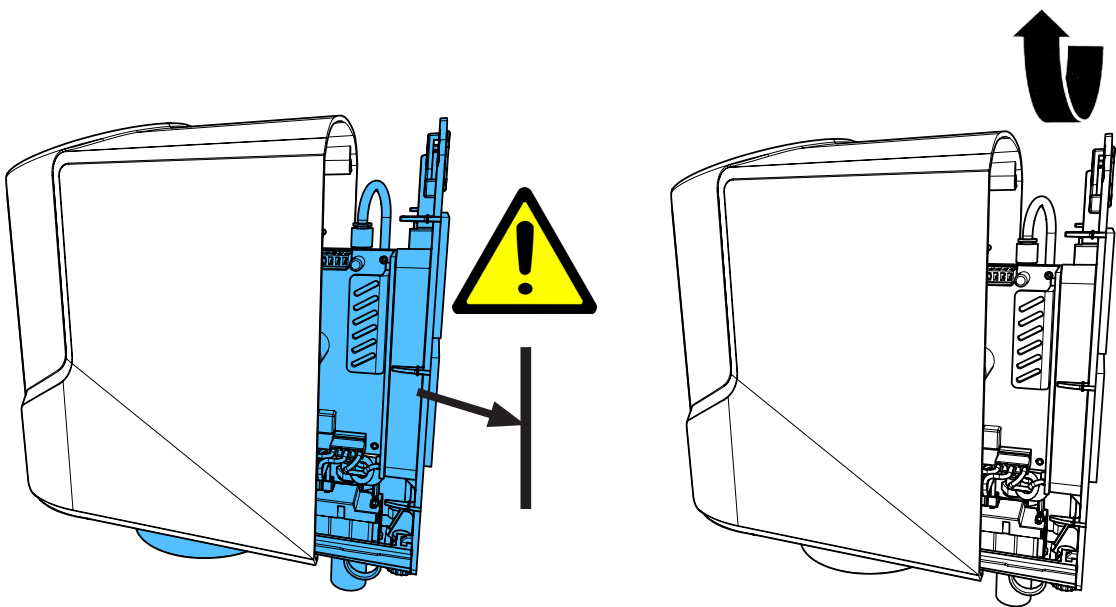


3

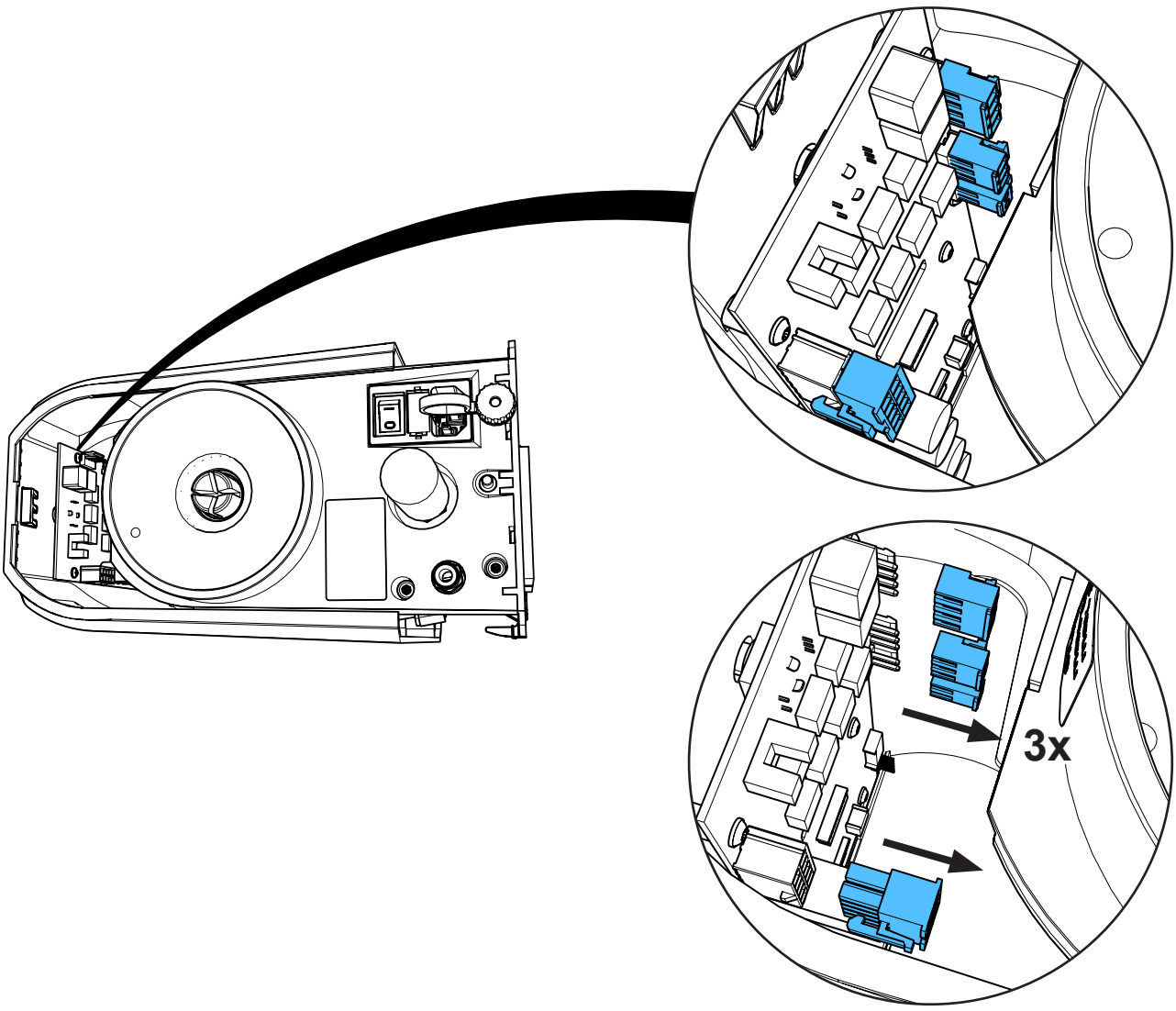


4

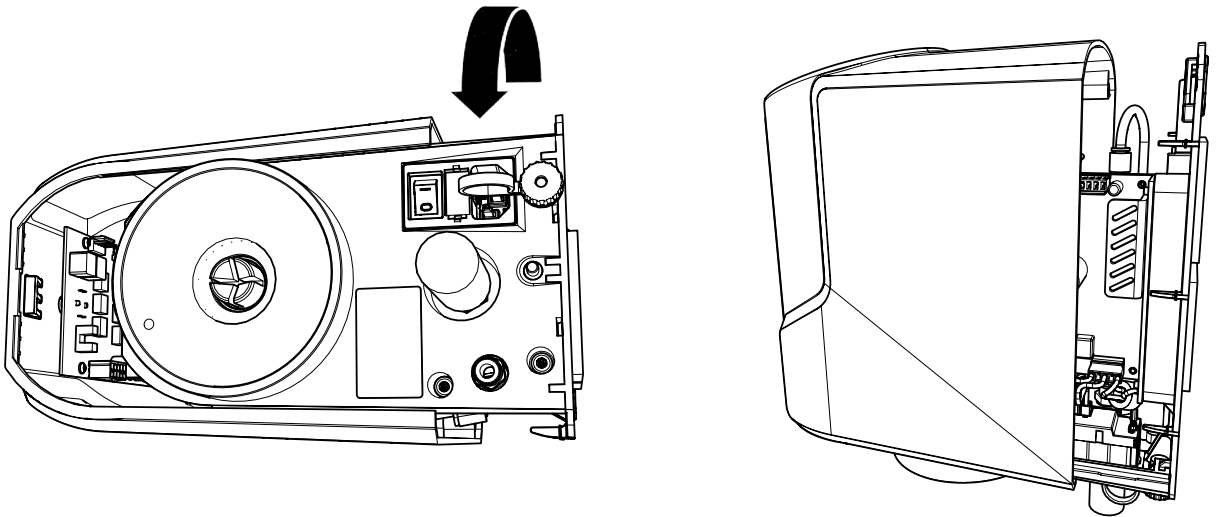


5

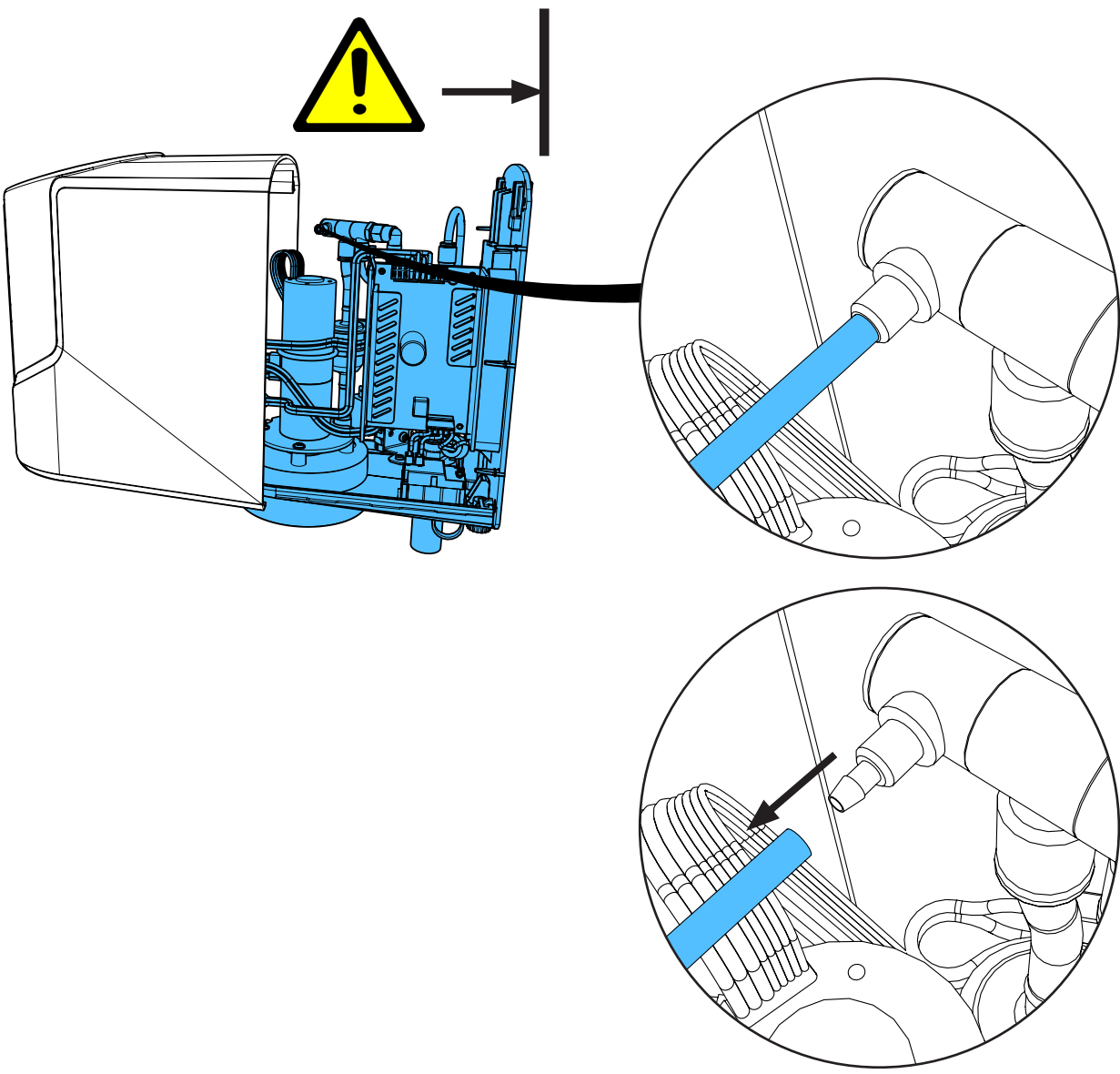


6

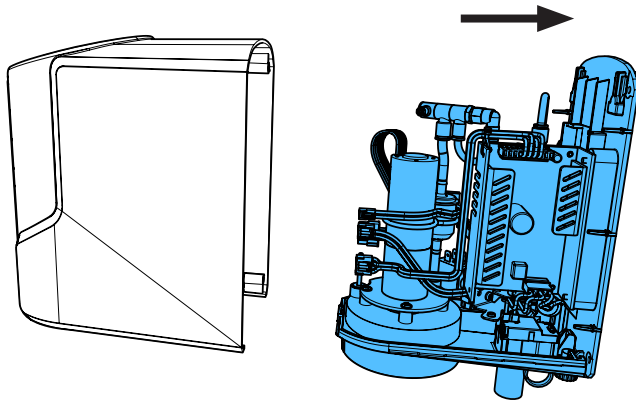




7

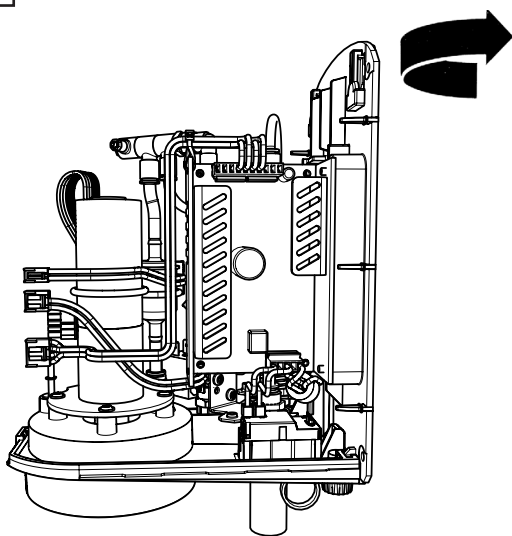




8

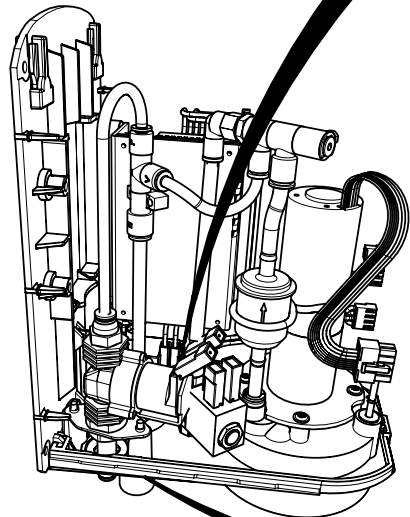




**i**


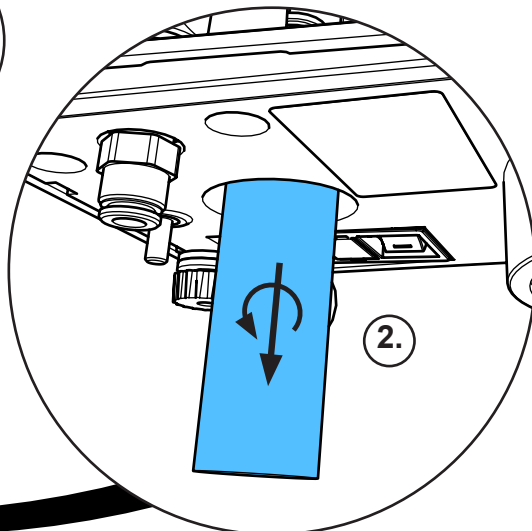
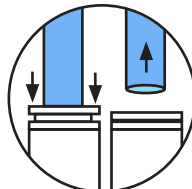
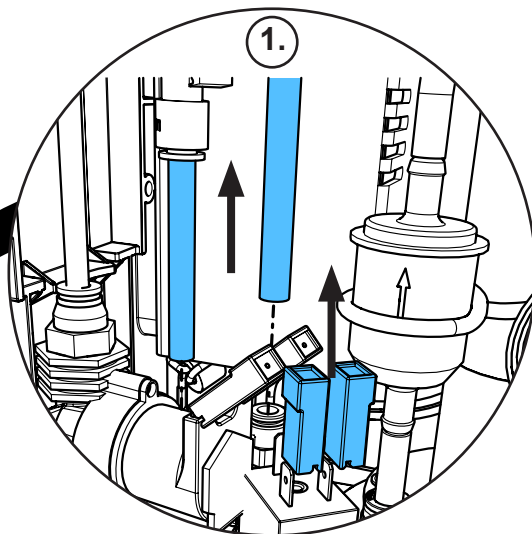


9

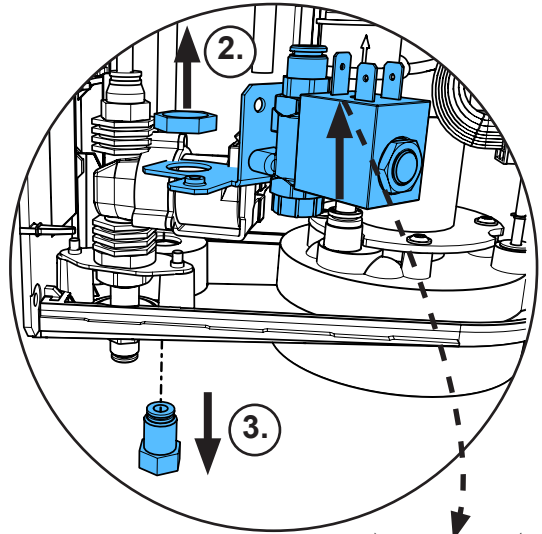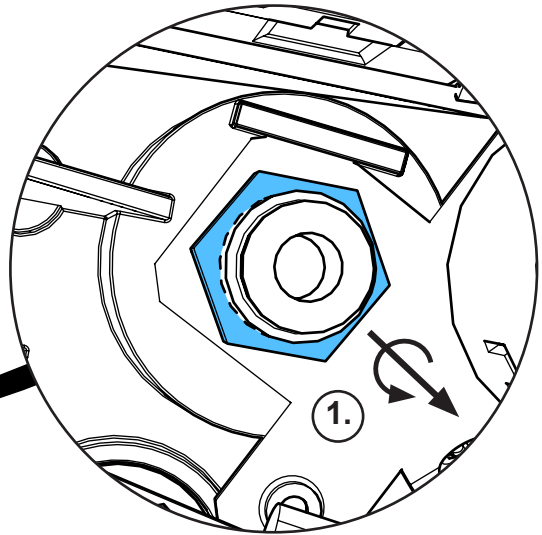
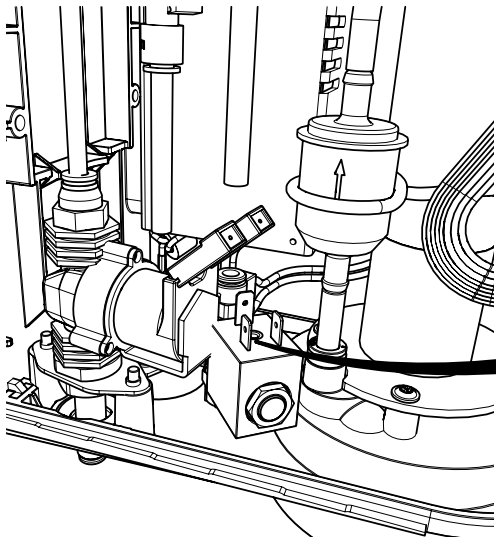


10

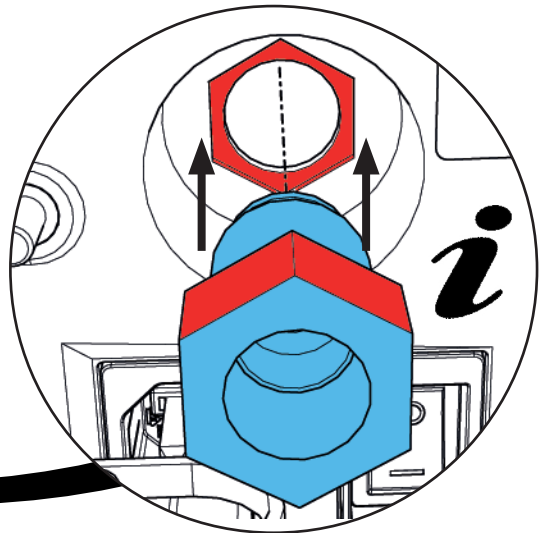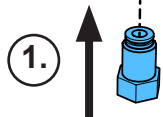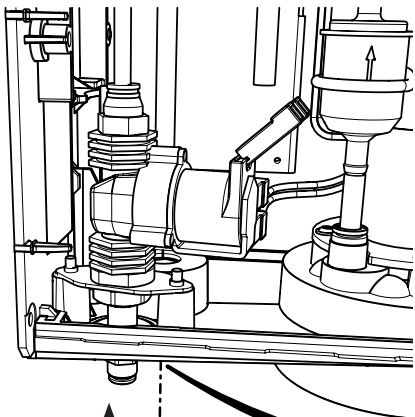


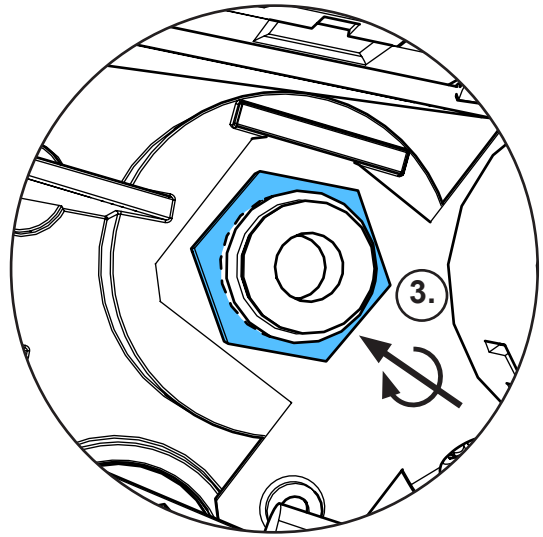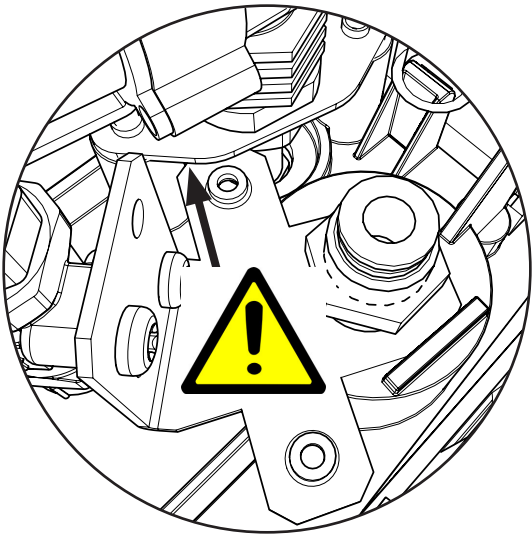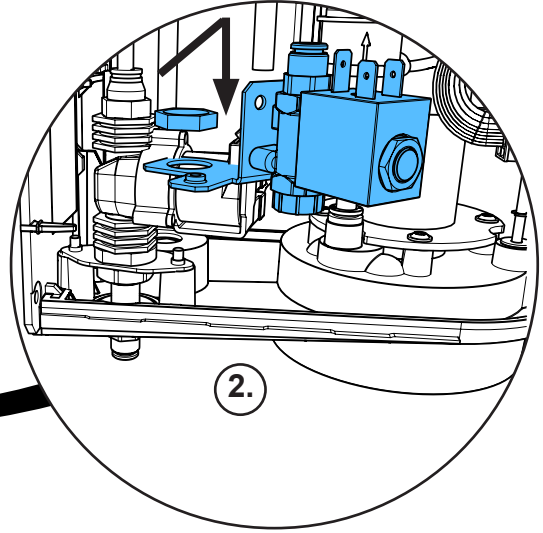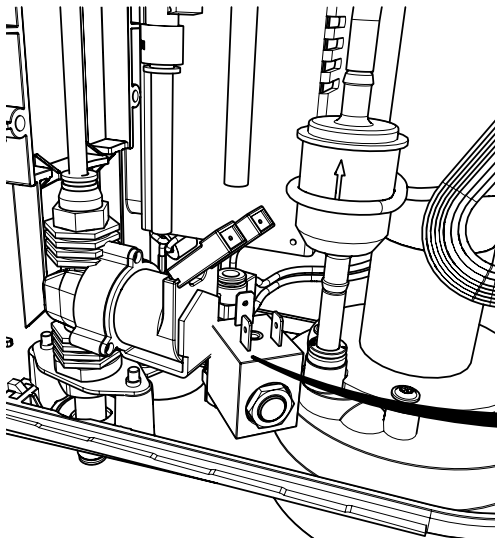


**4,9 - 5,1 Nm**

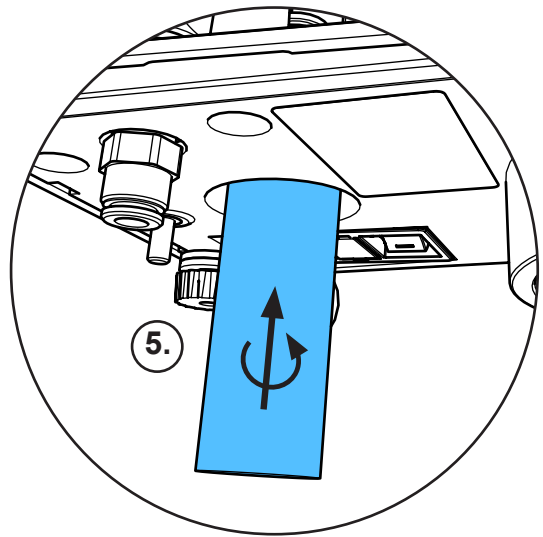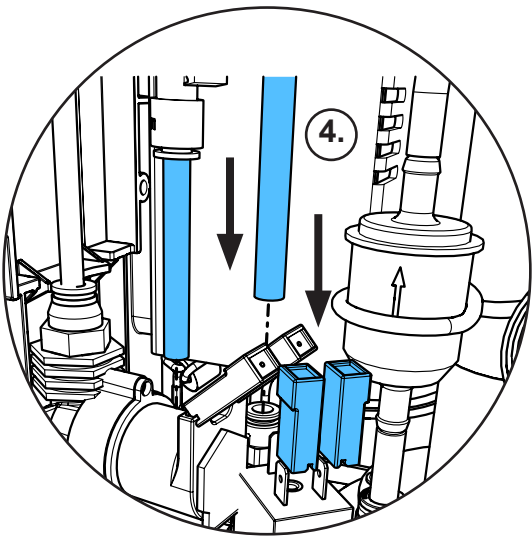


***i***

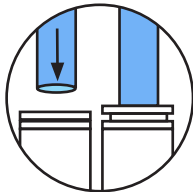


11

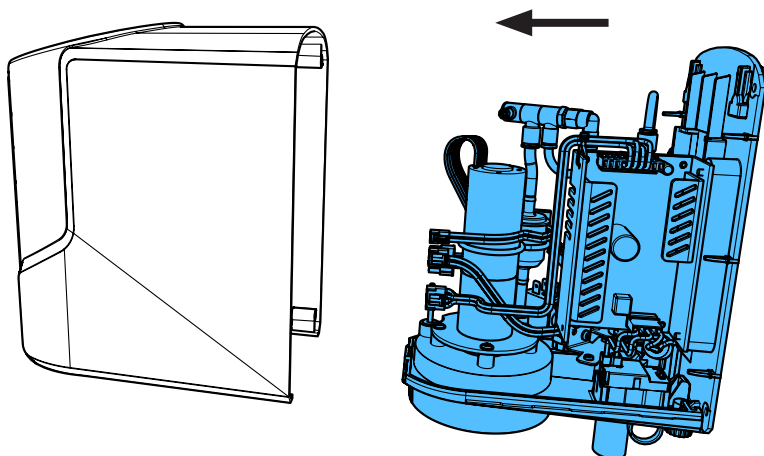


12

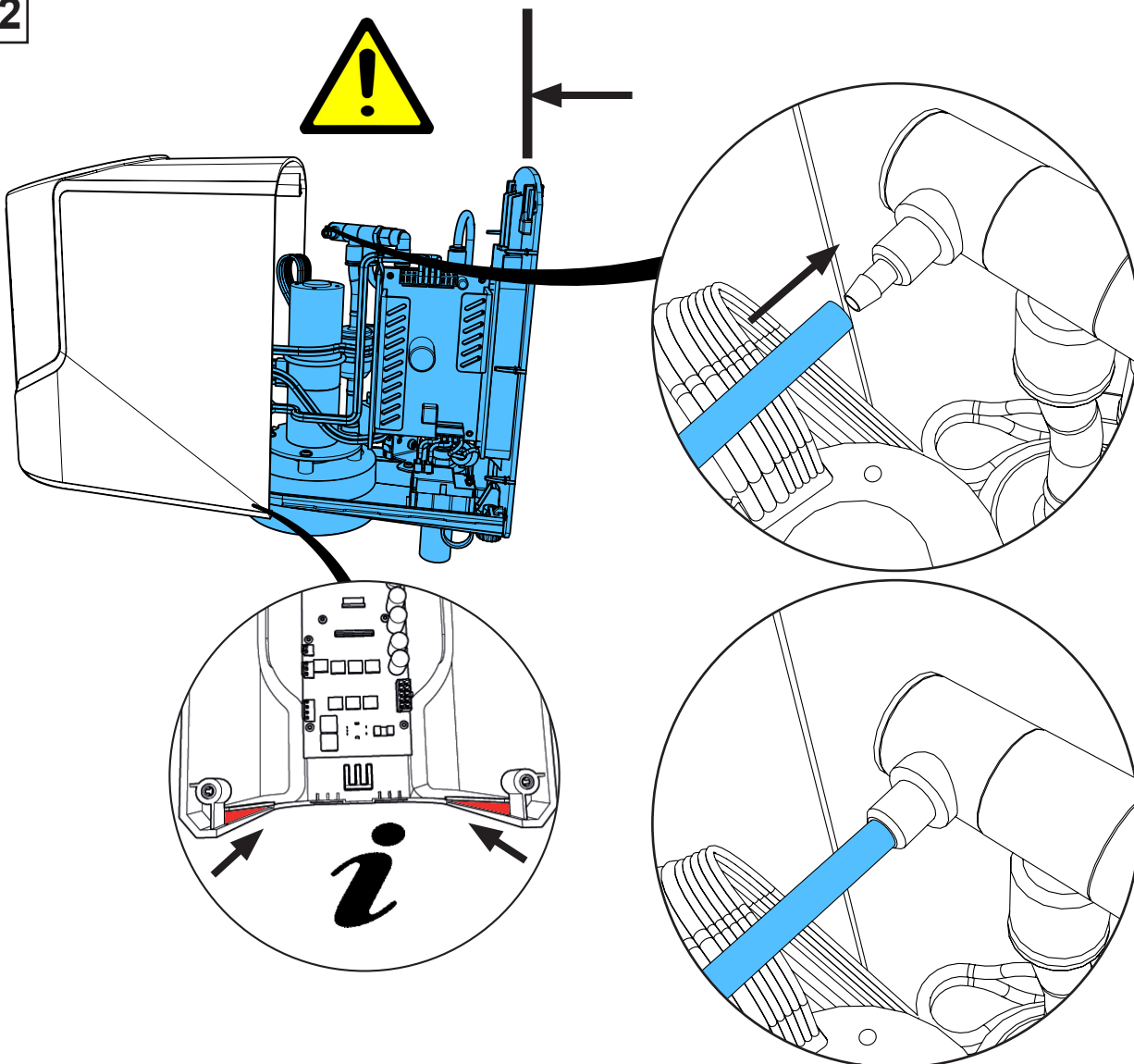

13

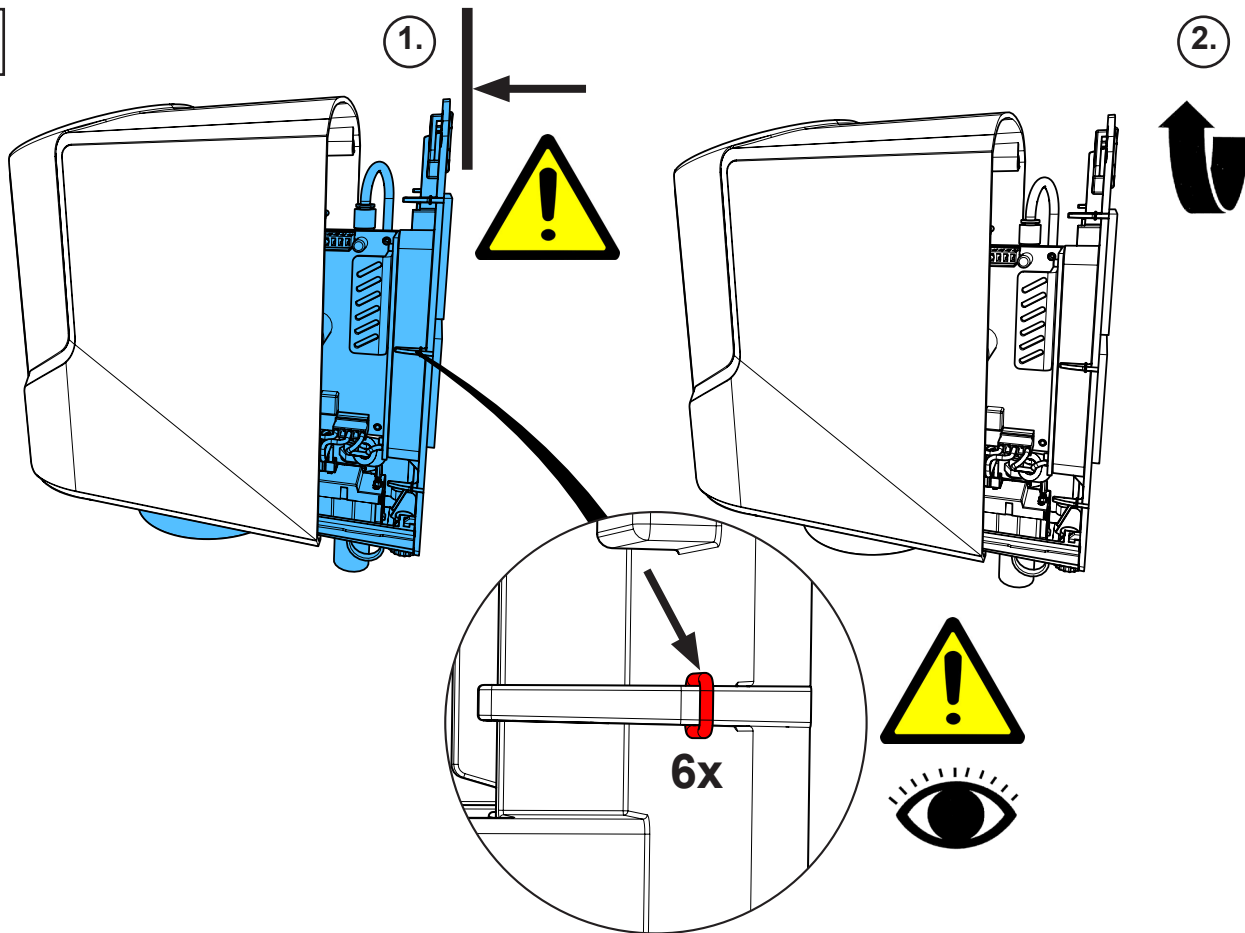


14

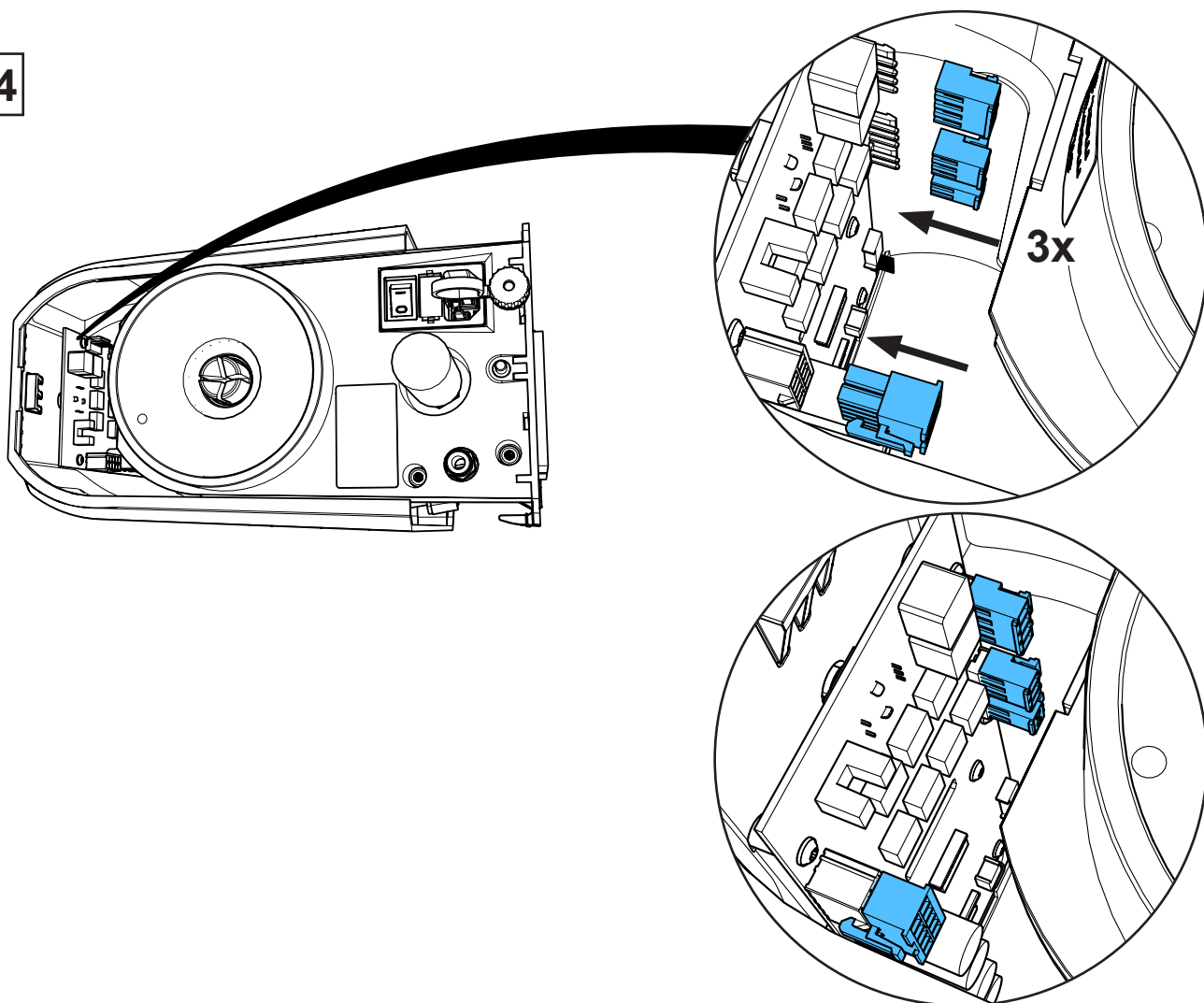





15

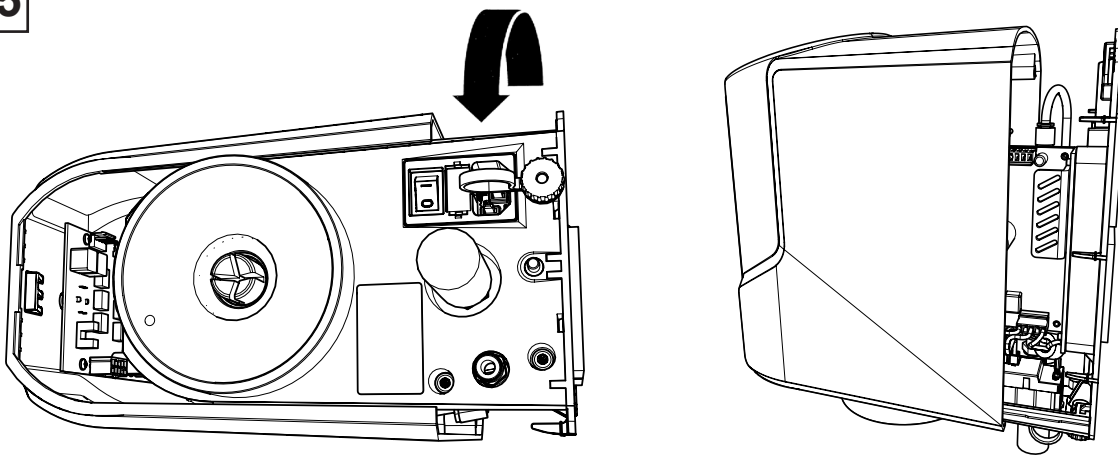

16

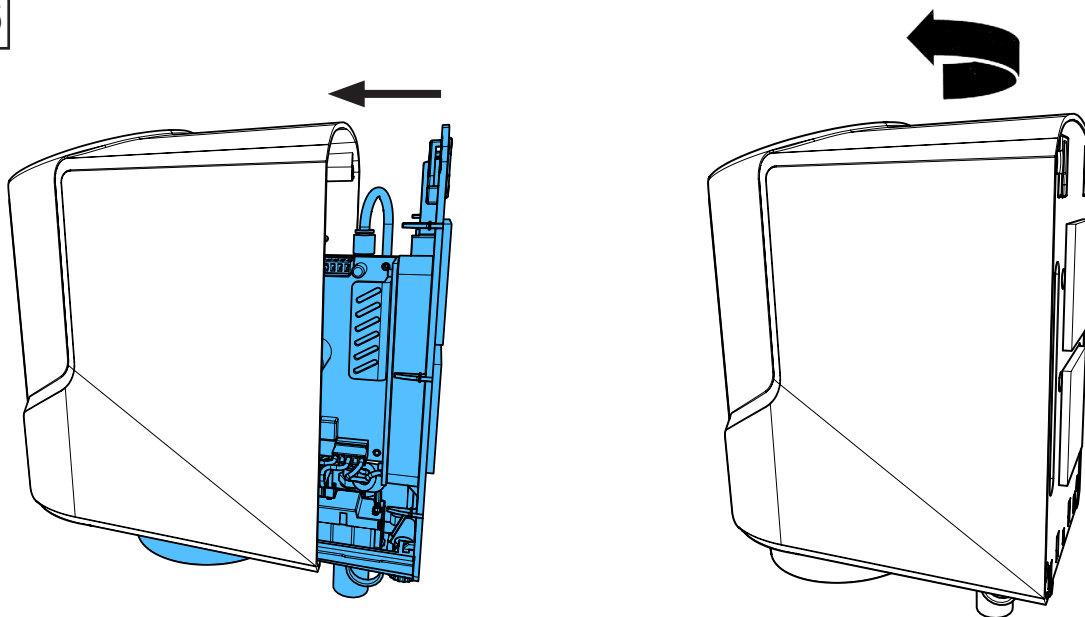




17

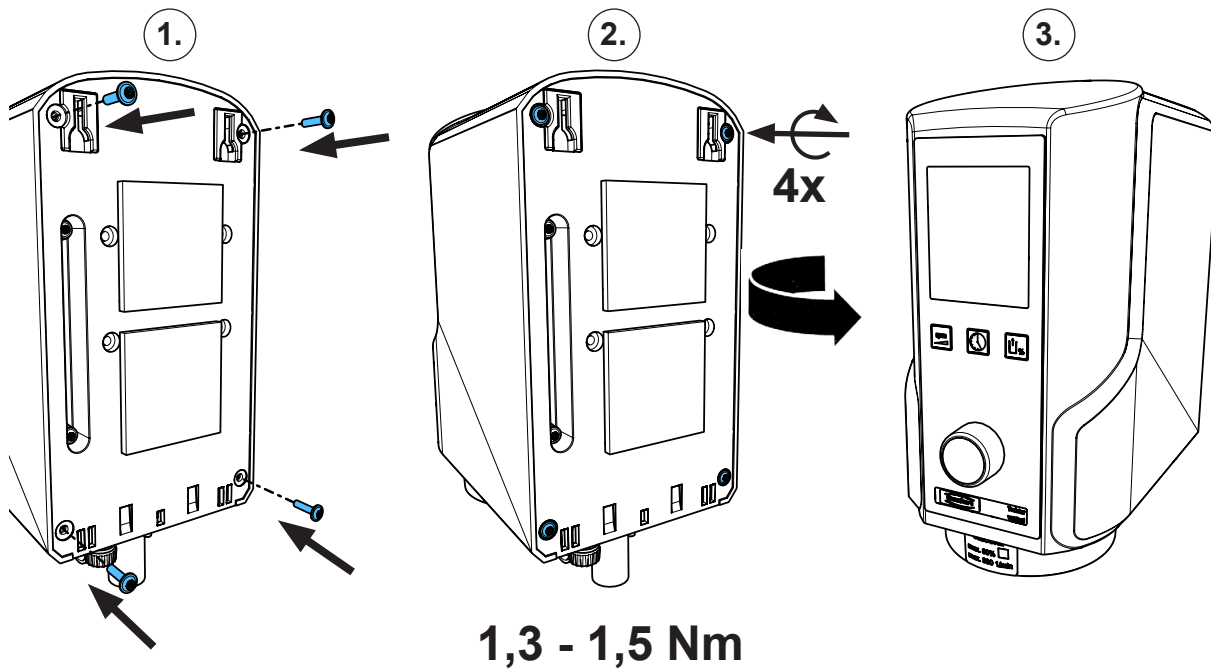




**90003 4273**


Venturidüse • Venturi nozzle • Buse Venturi • Ugello Venturi  
Tobera de Venturi • Сопло Вентури

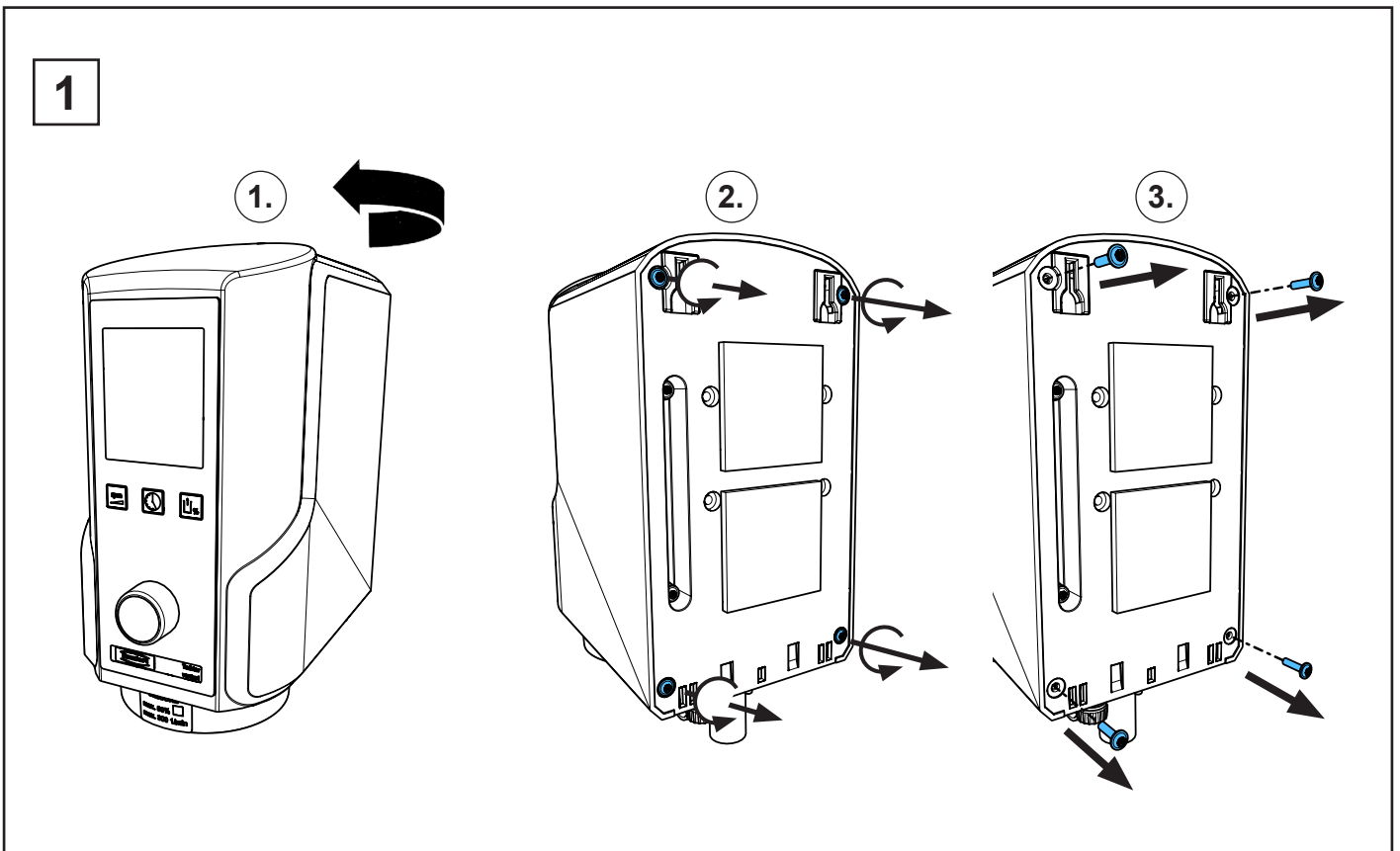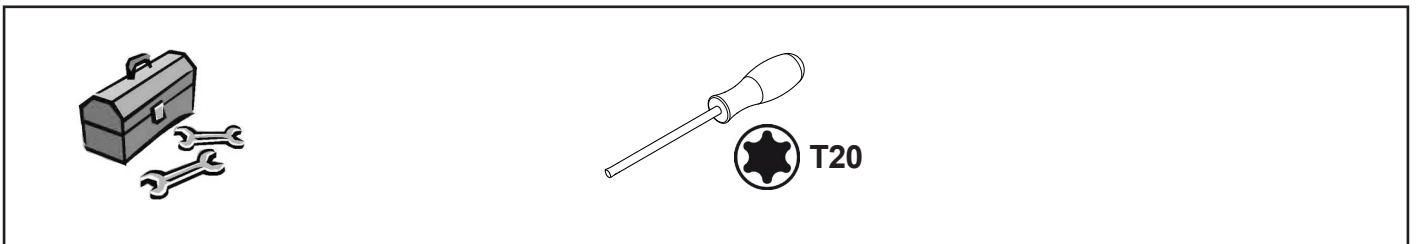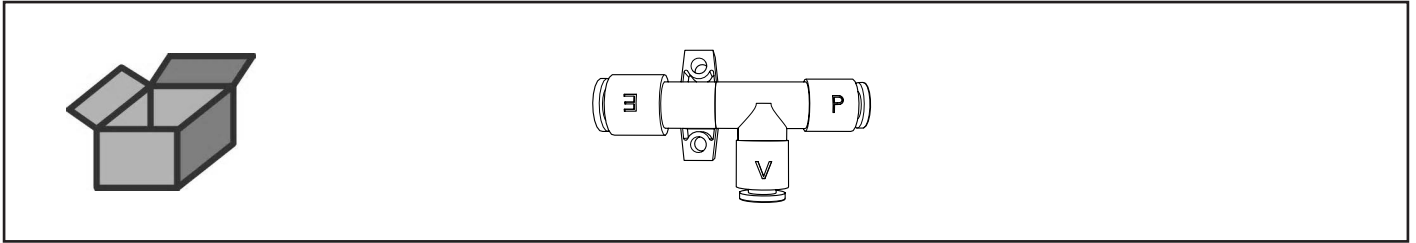


2

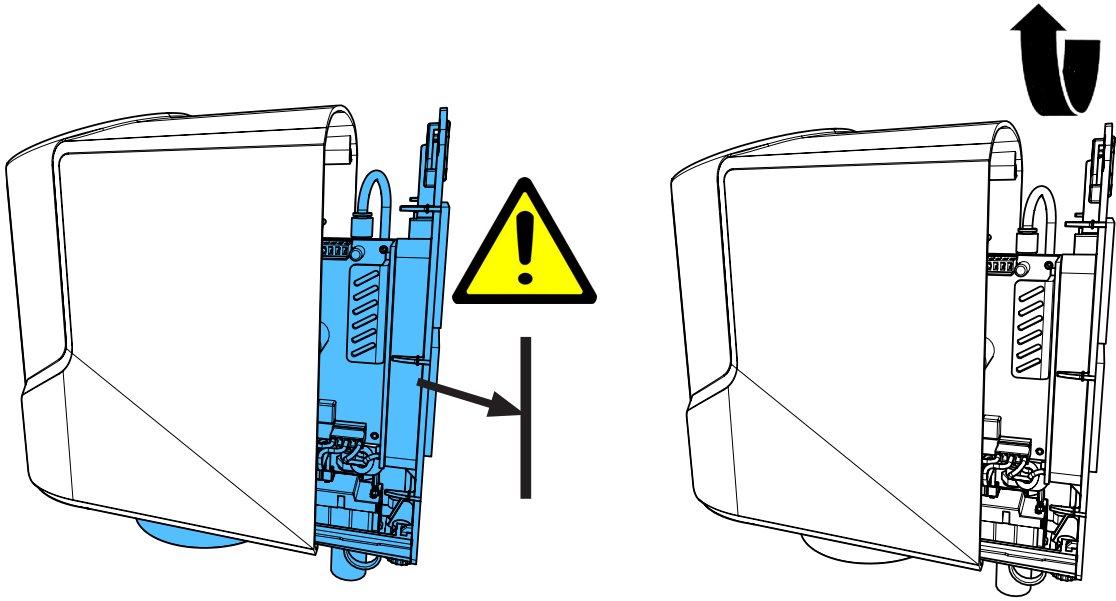


3

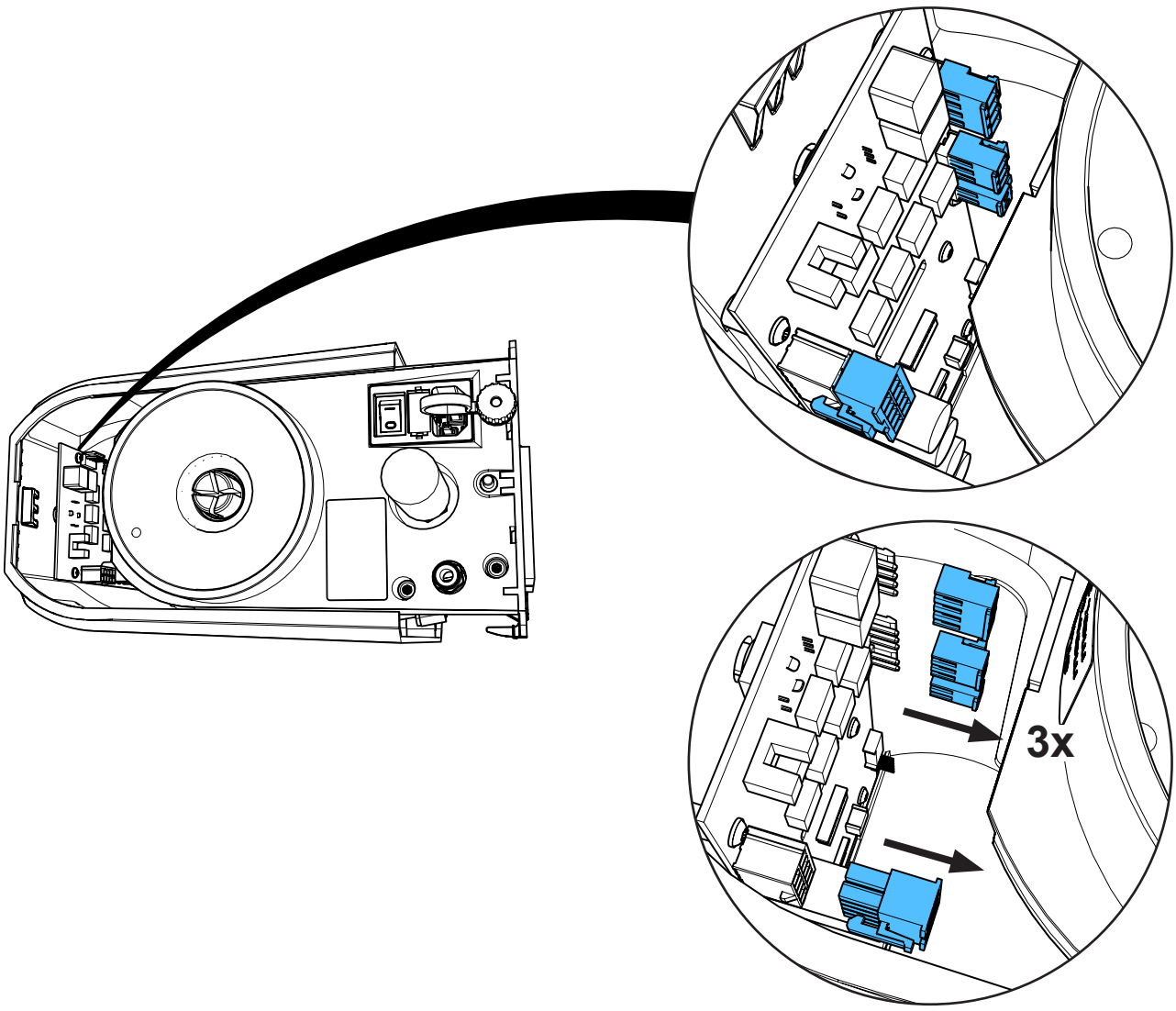


4

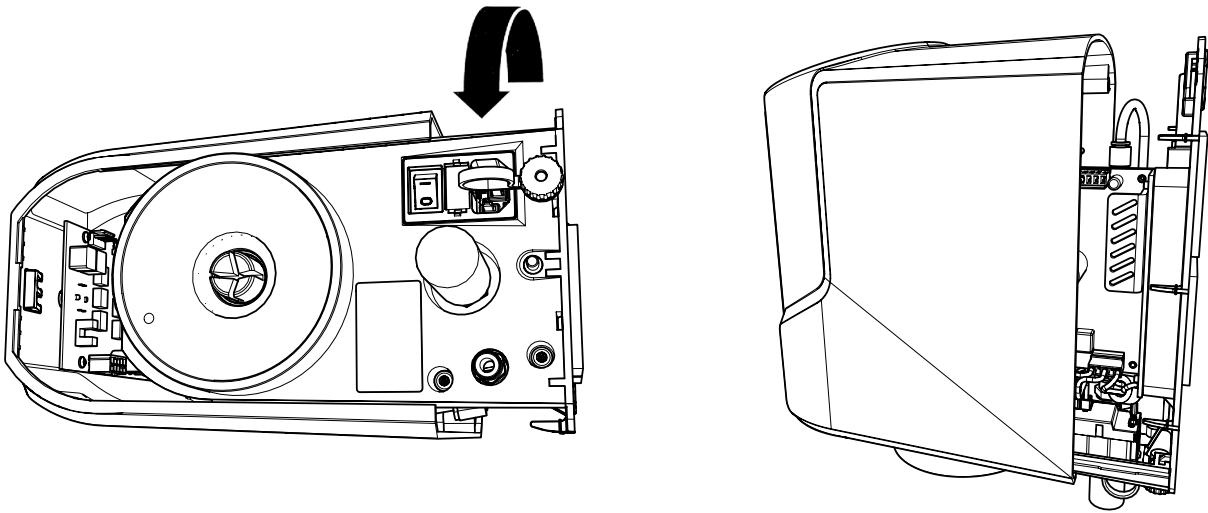


5

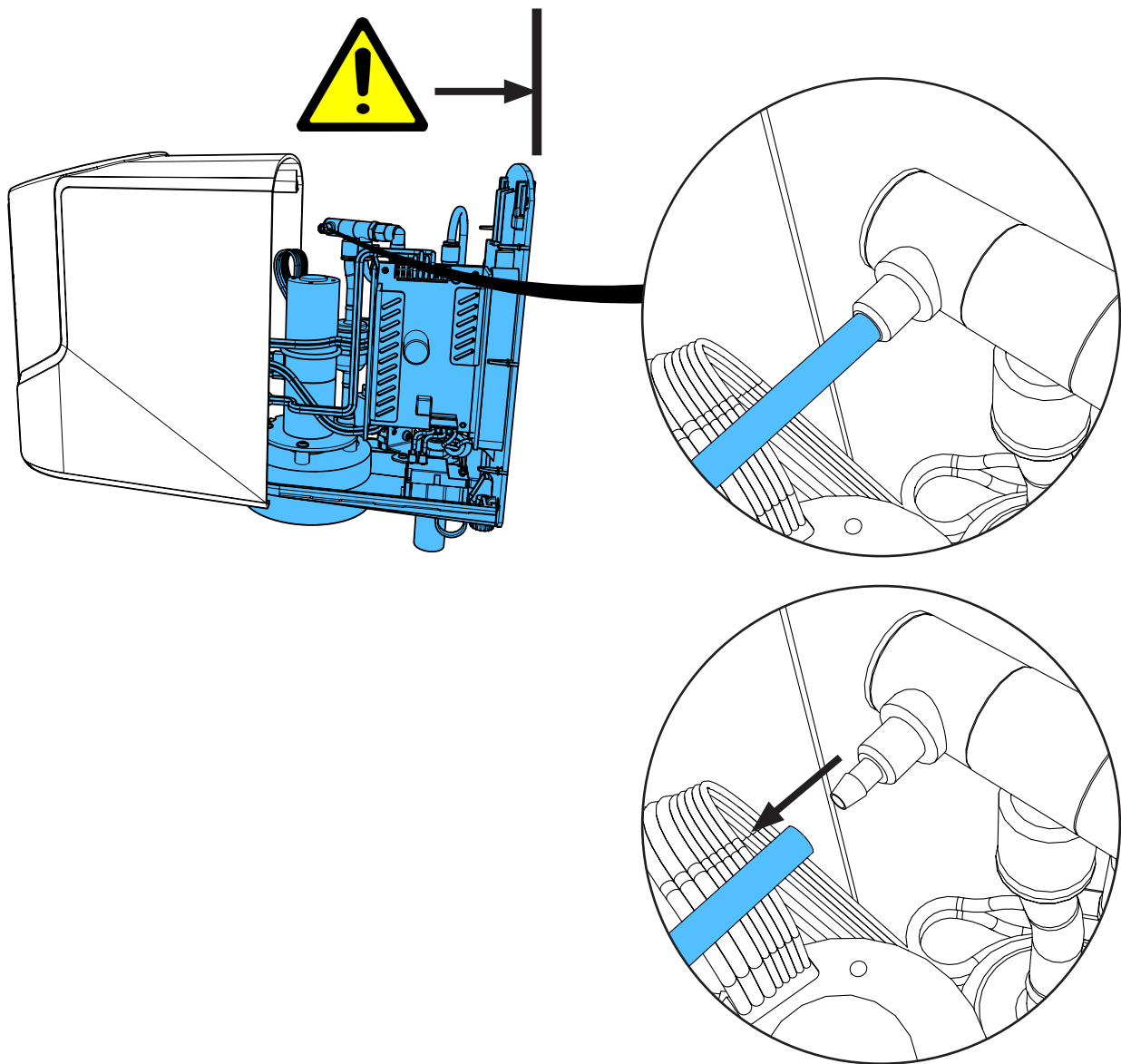



6

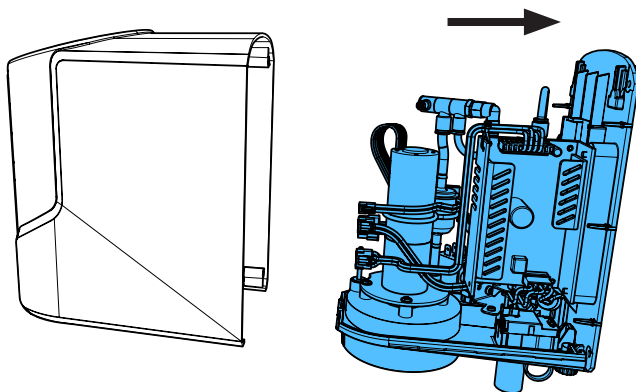



7

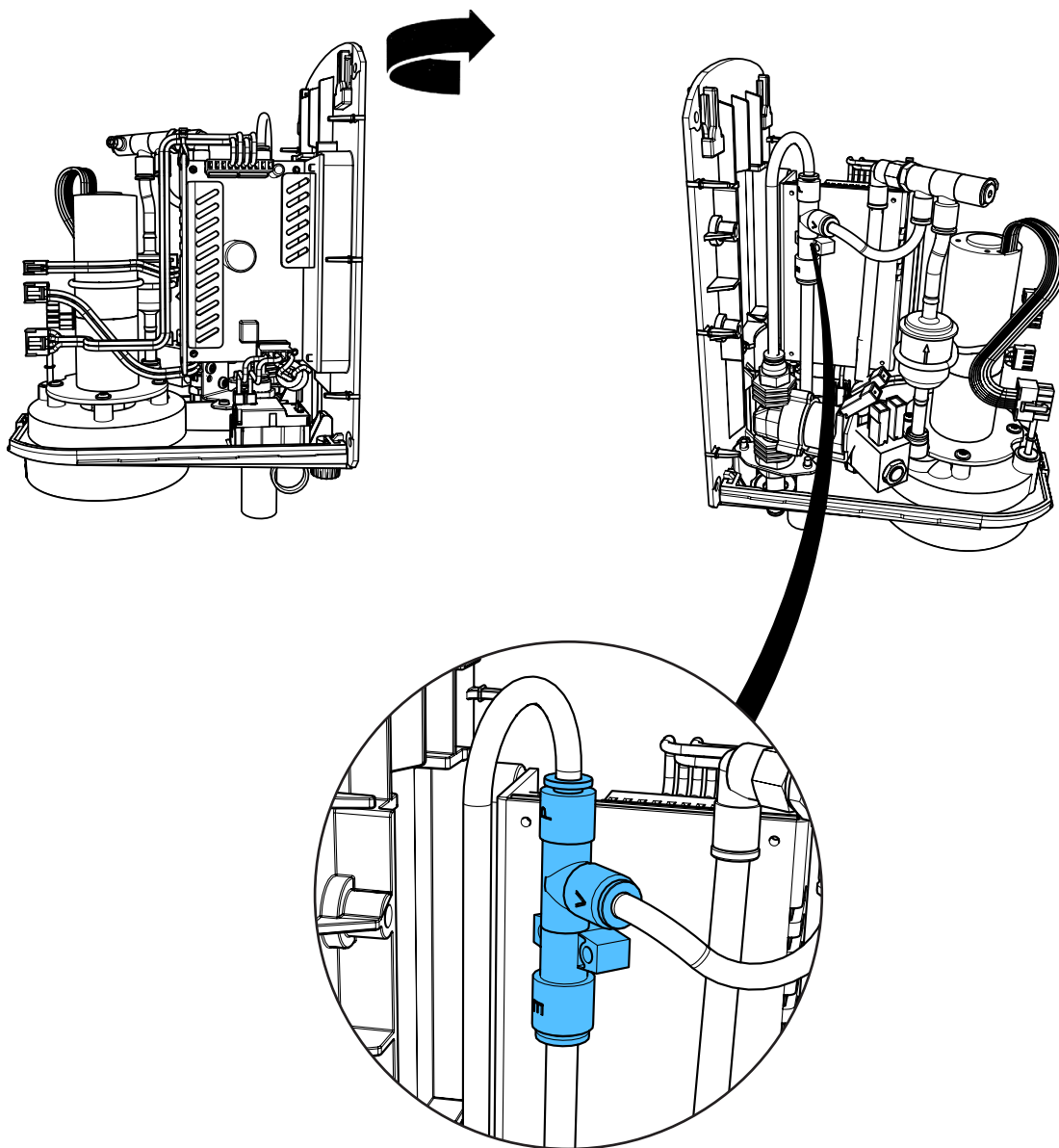



8

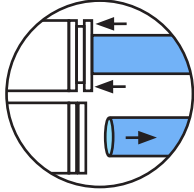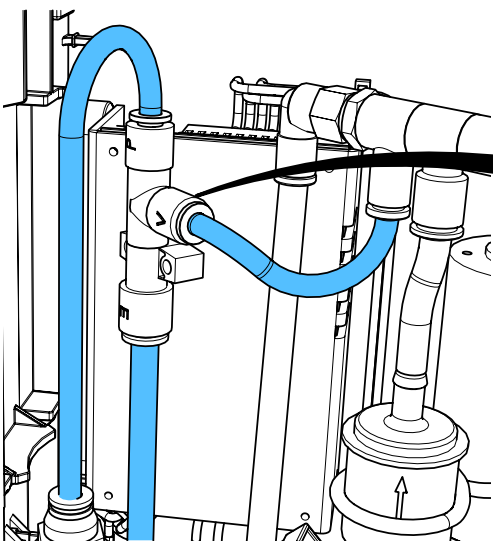


**i**

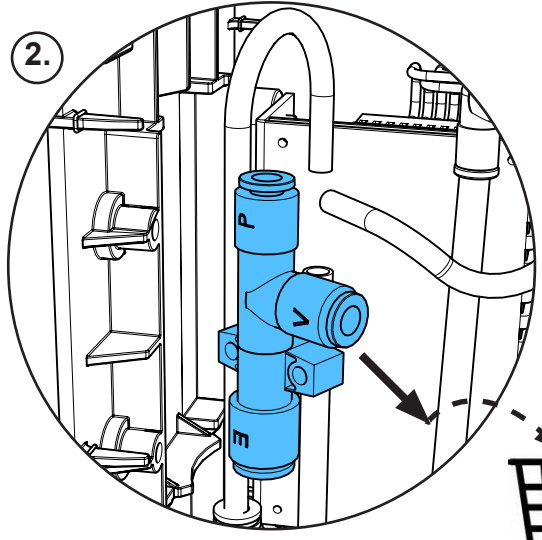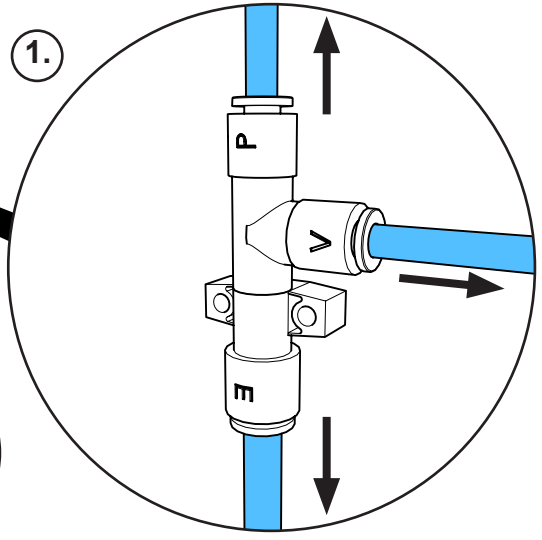


9

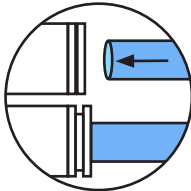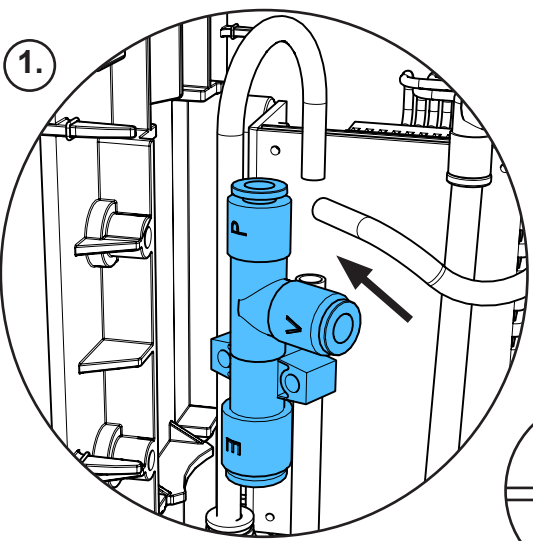


**i**

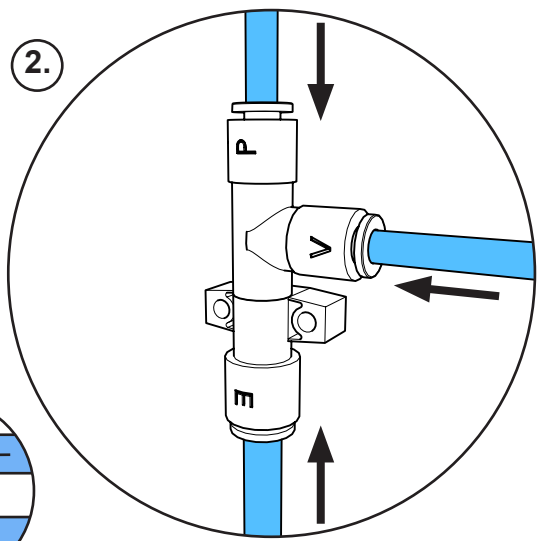

10

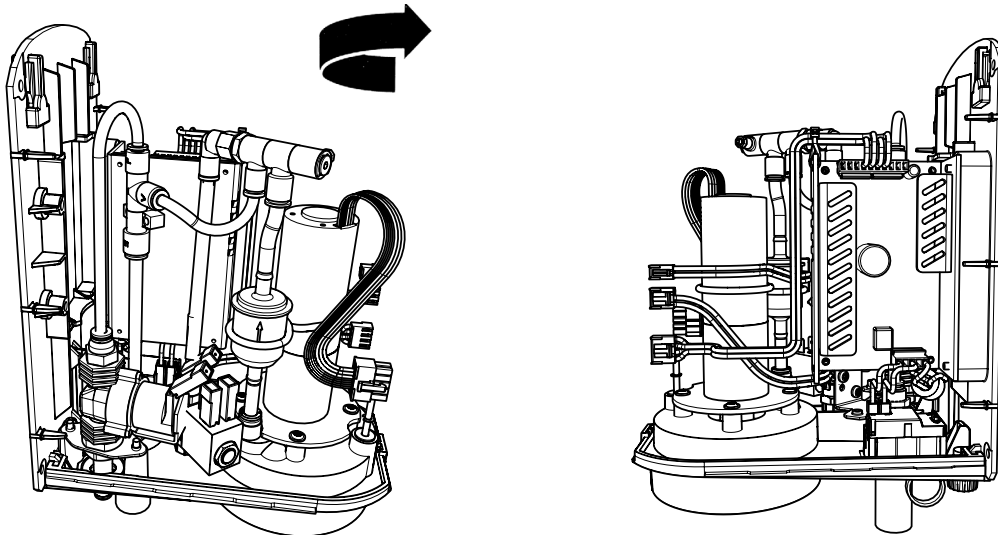


11

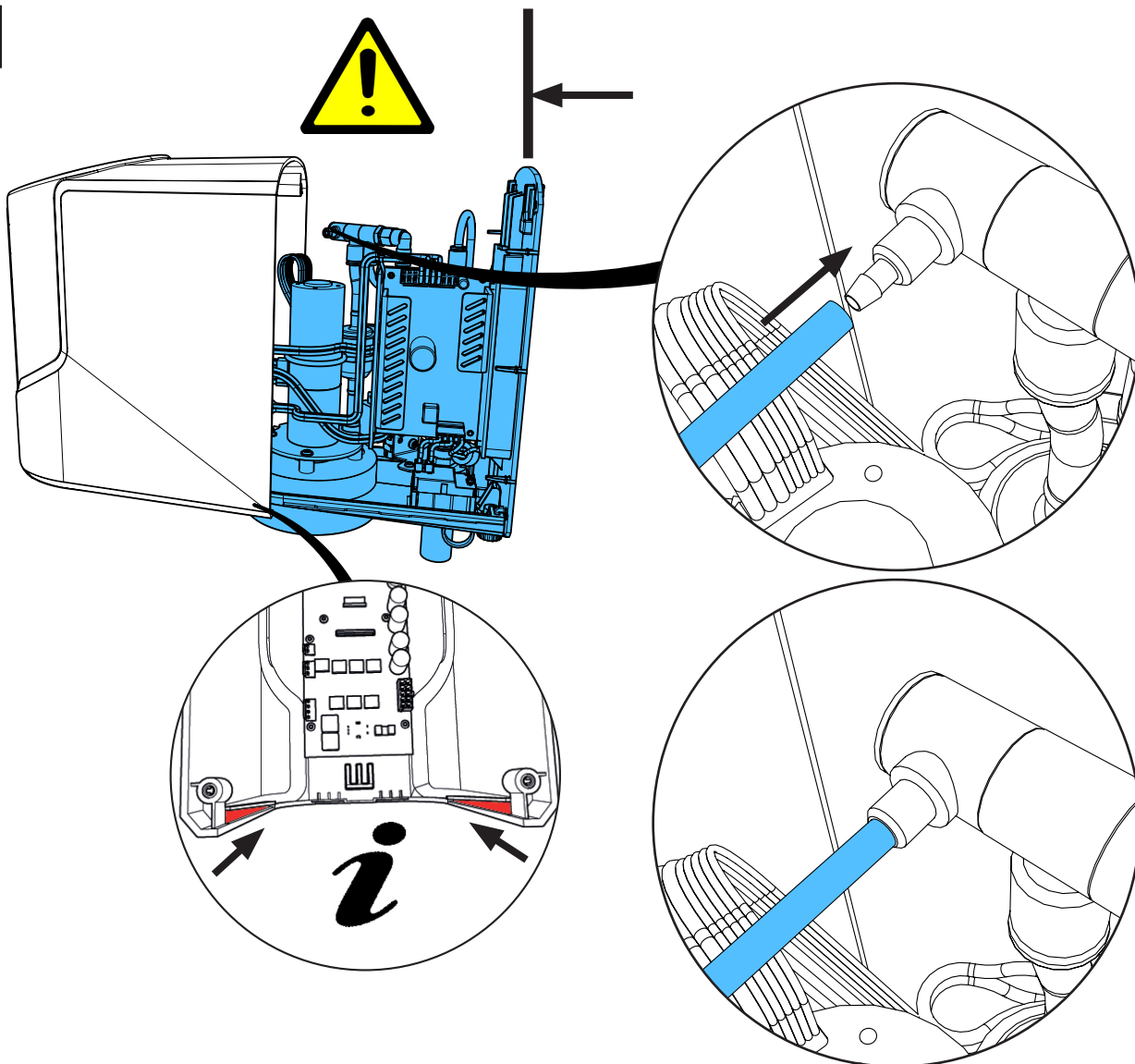





12

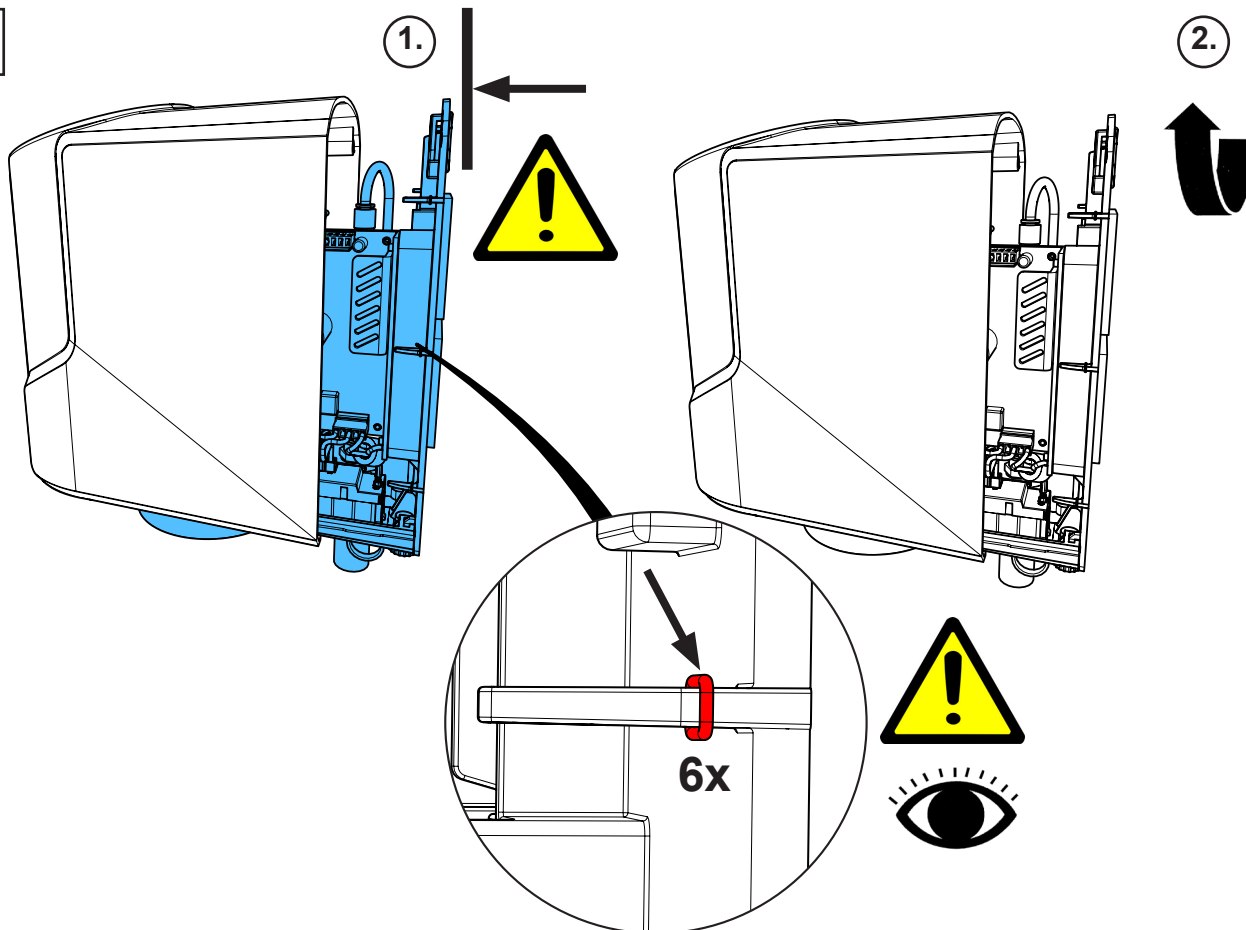


13

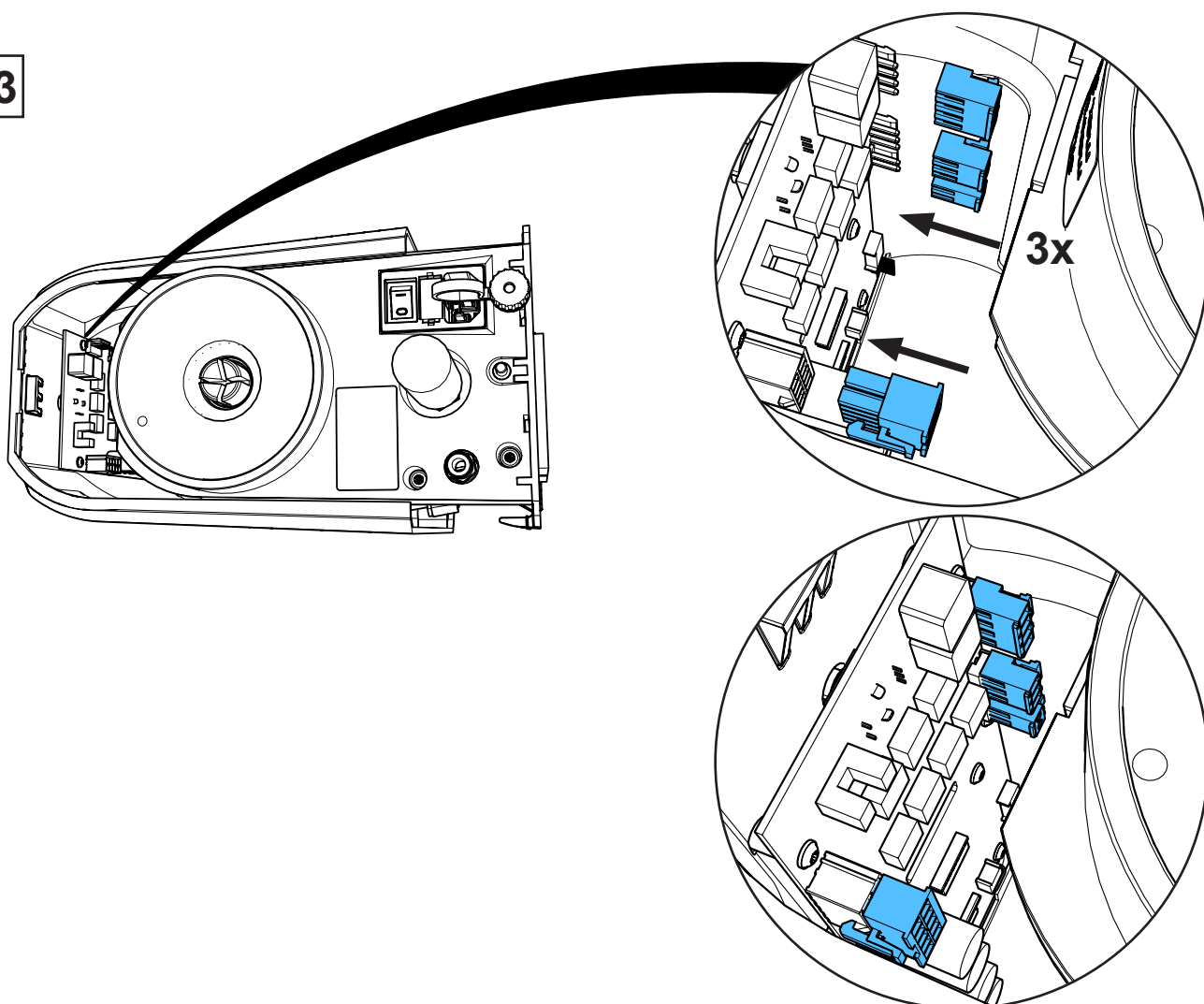


14

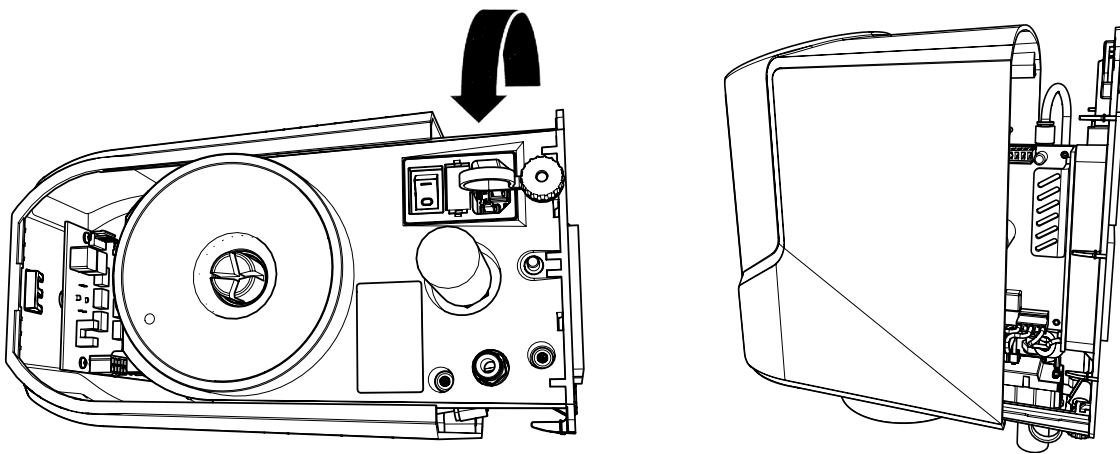

15



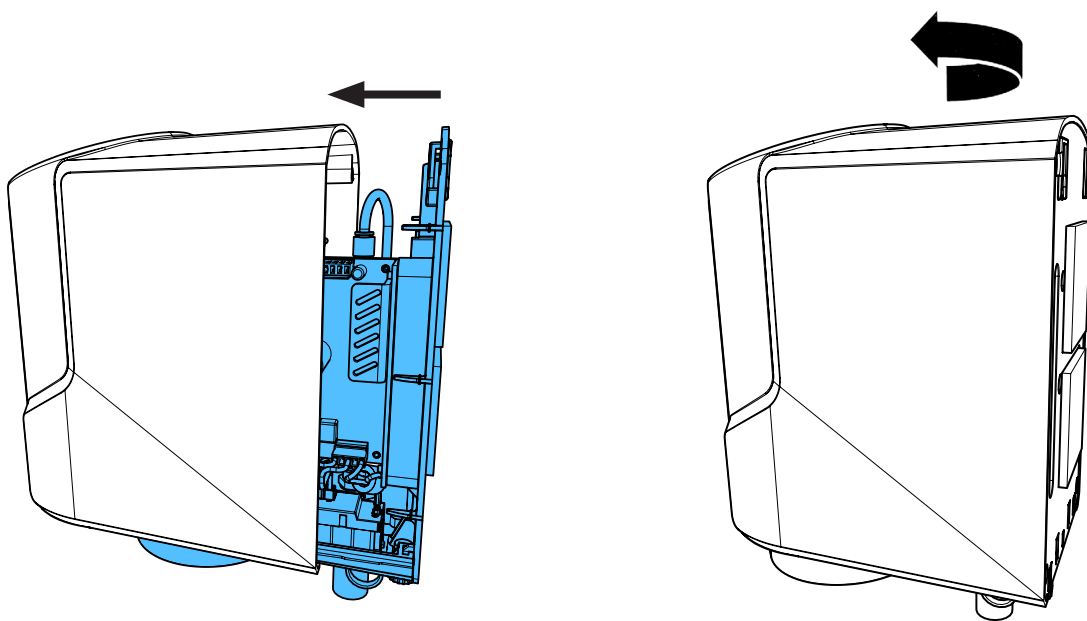


16

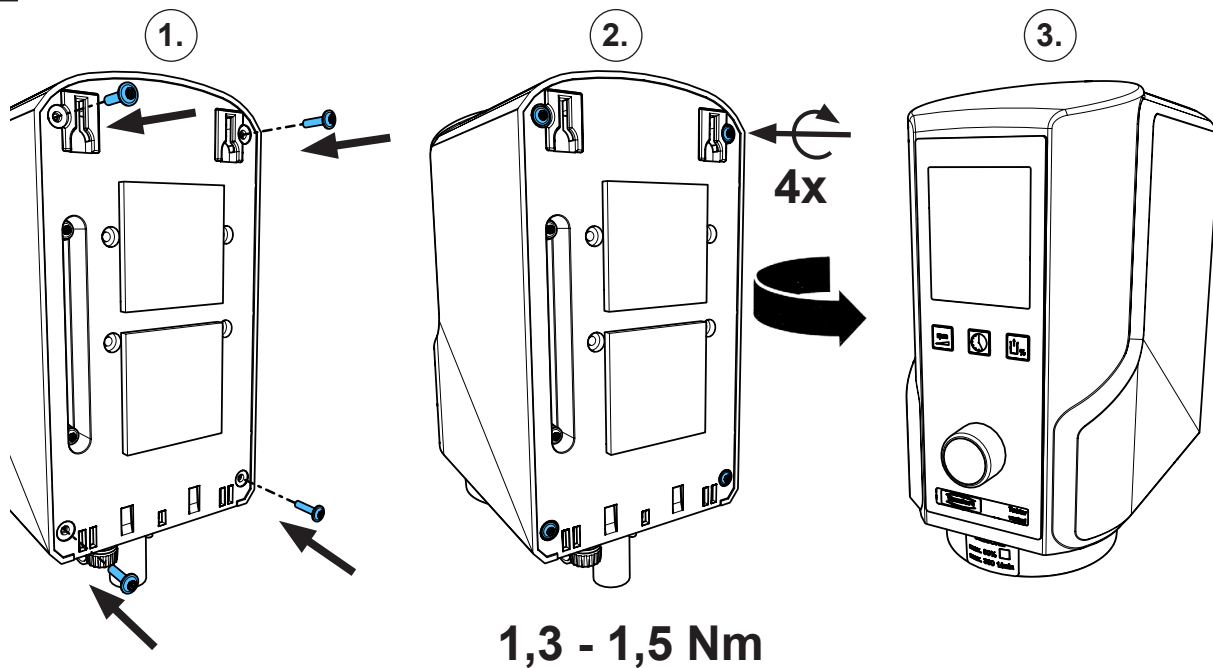


## 90003 4823


Display Twister • Display Twister • Afficheur Twister • Display Twister  
Pantalla de visualización Twister • Дисплей Twister

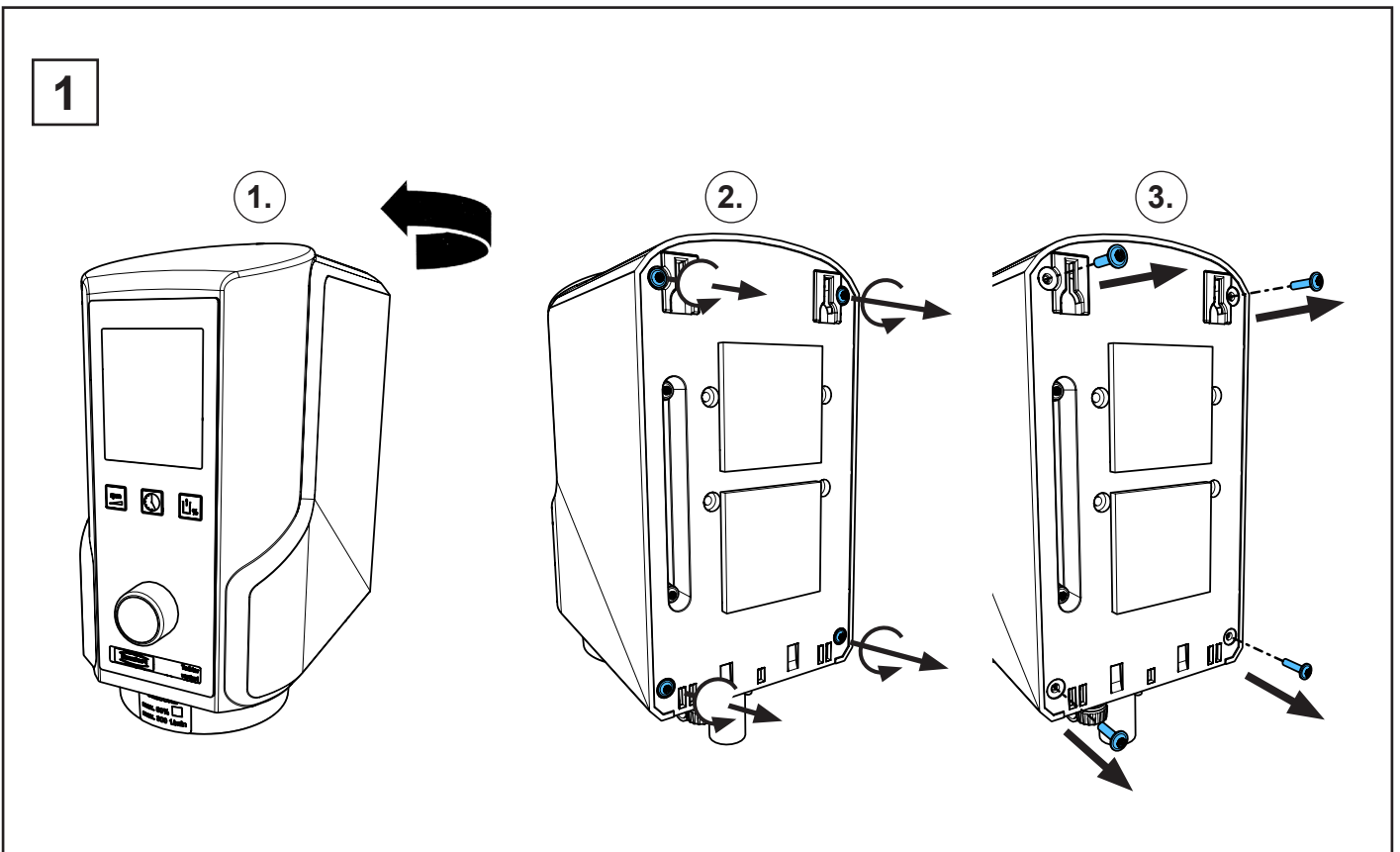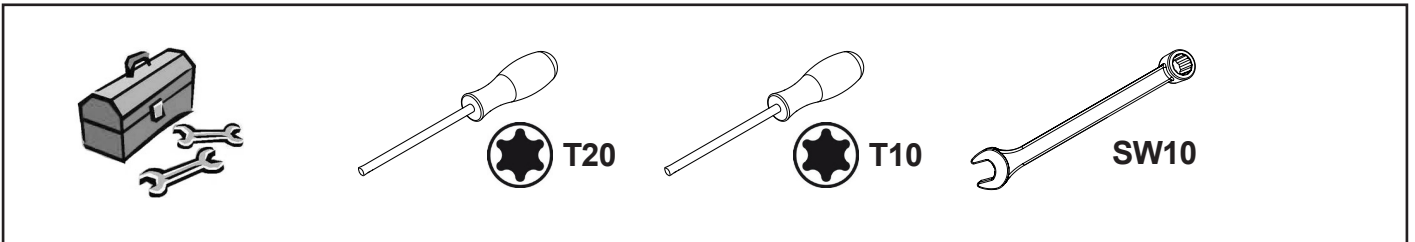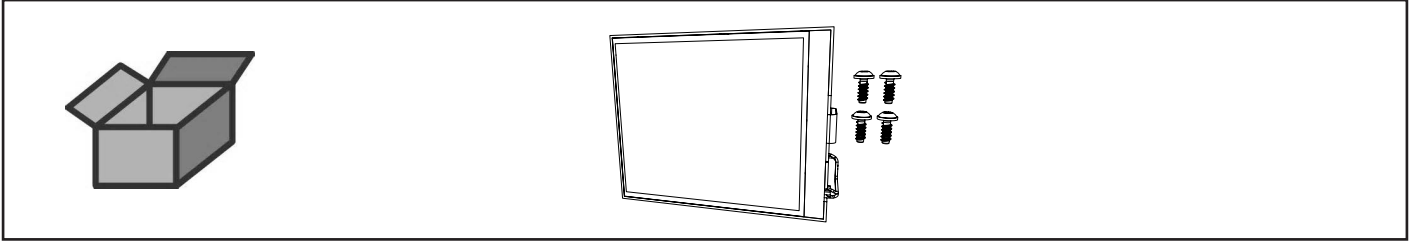



2

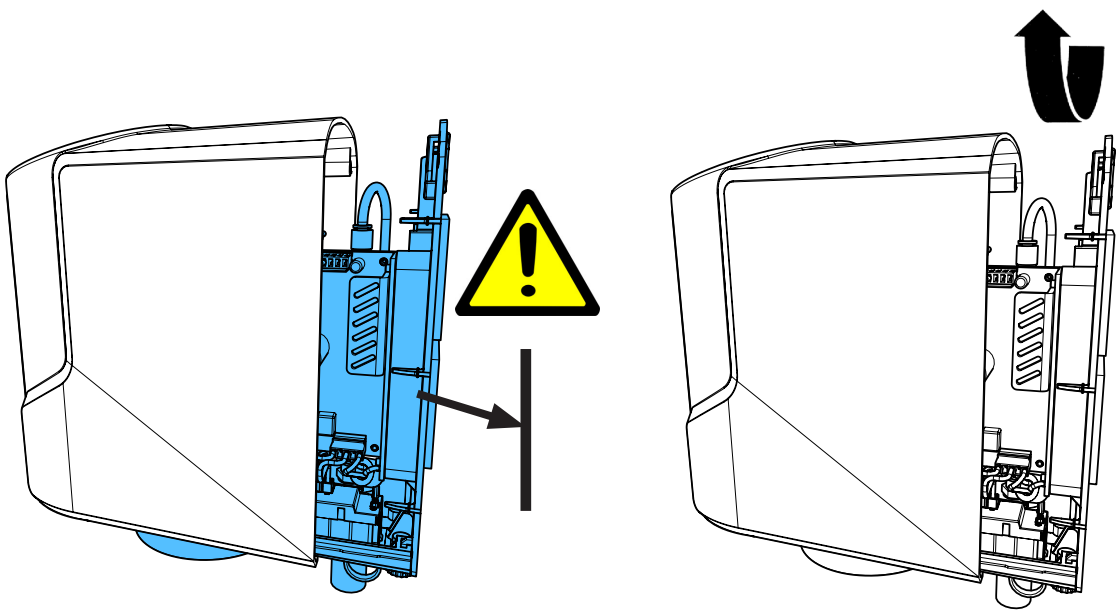





3

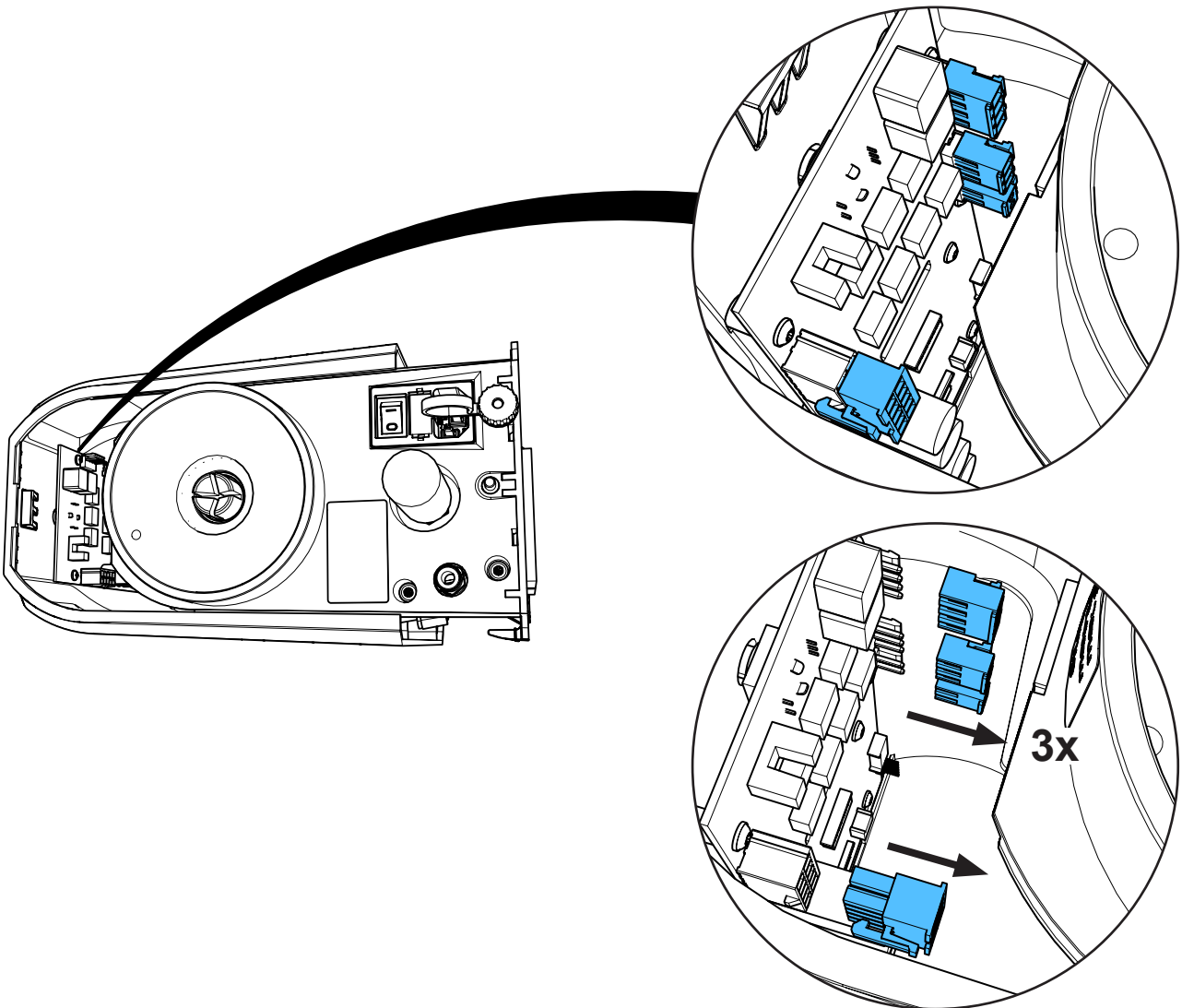


4

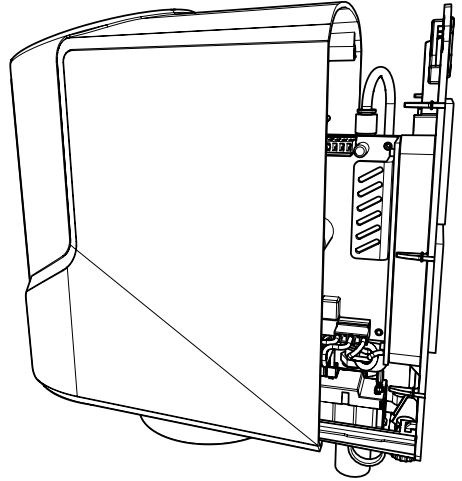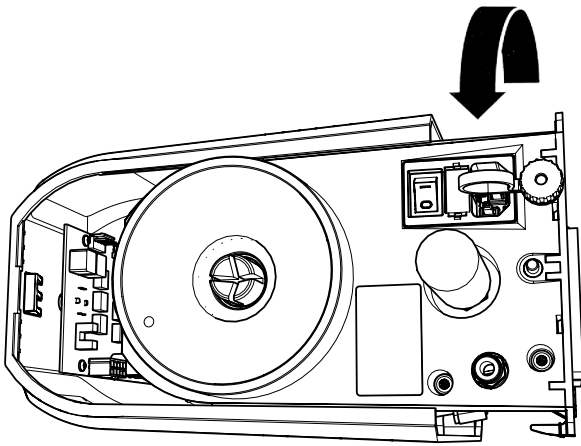


5

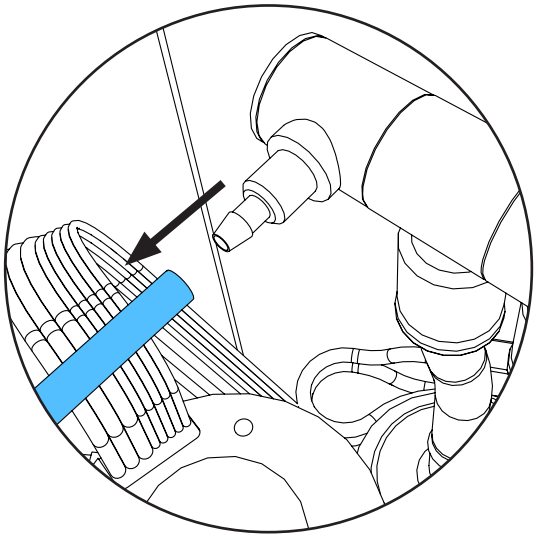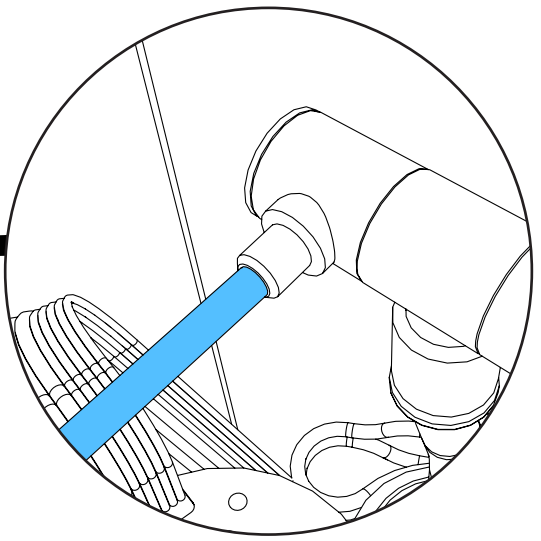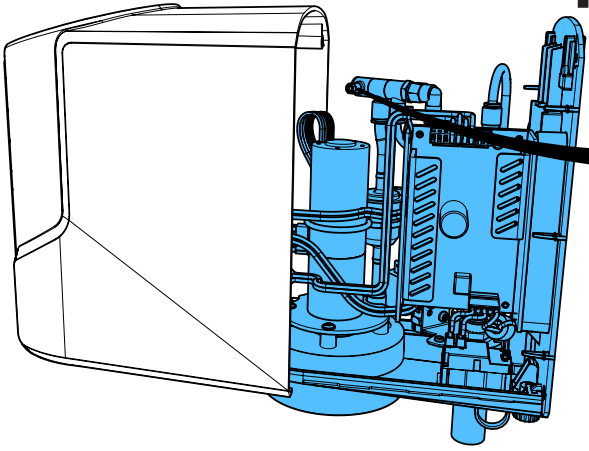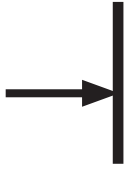


6

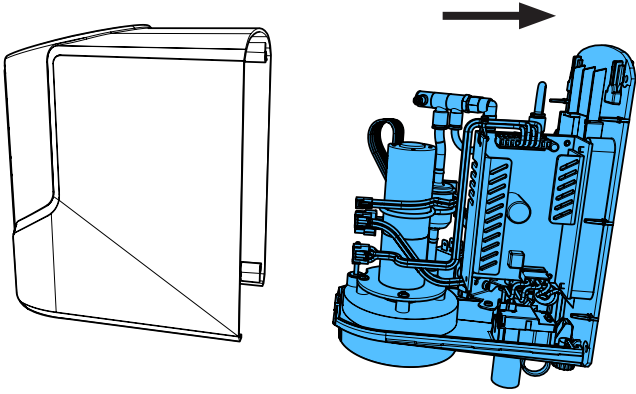





7

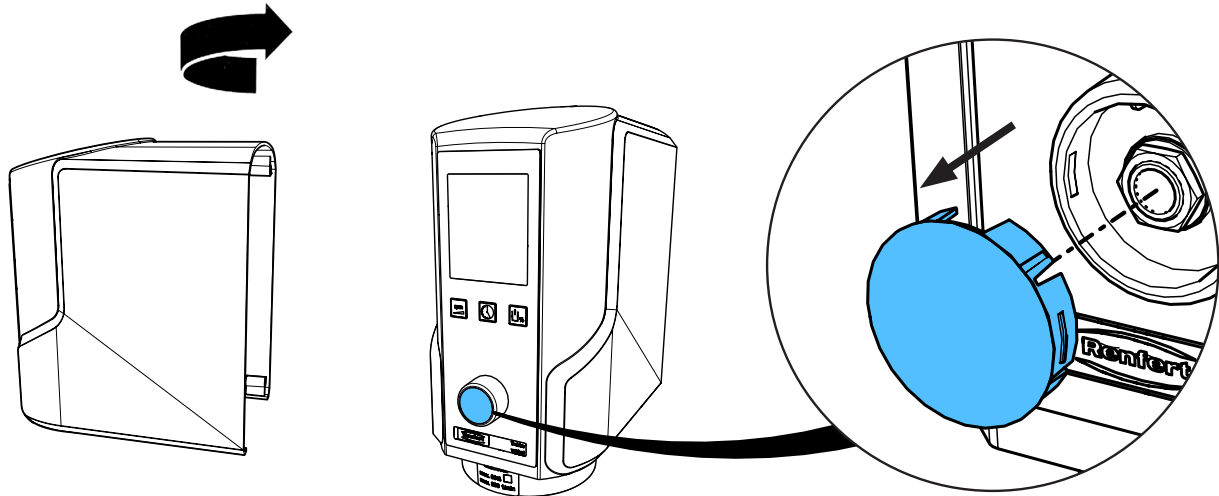




8

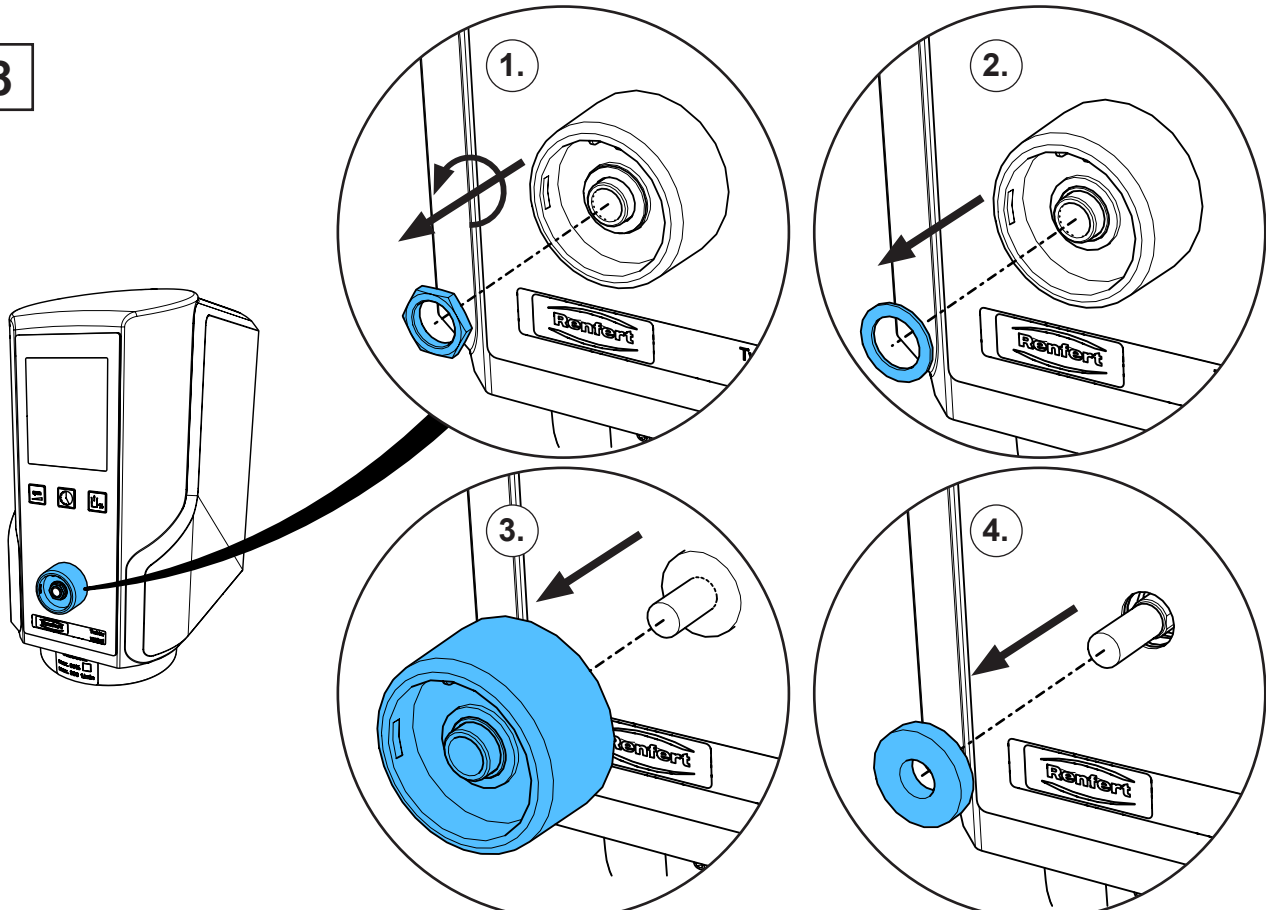




9

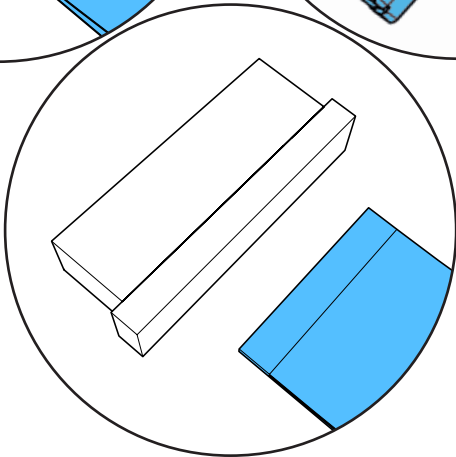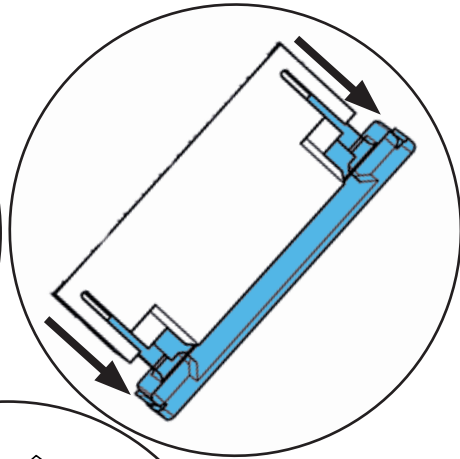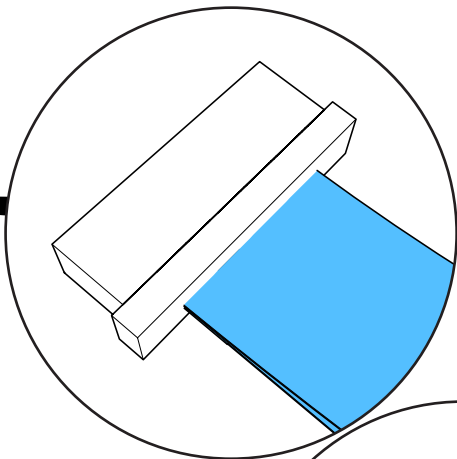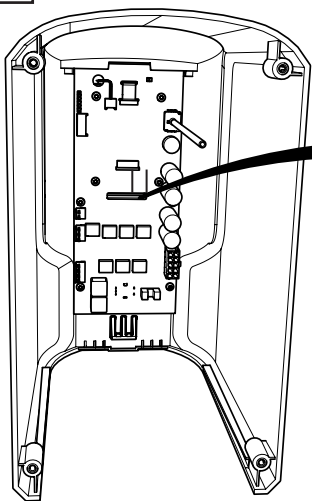


10

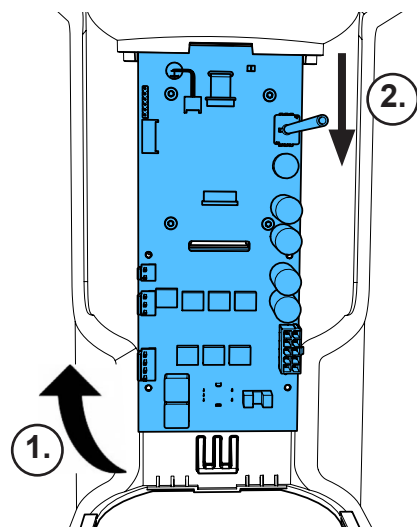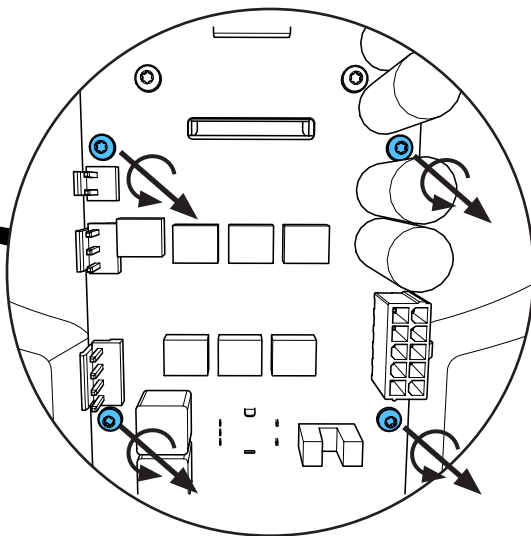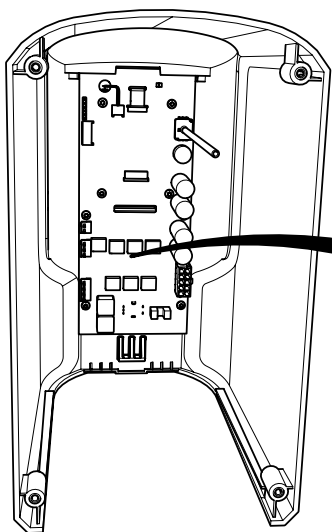


11

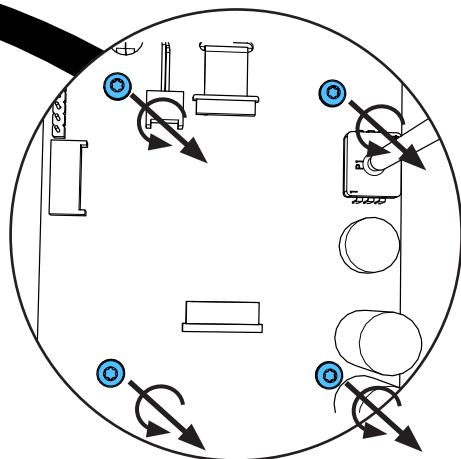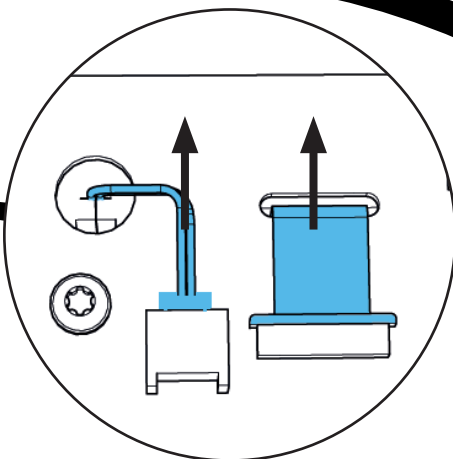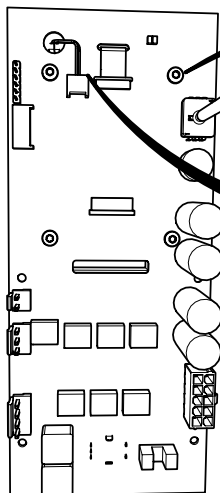




12

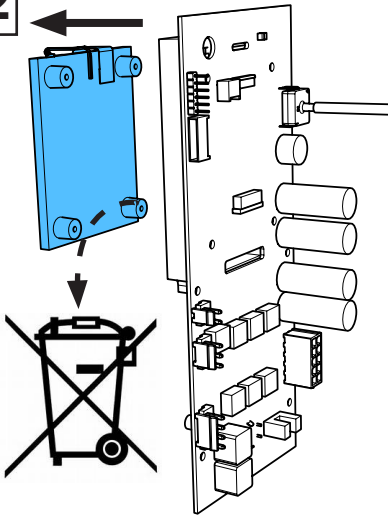

13

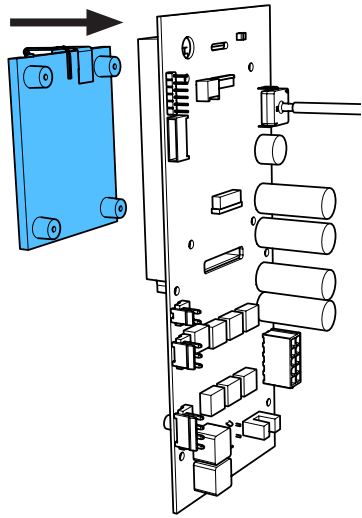



14

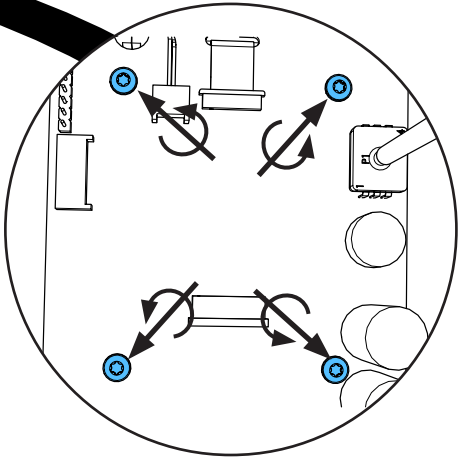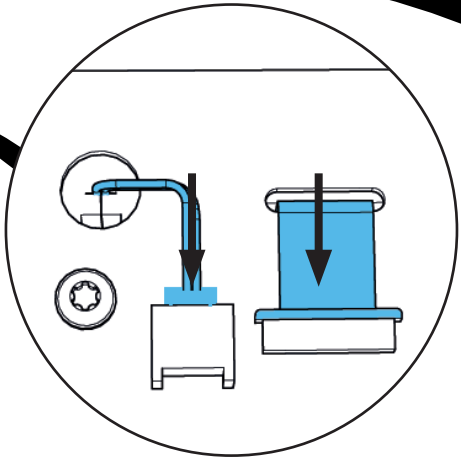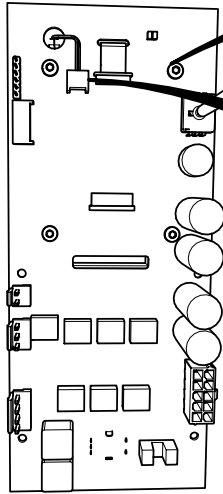

0,9 - 1,1 Nm



15

*i*

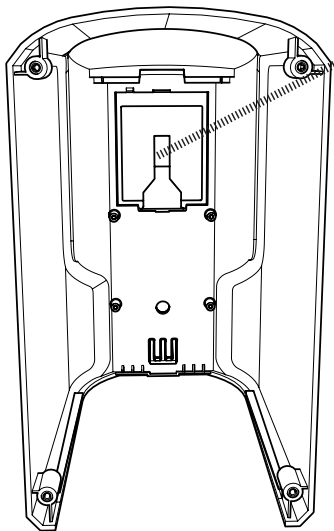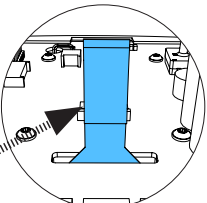



1.

2.

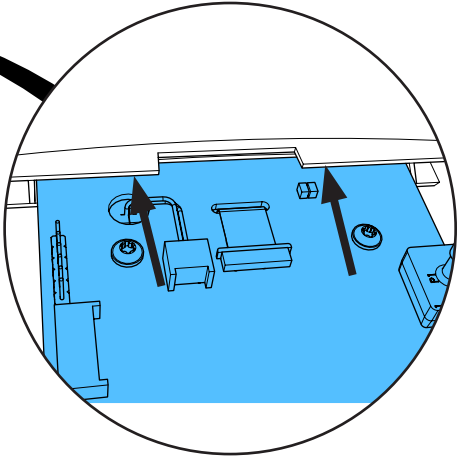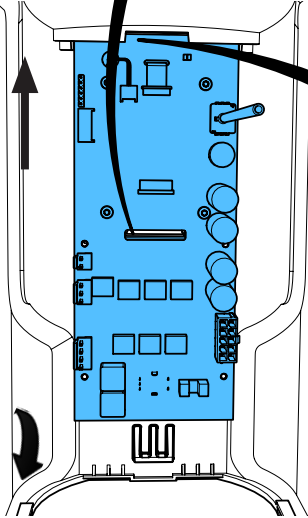





16

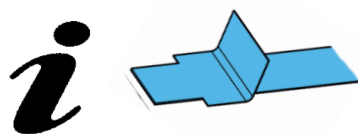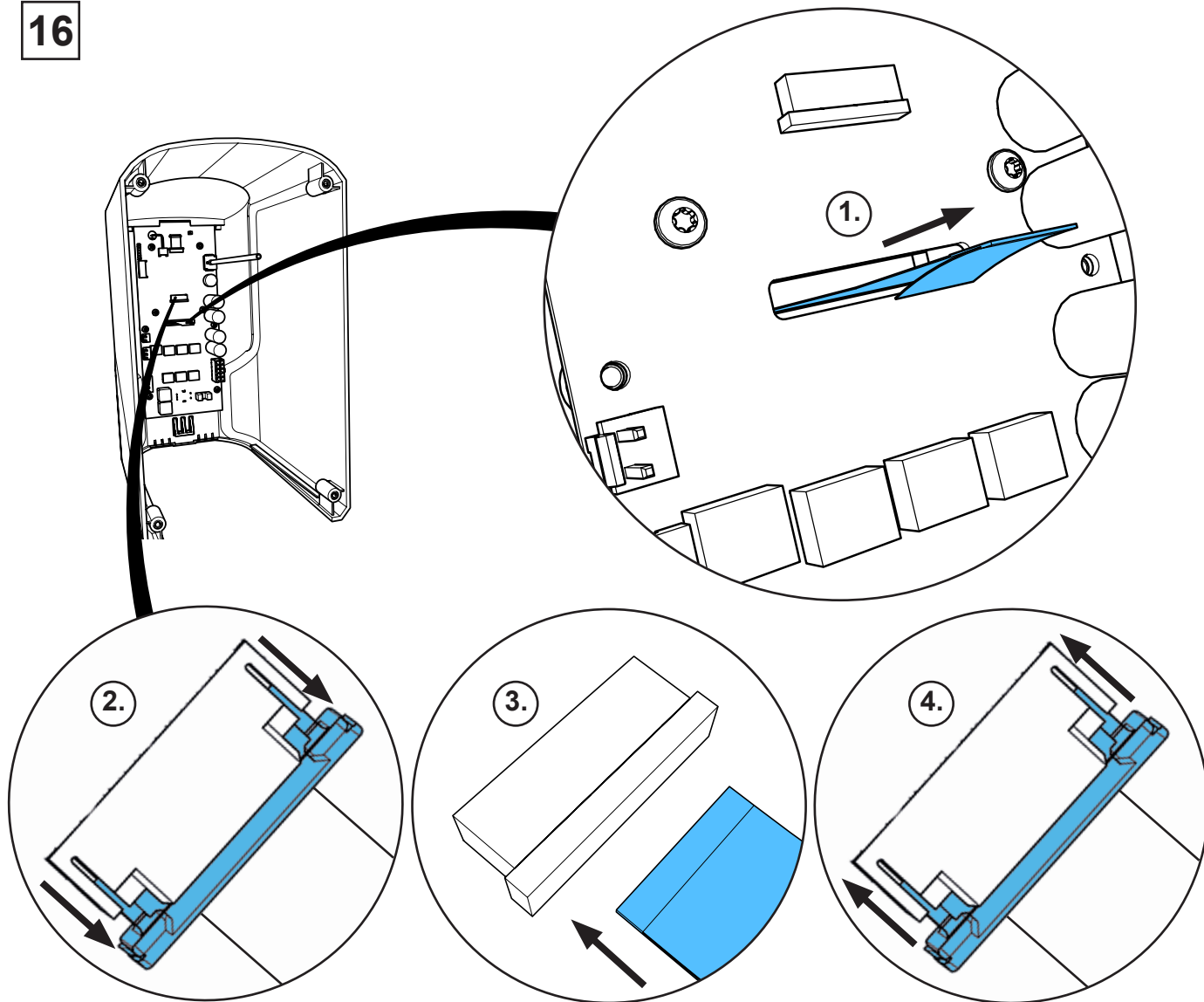


17

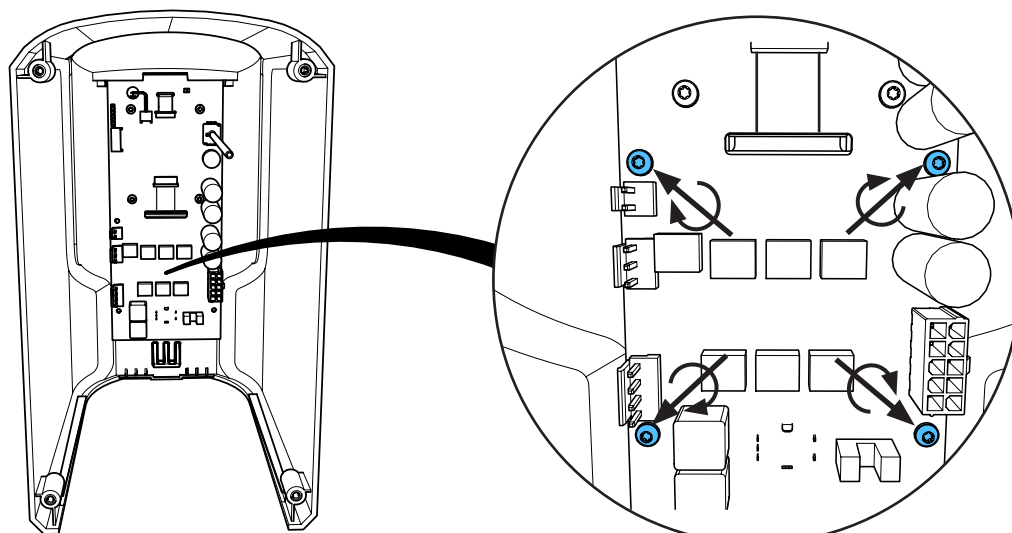


18

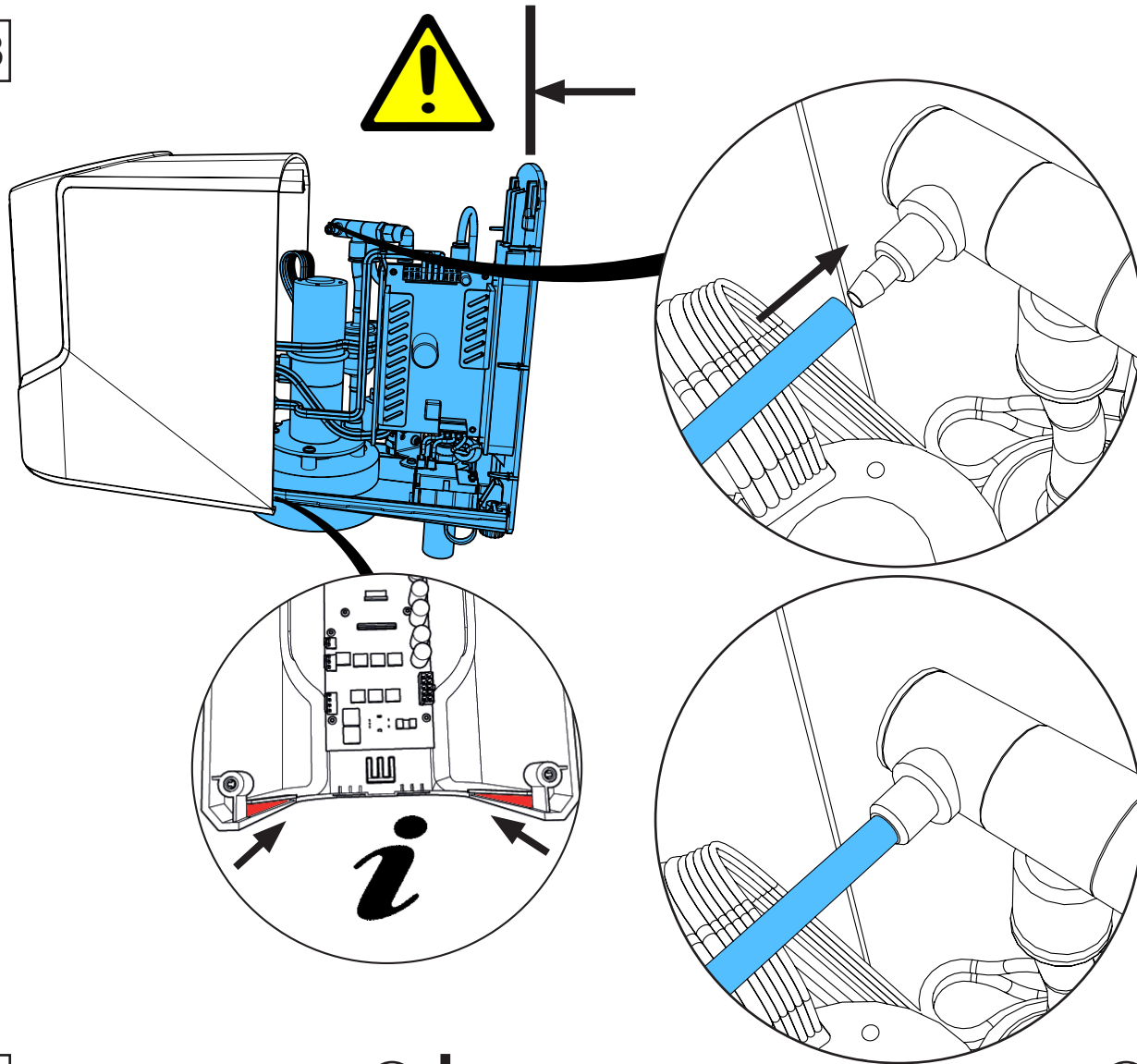


19

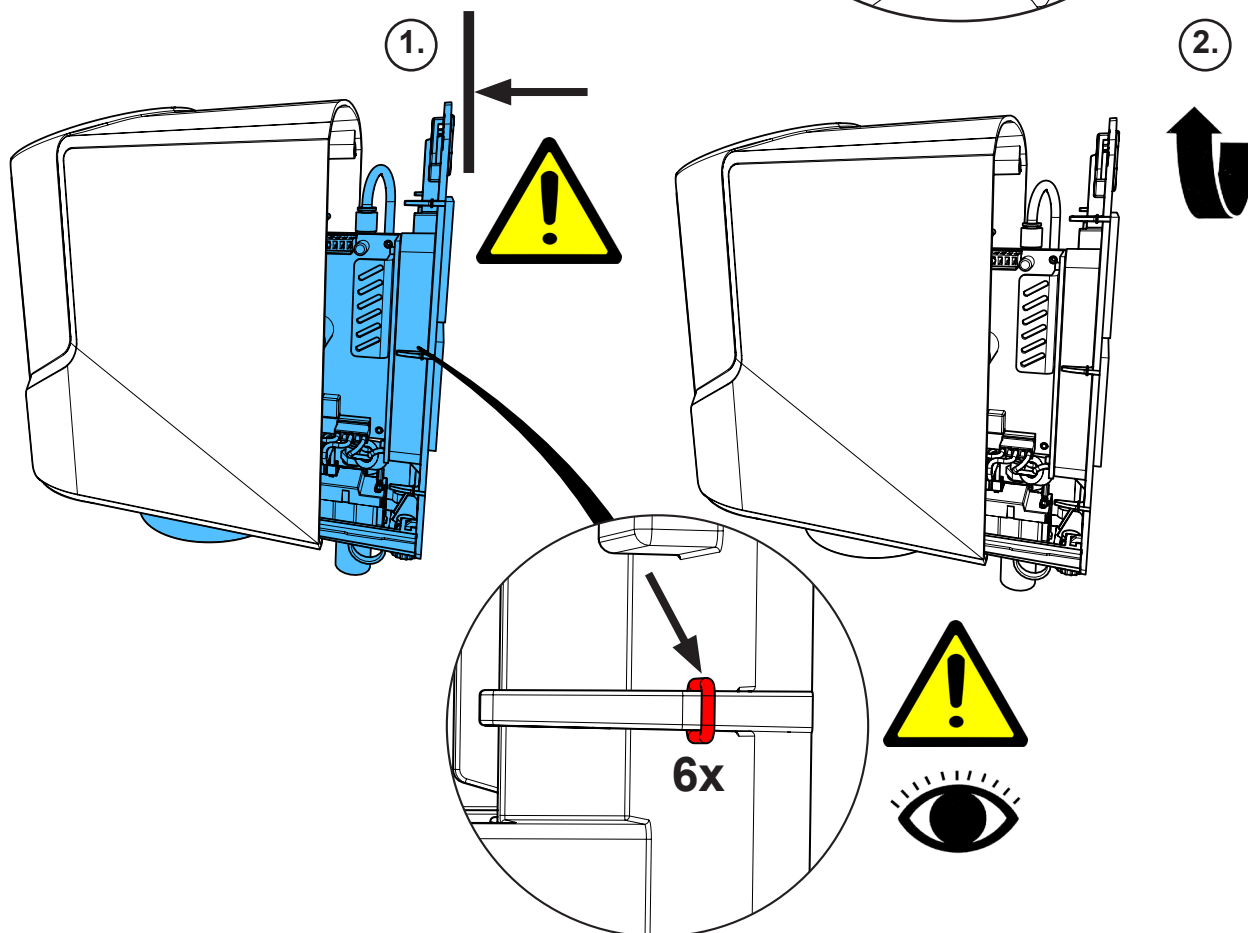


20

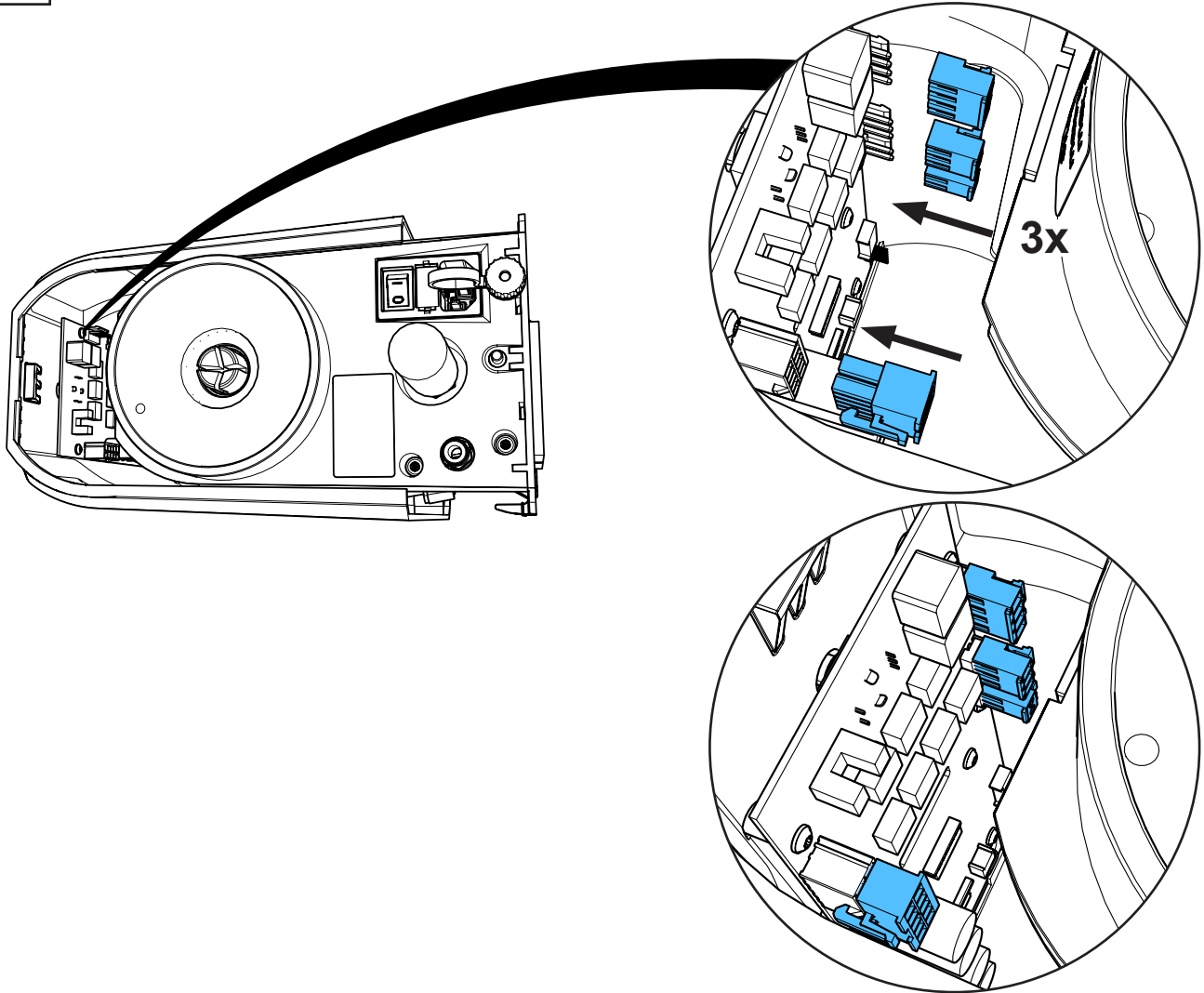


21

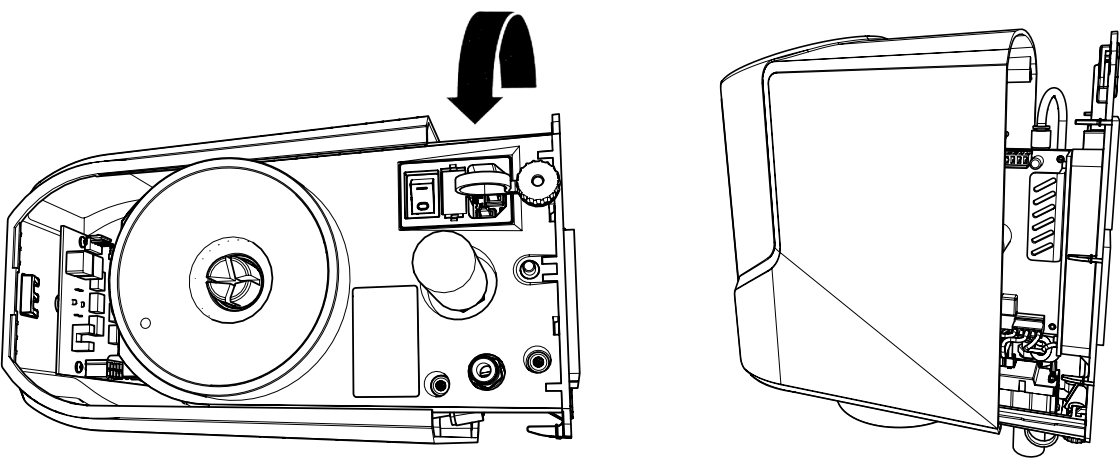

22

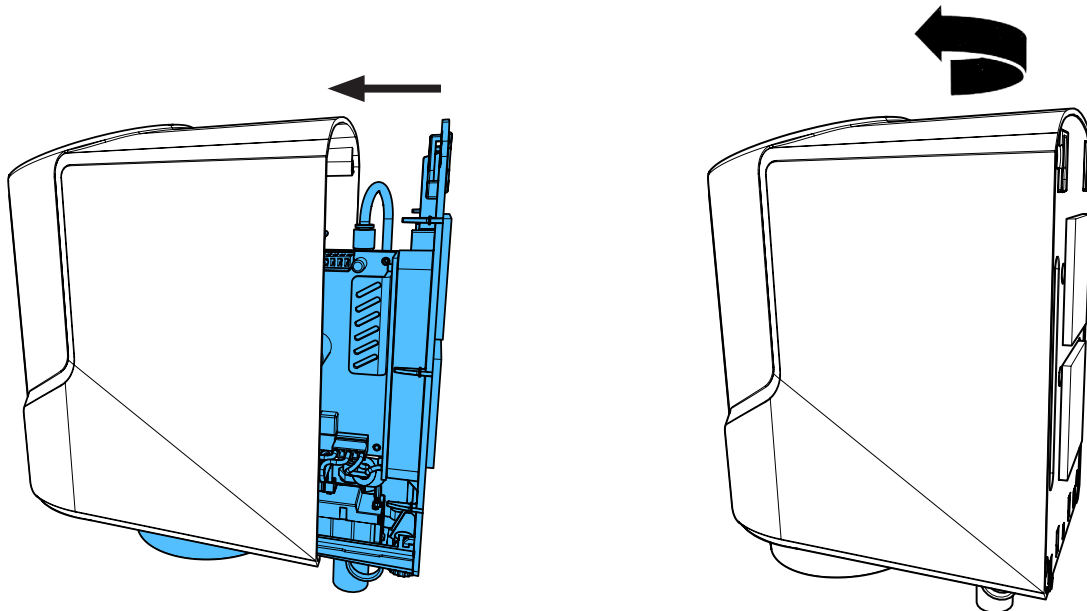

23

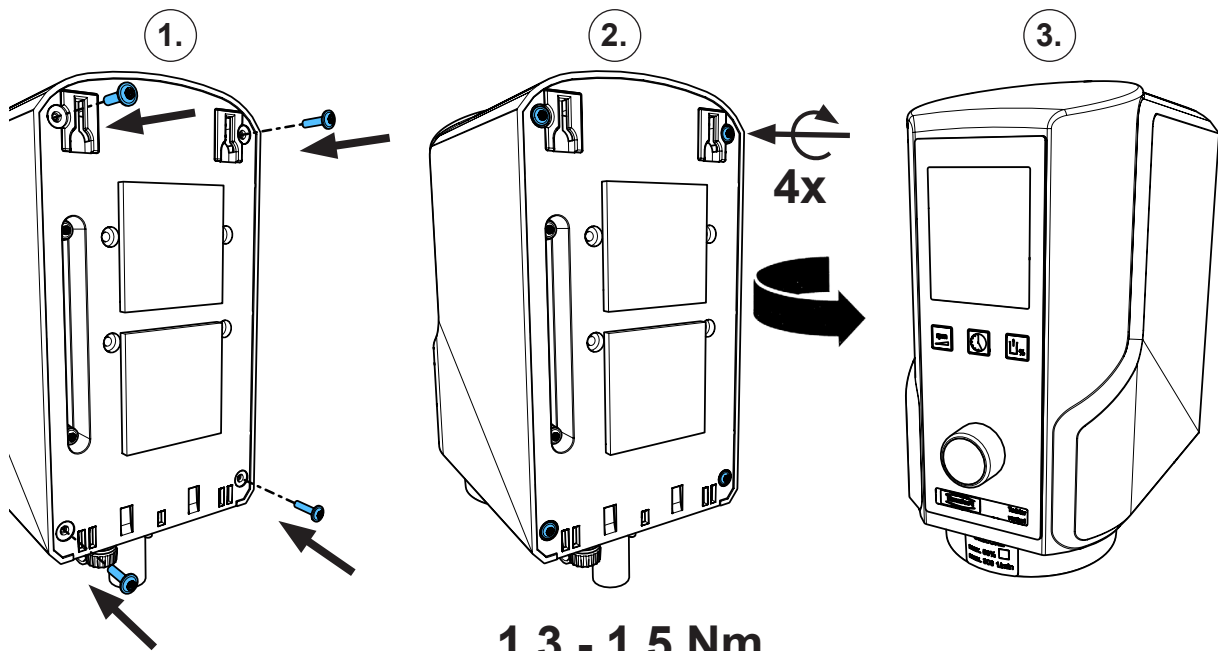



**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422